# ***SYMPROfluid Universal***

**No. 6500-0600**

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**No. 6500-0600**

## **Description:**

Special cleaner for use in the SYMPRO high-performance cleaning device from Renfert.

Ready-for-use solution for easy removal of stubborn deposits such as plaque, tartar, plaster residues, stains (tea, coffee and red wine, etc.) and denture adhesive.

## **Dosage and application:**

200 ml per application

## **Please observe the processing order!**

**Without denture adhesive deposits, continue directly with 2.**

1. When soiled with denture adhesive deposits, roughly clean the denture using a dry disposable cloth first.
2. Clean using 200 ml SYMPROfluid Universal in high-performance cleaning device for approx. 10 – 30 min. (white beaker).

**With nicotine stains (otherwise continue with 4.)**

3. Place into the grey beaker and immerse entirely with SYMPROfluid Nicoclean. Immersion time approx. 5 – 10 min (without the high-performance cleaning device).

*With especially stubborn stains it may be necessary to alter to a longer running time (+ max. 10 min) and then short, manual cleaning, under running water, using a hard toothbrush.*

4. Rinse and disinfect the cleaned object thoroughly before transferring to the patient.

*The cleaning liquid can become contaminated with bacteria from the inserted object. For this reason: Always use fresh cleaning liquid for every new cleaning sequence! The cleaning needles can be used more than once.*

5. Empty the beaker:
  - Place holding magnet at the base of the cleaning beaker.
  - Carefully pour the cleaning liquid out.
  - Rinse the beaker and cleaning needles with clean water.

*If the denture was soiled with denture adhesive, clean the beaker including the needles thoroughly in hot water. This is the only way to remove loose denture adhesive particles from the beaker.*

**Hotline application support:**

Workdays tel.: +49 7731 82 08-961

Email: didicher@renfert.com

**Warnings and safety advice:**

*SYMPROfluid Universal* is classed as a harmful material according to GefStoffV / EC regulations.

**Hazard information:**

Irritant, irritates the eyes, respiratory organs and skin.

**First Aid measures:**

- Remove soiled, soaked clothes immediately.
- After inhaling, ensure of fresh air.
- After skin contact, wash hands immediately using plenty of water and soap. If skin is irritated please consult a doctor.
- After eye contact, rinse immediately with plenty of water and consult a doctor.
- After swallowing, rinse mouth out immediately and drink plenty of water. Do not induce vomiting. Consult a doctor immediately.

**In every case, please observe the information on the safety data sheet. (Download under [www.renfert.com](http://www.renfert.com)).**

**Personal protective equipment:**

- Wear certified protective gloves.
- Airtight protective goggles.

**The liquids must not come into contact with the following surfaces:**

PVC, linoleum, natural stone, stainless steel, wooden surfaces

**Physical and chemical properties:**

Acid cleaning liquid, pH=1,

density: 1040g/l

Identification of constituents according to EC regulation no.648/2004, Annex VII:

< 5% nonionic tensides, fragrance

**Information on disposal:**

The used cleaning liquid can be disposed of in the waste water system after having been neutralised, as long as the contaminated deposits allow.

Neutralise with a mixture ratio of 1:1 *SYMPROfluid Universal* and *SYMPROfluid Nicoclean* or request information from your local authorities about proper disposal.

Waste code: 20 01 29 cleaning liquid, which contains harmful materials.

The tensides are bio-degradable according to EC regulation for detergents No. 648/2004

**Delivery, storage:**

2000 ml container

Store only in the closed original container between -5°C and 40°C. Keep cool, dark and in an upright position.

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