# ***SYMPROfluid Nicoclean***

**No. 6500-0610**

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**No. 6500-0610**

## **Description:**

Special cleaner for the removal of stubborn nicotine stains.

Ready-for-use solution for the effective removal of stubborn nicotine deposits and stains.

## **Application:**

*SYMPROfluid Nicoclean* is for use after the *SYMPROfluid Universal*.

1. Place object into the grey beaker and immerse completely in *SYMPROfluid Nicoclean*. Immersion time approx. 5 – 10 min (without high performance cleaning device).  
To remove the remaining residues, simply brush clean in warm water with a toothbrush or denture brush.
2. Rinse the cleaned object thoroughly in water and disinfect before transferring back to patient.

*The cleaning liquid can become contaminated with bacteria from the inserted object. For this reason: Always use fresh cleaning liquid for every new cleaning sequence!*

3. Empty the beaker:
  - Carefully pour the cleaning liquid out.
  - Rinse with clean water.

## **Hotline application support:**

Workdays tel.: +49 7731 82 08-961

Email: didicher@renfert.com

## **Warnings and safety advice:**

*SYMPROfluid Universal* is classed as a harmful material according to GefStoffV / EC regulations.

## **Hazard information:**

Irritant, irritates the eyes, respiratory organs and skin.

## **First Aid information:**

- Remove soiled, soaked clothes immediately.
- After inhaling, ensure of fresh air.
- After skin contact, wash hands immediately using plenty of water and soap. If skin is irritated please consult a doctor.
- After eye contact, rinse immediately with plenty of water and consult a doctor.
- After swallowing, rinse mouth out immediately and drink plenty of water. Do not induce vomiting. Consult a doctor immediately.

**In every case, please observe the information on the safety data sheet. (Download under [www.renfert.com](http://www.renfert.com)).**

**Personal protective equipment:**

- Wear certified protective gloves.
- Airtight protective goggles.

**The liquids must not come into contact with the following surfaces:**

PVC, linoleum, natural stone, stainless steel, wooden surfaces.

**Physical and chemical properties:**

Acid cleaning liquid, pH=12, density: 1020g/l

Identification of constituents according to EC standard no. 648/2004, Annex VII: < 5% nonionic tensides, fragrance

**Information in disposal:**

The used cleaning liquid can be disposed of in the waste water system after having been neutralised, as long as the contaminated deposits allow.

Neutralise with a mixture ratio of 1:1 *SYMPROfluid Universal* and *SYMPROfluid Nicoclean* or request information from your local authorities about proper disposal. Waste code: 20 01 29 cleaning liquid, which contains harmful materials. The tensides are bio-degradable according to EC regulation for detergents No. 648/2004

**Delivery, storage:**

2000 ml container

Store only in the closed original container between -5°C and 40°C. Keep cool, dark and in an upright position.

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