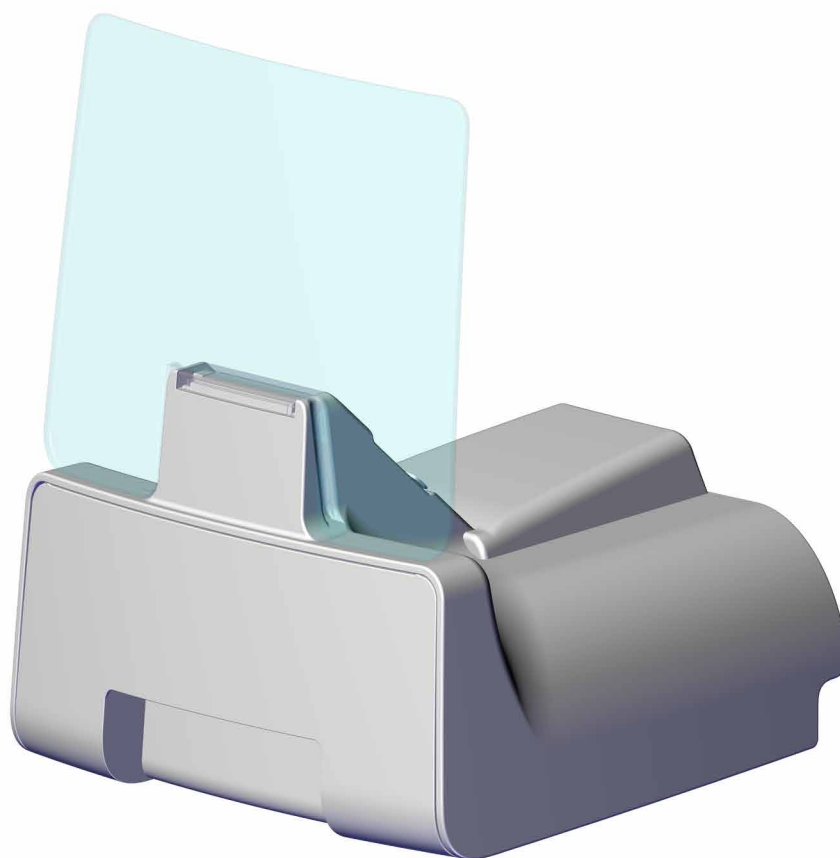
making work easy

de
en

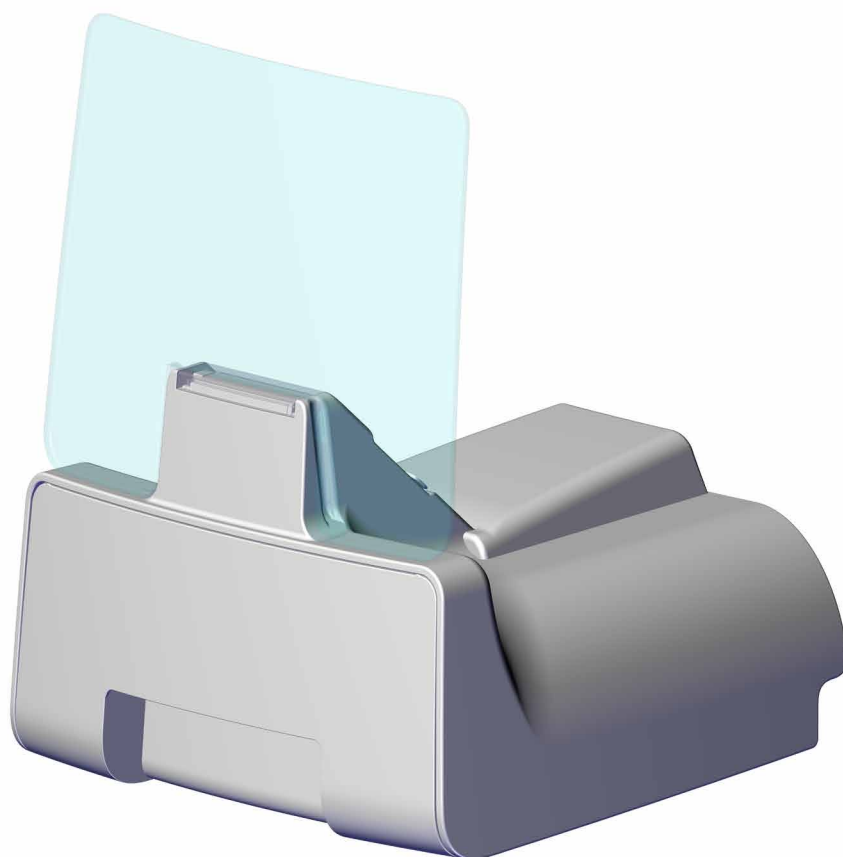
Hygieneleithaden

Made in Germany

205182 15092026

making work easy

Hygieneleithaden

Inhalt

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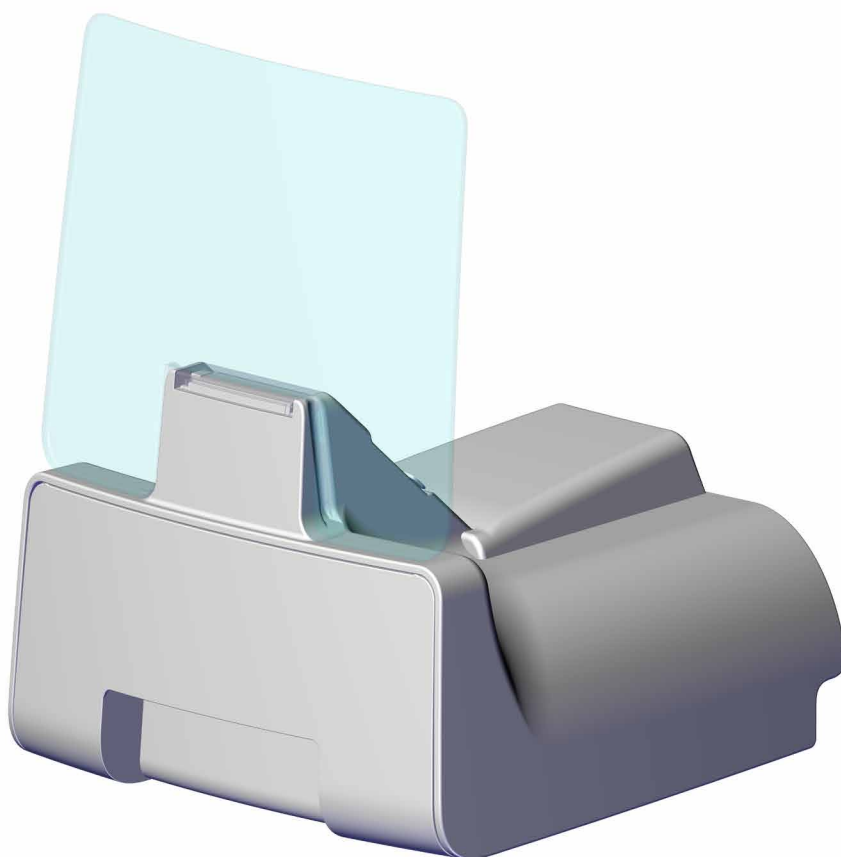
<https://www.renfert.com/silent-xs>

en

Here you will find the following documents and information for download:

- Instructions For Use
- Quick Start Guide
- Spare parts lists and spare parts drawings
- Repair instructions
- FAQs, cause and rectification of faults
- How-to videos
- Declarations of conformity

Please note: The documents and information listed are not available for all products.

Intended use

This unit is intended for commercial use in dental labs or in the environment of dental offices in accordance with the IFU (Instructions For Use). The unit is intended for use in the following tasks:

- Suction of dry, non-explosive dust during minor fitting work on disinfected objects.

Accessories:

The block is only intended for use in a dental technician's environment and is not suitable for use with patients.

Unpacking & startup:

CAUTION: The unit must never be operated without filter elements

See separate Quick Start Guide!

After fully assembling and charging the unit, carry out the disinfection steps described in Section "B".

- ▶ Disinfect the dental restoration with a suitable product listed by the VAH (Association for Applied Hygiene) before using the suction unit (repeat disinfection before and after each patient contact).
- ▶ It is the user's responsibility to separately comply with country-specific regulations regarding compliance with reprocessing procedures.

A: Cleaning

Danger of electrical shock

⇒ Only clean and disinfect the unit when it is switched off (disconnect the mains plug).

Health hazard due to contamination and dust

⇒ Clean and disinfect the surface after each treatment.

⇒ Check the filter fill level daily.

⇒ Do not use compressed air for cleaning (this may result in a defective fan motor)

Grinding residues can be hazardous to health.

Wear appropriate personal protective equipment (mouth/nose protection, gloves) when cleaning and disposing of dirt and other contaminants.

Housing / pane / grille

⇒ Remove pane.

⇒ Use a dry cloth to wipe away any loose dirt or coarse particles from the entire surface of the unit, including pane, pane holder and lighting.

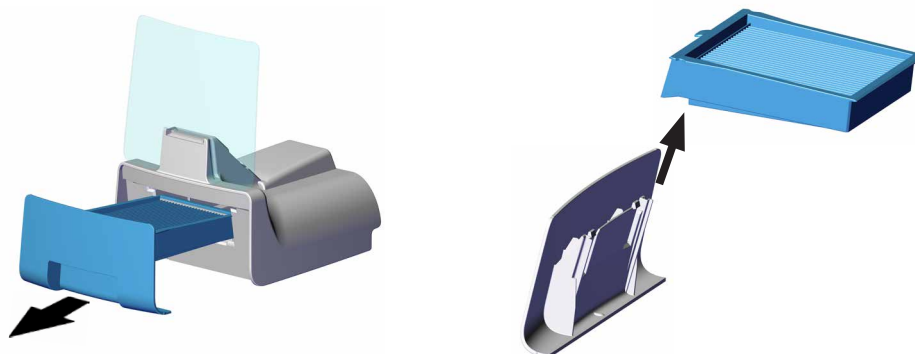
Use a soft cloth and avoid applying too much pressure to prevent scratches on the pane.

⇒ Remove the suction grille and manually remove any stuck particles.

⇒ Dispose of particles, dirt and other contaminants appropriately. *

Dust drawer panel / filter element

⇒ Remove the dust drawer panel, including the filter element.

⇒ Properly dispose of any material that has accumulated in the filter.

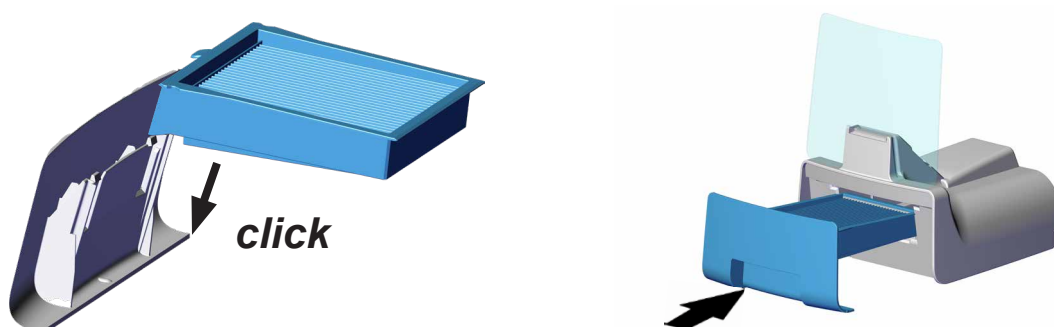
⇒ Separate the filter element from the drawer panel.

⇒ Additionally, use a dry cloth to clean the inner surface of the drawer panel.


⇒ Replace the filter element once it has reached the end of its service life, and dispose of the used filter properly.

⇒ Attach the filter to the dust drawer panel by clicking it into place.

⇒ Slide in the dust drawer. Pay attention to correct fit.

Exhaust air grille / exhaust air filter

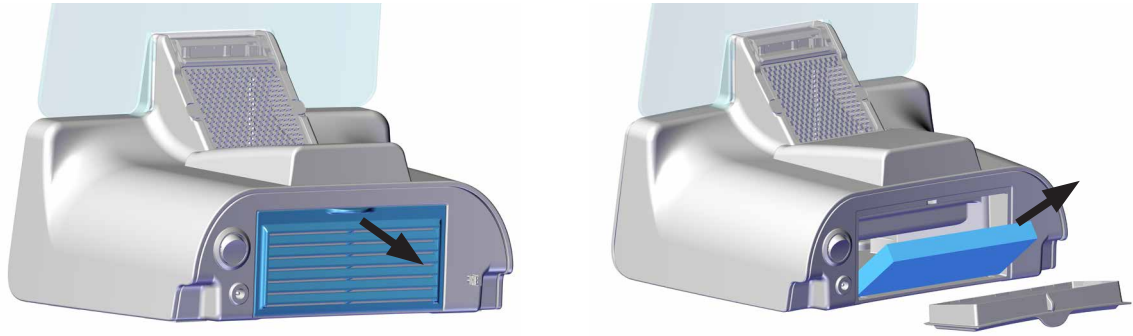- ⇒ Remove the exhaust air cover.
- ⇒ Remove the filter fleece and replace it after it has reached the end of its service life. Dispose of the used filter fleece properly.
- ⇒ Clean the inner surface of the exhaust air cover with a dry cloth.
- ⇒ Clean accessible areas of the duct to the fan with a dry cloth.
- ⇒ Insert a new filter fleece into the exhaust air cover.
- ⇒ Replace the exhaust air cover with the inserted filter fleece. Pay attention to correct fit!

B: Desinfection

Do not use cleaners that contain solvents, are alkaline, or abrasive!
Do not use solvent-based disinfectants (e.g., BIB forte / Alpro Medical)!

Caution: Never spray disinfectant directly onto the unit or any of its parts!

For disinfection, use a disinfectant wipe listed by the VAH (Association for Applied Hygiene), e. g. Caviwipes (Metrex), for disinfection.

Disinfection must be performed before initial use and after each subsequent treatment.

Procedure:

- ⇒ Remove suction grille and wipe with disinfectant wipe. Disinfect carefully, paying particular attention to any perforations or cavities.
- ⇒ Wipe the entire surface of the housing, including the pane, with a disinfectant wipe.
Pay particular attention to areas that are difficult to access, such as:
Lighting; cavity at the pane/grille holder; recessed grip on the front of the dust drawer; exhaust air grille; feet; and screws. Disinfect these items carefully.
- ⇒ Insert grille and pane
- ⇒ If necessary, move the pane to its parking position.

Your unit is now ready for its next use!

Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany