making work easy

de

en

fr

it

es

pt

ru

ja

ko

sv

fi

SILENT XS

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Made in Germany

205138 15092026

making work easy

SILENT XS

Inhalt

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Contents

en

1	Online help and information	3
1.1	Symbols used	3
2	Safety	3
2.1	Intended use	3
2.2	Meaning of these Instructions For Use (IFU)	3
2.3	Requirements for operators	4
2.4	Safe condition of the unit	4
2.5	Hazards as a result of modifications and repairs	4
2.6	Risk as a result of electric shock and fire	4
2.7	Risk of injury to the eyes or skin	4
2.8	Risk to health and fire hazard due to dust	4
2.9	Risk to health due to infection	4
3	Product description	5
3.1	General description	5
3.2	Hygiene guidelines	5
3.3	Assemblies and functional elements	5
3.4	Scope of delivery	6
3.5	Accessories	6
4	Startup	6
4.1	Unpacking	6
4.2	Insert batteries	6
4.3	Connecting the charger / charging the batteries	7
4.4	Installing the inspection window	7
4.4.1	Inspection window in park position	7
4.5	Installing the clamp	7
5	Operation	8
5.1	Switching on/off	8
5.2	Charging the unit	8
5.2.1	Charging status	8
5.3	Emptying the dust drawer	8
6	Cleaning / Maintenance	9
6.1	Cleaning	9
6.2	Disinfection	9
6.3	Removing/inserting the suction grille	9
6.4	Changing the fine filter	10
6.5	Replacing the exhaust air filter	10
6.6	Spare parts	10
7	Technical data	11
8	Warranty	12
8.1	Exclusion of liability	12
9	Obligations regarding information	12
9.1	Disposal instructions	12
9.1.1	Disposal of rechargeable batteries / batteries	12
9.1.2	Disposal information for countries in the EU	12

1 Online help and information

<https://www.renfert.com/silent-xs>

en

Here you will find the following documents and information for download:

- Instructions For Use
- Quick Start Guide
- Spare parts lists and spare parts drawings
- Maintenance manual
- FAQs, cause and rectification of faults
- How-to videos
- Declarations of conformity

Please note: The documents and information listed are not available for all products.

1.1 Symbols used

In these Instructions For Use (IFU) or on the equipment you will find symbols with the following meanings:

Danger

Immediate risk of injury. Consult accompanying documents!

Electrical voltage

Danger due to electrical voltage.

Caution

Risk of damage to the equipment if the instruction is not observed.

Note

Provides useful information to make operation easier.

The equipment complies with applicable EU directives.

This product complies with the relevant UK legislation.

See the UKCA declaration of conformity on the Internet at www.renfert.com.

The equipment is subject to the provisions of the EU directive 2002/96/EC (WEEE Directive).

- List, special attention should be paid to
 - List
 - Subordinate list

⇒ Instructions / required action / input / operational sequence:

You are asked to perform the specified action in the given order.

- ◆ Result of an action / response by the equipment / response by the program:

The equipment or program responds as a result of your actions or when a specific incident occurs.

Other symbols are explained as they occur in the text.

2 Safety

2.1 Intended use

The unit is intended for commercial use in dental labs or in the environment of dental offices in accordance with the IFU (Instructions For Use). The unit is intended for use in the following tasks:

- Suction of dry, non-explosive dust during minor fitting work on disinfected objects.

2.2 Meaning of these Instructions For Use (IFU)

These IFU will help you use the unit safely.

If these IFU are not observed, accidents may lead to life-threatening electric shock, injury, or material damage.

⇒ Only use the unit in accordance with these IFU.

⇒ Keep these IFU ready with the unit.

⇒ Provide these IFU to all subsequent users of the unit.

2.3 **Requirements for operators**

- ⇒ The product may only be used by persons aged 14 or over who are familiar with the procedures and rules in a dental laboratory.

2.4 **Safe condition of the unit**

Defective, broken or leaking components may cause injuries.

- ⇒ Check the power cord, housing, and other components such as connection cables, tubes, and the operating panel pad for damage, such as kinks, cracks, porosity, and signs of wear and tear.
- ⇒ Defective equipment must be taken out of service immediately. Disconnect the power plug. Send the unit in for repair or dispose of it safely. Until then, secure the unit to prevent it from being switched on again so that it is not used inadvertently.
- ⇒ Do not expose the unit to mechanical impact. Do not drop it.

2.5 **Hazards as a result of modifications and repairs**

Improper modifications and repairs to the unit may cause accidents, fire, and electric shock.

- ⇒ Only have the unit repaired by qualified specialist dealers or return it.

The use of unsuitable accessories or spare parts may lead to unpredictable accidents and damage.

- ⇒ Only use accessories and spare parts supplied or approved by Renfert GmbH.

2.6 **Risk as a result of electric shock and fire**

Risk of electric shock and of fire as a result of excessive operating voltage or if there is no protective conductor.

- ⇒ Ensure that the power plug is easily accessible.
- ⇒ Only operate the unit using a power cord with a plug that matches the local power supply. Only have the power cords replaced by a qualified electrician.
- ⇒ Only operate the unit if the information on the type plate matches the power supply.

Risk of fire if the wrong charger or batteries are used.

- ⇒ Use only the charger supplied to charge the batteries.
- ⇒ Do not use rechargeable lithium-ion batteries.
- ⇒ Do not use normal batteries (non-rechargeable).

2.7 **Risk of injury to the eyes or skin**

Risk of glare!

- ⇒ Do not look directly into the object illumination!

2.8 **Risk to health and fire hazard due to dust**

Dust emissions may result in damage to health.

- ⇒ The unit is not suitable for sucking in dust classified as hazardous, such as cristobalite and quartz (investment material), or CrCo.
- ⇒ When disposing of the sucked in material or used filters, observe local provisions and accident prevention regulations!
- ⇒ Depending on the type of sucked in material, wear appropriate personal protective equipment when emptying the dust drawer or during cleaning.

Sucking in non-approved types of dust can cause a fire.

- ⇒ Do not suck in flammable or explosive gasses, fumes or dust.
- ⇒ Do not suck in hot materials.

2.9 **Risk to health due to infection**

Infection due to cross-contamination may result in damage to health.

- ⇒ Disinfect the dental restoration with a suitable product listed by the VAH (Association for Applied Hygiene) before using the suction unit (see "Hygiene guidelines").
- ⇒ Repeat disinfection before and after each patient contact (see "Hygiene guidelines").
- ⇒ Disinfect the unit before use and after each treatment (see "Hygiene guidelines").
- ⇒ It is the user's responsibility to separately comply with country-specific regulations regarding compliance with reprocessing procedures.

3 Product description

3.1 General description

SILENT XS is a mobile suction unit. Thanks to battery operation, it is not tied to a single workstation and can easily be used in different locations.

3.2 Hygiene guidelines

The hygiene guidelines must be observed during startup and cleaning/maintenance.

The hygiene guidelines can be found online.

Scan this QR code.

3.3 Assemblies and functional elements

- | | |
|----------------------------------|--|
| 1 On/off switch | 8 Exhaust air filter (behind exhaust air grille) |
| 2 Charging socket | 9 Operating and charging status indication |
| 3 Object illumination | 10 Inspection window |
| 4 Suction grille | 11 Dust drawer with fine filter |
| 5 Suction opening | 12 Charger, desktop power supply with power cord |
| 6 Clamp (available as accessory) | alternatively |
| 7 Exhaust air grille | 13 Charger, plug-in power supply with plug set |

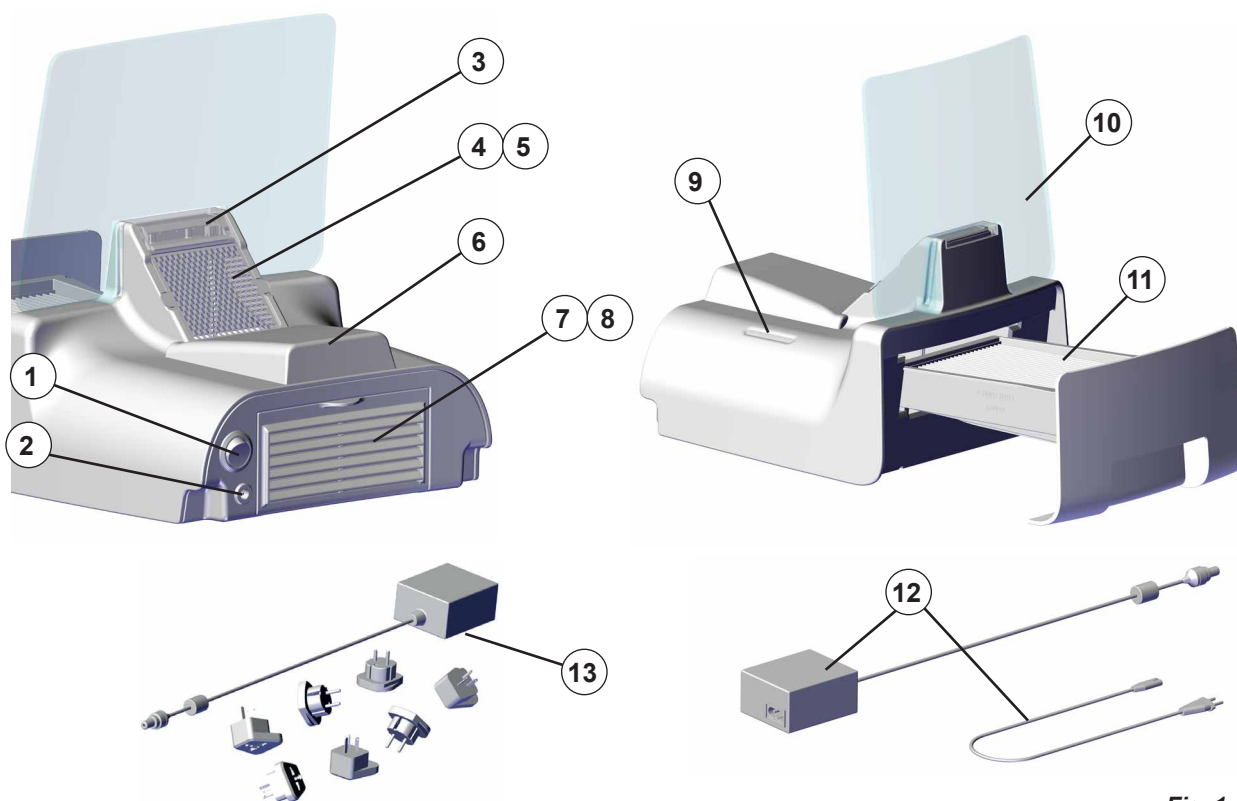
Fig. 1

3.4 Scope of delivery

- 1 Suction unit SILENT XS
- 1 Inspection window
- 1 Charger (desktop power supply or plug-in power supply)
- 1 Power cord (only with desktop power supply)
- 1 Quick Start Guide

3.5 Accessories

- 2922 0001 Clamp
- 2922 0002 Set of filter (3 pcs.)

4 Startup

4.1 Unpacking

- ⇒ Remove the unit and accessories from the packaging.
- ⇒ Check the delivery for completeness (compare with the scope of delivery).

4.2 Insert batteries

Do not use rechargeable lithium-ion batteries!

Rechargeable batteries are not included in the scope of delivery and must be obtained locally prior to startup.

Recommended batteries:

- Type: Varta Endless
- Duracell Recharge
- Capacity: 2500 mAh

The fully charged batteries provide up to 45 minutes of run time.

Battery life is significantly affected by charging status, use and charging cycles, age, manufacturer and production batches.

Use of non-recommended batteries may result in significant variations in run time.

Please refer to the technical data for the minimum battery requirements.

- ⇒ Use the on/off switch (1, Fig. 1) to turn off the unit.
- ⇒ Turn the unit over.
- ⇒ Remove the screws from the battery compartment.
- ⇒ Slide the battery compartment cover in the direction of the printed arrow and remove it.
- ⇒ Slide the holding clip (Fig. 2) to the side and remove the battery holder.

Caution

Do not pull on the battery holder cable!

- ⇒ Insert the batteries.

Pay attention to correct polarity!

The negative pole of the batteries always points towards the contact spring in the battery holder.

When inserting the batteries, first insert the batteries with the positive pole next to the contact into the battery holder and push them up to the contact.

Then insert the batteries whose negative pole is pointing towards the contact spring.

- ⇒ Insert the battery holder until it rests under the retaining clip.
- ⇒ Replace the battery cover and slide it against the direction of the arrow until the screw holes are accessible.
- ⇒ Screw the screws back in.

Do not overtighten the screws!

Tighten the screws finger-tight.

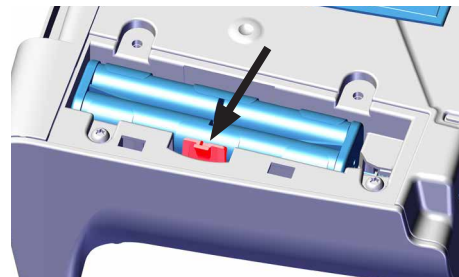
Fig. 2

4.3 Connecting the charger / charging the batteries

Observe the instructions in chapter 2.6 “Risk as a result of electric shock and fire!”

Desktop power supply (12, Fig. 1):

- ⇒ Plug the power cord into the desktop power supply.
- ⇒ Connect the power plug.

Plug-in power supply (13, Fig. 1):

- ⇒ Select the appropriate country-specific plug adapter from the plug set.
- ⇒ Insert the plug adapter into the power supply until it clicks into place.

- ⇒ Connect the charging plug to the charging socket (2).

- ◆ The batteries are charged.
- ◆ It takes approx. 5 hours to recharge empty batteries.

The unit can also be operated while connected to the charger and the batteries are charging.

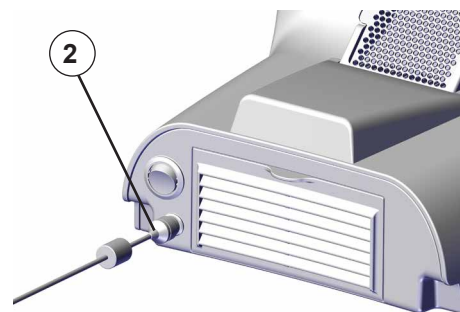
Fig. 3

4.4 Installing the inspection window

- ⇒ Slide the inspection window (10) straight from above into the grooves on the side of the suction opening.

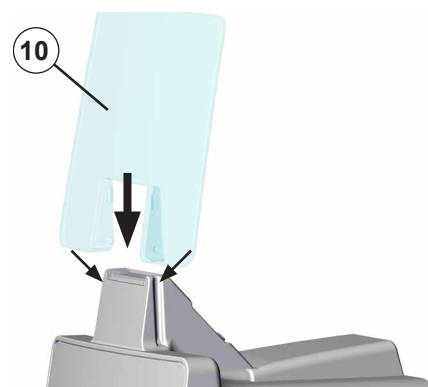
Fig. 4

4.4.1 Inspection window in park position

The inspection window can be folded down to store the unit in a drawer, for example.

To do this, pull the inspection window up a bit and fold it back.

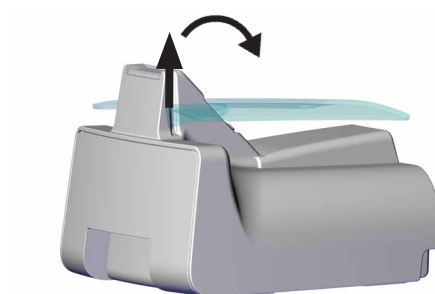
Fig. 5

4.5 Installing the clamp

The clamp (6) is available as an accessory.

It is held on the top of the housing by magnets.

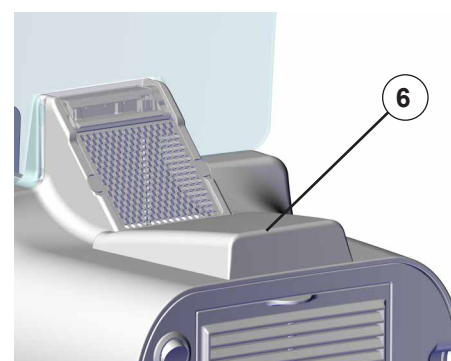
Fig. 6

5 Operation

en

The unit must be disinfected before first use (see chapter „Disinfection“).
The procedure is described in the hygiene guidelines.

5.1 Switching on/off

Use the On/Off switch (1, Fig. 1) to switch the unit on and off.

⇒ Switching on

- ♦ The suction fan starts to run.
- ♦ The object illumination is switched on.

5.2 Charging the unit

When not in use, the unit should be connected to the charger to keep it fully charged.

5.2.1 Charging status

The operating and charging status is indicated by 4 LEDs (9).

LED 1 (green):

- Is lit when the charger is connected.
- The inserted batteries are charging.

LED 2-4 (white):

- Flash during the charging process depending on the charging status.
- All LEDs will light continuously when all batteries are fully charged.

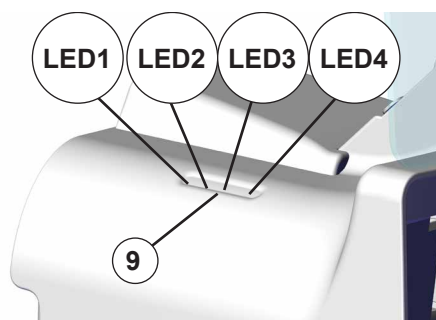
Fig. 7

During operation:

LED 2-4: Indication of the charging status.

LED 4 indicates a remaining run time of approx. 5–10 minutes.

5.3 Emptying the dust drawer

Empty the dust drawer after every grinding operation.

Caution

Do not beat out the dust, as this will only increase the dust accumulation!

⇒ Switch the unit off.

⇒ Pull the dust drawer (11) straight out.

⇒ Take the dust drawer to the disposal site and empty it carefully so that no dust is stirred up.

⇒ Insert the dust drawer again and push it all the way in.

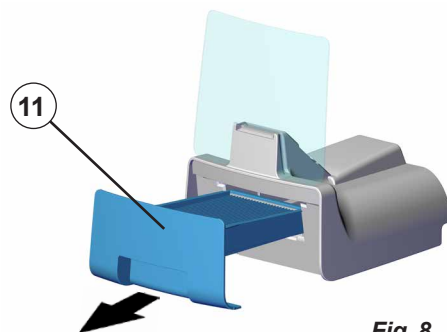
Fig. 8

6 Cleaning / Maintenance

The unit does not contain any internal parts that require maintenance.
Do not open the unit except in the situations described below!

Before cleaning and maintenance work:

- Switch off the unit and disconnect the charging plug!

6.1 Cleaning

Do not steam clean the unit under any circumstances.
Do not use compressed air.

To clean the unit, wipe the entire surface (e. g. including the window) with a dry cloth.

Use a soft cloth and avoid applying too much pressure to prevent scratches on the window.

Do not use any solvent-based, alkaline or abrasive cleaning agents.

Do not use solvent-based disinfectants (e. g. BIB forte/ Alpro Medical).

6.2 Disinfection

Disinfect the unit in accordance with hygiene guidelines.
The following steps are only an excerpt from the hygiene guidelines (see chapter 3.2).

Caution

Do not spray disinfectant directly onto or into the unit.

Use disinfectant wipes.

For disinfection, use a disinfectant wipe listed by the VAH (Association for Applied Hygiene), e. g. minutenwipes / Alpro Medical or Caviwipes / Metrex.

It is recommended to disinfect the unit before initial use and after each use.

To do this, carefully wipe the entire surface of the housing (including the grilles and window) with the disinfectant wipe.

Follow the manufacturer specifications for handling and exposure time.

6.3 Removing/inserting the suction grille

Removal

- ⇒ Slide the suction grille (4) upwards (a).
- ⇒ Swing the suction grille out at the bottom (b) and remove it.
- ⇒ Remove any stuck particles.

Insertion:

- ⇒ Slide the suction grille up under the object illumination cover.
- ⇒ Fold down the suction grille at the bottom and slide it straight down until it clicks into place.

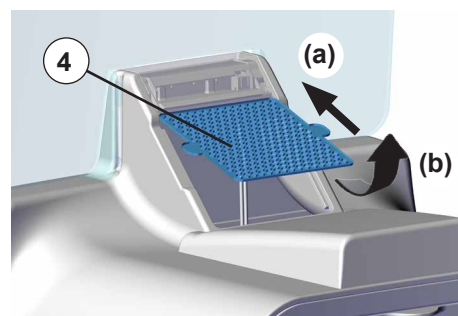
Fig. 9

6.4 Changing the fine filter

If the suction power decreases significantly despite emptying, the fine filter in the dust drawer (11) must be replaced.

With frequent use, this step may be necessary after approx. two weeks.

- ⇒ Pull the dust drawer straight out.
- ⇒ Take the dust drawer to the disposal site and empty it carefully so that no dust is stirred up.
Do not beat out the dust, as this will only increase dust accumulation.
- ⇒ Pull the fine filter element up and out of the dust drawer panel.
- ⇒ Dispose of the fine filter element.
- ⇒ Insert the new fine filter element into the grooves on the panel from the top and push down until it clicks into place.
- ⇒ Insert the dust drawer again and push it all the way in.

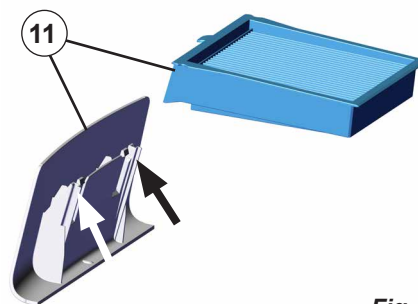
Fig. 10

6.5 Replacing the exhaust air filter

The exhaust air filter (8) must be replaced every six months.

- ⇒ Pull the exhaust air grille (7) straight out by the handle tab.
- ⇒ Remove and dispose of the exhaust air filter.
- ⇒ Insert the new exhaust air filter into the exhaust air grille.
- ⇒ Place the lower edge of the exhaust air grille in the housing.
- ⇒ Press the top edge until it clicks into place.

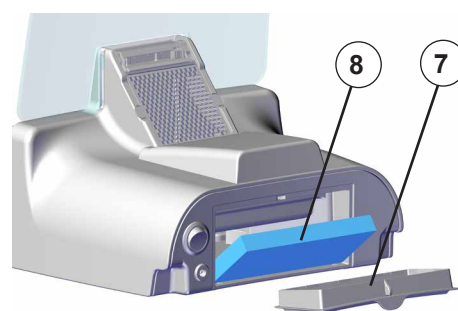
Fig. 11

6.6 Spare parts

You can find information on wear parts and spare parts in the spare parts list available online, see chapter "Online help and information".

The components excluded from the warranty (wear parts, consumables) are marked in the spare parts list.

The serial number, date of manufacturing and equipment version are shown on the nameplate of the equipment.

7 Technical data

Item number:	2922 0000 / 2922 1000 / 2922 3000
Nominal voltage / VDC:	12
Power consumption ^{*1)} / A:	1.4
LpA ^{*2)} (at max. flow rate) / dB(A):	52.7
Flow rate, max. / l/min [cfm]:	510 [18]
Fine filter: – Filter surface, approx. / m ² [sq inch]: – Filter quality:	0.11 [170.5] HEPA H13
Fill volume dust drawer, approx. / l [US fl oz]:	0.22 [7.44]
Rechargeable battery: Quantity Type Size Voltage / V Capacity, minimum / mAh	8 NiMH (rechargeable) AA 1.2 1200
Dimensions (width x height x depth) / mm [inch] (incl. Inspection window):	223 x 248 x 193 [8.8 x 9.8 x 7.6]
Weight (empty) / kg [lbs]:	1.12 [2.47]
This product contains a light source of energy efficiency class:	E
Charger	
Permissible mains voltage / VAC:	100 - 240
Mains frequency / Hz:	50 / 60
Output voltage / VDC:	12
Output power / W:	25
Ambient conditions for operation ^{*3)}	
Ambient temperature / °C [°F]:	5 - 30 [41 - 86]
Ambient conditions for storage and transport	
Ambient temperature / °C [°F]:	- 20 to + 60 [- 4 to + 140 °F]
Maximum relative humidity:	80 %
Ambient conditions according to EN 61010-1 ^{*3)}	
Ambient conditions:	Indoors
Installation height, max.:	2,000 m above sea level
Ambient temperature:	5 - 40 °C [41 - 104 °F]
Relative humidity:	Max. 80 % at 31 °C [87.8 °F], decreasing linearly to 50 % at 40 °C [104 °F] **)
Voltage fluctuations in the mains power supply from the nominal value, max.:	10 %
Contamination level:	2
Over-voltage category:	II

^{*1)} Max. 1.8 A in suction mode and when charging empty batteries.

^{*2)} Noise level in accordance with EN ISO 11202

^{*3)} The product performs as specified in the technical data when operated within ambient conditions. If the unit is operated outside these ambient conditions, safety devices contained in the product may stop operation, the optimum performance specified in the technical data may not be achieved, and the service life of components may be reduced.

The product fulfills the safety requirements of standard EN 61010-1 when operated within ambient conditions. Operation outside of these ambient conditions is not permitted, as electrical safety may be compromised.

8 Warranty

Renfert offers a **3-year warranty** on all parts of the equipment, provided that the equipment is properly used.

Warranty claims may only be made upon presentation of the original sales receipt from the authorized dealer.

Parts that are subject to natural wear and tear (wear parts) and consumables are excluded from the warranty. These parts are marked in the spare parts list.

The warranty is voided in the case of improper use; failure to observe the operating, cleaning, maintenance, and connection instructions; in the case of independent repairs or repairs that are not carried out by an authorized dealer; if spare parts from other manufacturers are used, or in the case of unusual influences or influences not in compliance with the utilization instructions.

Warranty claims shall not extend the warranty period.

8.1 Exclusion of liability

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- ▶ The product is employed for any purposes other than those specified in the instructions for use.
- ▶ The product is altered in any way other than those alterations described in the instructions for use.
- ▶ The product is not repaired by an authorized dealer or if original Renfert spare parts are not used.
- ▶ The product continues to be used despite obvious safety defects or damage.
- ▶ The product is subjected to mechanical impact or is dropped.

9 Obligations regarding information

- ▶ Information on REACH and SVHC can be found on our website in the service area (www.renfert.com/service)

9.1 Disposal instructions

The equipment must be disposed of by a specialist company. If there are any residues in the equipment that are hazardous to health, the specialist company must be informed.

9.1.1 Disposal of rechargeable batteries / batteries

The product contains rechargeable batteries or batteries.

They must be removed before disposal and then disposed of properly.

These may not be disposed of with household waste.

Rechargeable batteries / batteries are not supplied with the product and must be disposed of by the operator on their own responsibility.

Dispose of rechargeable batteries / batteries according to local regulations!

9.1.2 Disposal information for countries in the EU

To conserve and protect the environment, prevent environmental pollution, and improve the recycling of raw materials, the European Commission has adopted a directive that requires the manufacturer to accept the return of electrical and electronic equipment for proper disposal or recycling.

For this reason, equipment with this symbol may not be disposed of in unsorted domestic waste within the European Union.

Please contact your local dealer for more information on proper disposal.

Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany