temp:ex

Temporary Cement Remover | REF 6600 0000

DEUTSCH

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ENGLISH

Product description/Intended purpose:

temp:ex is a cleaning fluid for dissolving and removing temporary zinc oxide cements. It dissolves a uniform layer of the temporary cement in the contact time. This can then be easily removed. In comparison with conventional procedures, this method can reliably reduce the risk of injury to the user, as well as the risk of breakages or perforations in the restorations.

Indication:

temp:ex frees temporary dental cement (based on zinc oxide) from dental restorations (temporary and permanent) made from a wide range of materials.

Contraindication:

Any oral use of temp:ex is contraindicated. temp:ex is contraindicated for patients with known allergies to individual ingredients.

Restorations made from titanium alloys (e.g. abutments), aluminum and tin must not be treated with temp:ex (risk of oxidation). The solution has a considerably reduced effect on silicon-based and polymer-reinforced temporary cements.

Special hazard warnings for use:

Before using temp:ex, please read the safety data sheet. Download from www.renfert.de

temp:ex is very corrosive. When temp:ex comes into contact with the surface of the skin and mucous membranes, there is no immediate warning pain, however a more deeply painful effect is felt later on.

Therefore, avoid contact with skin, eyes, and clothing under all circumstances. In the event of contact with skin, rinse immediately with running water. In the event of contact with the eyes, rinse well with water and consult an ophthalmologist. Change contaminated clothes. Keep away from food and drink.

To protect working surfaces, the use of an alkali-resistant surface covering is recommended. Wear alkali-resistant protective gloves, protective clothing and protective goggles during processing. Maximum application temperature approx. 35°C / 95°F.

Seal the container immediately after use. Dilute and rinse spilled temp:ex with plenty of water.

Application:

To prevent cross contamination and diminished dissolution performance, temp:ex is for single-use only!

- ▶ Seal the container immediately after use.
- ▶ Before applying, contaminants must be rinsed from the restoration under running water.

**Insertion
into temp:ex**

- ▶ Select a plastic container suitable for the size of the restoration (e.g. mouth-washing beaker – **recommended material: polypropylene, polystyrene**) and place the restoration in it. **Glass containers may exhibit an irreversible opacity after several applications and longer periods of contact.** **To prevent damage to the work surfaces, an alkali-resistant surface covering is recommended for use in the work area.**

- ▶ Cover restoration completely with temp:ex. To prevent air pockets, the restoration openings must point upward.
- ▶ Depending on the strength of the cement, a treatment time of approx. 10 minutes is recommended. **The required treatment time is strongly influenced by the thickness of the layer and the different compositions of the temporary cements. A maximum treatment time of 60 minutes must not be exceeded.**

**As an alternative to increase efficiency,
you can use the SYMPRO cleaning device:**

The operating instructions for the SYMPRO incl. accessories must also be observed before application!

- ▶ Select one cleaning beaker suitable for the size of the restoration (e.g. small SYMPRO cleaning beaker: for max. 4 parts, insert individually or connected) and place the restoration in it. **temp:ex must not be used on objects made from zirconium dioxide in the SYMPRO cleaning device: darker needle abrasion could occur on the zirconium dioxide surfaces.**
- ▶ The adjusted needle amount for the cleaning beaker must be observed.
- ▶ Recommended quantity of temp:ex: 10 ml / 0.33 oz. (for small cleaning beaker).
- ▶ Recommended cleaning time: approx. 5 min at 2000 RPM.
- ▶ **The maximum cleaning time of 15 minutes must not be exceeded.**
- ▶ After the treatment time in a plastic beaker or after a cleaning time in the SYMPRO, the result should be checked by carefully and thoroughly rinsing (min. 6 seconds) the restoration under running water with the aid of tweezers. The rinsing process can be supported by using a multi-function nozzle (water/air). The risk of the spraying jet deflecting must be taken into consideration here: **wear protective goggles at all times!**
- ▶ If necessary/desired, the restoration can be put in temp:ex again or cleaned once more in the SYMPRO to fully remove the cement layer.
- ▶ The rinsing/spraying process described above must then be repeated.
- ▶ To apply cement again, the restoration must be hygienically prepared in accordance with the national hygiene regulations.
- ▶ To achieve an optimum adhesive bond when reapplying cement, the package information leaflet from the respective cement manufacturer must be observed! In case of complaints and events in connection with the medical device, please inform Renfert immediately. Renfert will investigate the matter and report serious incidents to the authorities in the member states concerned.

Disposal:

Pour used liquid into a collection container. Dispose of in accordance with official regulations.

Storage:

Avoid direct sunlight. Seal the container shut immediately after use. temp:ex can be stored up to (see label) if protected from light and in the original packaging at 23°C / 73.4°F.

temp:ex

Temporary Cement Remover | REF 6600 0000

ENGLISH

Shelf life:

Do not use after the expiry date (see label). Can be used for 6 months after opening.

Ingredients:

H₂O, potassium hydroxide, triarylmethane dye.

Disclaimer:

This product has been developed and is intended exclusively for professional use by trained dental professionals. It must be handled in accordance with the information provided in this instruction document. The manufacturer assumes no liability for damages that arise from improper handling or processing.

Rx only Professional use only

Please observe the instructions for use *

EC conformity

Manufacturer specifications

Not intended for reuse

Batch designation

Attention: see accompanying document

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Renfert GmbH