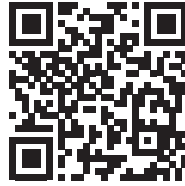
Quick Start Guide

SIMPLEX sliceware / SIMPLEX print

SIMPLEX sliceware

myRenfert
Login

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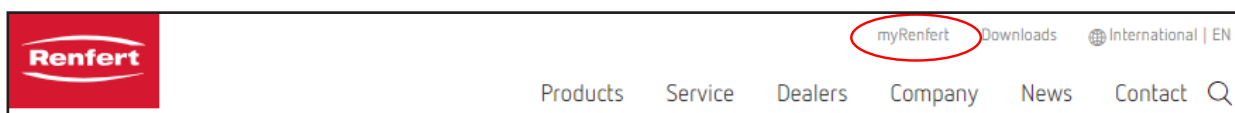
To download the software, you must log in to “myRenfert” and register your SIMPLEX 3D filament printer.

Login


⇒ Log in to “myRenfert” on the Renfert website www.renfert.com with your credentials.

⇒ If you do not yet have a “myRenfert” account, you must first register via “myRenfert”.

Register your SIMPLEX 3D filament printer

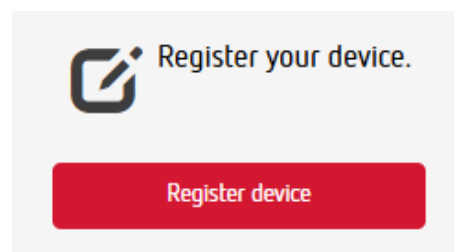- ⇒ Select „Register device“.
- ⇒ Enter the serial number of your equipment. You can find it on the type plate of your equipment.
- ⇒ If desired, you can also enter a comment about the equipment.
- ⇒ Click on “Submit”.
 - ◆ The registered equipment is displayed under „My Devices“.

Download

After registering the product, you can download the software package with the following components:

- SIMPLEX sliceware;
 - SIMPLEX sliceware instructions;
 - SIMPLEX print;
 - Version information
- ⇒ Select “My Devices”.
- ⇒ At your device click on “Details” - „Software Download“ - „Go to Download“ - „Download File“.
- ◆ The software package is downloaded to your computer as a ZIP file.
- ⇒ Open the download directory on your PC.
- ⇒ Unzip the ZIP file.
- ⇒ Click on “Simplex_Print_....._installer.exe” and follow the instructions.
- ◆ The SIMPLEX print software will be installed on your PC.
- ⇒ Click on “sliceware_....._installer.exe” and follow the instructions.
- ◆ The SIMPLEX sliceware software will be installed on your PC.

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