making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - English
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Polski
- FAQ - Українська
- FAQ - 日本語
- FAQ - 한국어
- FAQ - اللغة العربية

# DUSTEX master plus

FAQ - frequently asked questions

# FAQ - Deutsch

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# FAQ - English

For chapter indications, refer to the complete instruction manual on the internet.

Problem	Possible cause	Corrective action
<b>Insufficient extraction power.</b>	<ul style="list-style-type: none"><li>• Extraction duct blocked.</li><li>• Extraction grate blocked.</li></ul>	<ul style="list-style-type: none"><li>• Remove and clean the extraction duct.</li><li>• Remove and clean the extraction grate.</li></ul>
<b>Lamp does not switch on.</b>	<ul style="list-style-type: none"><li>• Power cord not plugged in.</li><li>• Illumination defect.</li><li>• Faulty electronics.</li></ul> <p><b>The device with fluorescent tubes</b></p> <ul style="list-style-type: none"><li>• The fluorescent tube is not situated correctly in the socket.</li><li>• Fluorescent tubes defect.</li></ul>	<ul style="list-style-type: none"><li>• Plug the power cord in.</li><li>• Change light using the service or see spare part list.</li><li>• Have Customer Service replace the electronics or refer to the Spare Parts List.</li></ul> <p>• Check that the fluorescent tube is fitted correctly (see chapter 5.4.2).</p> <p>• Changing the fluorescent tubes (see chapter 5.4.2).</p>

# FAQ - English

For chapter indications, refer to the complete instruction manual on the internet.

Problem	Possible cause	Corrective action
<b>Insufficient extraction power.</b>	<ul style="list-style-type: none"><li>• Extraction duct blocked.</li><li>• Extraction grate blocked.</li></ul>	<ul style="list-style-type: none"><li>• Remove and clean the extraction duct.</li><li>• Remove and clean the extraction grate.</li></ul>
<b>Lamp does not switch on.</b>	<ul style="list-style-type: none"><li>• Power cord not plugged in.</li><li>• Illumination defect.</li><li>• Faulty electronics.</li></ul> <p><b>The device with fluorescent tubes</b></p> <ul style="list-style-type: none"><li>• The fluorescent tube is not situated correctly in the socket.</li><li>• Fluorescent tubes defect.</li></ul>	<ul style="list-style-type: none"><li>• Plug the power cord in.</li><li>• Change light using the service or see spare part list.</li><li>• Have Customer Service replace the electronics or refer to the Spare Parts List.</li></ul> <p>• Check that the fluorescent tube is fitted correctly (see chapter 5.4.2).</p> <p>• Changing the fluorescent tubes (see chapter 5.4.2).</p>

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