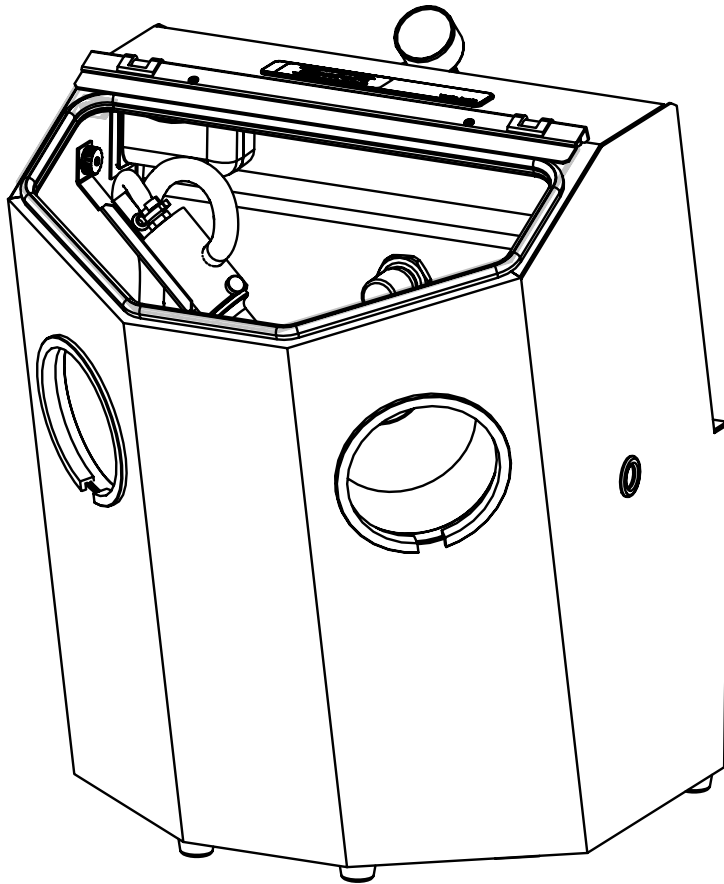
making work easy

# VARIO basic

Service Instructions

Made in Germany

21-6904 05122025

# Contents

## 90002 1428

Lampe kpl. • Lamp complete • Lampe complète • Lampada LED completa • Lámpara completa  
Светильник полный

## 90003 6659

Lampenabdeckung • Cover for lamp • Protection pour lampe • Protezione per la lampada  
Cobertura de lámpara • Крышка для светильника

## 90003 6121 / 90003 6122 / 90003 5938

Geräteanschlussleitung • Unit connector cable • Câble raccordement appareil  
Cavo di raccordo apparecchio • Cable de conexión aparato • Сетевой кабель

## 92960 0004

Mischkammer • Mixing chamber • Chambre de malaxage • Camera di miscela  
Camara de mezcla • Смесительная камера

## 90003 5431

Strahldüse kpl. 2960 • Blasting nozzle compl. 2960 • Buse de sablage compl. 2960  
Ugello di sabbiatura compl. 2960 • Tobera compl. 2960 • Сопло для пескоструйного аппарата  
в комплекте 2960

## 92960 0001

Schlauchset kpl. 2960 • Hose set cpl. 2960 • Jeu de tuyaux complete 2960 • Kit di tubi cpl. 2960  
Juego completo de tubos 2960 • Набор шлангов в комплекте 2960

## 90002 1428

Ansaugfilter kpl. • Extraction filter cpl • Filtre d'aspiration complet • Filtro dell'aria cpl.  
Filtro aspirador completo • Фильтр входящего воздуха в комплекте

## 2 1119

Manometer mit Verschraubung • Manometer mit Verschraubung • Manomètre avec raccord  
Manometro con filettatura • Manómetro con atornilladura • Манометр с резьбовым  
соединением

## 90003 4109

Druckluftfilter komplett 2960 • Compressed air filter complete 2960 • Filtre pour air comprimé  
complet 2960 • Regolatore filtrante completo 2960 • Filtro aire comprimido 2960 • Фильтр  
сжатого воздуха в комплекте 2960

## 90003 4109

Druckregler komplett 2960 • Pressure regulator compl. 2960 • Régulateur de pression compl.  
2960 • Regolatore di pressione compl. 2960 • Regulador de presión compl. 2960 • Регулятор  
давления полный 2960

## 92960 0005

Vierfachverteiler Vario basic • Quadruple distributor Vario basic • Distributeur quatre voies Vario  
basic • Distributore quadruplo Vario basic • Distribuidor cuádruple Vario basic  
Распределитель четырехсторонний

## 90003 4516

Lochblech komplett 2960 • Grate complete 2960 • Tôle perforée complète 2960  
Grata di fondo perforata 2960 • Chapa de fondo perforada 2960 • Лист перфорированный в  
комплекте 2960

## 90003 0135

Handlochmanschetten grau • Cuffs for Microblasters, grey • Manchettes pour sableuses, grises  
Manicotti per sabbiatrici, grigi • Puños para chorreadores, gris  
Манжета для пескоструйных аппаратов

**90003 0923**

Handstulpenring geschlitzt • Handcuff ring - slit • Anneau fendu pour gants • Anelli intagliati per manicotti • Anillo para puños ranurado • Кольцо манжеты

**90003 4523**

Schutzhandschuhe 2960 • Protective gloves 2960 • Gants de protection 2960 • Guanti protettivi 2960 • Guantes de protección 2960 • Перчатки защитные 2960

**90002 1316**

Glasscheibe beschichtet • Coated glass screen • Vitre revêtu  
Vetro rivestito Cristal con revestimiento • Запасное стекло с покрытием

**90003 5756**

Glasscheibe unbeschichtet • Uncoated glass screen • Vitre non revêtu  
Vetro senza rivestimento • Cristal sin revestimiento • Запасное стекло без покрытия

**90003 4103**

Glashalter kpl. • Glass holder cpl. • Porte-vitre complet • Supporto per lastra compl.  
Sujetador para cristal • Держатель стекла в комплекте

**90003 43005**

Gegenhalter mit Schrauben • Locking devices with screws • Arrêt avec vis • Conrostampo con viti • Contrabuterola con tornillos • Держатель с болтами

**92931 0000**

Fußschalter pneumatisch • Pneumatic foot switch • Commande à pied pneumatique  
Interruttore a pedale pneumatico • Pedal interruptor neumático • Педаль пневматическая

**90003 4590**

Halter Strahldüse • Retainer for blasting nozzle • Support pour buse de sablage  
Supporto per ugello di sabbiatura • Soporte para tobera de chorreado • Держатель струйного сопла

**90003 2729**

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Fault	Cause	Remedy
Compressed air escapes from the switch.	• The pneumatic hoses have been connected incorrectly.	• Change the connectors.
The foot-switch does not switch on.	• No pressure is being applied to the compressed air inlet. • The pneumatic hoses have been kinked.	• Check the compressed air hoses and compressor. • Re-arrange the hoses.

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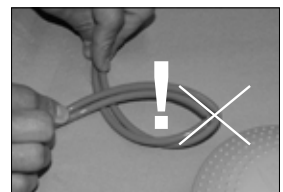
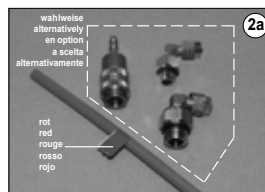
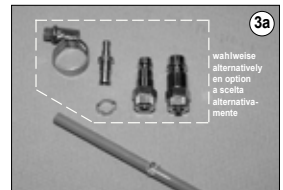
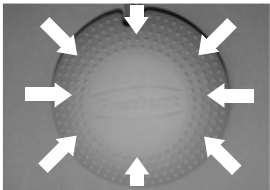
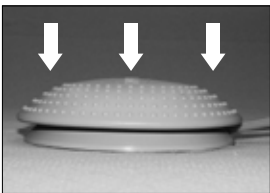
**As long as only the compressed air supply hose (P) is connected, do not operate the foot-switch as the hose (red) will thrash around and quite possibly cause injury.**

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The foot switch is maintenance-free. Surface should be cleaned with a damp cloth from time to time. Do not wash out the foot switch, do not use any cleansing agent containing solvents. Air hoses should be checked occasionally on damages so that unnecessary air leakage can be avoided.

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When used properly Renfert guarantees all parts of the foot-switch for 3 years. Parts given to natural wear and tear are excluded from this guarantee. The guarantee is void for damage caused by misuse, disregard of the instructions for use, cleaning, maintenance and connection, for self-repaired units or repair by untrained personnel, for use of spare parts from other manufacturers and on influences outside the normal practice or unauthorised in the instruction manual. Services under guarantee do not extend the guarantee.

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany