Renfert

making work easy

BASIC prebonder

Quick Start Guide

Made in Germany

de

en

fr

it

es

pt

tr

ru

zh

ja

ko

ar

20-5193 08012026

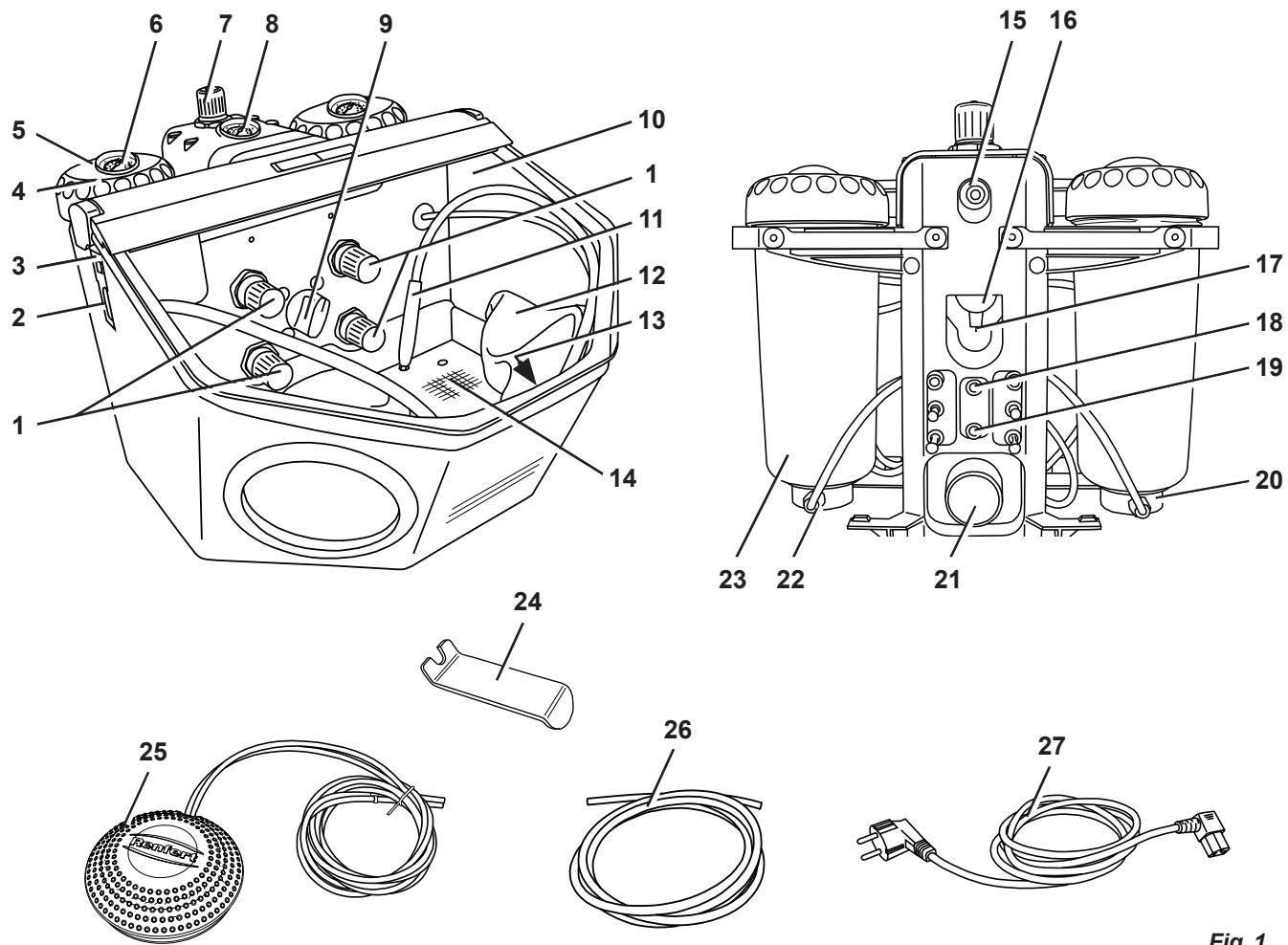
Fig. 1

<https://www.renfert.com/basic-prebonder>

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<https://www.renfert.com/basic-prebonder>

en

Here you will find the following documents and information for download:

- Instructions For Use
- Quick Start Guide
- Spare parts lists and spare parts drawings
- Repair instructions
- FAQs, cause and rectification of faults
- How-to videos
- Declarations of conformity

Please note: The documents and information listed are not available for all products.

Employed Symbols

In these instructions or on the unit itself you will find symbols which have the following meaning:

Danger

This indicates an imminent risk of injury. In these instructions or on the unit itself you will find symbols which have the following meaning.

Electrical hazard

There is a risk of electrical shock.

Attention

Failure to observe the associated information can result in damage to the unit.

Note

The operating instructions contain useful information to make handling easier.

Attention!

Wear eye protection.

The unit complies with the relevant EC guidelines.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under www.renfert.com.

The unit is subject to the EC guidelines 2002/96/EG (WEEE Directive).

Other symbols are explained as they occur.

Safety

Intended use

The Basic sandblaster is intended for commercial use in dental applications in dental offices and in-office labs in accordance with the Instructions For Use (IFU). The unit is intended for use in the following tasks:

- Prebonder tank:
Surface conditioning prior to inserting fixed dental restorations in the mouth – suitable for zirconium dioxide / metals / hybrid materials (ceramic / resin) and polymers.
- Classic tank (25–70 µm / 70–250 µm):
All dental activities that have been carried out in the dental office so far:
 - Removal of dental cement residues
 - Cleaning dentures
 - Satin finishing
 - All classic dental activities

Meaning of these instructions

These instructions for use act as a guide for safe usage of the unit.

If the instructions for use are not observed, accidents may lead to life-threatening electric shock, injury, or damage.

- ⇒ Only use the unit in accordance with these instructions for use.
 - ⇒ Keep the instructions for use ready with the unit.
 - ⇒ Provide the instructions for use to all subsequent users of the unit.
-

Requirements that the operator must meet

- ⇒ This product is intended for use by persons 14 years of age or older who are familiar with the procedures and regulations used in an in-office lab or any other dental lab.
-

Safe condition of the unit

Faulty or broken components, or those that are not sealed as required, may cause injuries.

- ⇒ Check the power cord, housing, and other components such as connection cables, tubes, and the key pad for damage, for example such as kinks, cracks, porosity, and signs of aging.
 - ⇒ Defective equipment must be taken out of service immediately. Disconnect the power plug. Send the unit in for repair or dispose of it safely. Until this happens, secure the unit to prevent it from being switched on again so that it is not used inadvertently.
 - ⇒ Do not expose the unit to mechanical impact. Do not drop it.
-

Risks as a result of modifications and repairs

Incorrect modifications and repairs to the unit may cause accidents, fire, and electric shock.

- ⇒ Only have the unit repaired by qualified specialist dealers or return it.
-

Risk as a result of electric shock and fire

Risk of electric shock and of fire as a result of excessive operating voltage or if there is no protective conductor.

- ⇒ Ensure that the power plug is easily accessible.
 - ⇒ Only operate the unit using a power cord equipped with a plug appropriate to the local power supply. Only allow qualified electricians to modify power cords.
 - ⇒ Only start the unit if the information on the nameplate matches the power supply.
 - ⇒ Only connect the unit to electrical sockets that are connected to the protective conductor system.
-

Risk of injury to the eyes or skin

Injury may occur to the eyes or skin as a result of high pressure or of components fracturing that are subject to high pressure.

- ⇒ Always wear appropriate protective goggles when carrying out work on the unit.
 - ⇒ Never direct the blasting material towards the eyes or unprotected areas of skin.
 - ⇒ Never perform sandblasting with the viewing panel open.
-

Risk to health as a result of dust

Dust emissions may result in damage to health.

- ⇒ Operate the unit with a suitable extraction unit. The extraction unit must be adapted to the dust that occurs.
-

Product description

General description

The BASIC prebonder sandblaster from Renfert features a specially developed concept for blasting before bonding. The blasting parameters and the surface pro abrasive have been precisely tailored for surface conditioning of fixed dental restorations based on scientific studies conducted by Ludwig Maximilian University (LMU) of Munich in Germany. The unit guarantees maximum and reliable adhesive strength for restoration materials, such as zirconium oxide, metals, hybrid materials (ceramic/resin), and modern polymers. It is therefore ideal for preparing materials prior to final insertion into the mouth.

The BASIC prebonder has an automatic quick stop function and is therefore especially suitable for exact and selective use of the sandblasting material.

Components and functional elements

See Fig. 1 (on inside cover)

- | | | |
|---------------------------------------|-------------------------------|-------------------------------|
| 1 Pressure regulator | 9 Selector switch | 19 Foot switch connection (B) |
| 2 Appliance socket | 10 Protection screen | 20 Mixing chamber |
| 3 ON / OFF switch | 11 Hand piece | 21 Suction tube |
| 4 Tank lid | 12 Hand sleeves | 22 Filter cartridge |
| 5 Ventilation valve | 13 Air nozzle | 23 Sandblasting tank |
| 6 Pressure gauge
(Flow pressure) | 14 Base grid | 24 Releasing tool |
| 7 Pressure regulator (fixed) | 15 Air connection | 25 Foot switch |
| 8 Pressure gauge
(Outlet pressure) | 16 Water trap | 26 Compressed air tube |
| | 17 Water trap valve | 27 Power cord |
| | 18 Foot switch connection (A) | |

Standard Delivery

- 1 BASIC prebonder
- 1 Foot switch
- 1 Connection set
- 1 Prebonder control pad 50 µm
- 5 Prebonder control tip (1 + 4)
- 1 Prebonder surface pro 50 µm
- 1 Set of Operating Instructions, with attachment
- 1 Power cord

Commissioning

Risk of injury!

Loose tubes may hit objects around them.

Tubes must be connected first to the unit.

Only press the foot switch after all tubes have been connected.

Installation

Choose a stable and non-slip surface to stand the unit on.

Foot switch connection

- ⇒ Remove the foot switch from its shipping location in the blasting chamber.
- ⇒ Insert the end of the tube marked with an „A“ (25) as far as it will go into the tube connection point also marked with an „A“ (18) on the blasting unit.
- ⇒ Insert the end of the tube marked with an „B“ (25) as far as it will go into the tube connection point also marked with an „B“ (19) on the blasting unit.

When inserting the tubes you will notice a slight resistance which must be overcome.

Compressed air connection

- ⇒ Push the compressed air tube (26) onto the tube connection point (15) as far as it will go. In doing so there will be a slight resistance to overcome.
- ⇒ To connect to the compressed air system, select the relevant tube coupling from the supplied set and attach this to the other end of the tube.
- ⇒ Connect to the compressed air network.
- ⇒ Check whether the pressure is at the required level (see tech. data). Display will be shown on the manometer (8).

Never press the pressure regulator (7) on the maintenance unit. This has been set and fixed to max. 6 bar in order to guarantee the best possible function of the pinch valves.

Electrical connection

Note the instructions in Chap. Risk as a result of electric shock and fire!

- ⇒ Plug the mains cable (27) into the socket (2).
- ⇒ Plug in the mains cable.

Connection to suction device

- ⇒ Plug the suction tube from the extraction unit to the suction pipe (21).
- For suction devices with automatic activation:
 - ⇒ Connect the mains plug from the sandblasting unit into the user socket on the suction device.

Filling the Sandblasting Tanks

Do not press the foot switch while filling is in progress!

When using Prebonder tank abrasives, only fill in and use a quantity of 500 g at a time.

Always use only clean, dry abrasive of the appropriate grain size.

- ⇒ Ventilate the blasting tank completely by pressing the ventilation valve (5).
- ⇒ Unscrew the tank lid (4). (If it is not possible to loosen the lid, there is still pressure within the blasting tank >>> Press the ventilation valve again!)
- ⇒ Add abrasive up to the maximum fill level (the point where the tank jacket thickens just below the thread).
- ⇒ Clean the thread on the sandblasting tank and tank lid as well as the seal.
- ⇒ Screw the tank lid closed firmly by hand.

Warning: Do not tilt the tank lid when screwing it on.

Check that the tank lid is seated securely prior to use.

Tank lids that are not closed securely can suddenly blow off.

There is a risk of injury as a result of flying parts and sudden emission of abrasive sand.

Authorised sandblasting material

- Prebonder surface pro
- Cobra (Renfert GmbH)
- Rolloblast (Renfert GmbH)
- ROCATEC (3M ESPE)

Sandblasting materials from other manufacturers may be used providing the grain size, grain shape and degree of purity comply with the above mentioned materials.

The "Prebonder surface pro" abrasive has undergone comprehensive scientific testing at LMU Munich. These tests covered grain size distribution, geometric shape, hardness, and degree of purity.

The company Renfert GmbH assumes no liability for function and durability of the units if other sandblasting material is used.

Adjusting the Operating Pressure

The Prebonder tank is equipped with a very sensitive pressure gauge to permit a more precise pressure adjustment.

Ensure that the pressure gauge is within the green area (0.6–1 bar).

⇒ Activate the foot switch and set the blasting pressure at the pressure regulator (1) of the respective blasting tank.

♦ The blasting pressure will be shown on the pressure gauge (6).

During longer downtimes, the pressure within the blasting tanks will reduce. By short activation of the foot switch the set pressure within in the blasting tanks will be restored.

Only blast restorations when the indicator is in the green area.

Operation

If the membrane key is damaged or defect, please disconnect the device and send away for repair! If the membrane is removed there is the risk of an electrical shock!

Do not operate the foot switch during tank selection.

⇒ Switch the illumination on (3).

⇒ Select the blasting tank at the selection switch (9).

⇒ Take the respective hand piece from the selected blasting tank (see coloured marking).

⇒ Activate the foot switch (25).

♦ The sandblasting material will flow as long as the foot switch is activated.

Prebonder blasting concept

Control tip

The Prebonder handpiece features a specially designed ergonomic handle and an integrated control tip. This ensures the following advantages:

- It helps to keep the optimum distance from the surface during the blasting process.
The distance has a significant impact on the energy input:
 - If the energy input is too high, it may damage the material structure and/or reduce the surface's wettability.
 - If the energy input is too low, surface activation will be insufficient, which will in turn reduce the adhesive strength.

- The precise blasting indicator permits to see exactly where the abrasive is applied. This helps to protect sensitive areas, such as crown margins or already polished surfaces, and to avoid unintentional sandblasting.

Fig. 1

Fig. 2

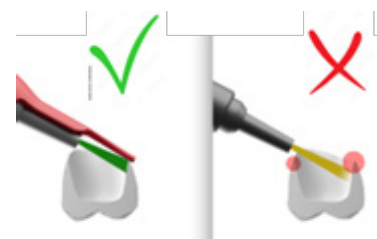
Fig. 3

Jet nozzle

Unlike conventional nozzles, which are designed primarily for maximum material removal, the prebonder jet nozzle has been optimized specifically for the requirements of adhesive surface pretreatment.

Thanks to its unique geometric design, the focus is on targeted transformation of the surface for optimal adhesion, rather than removal of material.

Function check with the control pad

We recommend using the enclosed control pad to ensure the correct setting of blasting energy before sandblasting the dental restoration.

If the color of the pad changes after 1-2 seconds of sandblasting, then the energy level is set optimally for surface conditioning.

Cleaning / Maintenance

Before cleaning or servicing:

- ▶ Switch off and unplug the device!
- ▶ Disconnect the appliance from the compressed air supply!

Sandblasting tank

Check the sandblasting tank or tank lid regularly, at least 1 x a year, by visually inspecting for damage or changes to the material (e.g., cracks ...).

If in doubt, replace!

The sandblasting tank and tank lid are subject to a natural aging process depending on ambient conditions and must be replaced as standard after every 15 years of operation.

Note the year of manufacture on the round stamp engraved on the sandblasting tank and in the tank lid; Our customer service team are happy to advise you.

To clean:

DO NOT use solvent-based cleaning agents or disinfectants.

Solvents and tensides can result in microcracking in the plastic (danger of explosion!).

⇒ Only clean the sandblasting tank and tank lid by rubbing with a dry cloth.

Cleaning the Interior

- ▶ DO NOT use solvent-containing cleaning agents or disinfectants. (use e.g., soapy water).
- ⇒ Remove the grate and vacuum the blasting chamber.

Condensation

⇒ Inspect the water separator (16).

⇒ Drain any water off through the valve (17).

Blasting Nozzle Replacement

For installation see accessory supplement.

Spare Parts

You can find information on wear parts and spare parts in the spare parts list available online, see chapter "Online help and information".

The components excluded from the warranty (wear parts, consumables) are marked in the spare parts list. Serial number and date of manufacturing are shown on the type plate of the unit.

Obligations regarding information

- ▶ Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

Information on disposal for countries within the EC

The unit must be disposed of by an authorized recycling operation. The selected firm must be informed of all possibly health-hazardous residues in the unit.

Within the European Union units with this symbol should not therefore be disposed of in unsorted domestic waste.

Please contact your local dealer for more information on proper disposal.

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declare under our sole responsibility, that the product(s)

Name	Part No.
Vario basic	2960 0005
Basic quattro IS	2959 0000 / 2959 0000 03
Basic quattro	2958 0000 / 2958 0000 03
Basic master	2948 2000 / 2948 2000 03 2948 2025 / 2948 2025 03 2948 2250 / 2948 2250 03
Basic classic	2947 1050 / 2947 1050 03 2947 1250 / 2947 1250 03 2947 2000 2947 2025 / 2947 2025 03 2947 2250
Basic eco	2949 1050 / 2949 1050 03 2949 1250 / 2949 1250 03 2949 2025 / 2949 2025 03
Basic mobil	2914 3050 2914 3250
Dustex master plus	2626 0105 / 2626 0105 03
Basic prebonder	2969 1050 / 29691250

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Electrical Equipment (Safety) Regulations 2016

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Ecodesign for Energy-Related Products Regulations 2010

Designated standards

EN 61326-1:2013

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.10.2025

The Technical Documentation has been retained by Renfert GmbH

Technical Data

Article number	2969 1xxx	2969 4xxx
Nominal voltage / V	230	120
Permissible mains voltage / V:	220 - 240	100 - 120
Mains frequency / Hz:	50/60	
Power consumption lighting / W:	16	
Working pressure / hPa (bar) [psi]: - Standard Tank - Prebonder Tank	1000 - 6000 (1 - 6) [14.5 - 87] 600 - 1000 (0,6 - 1) [8.7 - 14.5]	
Connecting pressure, external / hPa (bar) [psi]:	6000 - 8000 (6 - 8) [87 - 116]	
Air consumption / l/min [cfm]:	98 bei 6000 hPa [3.46 bei 87 psi]	
Cable length, approx. / m [inch]:	2 [78]	
Ø Compressed air hose / mm [mil]: interior exterior	4 [157] 6 [236]	
Ø Size of pipe union for external extraction unit / mm [inch]: interior exterior	35 [1.38] 40 [1.57]	
Dimensions (W x H x D) / mm [inch]:	395 x 320 x 510 [15.6 x 12.6 x 20.1]	
Blasting chamber volume / l [US gal]:	20 [5.3]	
Tank capacity, each/ ml [fl oz]:	1000 [34]	
Weight (empty), approx. / kg [lbs]:	11.5 [25.4]	
This product contains a light source of energy efficiency class:	F	
Environmental conditions for operation *)		
The ambient conditions according to EN 61010-1 apply.		
Environmental conditions for safe operation		
Ambient conditions:	Indoors	
Installation height, max.:	2,000 m [6,500 ft] above sea level	
Ambient temperature:	5 - 40 °C [41 - 104 °F]	
Relative humidity:	Max. 80 % at 31 °C [87.8 °F], decreasing linearly to 50 % at 40 °C [104 °F] *)	
Voltage fluctuations in the mains power supply from the nominal value, max.:	10 %	
Contamination level:	2	
Over-voltage category:	II	

- *) The product performs as specified in the technical data when used under ambient conditions. If the unit is operated outside these ambient conditions, safety devices contained in the product may stop operation, the optimum performance specified in the technical data may not be achieved, and the service life of components may be reduced. The product fulfills the safety requirements of standard EN 61010-1 when operated within ambient conditions. Operation outside of these ambient conditions is not permitted, as electrical safety may be compromised.

**Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...**

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany