making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Українська
- FAQ – Svenska
- FAQ - 中文
- FAQ - 日本語
- FAQ - 한국어
- FAQ - اللغة العربية

EASYCLEAN

FAQ - frequently asked questions

FAQ - Deutsch

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FAQ - English

For chapter indications, refer to the complete instruction manual on the internet.

Fault	Possible cause	Remedy
Casing damaged	<ul style="list-style-type: none"> • damage by third party, transport damage 	<ul style="list-style-type: none"> • return unit to supplier or manufacturer
Mains cable damaged	<ul style="list-style-type: none"> • damage by third party, transport damage 	<ul style="list-style-type: none"> • obtain original spare mains cable from manufacturer or supplier
No operating functions; all LEDs dark	<ul style="list-style-type: none"> • mains cable not plugged in • socket dead • mains cable damaged/interrupted • fault of electronics 	<ul style="list-style-type: none"> • plug in mains cable • check socket/fuse • replace mains cable • return unit to supplier or manufacturer
No ultrasonic function; LED ultrasound dark	<ul style="list-style-type: none"> • turning knob for ultrasonic operation in „0“ position • unit is switched off • key ►■ (ultrasound) not pressed • fault of electronics 	<ul style="list-style-type: none"> • switch on the turning knob for ultrasonic operation • switch on the unit at key on/off • press key ►■ • return unit to supplier or manufacturer
No ultrasonic operation; LEDs of LED cleaning period blink alternately ("running light") = fault indication ultrasound	<ul style="list-style-type: none"> • adverse filling level • fault of electronics 	<ul style="list-style-type: none"> • change filling level, switch unit and off and on • switch unit off and on • if fault is indicated again: return unit to supplier or manufacturer
Unsatisfactory cleaning results	<ul style="list-style-type: none"> • no or unsuitable cleaning medium used • cleaning temperature not sufficient • cleaning period too short 	<ul style="list-style-type: none"> • use suitable cleaning medium • heat up cleaning liquid • repeat cleaning interval
Unit does not heat up; LED temperature dark	<ul style="list-style-type: none"> • turning knob temperature in „0“ position • unit is switched off • fault of electronics 	<ul style="list-style-type: none"> • switch on turning knob temperature • switch on unit with key on/off • return unit to supplier or manufacturer
No heating function; LEDs of LED temperature blink alternately ("running light") = fault indication heating	<ul style="list-style-type: none"> • fault of electronics 	<ul style="list-style-type: none"> • switch unit off and on • if fault is indicated again: return unit to supplier or manufacturer
Unsatisfactory heating-up period	<ul style="list-style-type: none"> • loss of heating energy • no mixing of cleaning liquid 	<ul style="list-style-type: none"> • use cover (optional accessory equipment) • e.g. switch on ultrasound (see section 7.3)
Unit produces boiling noise during heating-up	<ul style="list-style-type: none"> • no mixing of cleaning liquid 	<ul style="list-style-type: none"> • e.g. switch on ultrasound (see section 7.3)
Set temperature is exceeded	<ul style="list-style-type: none"> • temperature sensor does not measure the average temperature (no revolution) • set temperature too low, • ultrasonic energy heats up the liquid more than required (physical process) 	<ul style="list-style-type: none"> • mix liquid manually or by means of ultrasound • for low set temperatures do not switch on heating • switch on ultrasound for short periods only
No operational functions; LEDs of LED ultrasound and LED temperature blink alternately ("running light") = fault indication programme control	<ul style="list-style-type: none"> • fault of electronics 	<ul style="list-style-type: none"> • switch unit off and on • if fault is again indicated: return unit to supplier or manufacturer

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