**Renfert**

making work easy


- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Polski
- FAQ - Dansk
- FAQ - Українська
- FAQ - 中文
- FAQ - 日本語
- FAQ - 한국어
- FAQ - Nederlands
- FAQ - Suomalainen
- FAQ - Norsk
- FAQ - Svenska
- FAQ - Eesti keel
- FAQ - Latviski
- FAQ - Lietuvių
- FAQ - čeština
- FAQ - Magyar
- FAQ - Slovenský
- FAQ - Ελληνικά
- FAQ - Hrvatski
- FAQ - Română
- FAQ - Slovenščina
- FAQ - български

# SYMPRO

FAQ - frequently asked questions

Made in Germany

21-7190 18122017

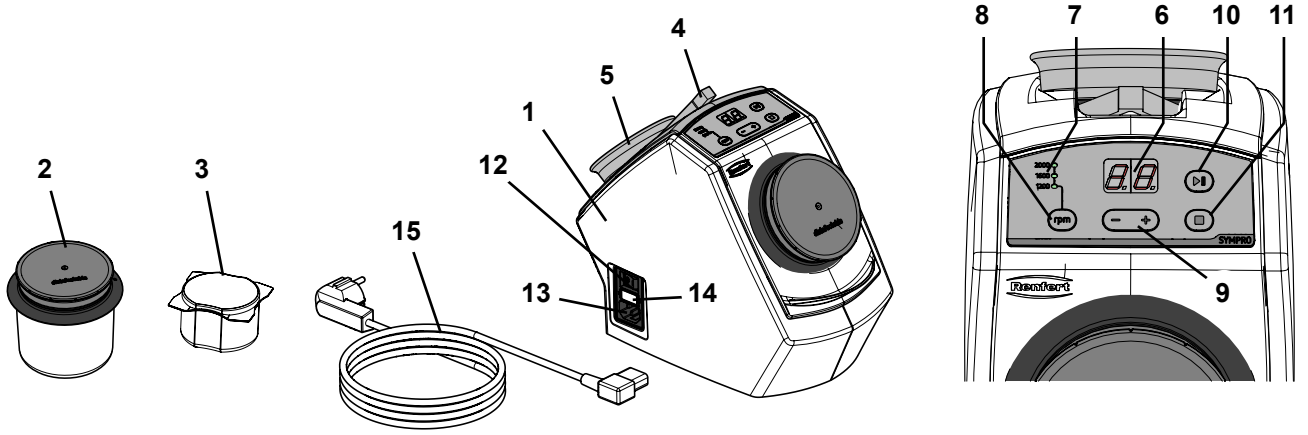**Fig. 1**

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## FAQ - English

For information on operating elements, see [Fig. 1](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

Trouble	Possible Cause	Corrective Action
<b>No display.</b>	<ul style="list-style-type: none"> <li>• Appliance not connected to mains.</li> <li>• Appliance not turned on.</li> <li>• No power from the socket.</li> <li>• Faulty fuse.</li> </ul>	<ul style="list-style-type: none"> <li>• Check correct fit of power cable in the appliance socket and wall socket.</li> <li>• Turn on appliance at the mains switch (<a href="#">12. Fig. 1</a>).</li> <li>• Check power socket, for example, with another appliance.</li> <li>• Replace the fuse (see section 6.8). If the fuse blows repeatedly, have the appliance inspected by a service technician.</li> </ul>
<b>Cannot change cleaning time.</b>	<ul style="list-style-type: none"> <li>• Appliance still running.</li> </ul>	<ul style="list-style-type: none"> <li>• Interrupt the cleaning process with the Start / Pause button, and then adjust cleaning time.</li> </ul>
<b>Cannot save cleaning time.</b>	<ul style="list-style-type: none"> <li>• Cleaning time was entered during an interrupted cleaning process.</li> </ul>	<ul style="list-style-type: none"> <li>• Stop the cleaning process with the Stop button, and then enter new cleaning time.</li> </ul>
<b>Insufficient cleaning performance.</b>	<ul style="list-style-type: none"> <li>• Cleaning time too short.</li> <li>• Active ingredients of the cleaning fluid spent.</li> <li>• Insufficient number of cleaning needles.</li> <li>• Cleaning needles spent.</li> </ul>	<ul style="list-style-type: none"> <li>• See chapt. 5.4.</li> <li>• Prepare fresh cleaning fluid after each application.</li> <li>• Fill up the cleaning needles (see section 6.5).</li> <li>• Replace the cleaning needles (see section 6.6).</li> </ul>
<b>Cleaning needles have corroded.</b>	<ul style="list-style-type: none"> <li>• Cleaning needles have been insufficiently rinsed.</li> <li>• Cleaning needles have been stored incorrectly.</li> <li>• Cleaning needles in use for longer period of time.</li> </ul>	<ul style="list-style-type: none"> <li>• Rinsing and storage see chapter 6.1.</li> <li>• Removing the corrosion: Clean the needles with 100 ml SYMPROfluid Universal, without contents, for 5 minutes at 2000 rpm. Dispose of cleaning liquid.</li> <li>• The needles are so-called consumable goods. If it is not possible to regenerate the needles, or the needles' performance deteriorates, see chapter 6.6.</li> </ul>

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