Renfert-Scanspray classic

REF 17311000

ENGLISH | INSTRUCTIONS FOR USE


Titanium dioxide-free

Congratulations! On choosing this Renfert-Scanspray classic, you have purchased a high quality, innovative product.

Description:

Renfert-Scanspray classic is finely-granulated 3D matting spray for extra-oral scanning of shiny or highly-reflective surfaces (e.g., models, primary crowns, frameworks). As well as offering an extra-large can for power users, it is also impressively cost-effective and long-lasting. The Renfert-Scanspray classic is suitable for all conventional optical scanner systems and produces exact optical image quality.

Application:

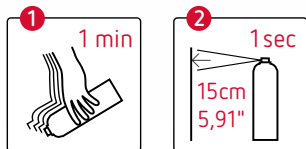Before use, shake can well for approx. 1 min. so that you can clearly hear the bead inside. Hold the can as upright as possible and spray a thin layer onto the surface to be scanned from a distance of approximately 15 cm. It is advisable to apply the product in short sprays (approx. 1 sec.). This way, only small amounts of powder are distributed, at the same time the nozzle system cleans itself during this procedure. The surface to be sprayed must be clean and dry. When spraying impression material, it is essential to ensure that the material is completely set first.

Removal:

After scanning, the sprayed surface can be cleaned easily with water and steam cleaning.

Cleaning the nozzle:

The special spray technique means the nozzle will not become blocked, even after frequent use. If necessary, the nozzle can be cleaned with water or compressed air.

Hazard warnings and safety advice:

Extremely flammable aerosol. Pressurized container: it may burst if exposed to heat. May cause drowsiness and dizziness. Harmful to aquatic organisms with long-term effects. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not

spray in the direction of an open flame or other source of ignition. Do not pierce or burn, even after use. Avoid inhalation of aerosols. Store in a well-ventilated place. Keep container tightly closed.

Protect against direct sunlight and do not expose to temperatures above 50 °C/122 °F.

Please always observe the information on the safety data sheet.

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