making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - 中文
- FAQ - 한국어
- FAQ - اللغة العربية

# SILENT powerCAM TC

FAQ - frequently asked questions

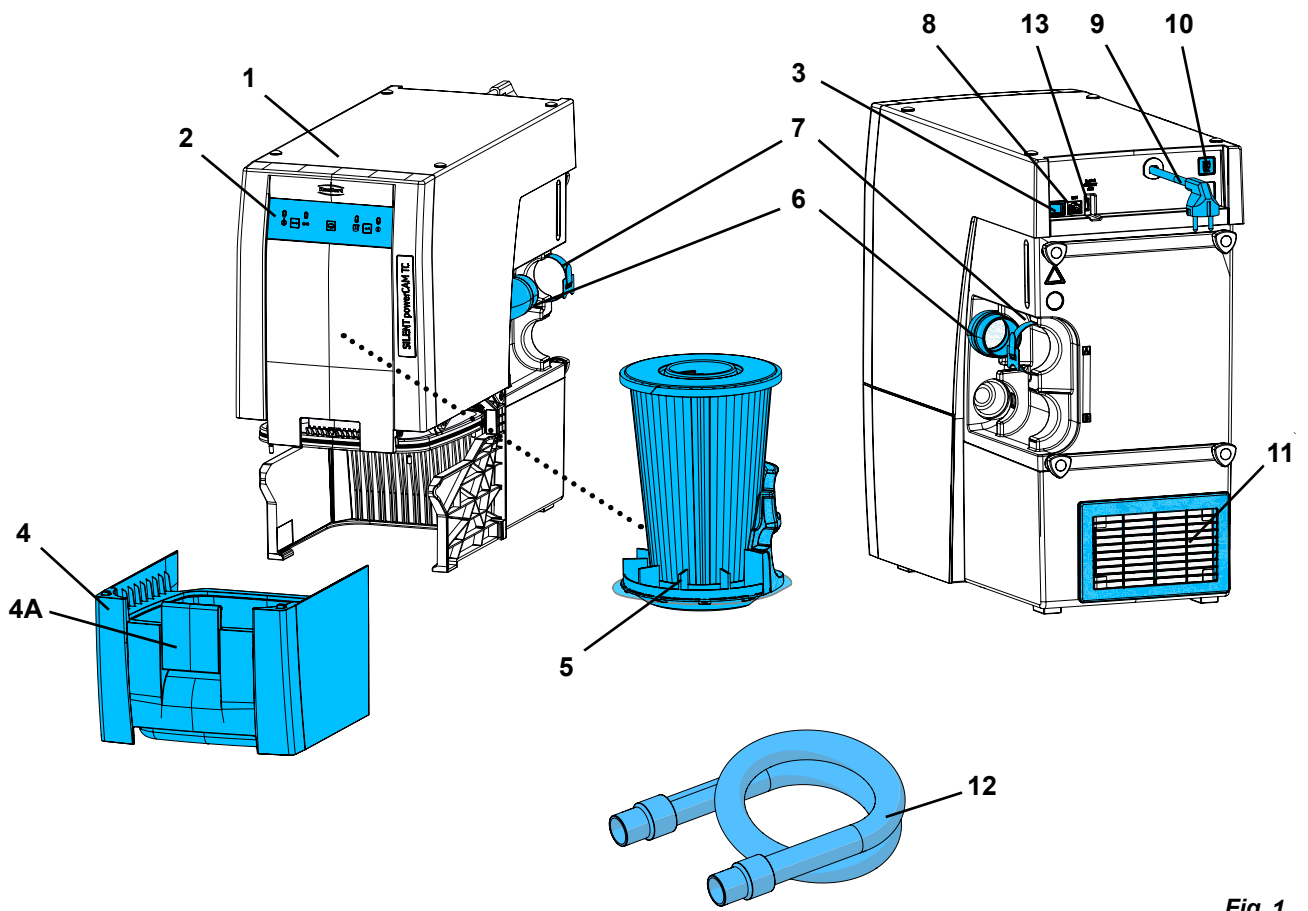
Fig. 1

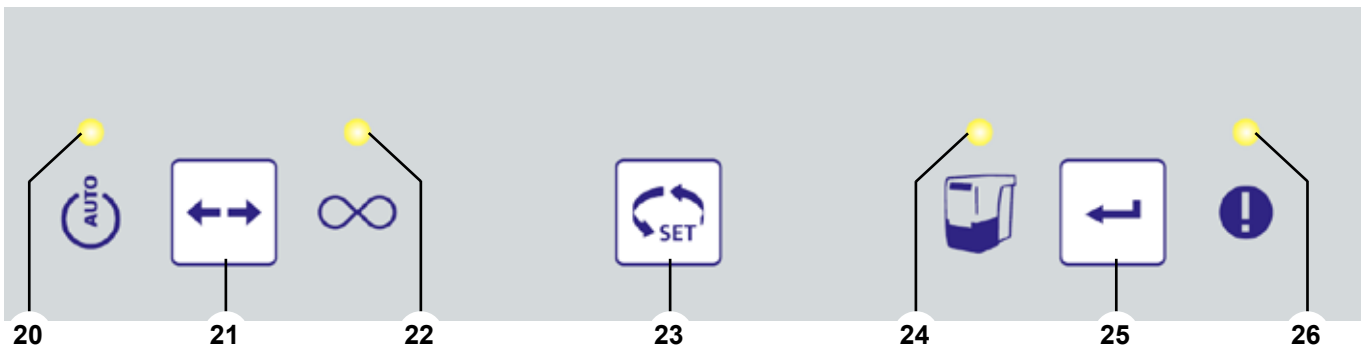
Fig. 2

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## FAQ - English

For information on operating elements, see [Fig. 1](#) and [Fig. 2](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

en

Trouble	Cause	Corrective action
<b>Error notification display (26, <a href="#">Fig. 2</a>) flashes.</b>	<ul style="list-style-type: none"> <li>The electronics have overheated.</li> </ul>	<ul style="list-style-type: none"> <li>Switch off the unit and let it cool down.</li> <li>Provide sufficient cooling, e.g., using:                             <ul style="list-style-type: none"> <li>Note Chap. 4.2 Setup.</li> <li>Use external air duct (see Chap. 4.6).</li> </ul> </li> </ul>
<b>The error notification display (26, <a href="#">Fig. 2</a>) lights up.</b>	<ul style="list-style-type: none"> <li>The fine filter is so heavily clogged that cleaning the filter does not have a sufficient impact.</li> </ul>	<ul style="list-style-type: none"> <li>Replace fine filter (see Chap. 6.3).</li> </ul>
<b>The suction power is not sufficient.</b>	<ul style="list-style-type: none"> <li>Blockage or leakage in the suction hose.</li> <li>Dust drawer not airtight.</li> <li>Fine filter blocked.</li> </ul>	<ul style="list-style-type: none"> <li>Check suction hose.</li> <li>Please also note the information in Chap. 4.4.</li> <li>Check that the dust drawer fits correctly (see Chap. 6.2).</li> <li>Switch the unit off and then on again so that the filter is cleaned. Replace the fine filter (see Chap. 6.3) if cleaning the filter does not improve suction power.</li> </ul>
<b>Dust drawer is overflowing.</b>	<ul style="list-style-type: none"> <li>The selected time interval for emptying the dust drawer is too long.</li> </ul>	<ul style="list-style-type: none"> <li>Set a shorter time interval (see Chap. 6.2.1).</li> </ul>
<b>The signal to empty the dust drawer is output, even though it is not full yet.</b>	<ul style="list-style-type: none"> <li>The time interval set for emptying the dust drawer is too short.</li> </ul>	<ul style="list-style-type: none"> <li>Set a longer time interval (see Chap. 6.2.1).</li> </ul>
<b>The display to empty the dust drawer flashes and an acoustic signal sounds three times.</b>	<ul style="list-style-type: none"> <li>The flow rate is no longer sufficient and filter cleaning is performed.</li> </ul>	<ul style="list-style-type: none"> <li>Wait until the filter has been cleaned.</li> </ul>
<b>The extraction unit unexpectedly stops suctioning during continuous or automatic operation and the corresponding displays (20/22, <a href="#">Fig. 2</a>) remain on.</b>	<ul style="list-style-type: none"> <li>Suction turbine has overheated.</li> <li>Suction turbine is faulty.</li> </ul>	<ul style="list-style-type: none"> <li>Switch off the unit and leave to cool for at least 60 minutes.</li> <li>Check whether suction hose is blocked, eliminate blockage.</li> <li>Switch the unit off and then on again so that the filter is cleaned. Replace the fine filter (see Chap. 6.3) if cleaning the filter does not improve suction power.</li> <li>Replace suction turbine.</li> </ul>
<b>The extraction unit does not respond to the signals of the CAM system.</b>	<ul style="list-style-type: none"> <li>The connector to the CAM system or the extraction unit is not correctly inserted.</li> <li>Interface cable damaged.</li> <li>Connector assignment of the interface cable does not match that of the CAM system.</li> </ul>	<ul style="list-style-type: none"> <li>Check the connector on the extraction unit and CAM system.</li> <li>Check the interface cable for damage, replace if required.</li> <li>Compare the connector assignment of the interface cable with that of the CAM. Use a suitable interface cable if required.</li> </ul>
<b>After switching on, not all displays light up briefly.</b>	<ul style="list-style-type: none"> <li>The display that does not light up is faulty.</li> </ul>	<ul style="list-style-type: none"> <li>Contact Renfert / Service.</li> </ul>

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