making work easy

FAQ - Deutsch

FAQ - English

FAQ - Français

FAQ - Italiano

FAQ - Español

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FAQ - Türkçe

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FAQ - English

For chapter indications, refer to the complete instruction manual on the internet.

Error	Possible cause	Corrective action
The device generates no vibrations.	• Power cord not properly plugged in at the device. • Blown fuse. • No power at the wall outlet.	• Check that the power cord is securely seated in the socket. • Replace the fuse (refer to Sec. 3.4). • Test the wall outlet for power.
No difference in the vibration intensity between the switch settings Fine / Coarse.	• Faulty diode (short circuit).	• Return the device for service.
No function in coarse oscillation mode.	• Faulty diode (interruption). • Faulty switch. • Faulty switch connection wiring.	• Return the device for service.
No function in fine oscillation mode.	• Faulty switch. • Faulty switch connection wiring.	• Return the device for service.
No effect at the different power level settings.	• Faulty switch. • Faulty switch connection wiring.	• Return the device for service.
Device extremely loud during operation.	• Loose internal components. • Loose rubber mat.	• Return the device for service. • Attach the rubber mat correctly (see Chap. 1.2).

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