making work easy

SILENT XS

FAQ - frequently asked questions

Made in Germany

205212 03032026

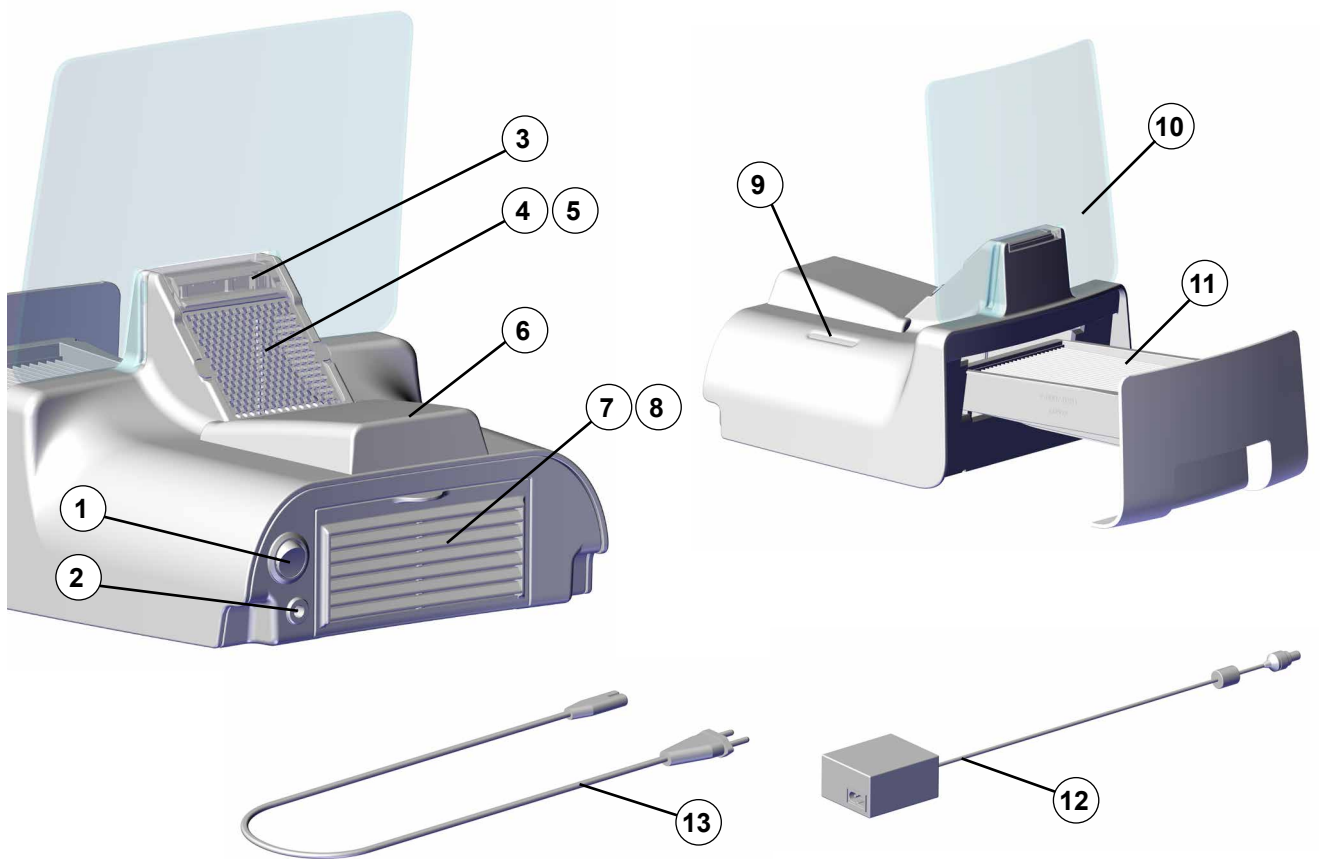
Fig. 1

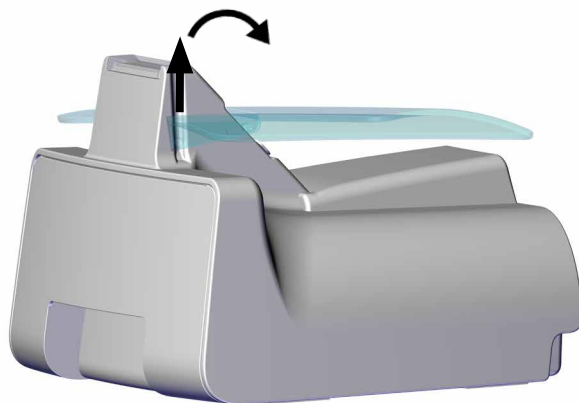
Fig. 2

www.renfert.com/td/prod/pp0010

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www.renfert.com/td/prod/pp0010

en

The following documents and information are available for download here:

- Instructions For Use (IFU)
- Quick Start Guide
- Spare parts lists and spare parts drawings
- Repair instructions
- FAQs, cause and rectification of faults
- How-to videos
- Declarations of conformity

Note: The documents and information listed are not available for all products.

FAQs – English

For more detailed information, please refer to the complete Instructions For Use (IFU) that can be found online.

Question / Error / Problem	Cause / Explanation	Answer / Troubleshooting / Solution
What is the best way to prepare the unit for its next use? (Storage / disinfection, etc.)		• Remove contaminations and grinding residue after each use. • Follow the instructions in the Hygiene Guide (see chapter 3.2 and the Hygiene Guide).
Why does it take a few seconds for the battery charge indicator to appear? (it shows white LEDs when batteries are inserted)	• This is no malfunction	• It takes up to 4 seconds to determine the charge status, and this process cannot be sped up.
Is it possible to adjust the suction power on the SILENT XS?	• This is no malfunction	• The suction power is not adjustable. • The suction power remains the same for both, battery and mains mode.
The batteries have been inserted, but the unit cannot operate on battery mode.	• The polarity of one or more of the cells in the battery holder is reversed.	• Use the +/- markings to check the polarity of all batteries in the battery holder and correct it if necessary.
	• Insufficient number of batteries.	• Insert the required number of batteries (8 x AA NiMH) into the battery holder. • Refer to chapter 4.2 for information on the recommended rechargeable batteries. • Refer to chapter 7 for information on the minimum requirements for rechargeable batteries.
	• Missing contact between the battery holder and one or more batteries.	• All batteries in the battery holder must make contact with each other and with the metal contact plates or springs. The easiest way to do this is to gently rotate each cell in the battery holder individually.
The charging time seems too long.	• Depending on the manufacturer, capacity, and age of the inserted batteries, charging can take up to 5 hours. • The charging time increases if the unit is used while connected to a power source during charging.	• Make sure the power supply unit is properly plugged into the charging socket, and check that the pulsating white LEDs indicate the charging status. • Use new rechargeable batteries. - Refer to chapter 4.2 for information on the recommended rechargeable batteries. - Refer to chapter 7 for information on the minimum requirements for rechargeable batteries.

Question / Error / Problem	Cause / Explanation	Answer / Troubleshooting / Solution
Is it possible to use Li-ion batteries in the Silent XS?		• No! Using them can damage the unit's electronics or batteries (due to an inappropriate charging process). • Refer to chapter 4.2 for information on the recommended rechargeable batteries. • Refer to chapter 7 for information on the minimum requirements for rechargeable batteries.
The unit was used in battery mode, ran out of power, and can no longer be charged, or has not been charged for a long time.	• The unit's deep discharge protection is enabled when the voltage is too low to protect the batteries from deep discharge. For safety reasons, the batteries can only be charged outside the unit in this case.	• Remove the batteries and charge them with an external charger (if available). • Alternatively, use new batteries if charging in an external charger was not successful or is not possible. • The unit can always be used with the power supply unit.
Light flickers when the unit is running on battery mode.	• This problem may occur with certain types of rechargeable batteries when their capacity decreases.	• Use the power supply unit to fully charge the batteries. • Refer to chapter 4.2 for information on the recommended rechargeable batteries. • Refer to chapter 7 for information on the minimum requirements for rechargeable batteries.
Is it possible to use batteries from different manufacturers and with different capacities?		• In principle, "yes," but: Deviations in unit runtime and charging time are to be expected. • Use batteries from the same manufacturer with the same capacity. • Refer to chapter 4.2 for information on the recommended rechargeable batteries. • Refer to chapter 7 for information on the minimum requirements for rechargeable batteries.
The charge status indicator still shows capacity, but the unit switches off.	• This may occur when using non-recommended batteries.	• Refer to chapter 4.2 for information on the recommended rechargeable batteries. • Refer to chapter 7 for information on the minimum requirements for rechargeable batteries.
Is it possible to use a different power supply unit to operate or charge the unit's batteries?	• No! Damage to the unit cannot be ruled out.	• Only use the original power supply unit.
Can the SILENT XS be used without the viewing pane? If so, does this affect the suction power or particle intake?	• The suction power does not decrease with use. • Without the viewing pane, detection becomes inaccurate, and particles cannot be detected properly.	• Always use the unit with the viewing pane.
How is a full dust drawer detected?	• There is no automatic detection.	• Empty the dust drawer after every grinding operation (see chapter 5.3).
How often do I need to change the fine filter?		• If the suction power decreases significantly despite emptying, the fine filter in the dust drawer (11) must be replaced (see chapter 6.4).
Which materials can be extracted using the SILENT XS?		• The SILENT XS is designed for the suction of dry, non-explosive dust during minor fitting work on disinfected objects (see chapters 2.1 and 2.8).

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany