making work easy

de

en

fr

it

es

pt

zh

ja

ko

# SIMPLEX 3D-Filament-Drucker

Setup - First Steps

Made in Germany

21-7141 15072022

making work easy

# SIMPLEX 3D-Filament-Drucker

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Made in Germany

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# Contents

en

<b>1</b>	<b>Introduction</b>	<b>3</b>
1.1	General description	3
1.2	Symbols used	3
<b>2</b>	<b>Safety</b>	<b>4</b>
2.1	Intended use	4
2.2	Improper use	4
2.3	Ambient conditions for safe operation	4
2.4	Ambient conditions for storage and transport	4
2.5	Hazard and warning information	4
2.5.1	General information	4
2.5.2	Specific information	5
2.6	Authorized persons	5
2.7	Disclaimer	5
<b>3</b>	<b>Product description</b>	<b>6</b>
3.1	Assemblies and functional elements	6
3.2	Scope of delivery	7
3.3	Accessories	7
<b>4</b>	<b>Startup</b>	<b>7</b>
4.1	Unpacking and setup	7
4.2	Printer assembly	8
4.2.1	Installing the filament spool holder	8
4.2.2	Installing the filament sensor (FMS)	8
4.2.3	Installing the filament guiding tube	8
4.2.4	Attaching the door	8
4.2.5	Attaching the cover	9
4.2.6	Connecting the power cord	9
4.3	Switching on	9
<b>5</b>	<b>Getting started</b>	<b>10</b>
5.1	Setting the language	10
5.2	Setting the temperature unit	11
5.3	Wi-Fi connection	11
5.4	Leveling the print bed	12
5.5	Moving the print head manually	13
5.5.1	Attaching the filament spool	14
5.6	Loading the filament	14
5.6.1	Selecting the filament / starting heating	14
5.6.2	Drawing in / extruding the filament	14
5.7	FMS system	16
5.7.1	Calibrating the FMS system	16
5.8	Printing an object	17
5.8.1	Loading an object from a USB flash drive	17

# 1 Introduction

## 1.1 General description

Congratulations on the purchase of your SIMPLEX 3D filament printer and welcome to the world of 3D printing!

The SIMPLEX 3D filament printing system was developed by Renfert with the goal of simplifying and perfecting the 3D printing process with filaments. It offers an excellent manufacturing experience without the need for prior knowledge of programming or 3D printing. To print a 3D object, you need the SIMPLEX 3D filament printer, SIMPLEX sliceware, and a SIMPLEX filament of your choice.

"App Store", "Apple", "Google", "Play Store", and "Android" are registered trademarks of Apple or Google.

## 1.2 Symbols used

Symbols with the following meanings are used in these instructions or on the equipment:

### **Danger**

**Immediate danger of injury. Consult accompanying documents!**

### **Electrical current**

**Danger due to electrical current.**

### **Hot surface**

**Danger of burns due to hot surfaces and hot components.**

### **Danger of hand injury**

**Danger of the user's hands getting crushed by moving parts.**

### **Caution**

**Risk of damage to the equipment if the instruction is not followed.**

**Wear protective gloves!**

### **Notice**

***Provides the operator with useful information that eases use.***

**The equipment complies with applicable EU directives.**

**The equipment is subject to the provisions of the EU directive 2002/96/EC (WEEE directive).**

### ► **List, particular attention should be paid**

- List
- Subordinate list

⇒ Instructions / appropriate action / input / operational sequence:

You are asked to carry out the specified action in the order indicated.

◆ Result of an action / response by the equipment / response by the program:

The equipment or program responds as a result of your actions or when a specific incident occurs.

Other symbols are explained as they occur.

## 2 Safety

Before using the equipment, read this user manual carefully and keep it for future reference. Renfert GmbH reserves the right to modify or revise this user manual at its own discretion at any time.

### 2.1 Intended use

The SIMPLEX 3D filament printer produces three-dimensional objects from melted filament in just one printing step. First you must create the objects using a CAD program and save them as \*.stl, \*.obj or \*.3mf files. You must then use SIMPLEX sliceware to convert your 3D CAD files into print objects for the SIMPLEX 3D filament printer. Transfer your files to your SIMPLEX 3D filament printer using a USB flash drive or a USB cable.

### 2.2 Improper use

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the equipment, increase the risk of serious injury and lead to damage to the environment or the product itself.

### 2.3 Ambient conditions for safe operation

The unit may only be operated:

- Indoors
- Up to an altitude of 2,000 m above sea level
- At an ambient temperature of between 15 - 32 °C [59 - 90 °F]
- At a maximum relative humidity of 80 % at 32 °C [90 °F]
- With mains power where the voltage fluctuations do not exceed 10 % of the nominal value
- Under contamination level 2 conditions
- Under over-voltage category II conditions

### 2.4 Ambient conditions for storage and transport

The following ambient conditions must be observed during storage and transport:

- Ambient temperature of -20 - +60 °C [-4 - +140 °F]
- Maximum relative humidity 80 %

### 2.5 Hazard and warning information

#### 2.5.1 General information

- ▶ If the unit is not used in accordance with the supplied operating instructions, the intended safety can no longer be guaranteed.
- ▶ The unit may only be operated using a power cord equipped with a plug appropriate to the local power supply. Any necessary modifications must be carried out by a qualified electrician.
- ▶ Detachable power cords must not be replaced with inadequately rated power cords. Only use original Renfert spare parts.
- ▶ Only operate the unit if the information on the nameplate corresponds to the specifications of your local mains power supply.
- ▶ The unit may only be plugged into electrical sockets that are connected to the protective conductor system.
- ▶ The power plug must be easily accessible.
- ▶ Disconnect the unit from the mains power supply before carrying out work on the electrical parts.
- ▶ It is the responsibility of the operator to ensure that national regulations are complied with during operation and regarding recurring safety inspection of electrical equipment. In Germany, these are DGUV Regulation 3 in conjunction with VDE 0701-0702.
- ▶ Check connection cables (such as power cords), tubes and housing (such as the key pad) regularly for damage (such as kinks, cracks and porosity) or signs of ageing. Do not operate equipment with damaged connection cables, tubes or housing parts or other defects!
- ▶ Defective equipment must be taken out of service immediately. Disconnect the power plug and secure the equipment against further use. Send the equipment in for repair!
- ▶ Please observe the national accident prevention regulations!
- ▶ Information on REACH and SVHC is available on our website at [www.renfert.com](http://www.renfert.com), in the Support area.

## 2.5.2 Specific information

- ▶ The cover and the door are equipped with a safety function. Operation without a cover and/or door is prohibited.
- ▶ Keep the power cord away from hot surfaces.
- ▶ Do not touch moving parts or heated elements. This can cause injury.
- ▶ Never reach into the unit while the unit is switched on during printing. Various parts of the machine (primarily the extruder and heated print bed) operate at very high temperatures and can cause severe burns.
- ▶ Before maintenance work, ensure that the unit is switched off and the power cord is disconnected from the mains power supply. After disconnecting the power plug, wait at least 5 minutes for the unit to cool before reaching into the printer for maintenance purposes.
- ▶ Always turn off the printer and disconnect it from the computer when not in use.
- ▶ The SIMPLEX 3D filament printer melts plastic filament during printing. Plastic odors/gases are released during printing. Ensure that the SIMPLEX 3D filament printer is placed in a well-ventilated location.
- ▶ Use SIMPLEX filaments, as they do not result in any gases during printing that are harmful to health.
- ▶ SIMPLEX-filaments are not suitable for the application in the patients mouth
- ▶ If using alternative filaments, ensure that no gases are released during printing that are harmful to health. Observe the Safety Data Sheets.
- ▶ When using filaments which are medical products, the used parameters must be coordinated with the manufacturer.
- ▶ This equipment is not suitable for use in locations where children may be present.

## 2.6 Authorized persons

The unit may

- only be used, cleaned and maintained by persons who have been instructed in its use and who have been expressly assigned to operate it,
- only be operated under supervision,
- not be used by children,
- not be used by persons who are unable to operate the unit safely due to limited physical, sensory, or mental capabilities.

Any repairs not described in these operating instructions may only be performed by a qualified electrician.

## 2.7 Disclaimer

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- ▶ The product is employed for any purposes other than those specified in the operating instructions.
- ▶ The product is altered in any way other than those alterations described in the operating instructions.
- ▶ The product is not repaired by an authorized dealer or if original Renfert spare parts are not used.
- ▶ The product continues to be used despite obvious safety defects or damage.
- ▶ The product is subjected to mechanical impact or is dropped.

# 3 Product description

## 3.1 Assemblies and functional elements

- 1 Touchscreen LCD display / main menu
- 2 Filament guiding tube
- 3 Nozzle
- 4 Removable glass plate
- 5 Leveling knobs
- 6 Z-Axis rod
- 7 Filament spool holder
- 8 Filament spool
- 9 Filament sensor
- 10 Extruder
- 11 USB A port for USB flash drive
- 12 USB B port for connection to a PC
- 13 ON/OFF switch / connection for mains power supply
- 14 Port for filament sensor
- 15 Port for cover fan
- 16 Distance card
- 17 Menu - TEMPERATURE 
- 18 Menu - EXTRUDER 
- 19 Menu - USB PRINT 
- 20 Menu - AXES 
- 21 Menu - SETTINGS 

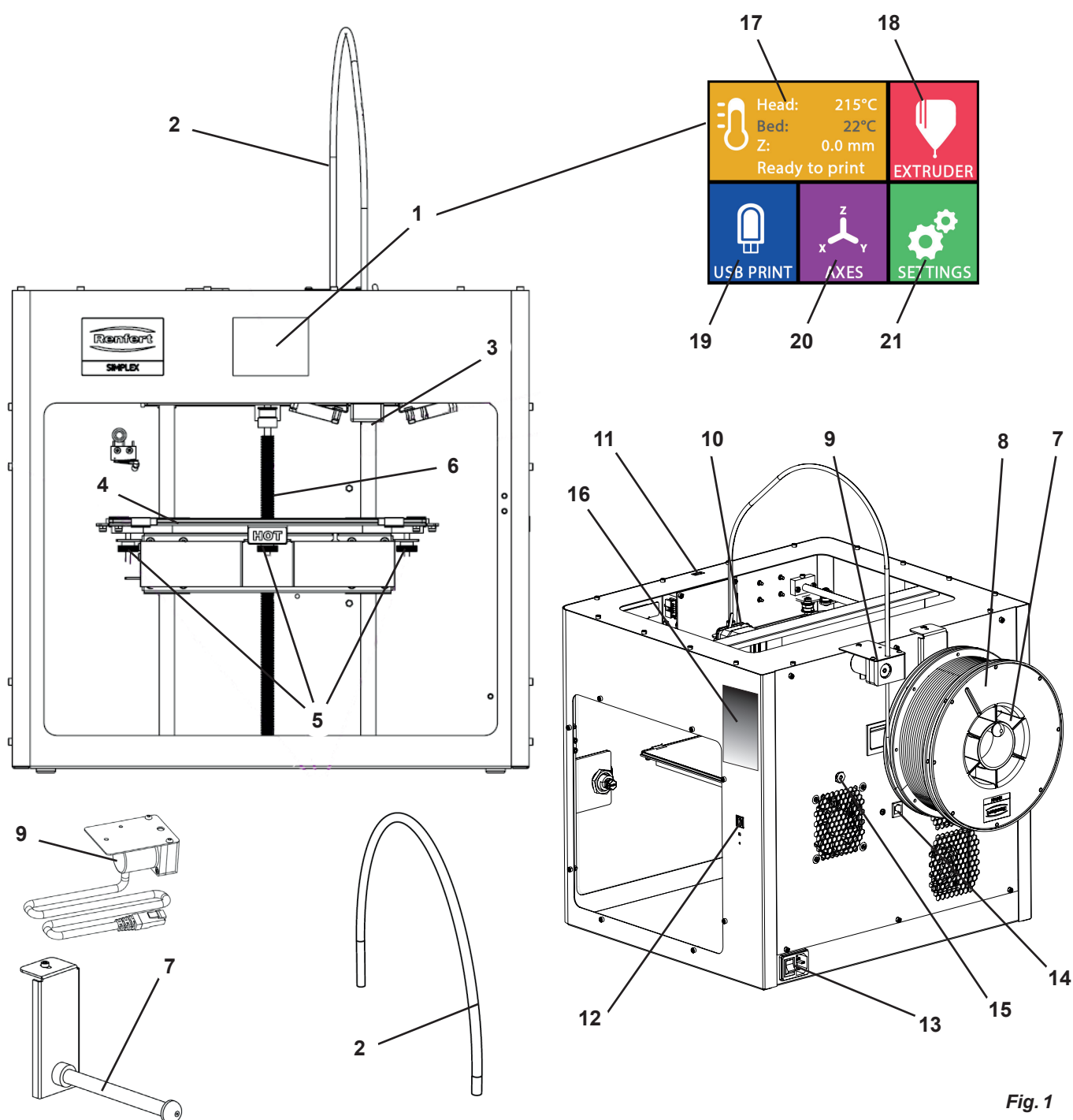
Fig. 1

## 3.2 Scope of delivery

1 SIMPLEX 3D filament printer	1 USB A-B cable
1 Filament guiding tube	1 Allen keys (set of 5)
1 Filament spool holder	1 Distance card
1 Filament spool: SIMPLEX study model	1 Replacement nozzle 0.4 mm
1 Filament sensor (FMS)	2 Nozzle keys
1 Cover	1 Spatula
1 Door including lock and key	1 Wire cutters
1 USB flash drive	1 Conductive paste (in dosing syringe)
1 Power cord	1 Operating instructions SIMPLEX 3D filament printer
1 Power adapter set	1 Operating instructions SIMPLEX sliceware

## 3.3 Accessories

1735 0100	SIMPLEX study model 800 g
1735 0200	SIMPLEX working model 800 g
1735 0300	SIMPLEX aligner model 800 g
1735 0600	SIMPLEX multi-use model 800 g

# 4 Startup

The unit must be stored at room temperature for at least 3 hours before startup.

Do not apply force to the unit during unpacking, setup, operation, or maintenance.

## 4.1 Unpacking and setup

A video on this topic is provided under the following link  
[www.renfert.com/simplex-1-en](http://www.renfert.com/simplex-1-en) and via the QR code shown.

Select a setup location where the printer is not exposed to direct sunlight or drafts.

- ⇒ Place the shipping box on a flat surface on the floor. Ensure that you have enough space available.
- ⇒ Open the shipping box. It contains three boxes:
  - Unit
  - Cover
  - Accessories box
- ⇒ Remove the three boxes.
- ⇒ Carefully open the box with the unit.
- ⇒ Remove the foam insert.
- ⇒ Carefully remove the accessories box provided. It protects the extruder during transport. This box contains the accessories for the SIMPLEX 3D filament printer.
- ⇒ Lift the SIMPLEX 3D filament printer out of the shipping box by gripping the insides of the frame.

Please note the weight of the printer and ask a second person to assist you if necessary.

When removing, be careful not to touch the extruder and the electronics.

The SIMPLEX 3D filament printer is not suitable for setup in the treatment room.

- ⇒ Place the SIMPLEX 3D filament printer on a stable and level surface with sufficient space around the unit.

Do not cover the ventilation openings on the rear of the unit

- ⇒ Ensure that the on/off switch and power cord are easily accessible.
- ⇒ Using the wire cutters, remove the two cable ties that secure the print bed.

**Caution: Do not damage the Kapton film!**

- ⇒ Remove the door, which is located under the print bed.
- ⇒ Carefully open the box with the cover so as not to damage the contents.
- ⇒ Remove the cover and set it aside carefully for subsequent assembly.

## 4.2 Printer assembly

### 4.2.1 Installing the filament spool holder

- ⇒ Remove the screw (31) on the top rear using the Allen key provided.
- ⇒ Secure the filament spool holder (7) with the screw in this position.

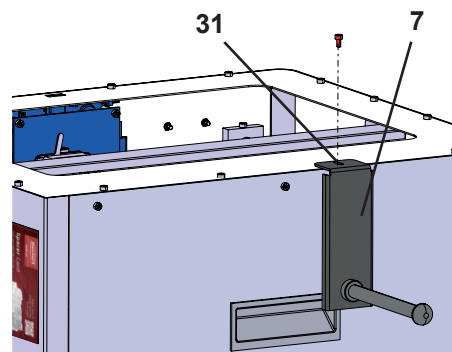
Fig. 2

### 4.2.2 Installing the filament sensor (FMS)

- ⇒ Remove the screw (30) on the top rear using the Allen key provided.
- ⇒ Secure the FMS (9) with the screw in this position.
- ⇒ Plug the FMS connection cable into the rear of the unit (14).

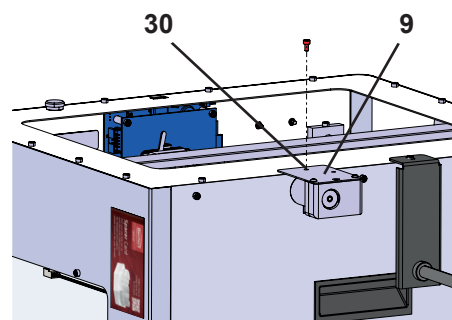
Fig. 3

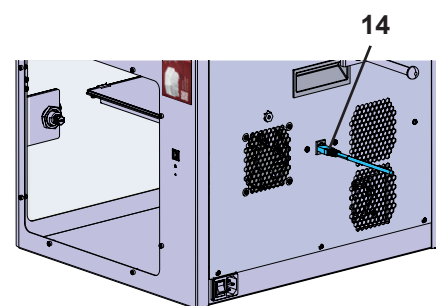
Fig. 4

### 4.2.3 Installing the filament guiding tube

- ⇒ Insert one end of the filament guiding tube (2) into the FMS (9) from above.
- ⇒ From above, insert the other end into the opening on the front of the extruder (10).

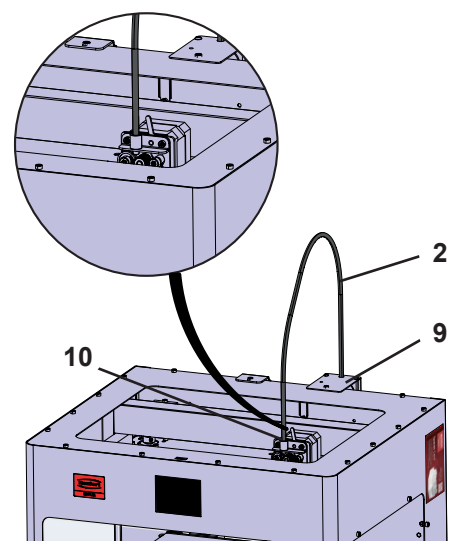
Fig. 5

### 4.2.4 Attaching the door

- ⇒ Hold the door at right angles to the unit on the hinge bolts.
- ⇒ Lower the door until the hinges thread into both hinge bolts.
- ⇒ Close the door.

### 4.2.5 Attaching the cover

- ⇒ Remove the four screws (32) on the top of the unit using the Allen key provided.
- ⇒ Place the cover into position.
- ⇒ Connect the fan to the socket (15) on the rear of the unit.

**Also ensure subsequently that the SIMPLEX unit is switched off when you connect the fan as otherwise a short circuit could occur.**

**The cover does not necessarily have to be secured again with the screws.**

**A safety system detects when the cover is removed and then prevents any of the axes from moving.**

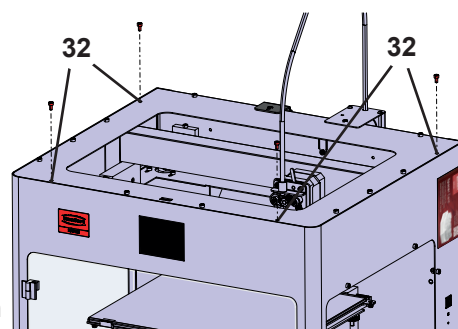
Fig. 6

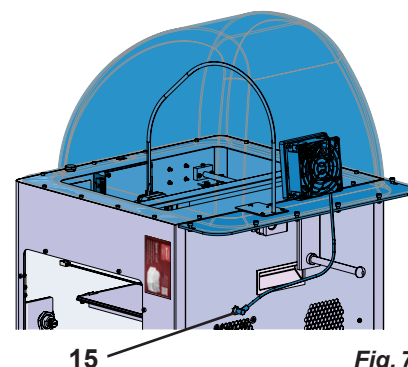
Fig. 7

### 4.2.6 Connecting the power cord

**Ensure that the on/off switch (13) is in the OFF position.**

- ⇒ Plug the power cord into the connection for the mains power supply (13) on the rear of the unit.
- ⇒ Insert the power plug into a building electrical socket.

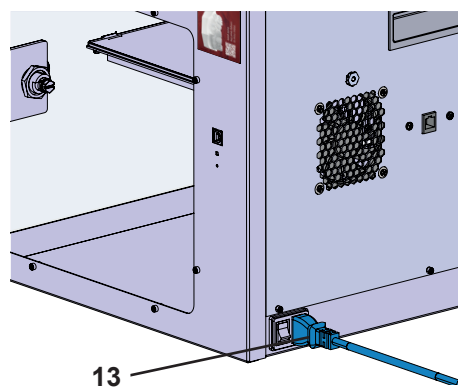
Fig. 8

## 4.3 Switching on

- ⇒ Switch the unit on using the on/off switch (13, Fig. 1) on the rear of the unit.
  - ◆ The LCD display on the front lights up.
  - ◆ The main menu is shown:
    - Menu - TEMPERATURE
    - Menu - EXTRUDER
    - Menu - USB PRINT
    - Menu - AXES
    - Menu - SETTINGS.

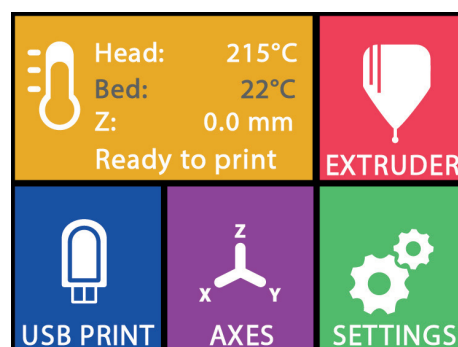
Fig. 9

## 5 Getting started

The following chapters describe the workflow that is required for initial startup in order to print an object that is provided on the supplied USB flash drive.

A prerequisite is the assembly of the printer as described above.

In addition, all of the following steps must be carried out in the order described.

The first steps include:

- Basic settings in the SETTINGS menu
  - Setting the language
  - Setting the temperature unit
  - Connecting the printer with a Wi-Fi network
- Leveling the print bed
- Moving the print head manually
- Loading the filament
  - Selecting a filament
  - Heating the print head
  - Drawing in the filament
  - Extruding the filament manually
- Calibrating the filament sensor (FMS)
- Loading the model from a USB flash drive
- 3D printing

Additional setting and control options offered by the printer are described in subsequent chapters.

### **Note on using the menu:**

**Use the lower left arrow ← to go back one page, exit a setting, cancel an operation, or return to the main menu.**

### 5.1 Setting the language

A video on this topic is provided under the following link  
[www.renfert.com/simplex-2-en](http://www.renfert.com/simplex-2-en) and via the QR code shown.

- ⇒ Select the SETTINGS menu. ⚙️.
- ⇒ Use the right arrow → to go to page 2 (as shown here in the image).
- ⇒ Press the button with the language icon .
- ⇒ Use the arrows ↑ / ↓ to select the required language.
- ⇒ Confirm your selection with OK.
- ⇒ Press the lower left arrow ← until you are back in the main menu.

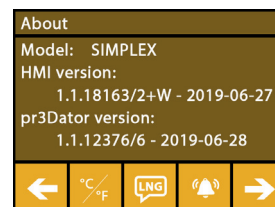
Fig. 10

Fig. 11

## 5.2 Setting the temperature unit

A video on this topic is provided under the following link [www.renfert.com/simplex-3-en](http://www.renfert.com/simplex-3-en) and via the QR code shown.

en

- ⇒ Select the SETTINGS menu. ⚙️.
- ⇒ Use the right arrow → to go to page 2 (as shown here in the image).
- ⇒ Press the button with the temperature icon °C/°F.
- ⇒ The unit can be changed by pressing the arrow buttons ← / → at the top.

**The factory setting is °Celsius.**

- ⇒ Press the lower left arrow ← until you are back in the main menu.

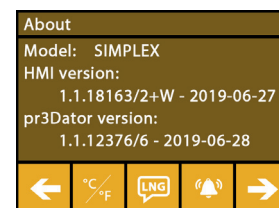
Fig. 12

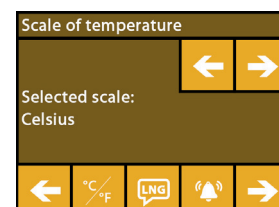
Fig. 13

## 5.3 Wi-Fi connection

A video on this topic is provided under the following link [www.renfert.com/simplex-8-en](http://www.renfert.com/simplex-8-en) and via the QR code shown.

**The Wi-Fi module is only available in EU member states and in the USA. The button with the Wi-Fi icon 📶 can only be operated if the Wi-Fi module is available.**

**The Wi-Fi module may only be used as an information tool for monitoring printing. Print jobs cannot be started, paused, or stopped using the Wi-Fi module.**

- ⇒ Select the SETTINGS menu. ⚙️.
- ⇒ Use the right arrow → to go to page 4 (as shown here in the image).
- ⇒ Press the button with the Wi-Fi icon 📶.
- ⇒ Press the button with the search icon 🔍 to search for a Wi-Fi network.
  - ♦ A list of Wi-Fi networks is shown
- ⇒ Use the up/down arrows ↑ / ↓ and select the required Wi-Fi network using the checkmark ✓.
- ⇒ Enter the password of the selected Wi-Fi network.
- ⇒ Confirm the password using the checkmark ✓.

**A connection was successfully established to the selected Wi-Fi network:**

- ♦ The name of the connected Wi-Fi network is shown
- ♦ The IP address (LAN IP) allocated for the SIMPLEX 3D filament printer is shown.

**A connection could not be established to the selected Wi-Fi network:**

- ♦ An error message is shown:  
"No connection to host <Name of the WiFi network>"

- ⇒ Press the lower left arrow ← until you are back in the main menu.

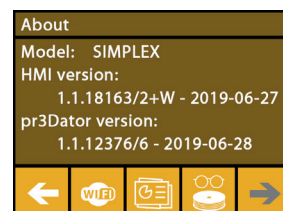
Fig. 14

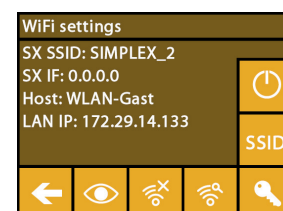
Fig. 15

## 5.4 Leveling the print bed


A video on this topic is provided under the following link [www.renfert.com/simplex-4-en](http://www.renfert.com/simplex-4-en) and via the QR code shown.

The SIMPLEX unit features a leveling process that ensures that the extruder and heated print bed are optimally spaced (0.3 mm) and leveled uniformly.

A smaller gap or uneven leveling can result in damage to the Kapton film (orange colored tape on the print bed), the print bed, the extruder, or in printing issues.

The gap between the extruder and the print bed is set using three adjustment screws under the print bed:

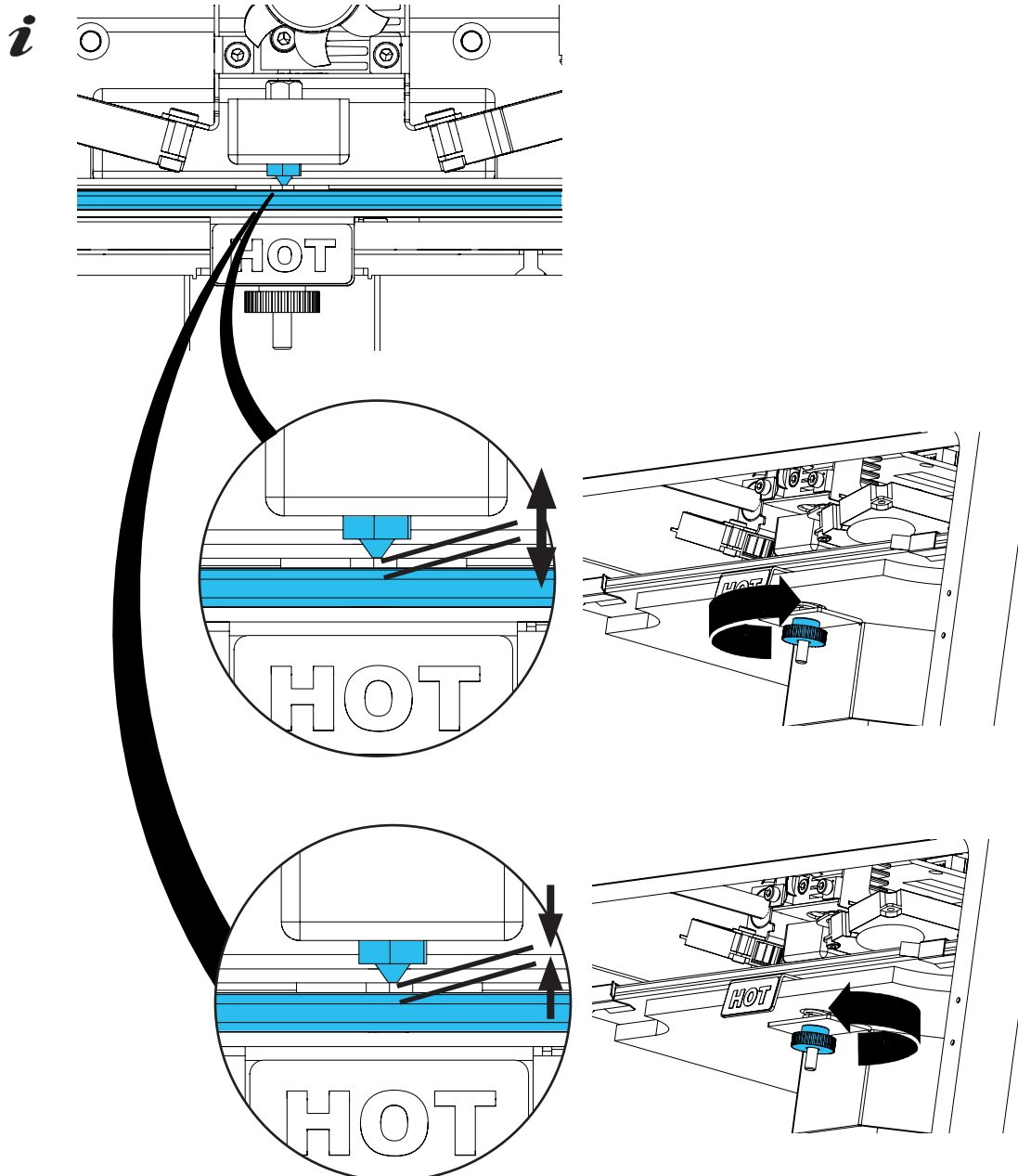*The print bed should be leveled after initial installation and approx. 1x every month.*

**i** You can ignore the "Please clean the nozzle!" message the first time the print bed is leveled.

**!** When you level the print bed again subsequently, ensure beforehand that a filament is not loaded, and that the nozzle is clean. Clean the nozzle carefully using a soft cloth. Wear gloves as the nozzle is at operating temperature at this point.

**i** You can cancel leveling at any time using the left arrow ← and return to the menu above.

- i** When prompted to do so, tighten the three leveling knobs (knurled screws) under the print bed. Do not overtighten them. You need to be able to adjust them for leveling.
- i** Alignment to three reference points is performed (one point after the other). Alignment to the three reference points is performed 2 x. You can correct leveling if required during the second run.

- ⇒ Ensure that there is no filament in the extruder!
- ⇒ Keep the distance card (16, Fig. 1) ready at hand. This card ensures the optimal distance between the build plate and the extruder. Keep this card for future use.
- ⇒ Select the SETTINGS menu. 
- ⇒ Press the button for the calibration menu .
- ⇒ Using the right arrow →, scroll through the messages on the display and follow the instructions.
- ⇒ If necessary, wait until the extruder has reached the set temperature.
- ⇒ Once the temperature has been reached, use the right arrow → to perform leveling step by step.
- ⇒ Follow the instructions on the LCD display.
  - ◆ Alignment to three reference points is performed (one point after the other).
- ⇒ Set the corresponding leveling knob in each case so that a distance card can slide between the nozzle and the print bed with minimum resistance.
- ⇒ Close the door after each step in order to proceed using the right arrow.
  - ◆ Alignment to the three reference points is performed 2 x.
- ⇒ At the second run, check the setting of the leveling knobs and correct it if required.
  - ◆ Once this operation has been completed, the print bed should be evenly leveled.
- ⇒ Using the right arrow →, exit leveling and return to the menu above.
- ⇒ Press the lower left arrow ← until you are back in the main menu.

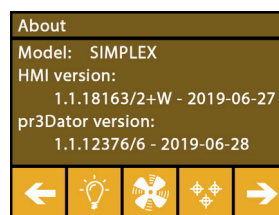
Fig. 16

Fig. 17

## 5.5 Moving the print head manually

A video on this topic is provided under the following link [www.renfert.com/simplex-5-en](http://www.renfert.com/simplex-5-en) and via the QR code shown.

The print head must be moved to an appropriate position for the following steps.

- ⇒ Select the AXES  menu.

- i** If the house icon is shown in black instead of white, first press the XYZ house icon  to move the print head to a defined home position.

- i** Ensure that there are no objects on the print bed when moving to the home position.

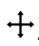
- ⇒ Press the button with the axis icon .
- ⇒ Press the Y+ and X+ arrow buttons to position the print head to the side outside of the print bed.
- ⇒ Press the lower left arrow ← until you are back in the main menu.

Fig. 18

Fig. 19

### 5.5.1 Attaching the filament spool

⇒ Place the filament spool (8) on the filament spool holder (7) in such a way that the filament can be inserted into the filament sensor from underneath.

**Ensure that the filament spool is placed on the filament spool holder in such a way that the filament can be inserted clockwise into the FMS from underneath (Fig. 20).**

**Only use Renfert filaments, as they do not result in any gases during printing that are harmful to health.**

**If using alternative filaments, ensure that no gases are released during printing that are harmful to health.**

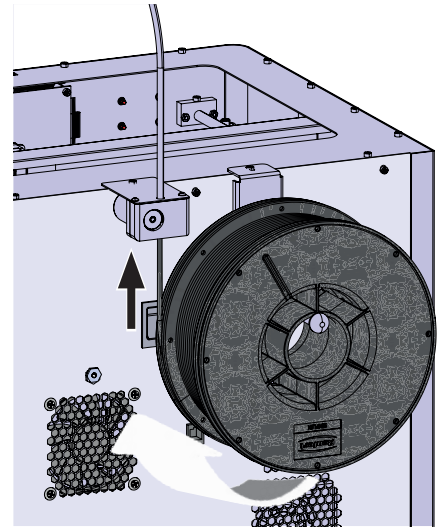
Fig. 20

## 5.6 Loading the filament

A video on this topic is provided under the following link [www.renfert.com/simplex-6-en](http://www.renfert.com/simplex-6-en) and via the QR code shown.

### 5.6.1 Selecting the filament / starting heating

**For loading, the extruder must be pre-heated to operating temperature (= printing temperature).**

The temperature values for the extruder (= head temp:) and the print bed (= bed temp:) of four different filaments are stored in the SIMPLEX 3D printer.

- TEMP1: SIMPLEX study model / working model / multi-use
- TEMP2: SIMPLEX aligner model
- USER1: Freely configurable
- USER2: Freely configurable

**Chap. 7.2. explains how you can store customer temperature values for the USER1 and USER2 filaments.**

⇒ Select the TEMPERATURE  menu.

⇒ Using the filament spool icon , select the required filament.

Select the TEMP1 filament for the study model filament that is included with the delivery.

⇒ Press the extruder icon  to start heating.

- ◆ On the display, the current temperature and the target temperature change from gray to white.

⇒ Press the lower left arrow  until you are back in the main menu.

Fig. 21

### 5.6.2 Drawing in / extruding the filament

⇒ Remove the printer cover. When doing so, pay attention to the cable of the fan.

⇒ Pull the end of the filament through the safety hole of the filament spool.

⇒ Cut off the end of the filament in such a way that the filament is straight without any damage, bends, kinks, or melted parts (Fig. 22).

Fig. 22

- ⇒ Remove the filament guiding tube (2) through the opening on top of the extruder (10).
- ⇒ Insert the end of the filament from underneath through the FMS into the filament guiding tube until it exits on the other side.

**Ensure that the filament does not spring off the spool or become tangled. The filament must be neatly wound on the spool. Improperly wound or knotted filaments can lead to printing errors or even misprints.**

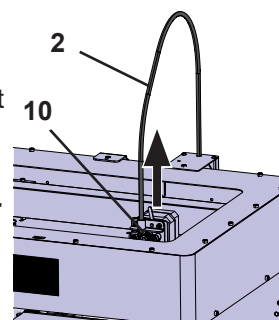
Fig. 23

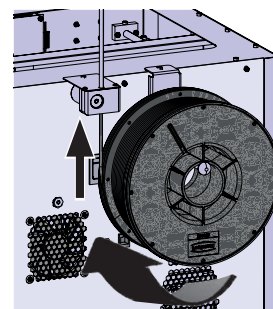
Fig. 24

- ⇒ Move the lever at the top of the extruder to the left (Fig. 25).

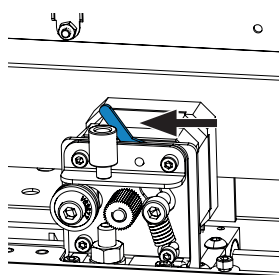
Fig. 25

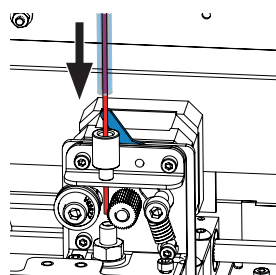
Fig. 26

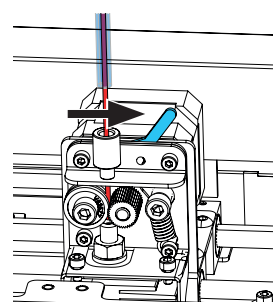
Fig. 27

- ⇒ Push the filament through the top opening of the extruder until the filament is between the two guide wheels (Fig. 26).
- ⇒ Now move the lever of the extruder to the right (Fig. 27)
- ⇒ Ensure that the filament is held by the two guide wheels by gently pulling on the filament and feeling resistance.

- ⇒ Heating of the extruder was already started in the previous step.

♦ The head temperature is shown in white in the TEMPERATURE menu.

- ⇒ Wait until "Ready to print" is shown in the TEMPERATURE menu.

- ⇒ Select the EXTRUDER menu.

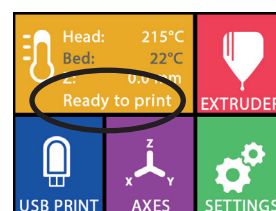
Fig. 28

If you select the EXTRUDER menu too early and the extruder has not yet reached the required temperature, a warning message is shown.

- ⇒ Skip the warning message with the right arrow →.

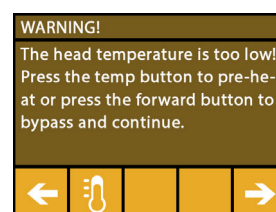
Fig. 29

⇒ Wait until the extruder has reached the required temperature. The current temperature and the required temperature are shown at the top of the display.

⇒ The filament can be extruded using the "EXTRUDE"  button. The cogwheel will only turn for as long as you press the button.

Alternatively:

⇒ Press the load filament  button. This automatically extrudes 15 cm of filament. This can be canceled by pressing the lower left arrow .

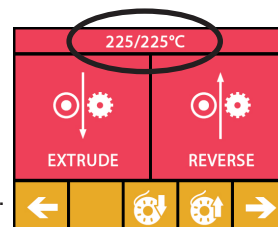
Fig. 30

⇒ Now insert the end of the filament guiding tube into the upper opening of the extruder again.

⇒ Place the cover on the printer. Pay attention to the cable of the fan.

**If the fan cable has come loose, first switch off the printer before you insert the connector into the unit again.**

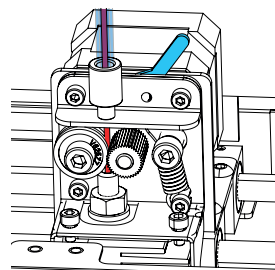
Fig. 31

## 5.7 FMS system

The filament sensor (FMS) controls the filament flow during printing.

If a jam is detected during printing, the printer attempts to fix this itself as part of an automatic fault correction process.

**You must ensure that the FMS system is always calibrated with the currently loaded filament and the corresponding temperature. If this is not the case, the filament flow cannot be properly monitored during printing.**

**The FMS system is negatively impacted, and printing is not correctly monitored.**

### 5.7.1 Calibrating the FMS system

A video on this topic is provided under the following link [www.renfert.com/simplex-7-en](http://www.renfert.com/simplex-7-en) and via the QR code shown.

The FMS system must be calibrated in order for it to operate correctly.

As a result of the previous steps, the extruder is already heated up and the filament is loaded.

⇒ Select calibration with the  icon.

**The FMS system must be calibrated with the filament used correctly selected.**

**If you replace the filament, i.e., use a different type, the FMS system must be recalibrated.**

⇒ If necessary, change the selected filament with the  icon.

⇒ Start the calibration using the start icon .

- ◆ The print head and print bed are moved to the home position.
- ◆ The print head is moved to a position beside the print bed.
- ◆ A preset quantity of material is extruded.

After successful calibration, this is shown on the display.

If calibration is not successful, an error message is shown.

⇒ In this case, repeat calibration.

**After calibration has been completed, the FMS system is adapted to the filament used and can detect and display errors in the filament flow.**

⇒ Press the lower left arrow  until you are back in the main menu.

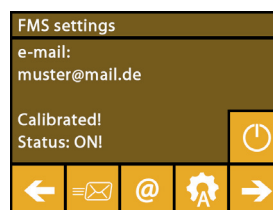
Fig. 32

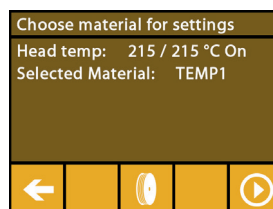
Fig. 33

## 5.8 Printing an object

A video on this topic is provided under the following link [www.renfert.com/simplex-10-en](http://www.renfert.com/simplex-10-en) and via the QR code shown.

en

To print a 3D object, SIMPLEX requires toolpath information that is generated from 3D object CAD files such as .obj or .stl. Toolpath information is known as a "G-code".

**Sample G-codes are stored on the USB flash drive provided. You can use these to immediately carry out test prints.**

**Make sure to use the G-code for the currently loaded filament. Using an incorrect G-code can cause misprints or damage to the printer.**

**The following G-codes are available:**

**G-Code SIMPLEX study model, working model, multi-use model ==> Filament TEMP1**

**G-Code SIMPLEX aligner model ==> Filament TEMP2**

### 5.8.1 Loading an object from a USB flash drive

⇒ Switch off the SIMPLEX unit (13).

⇒ Insert the USB flash drive into the USB port (11) on the SIMPLEX unit.

⇒ Switch on the SIMPLEX unit (13).

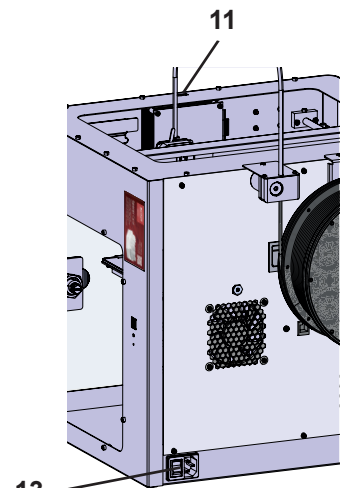
Fig. 34

⇒ Select the USB  menu.

⇒ Use the arrows  /  to select the file name of the object.

**Using the right arrow  and the "up directory"  icon, you can navigate through the directory structure of the USB flash drive.**

⇒ Press the printer icon  to start printing.

Fig. 35

♦ The unit warms the extruder and the print bed to the temperatures stored in the G-code.

♦ As long as the temperatures have not yet been reached, the two letters "H" and "B" are shown in red on the display.

H: Head = Print head

B: Bed = Print bed

♦ As soon as the stored temperatures are reached with sufficient accuracy, the color of the letters changes to white, and printing begins.

Fig. 36

**During printing, the following information is shown on the display:**

- Name of the file to be printed
- Head (H) and bed (B) temperature
- Z-position information
- Printing speed
- Filament: remaining / total
- Progress bar with information in % as well as elapsed time / total time required.

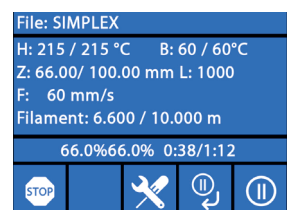
Fig. 37

### Printing is complete:

- ♦ For easier removal of the object, the print bed is automatically lowered a little and the print head is moved to the home position.
- ♦ A corresponding message is shown on the display. This can be confirmed with "OK".

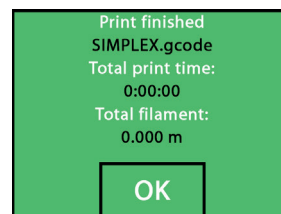
Fig. 38

### Caution! Danger of burns!

The print head and print bed could still be hot.

Wear protective gloves!

- ⇒ If necessary, you can use the "AXES" menu (see Chap. 5.5) to lower the print bed further.
- ⇒ Remove the object from the print bed. Carefully remove the object with one hand using a slight twisting motion while holding the print bed with the other hand. If the object is difficult to remove, use the spatula provided.

Take care not to scratch the Kapton film.

*So that removal of the object is as easy as possible, wait until the print bed has cooled.*

If necessary, you can also remove the build plate from the printer to remove the object.

To do this, loosen the fixing clasps on the front right and left of the print bed and remove the build plate. When reinserting, ensure that the build plate has been inserted in the correct end position and that the fixing clasps are closed.

*An incorrectly seated build plate can cause misprints or damage to the printer.*

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany