making work easy

- de
- en
- fr
- it
- es
- pt
- tr
- ru
- zh
- ja
- ko

Basic quattro

Quick Start Guide

Made in Germany

20-5196 20112025

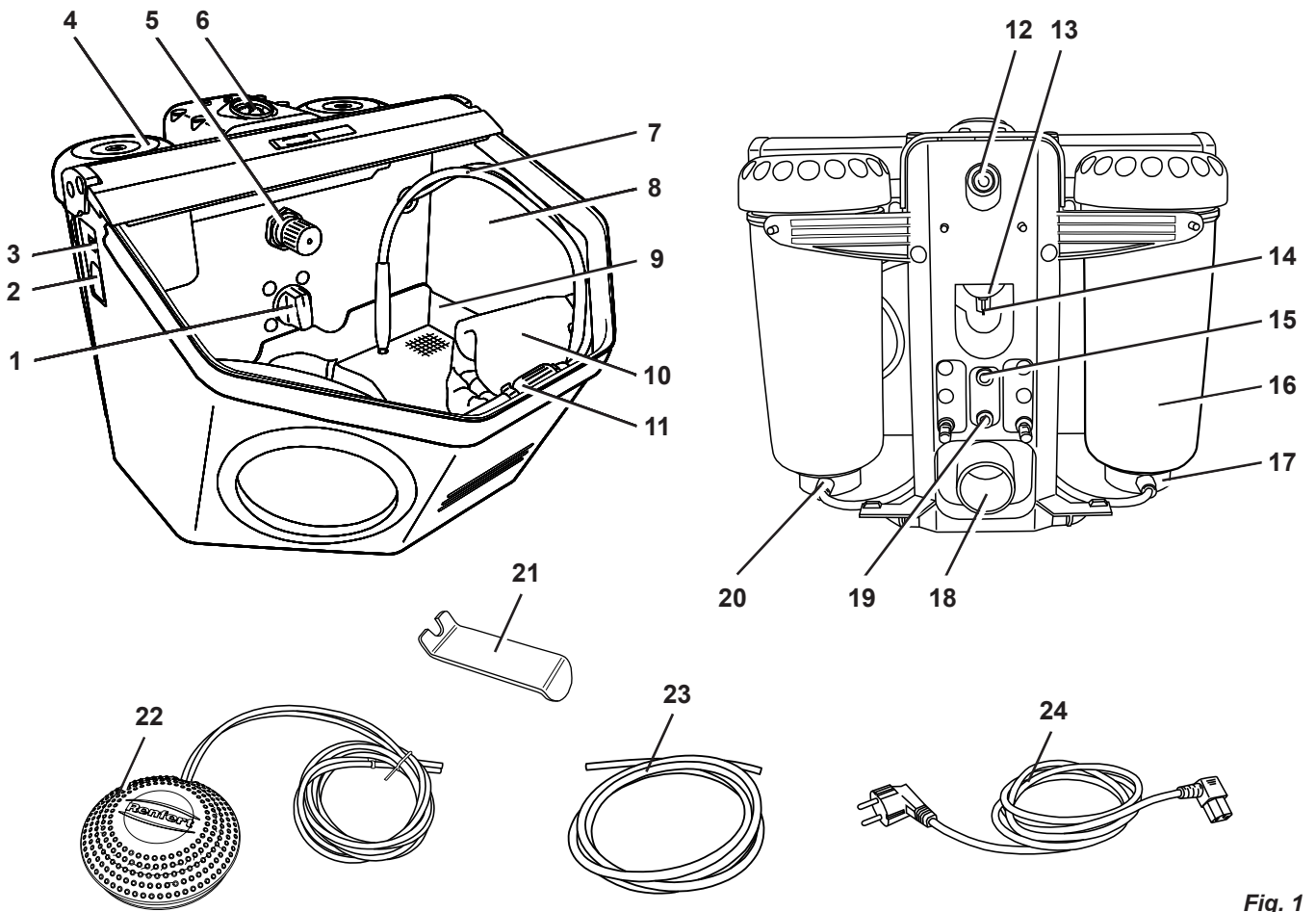
Fig. 1

www.renfert.com/29580000

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www.renfert.com/29580000

en

Here you will find the following documents and information for download:

- Operating Instructions
- Quick Start Guide
- Spare parts lists and spare parts drawings
- Maintenance manual
- FAQs, cause and rectification of faults
- How to - Videos
- Declarations of conformity

Please note: The documents and information listed are not available for all products.

Employed Symbols

In these instructions or on the unit itself you will find symbols which have the following meaning:

Danger

This indicates an imminent risk of injury. In these instructions or on the unit itself you will find symbols which have the following meaning.

Electrical hazard

There is a risk of electrical shock.

Attention

Failure to observe the associated information can result in damage to the unit.

Note

The operating instructions contain useful information to make handling easier.

Attention!

Wear eye protection.

The unit complies with the relevant EC guidelines.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under www.renfert.com.

The unit is subject to the EC guidelines 2002/96/EG (WEEE Directive).

Other symbols are explained when they are used.

Safety

Intended use

The Basic sandblaster is intended for commercial use in dental laboratories in accordance with the instructions for use. The unit is intended for use in the following tasks:

- Removal of residual investment material and of oxides on cast components
- Surface treatment
- Devestment of press ceramics
- Sandblasting of ceramic occlusal surfaces

Meaning of these instructions

These instructions for use act as a guide for safe usage of the unit.

If the instructions for use are not observed, accidents may lead to life-threatening electric shock, injury, or damage.

- ⇒ Only use the unit in accordance with these instructions for use.
- ⇒ Keep the instructions for use ready with the unit.
- ⇒ Provide the instructions for use to all subsequent users of the unit.

Requirements that the operator must meet

- ⇒ The product may only be used by persons aged 14 or over who are familiar with the procedures and rules in a dental laboratory.

Safe condition of the unit

Faulty or broken components, or those that are not sealed as required, may cause injuries.

- ⇒ Check the power cord, housing, and other components such as connection cables, tubes, and the key pad for damage, for example such as kinks, cracks, porosity, and signs of aging.
- ⇒ Defective equipment must be taken out of service immediately. Disconnect the power plug. Send the unit in for repair or dispose of it safely. Until this happens, secure the unit to prevent it from being switched on again so that it is not used inadvertently.
- ⇒ Do not expose the unit to mechanical impact. Do not drop it.

Risks as a result of modifications and repairs

Incorrect modifications and repairs to the unit may cause accidents, fire, and electric shock.

- ⇒ Only have the unit repaired by qualified specialist dealers or return it.

Risk as a result of electric shock and fire

Risk of electric shock and of fire as a result of excessive operating voltage or if there is no protective conductor.

- ⇒ Ensure that the power plug is easily accessible.
- ⇒ Only operate the unit using a power cord equipped with a plug appropriate to the local power supply. Only allow qualified electricians to modify power cords.
- ⇒ Only start the unit if the information on the nameplate matches the power supply.
- ⇒ Only connect the unit to electrical sockets that are connected to the protective conductor system.

Risk of injury to the eyes or skin

Injury may occur to the eyes or skin as a result of high pressure or of components fracturing that are subject to high pressure.

- ⇒ Always wear appropriate protective goggles when carrying out work on the unit.
- ⇒ Never direct the blasting material towards the eyes or unprotected areas of skin.
- ⇒ Never perform sandblasting with the viewing panel open.

Risk to health as a result of dust

Dust emissions may result in damage to health.

- ⇒ Operate the unit with a suitable extraction unit. The extraction unit must be adapted to the dust that occurs.

Product description

Components and functional elements

See Fig. 1 (on inside cover)

| | | |
|-----------------------------|--------------------------------------|--------------------------------------|
| 1 Selector switch | 9 Base grid | 17 Mixing chamber |
| 2 Appliance socket | 10 Hand sleeves | 18 Suction tube |
| 3 ON / OFF switch | 11 Air nozzle | 19 Foot switch connection (B) |
| 4 Tank lid | 12 Air connection | 20 Filter cartridge |
| 5 Pressure regulator | 13 Water trap | 21 Releasing tool |
| 6 Pressure gauge | 14 Water trap valve | 22 Foot switch |
| 7 Hand piece | 15 Foot switch connection (A) | 23 Compressed air tube |
| 8 Protection screen | 16 Sandblasting tank | 24 Power cord |

Standard Delivery

- 1 Basic quattro
- 1 Foot switch
- 1 Connection set
- 1 Power cord
- 1 Quick Start Guide

Commissioning

Risk of injury!

Loose tubes may hit objects around them. Tubes must be connected first to the unit. Only press the foot switch after all tubes have been connected.

en

Installation

⇒ Choose a stable and non-slip surface to stand the unit on.

Foot switch connection

⇒ Insert the end of the tube marked with an „A“ (22) as far as it will go into the tube connection point also marked with an „A“ (15) on the blasting unit.

⇒ Insert the end of the tube marked with an „B“ (22) as far as it will go into the tube connection point also marked with an „B“ (19) on the blasting unit.

When inserting the tubes you will notice a slight resistance which must be overcome.

Compressed air connection

⇒ Push the compressed air tube (23) onto the tube connection point (12) as far as it will go.

⇒ To connect to the compressed air system, select the relevant tube coupling from the supplied set and attach this to the other end of the tube.

⇒ Connect to the compressed air network.

Electrical connection

Please note the instructions in the chapter “Risk of electric shock and fire”!

⇒ Plug the mains cable (24) into the socket (2).

⇒ Plug in the mains cable.

Connection to suction device

⇒ Plug the suction tube from the extraction unit to the suction pipe (18).

◆ For suction devices with automatic activation:

⇒ Connect the mains plug from the sandblasting unit into the user socket on the suction device.

Adjusting the Operating Pressure

⇒ Activate the foot switch.

⇒ Set the sandblasting pressure at the pressure regulator (5).

◆ The amount of pressure will be shown on the pressure gauge (6).

Filling the Sandblasting Tanks

Do not press the foot switch while filling is in progress!

Regardless of the pressure display, the sandblasting tanks are only pressurized when the foot switch is pressed.

⇒ Use only clean, dry abrasive of the appropriate grain size (refer to the accessories).

⇒ Only fill with abrasives up to the max. fill height (thickening of the tank cover underneath the thread).

⇒ Clean the thread on the sandblasting tank and tank lid as well as the seal.

⇒ Screw the tank lid closed firmly by hand.

Warning: Do not tilt the tank lid when screwing it on.

Check that the tank lid is seated securely prior to use.

Tank lids that are not closed securely can suddenly blow off.

There is a risk of injury as a result of flying parts and sudden emission of abrasive sand.

Authorised sandblasting material

- Cobra (Renfert GmbH)
- Rolloblast (Renfert GmbH)
- ROCATEC (3M ESPE)

Sandblasting materials from other manufacturers may be used providing the grain size, grain shape and degree of purity comply with the above mentioned materials.

Operation

**If the membrane key is damaged or defect, please disconnect the device and send away for repair!
If the membrane is removed there is the risk of an electrical shock!**

- ⇒ Switch the illumination on (3).
- ⇒ Select the blasting tank at the selection switch (1).
- ⇒ Take the respective hand piece from the selected blasting tank (see coloured marking).
- ⇒ Activate the foot switch (22).

Cleaning / Maintenance

Sandblasting tank

**Check the sandblasting tank or tank lid regularly, at least 1 x a year, by visually inspecting for damage or changes to the material (e.g., cracks ...).
If in doubt, replace!**

**The sandblasting tank and tank lid are subject to a natural aging process depending on ambient conditions and must be replaced as standard after every 15 years of operation.
Note the year of manufacture on the round stamp engraved on the sandblasting tank and in the tank lid; Our customer service team are happy to advise you.**

To clean:

**DO NOT use solvent-based cleaning agents or disinfectants.
Solvents and tensides can result in microcracking in the plastic (danger of explosion!).**

- ⇒ Only clean the sandblasting tank and tank lid by rubbing with a dry cloth.

Cleaning the Interior

- ▶ DO NOT use solvent-containing cleaning agents or disinfectants. (use e.g. soapy water).
- ⇒ Remove the grate and vacuum the blasting chamber.

Condensation

- ⇒ Inspect the water separator (13).
- ⇒ Drain any water off through the valve (14).

Spare Parts

**Only use accessories and spare parts that have been supplied or approved by Renfert GmbH.
Using other accessories or other spare parts can lead to inadvertent accidents and damage.**

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

Obligations regarding information

- ▶ Information on REACH and SVHC is available on our website www.renfert.com/service in the Support area.

Disposal instructions

The unit must be disposed of by an authorized recycling operation. The selected firm must be informed of all possibly health-hazardous residues in the unit.

Within the European Union units with this symbol should not therefore be disposed of in unsorted domestic waste.

For more information regarding proper disposal please apply at your local authorities.

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declare under our sole responsibility, that the product(s)

| Name | Part No. |
|--------------------|--|
| Vario basic | 2960 0005 |
| Basic quattro IS | 2959 0000 / 2959 0000 03 |
| Basic quattro | 2958 0000 / 2958 0000 03 |
| Basic master | 2948 2000 / 2948 2000 03 2948 2025 / 2948 2025 03 2948 2250 / 2948 2250 03 |
| Basic classic | 2947 1050 / 2947 1050 03 2947 1250 / 2947 1250 03 2947 2000 2947 2025 / 2947 2025 03 2947 2250 |
| Basic eco | 2949 1050 / 2949 1050 03 2949 1250 / 2949 1250 03 2949 2025 / 2949 2025 03 |
| Basic mobil | 2914 3050 2914 3250 |
| Dustex master plus | 2626 0105 / 2626 0105 03 |

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Electrical Equipment (Safety) Regulations 2016

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

Technical Data

| | |
|--|---|
| Voltage | 220 - 240 V, 50 / 60 Hz 100 - 120 V, 50 / 60 Hz |
| Working pressure: | 1 - 6 bar [14.5 - 87 psi] |
| Connecting pressure, external: | 6 - 8 bar [87 - 116 psi] |
| Air consumption: | 98 l/min at 6 bar [3.46 cfm at 87 psi] |
| Lamp power consumption: | 16 W |
| Cable length, approx.: | 2 m [78 inch] |
| Ø Compressed air hose: interior: exterior: | 4 mm [157 mil] 6 mm [236 mil] |
| Ø Size of pipe union for external extraction unit: interior: exterior: | 35 mm [1.38 inch] 40 mm [1.57 inch] |
| Dimensions (W x H x D): | 395 x 285 x 460 mm [15.6 x 11.2 x 18.1 inch] |
| Blasting chamber volume: | 20 l [5.3 US gal] |
| Tank capacity, each: | 1000 ml [34 fl oz] |
| Weight (empty), approx.: | 8.9 kg [19.6 lbs] |
| This product contains a light source of energy efficiency class: | F |
| Environmental conditions for safe operation: | |
| Ambient conditions: | Indoors |
| Installation height, max.: | 2,000 m [6,500 ft] above sea level |
| Ambient temperature: | 5 - 40 °C [41 - 104 °F] |
| Relative humidity: | Max. 80 % at 31 °C [87.8 °F], decreasing linearly to 50 % at 40 °C [104 °F] *) |
| Voltage fluctuations in the mains power supply from the nominal value, max.: | 10 % |
| Contamination level: | 2 |
| Over-voltage category: | II |

*) Between 5 - 30 °C [41 - 86 °F] the device can be operated at a relative humidity of up to 80 %.
At temperatures between 31 - 40 °C [87.8 - 104 °F] the humidity must decrease proportionally in order to ensure operational readiness (e.g. at 35 °C [95 °F] = 65 % humidity, at 40 °C [104 °F] = 50 % humidity). The device may not be operated at temperatures above 40 °C [104 °F].

**Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...**

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany