making work easy

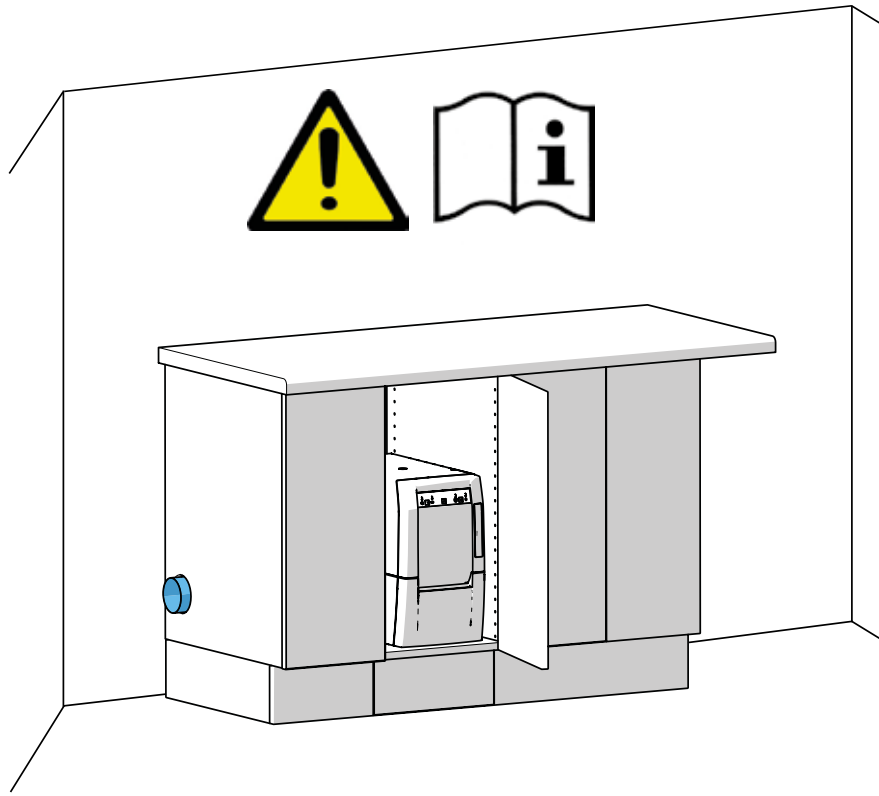
- de
- en
- fr
- it
- es
- pt
- tr
- ru
- pl
- cs
- zh
- ja
- ko
- ar
- uk
- ro

# SILENT compactCAM

Made in Germany

21-6815 25102023

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en

# SILENT compactCAM

TRANSLATION OF THE ORIGINAL INSTRUCTIONS FOR USE

Made in Germany

21-6815 25102023

# Contents

|           |   |           |
|-----------|---|-----------|
| <b>1</b>  | <b>Introduction</b>                           | <b>3</b>  |
| 1.1       | Symbols                                       | 3         |
| <b>2</b>  | <b>Safety</b>                                 | <b>4</b>  |
| 2.1       | Intended Use                                  | 4         |
| 2.2       | Improper Use                                  | 4         |
| 2.3       | Ambient Conditions for Safe Operation         | 4         |
| 2.4       | Ambient Conditions for Storage and Transport  | 4         |
| 2.5       | Hazard and Warning Information                | 4         |
| 2.5.1     | General Information                           | 4         |
| 2.5.2     | Specific Information                          | 5         |
| 2.6       | Authorized Persons                            | 5         |
| 2.7       | Disclaimer                                    | 5         |
| <b>3</b>  | <b>Product Description</b>                    | <b>6</b>  |
| 3.1       | General Description                           | 6         |
| 3.2       | Components and Functional Elements            | 6         |
| 3.3       | Scope of Delivery                             | 7         |
| 3.4       | Accessories                                   | 7         |
| <b>4</b>  | <b>Setting Up</b>                             | <b>8</b>  |
| 4.1       | Unpacking                                     | 8         |
| 4.2       | Setup   | 8         |
| 4.3       | Electrical Connection                         | 8         |
| 4.4       | Connection to the Extraction Point            | 8         |
| 4.5       | Connection to a CAM interface                 | 9         |
| 4.6       | External Exhaust Air Route                    | 9         |
| <b>5</b>  | <b>Operation</b>                              | <b>9</b>  |
| 5.1       | Switching the Unit On                         | 9         |
| 5.2       | Select Mode: CAM-mode / continuous operation  | 9         |
| 5.3       | Filter Cleaning                               | 10        |
| <b>6</b>  | <b>Cleaning / Maintenance</b>                 | <b>10</b> |
| 6.1       | Cleaning                                      | 10        |
| 6.2       | Empty Dust Drawer                             | 10        |
| 6.2.1     | Setting the Time Period "Empty Dust Drawer"   | 11        |
| 6.3       | Change Fine Filter                            | 11        |
| 6.4       | Safety Mechanism                              | 11        |
| 6.5       | Spare Parts                                   | 12        |
| 6.6       | Factory Settings                              | 12        |
| <b>7</b>  | <b>Troubleshooting</b>                        | <b>12</b> |
| <b>8</b>  | <b>Technical Data</b>                         | <b>13</b> |
| <b>9</b>  | <b>CAM interface</b>                          | <b>14</b> |
| 9.1       | Pin assignment CAM interface (12, Fig. 1)     | 14        |
| 9.1.1     | Outputs (Out1 / Out2)                         | 14        |
| 9.1.2     | Inputs (In1 / In2)                            | 14        |
| 9.2       | Control commands / Status information         | 14        |
| 9.3       | Pin assignment interface cable                | 14        |
| <b>10</b> | <b>Warranty</b>                               | <b>16</b> |
| <b>11</b> | <b>Disposal Information</b>                   | <b>17</b> |
| 11.1      | Disposing of Consumables                      | 17        |
| 11.2      | Disposing of the Unit                         | 17        |
| 11.2.1    | Disposal Instructions for countries in the EU | 17        |

# 1 Introduction

## 1.1 Symbols

In the instructions for use and on the unit itself you will find these symbols with the following meanings:

### **Danger**

This indicates a direct risk of injury. Consult accompanying documents!

### **Electrical current**

This indicates a risk of hazard due to an electrical current.

### **Attention**

Disregarding this warning may result in damage to equipment.

### **Note**

This provides the operator with useful information to improve and ease use.

The device complies with the requirements of the applicable EU directives.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under [www.renfert.com](http://www.renfert.com).

The device is subject to the EU directive 2002/96/EG (WEEE directive).

There is a FAQ video available on this topic in our video gallery under [www.renfert.com/p49](http://www.renfert.com/p49).

### ► List, particular attention should be paid

- List
- List

⇒ Instructions / appropriate action / input / operational sequence:

You will be asked to carry out the action in a specified order.

- ◆ Result of an action / reaction of the device / reaction of the program:

The unit or program reacts as a result of your actions or when a specific incident occurs.

Other symbols are explained as they occur.

# 2 Safety

## 2.1 Intended Use

This extraction unit is intended for use with dental milling machines for the suction of dry dust, generated whilst milling commercially available milling blocks and blanks.

The device is intended exclusively for commercial use.

The intended use also includes compliance with the instructions specified by the manufacturer concerning operation, servicing and maintenance.

## 2.2 Improper Use

Fire-promoting, easily flammable, hot, burning or explosive materials must not be suctioned into the device.

It is not permitted to suction liquids.

This device is not intended for private, household use.

Any use other than specified in these instructions is deemed improper and constitutes a misuse of the device.

The manufacturer shall not be liable for damages caused by improper use.

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.

## 2.3 Ambient Conditions for Safe Operation

The device may only be operated:

- Indoors
- Up to an altitude of 2,000 m above sea level,
- At an ambient temperature of between 5 - 40 °C [41 - 104 °F] \*),
- At a maximum relative humidity of 80 % at 31 °C [87.8 °F], dropping to a linear of up to 50 % relative humidity at 40 °C [104 °F] \*),
- With mains power where the voltage fluctuations do not exceed 10 % of the nominal value,
- Under contamination level 2 conditions,
- Under over-voltage category II conditions.

\*) Between 5 - 30 °C [41 - 86 °F] the device can be operated at a relative humidity of up to 80 %. At temperatures between 31 - 40 °C [87.8 - 104 °F] the humidity must decrease proportionally in order to ensure operational readiness (e.g. at 35 °C [95 °F] = 65 % humidity, at 40 °C [104 °F] = 50 % humidity). The device may not be operated at temperatures above 40 °C [104 °F].

## 2.4 Ambient Conditions for Storage and Transport

For storage and transport the following specifications to ambient conditions apply:

- Ambient temperature - 20 – + 60 °C [- 4 – + 140 °F].
- Maximum relative humidity 80 %.

## 2.5 Hazard and Warning Information

### 2.5.1 General Information

- ▶ If the device is not used in compliance with the supplied instructions, the safety of the device can no longer be guaranteed.
- ▶ The device may only be operated using a mains cable with the country-specific plug system. Any necessary alterations must be carried out by a qualified electrician.
- ▶ The device may only be operated if the information on the identification plate conforms to the specifications of your local mains power supply.
- ▶ The device may only be plugged into outlets which are connected to the protective conductor system.
- ▶ The mains plug must be easily accessible.
- ▶ Disconnect the device from the mains before carrying out work on the electrical parts.
- ▶ Check connection cables (such as power supply cords), tubes and housing (i.e. the key-pad) regularly for damage (i.e. kinks, cracks and porosity) or signs of ageing. Devices with damaged connection cables, tubes or housing parts or other defects must not be operated!
- ▶ Defective devices must be put out of service immediately. Remove the mains plug and ensure the device is not used. Send the device for repair!

- ▶ Please observe the national accident prevention regulations!
- ▶ It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical, Electronic and Information Technology).
- ▶ Information on REACH and SVHC is available on our website [www.renfert.com](http://www.renfert.com), in the Support area.

## 2.5.2 Specific Information

- ▶ When connecting to a CAM system, please observe the CAM device instructions for use and comply with their safety precautions.
- ▶ Please observe the national regulations and permitted exposure to dust in a working environment. Please ask the “National Institute for Occupational Safety and Health” or other responsible authority.
- ▶ Please observe the Safety Data Sheets of the materials to be extracted.
- ▶ Always wear protective gear, when extracting hazardous materials.
- ▶ It is necessary to wear suitable personal protective equipment when emptying the dust drawer or cleaning, depending on the type of extracted material.
- ▶ When disposing of the extracted material or used filter, please observe the local specifications and accident prevention regulations!
- ▶ Make sure the dust drawer is fully closed during operation.
- ▶ Do not operate without a suction hose.
- ▶ Do not extract easily flammable, explosive gasses or fumes.
- ▶ The following applications involve increased risks and are therefore not permitted:  
When cleaning milling chambers and SLM printing systems by suction, a sufficiently high concentration and purity (i.e. unmixed with other dental dusts such as gypsum, resins) can result in spontaneous ignition or the milling dust exploding or printing powder because of a potential exothermic reaction (e.g. due to oxidation). An alternative cleaning method must be selected for the following milling or printing media in particular (such as manual cleaning):
  - Wood
  - Titanium/titanium-aluminum
  - Light metals and light metal alloys (e.g. aluminum, magnesium)
  - Cobalt chrome powder (e.g. for use in SLM systems)
- ▶ If large quantities of light metals such as titanium alloy are processed (e.g. with sandpaper), producing very fine grinding dust, a sufficiently high concentration and purity may result in spontaneous combustion because of a potential exothermic reaction.
- ▶ Do not extract hot materials.
- ▶ Do not extract liquids.
- ▶ If the extraction unit is used to suction hazardous materials, appropriate personal protective gear must be worn and steps must be taken to ensure that the exhaust air is properly ventilated. Please refer to the associated safety data sheets for specific requirements.
- ▶ Dispose of extracted material according to local statutory regulations.

## 2.6 Authorized Persons

Operation and maintenance of the device may only be performed by qualified personnel.

Minors and pregnant women may only operate and service the device if they are wearing appropriate protective gear, in particular if the device is being used to extract hazardous materials.

Any repairs not specifically described in these operating instructions may only be carried out by a qualified electrician.

## 2.7 Disclaimer

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- ▶ The product is employed for any purposes other than those specified in the operating instructions.
- ▶ The product is altered in any way other than those alterations described in the operating instructions.
- ▶ The product is not repaired by an authorized facility or if non-original Renfert parts are implemented.
- ▶ The product continues to be used despite obvious safety faults or damage.
- ▶ The product is subjected to mechanical impacts or is dropped.

# 3 Product Description

## 3.1 General Description

This device is an extraction unit for vacuuming-up dust generated by dental CAM equipment. It can be manually operated or controlled by the CAM device.

The extraction unit is equipped with a bi-directional interface, which transfers status data to the CAM device and records control commands from the CAM device.

The suction performance is fixed and cannot be altered.

## 3.2 Components and Functional Elements

- |   |                             |
|---|-----------------------------|
| 1 SILENT compactCAM                       | 8 Suction port              |
| 2 Key-pad                                 | 9 On / off switch           |
| 3 Dust drawer                             | 10 Power supply             |
| 4 Fine filter                             | 11 Device protection switch |
| 5 Mains cable                             | 12 CAM interface            |
| 6 Suction hose                            | 13 Service flap             |
| 7 Exhaust air filter / exhaust air outlet |                             |

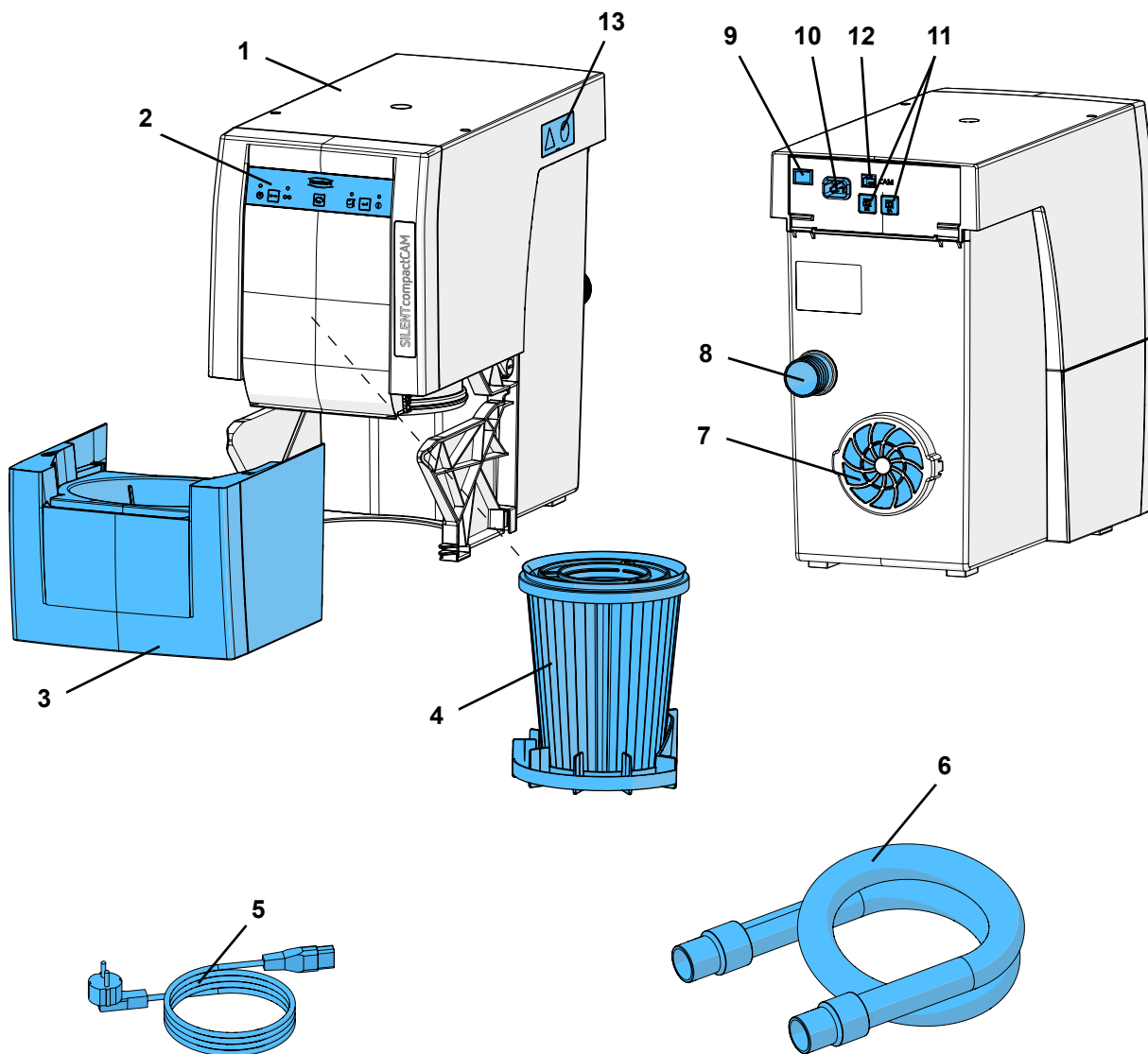
Fig. 1

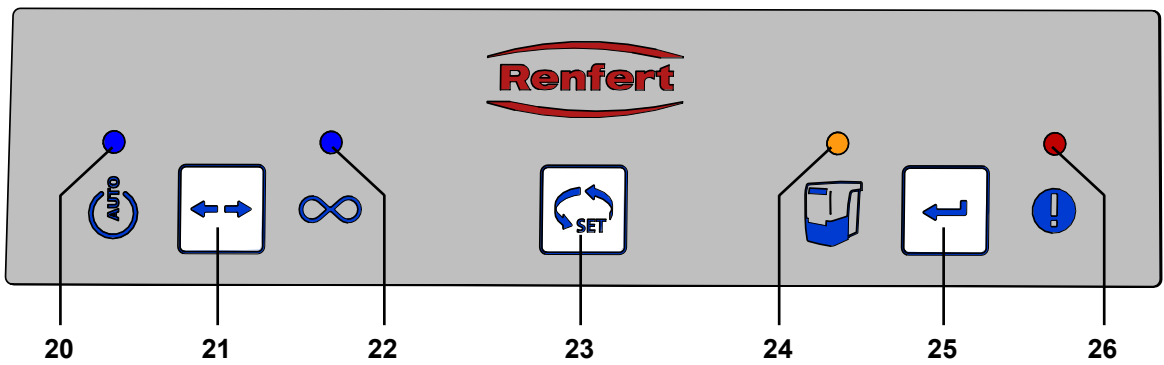
Fig. 2

- |   |  |
|---|--|
| <b>20</b> Display CAM-mode                                    | <b>23</b> Selection key                |
| <b>21</b> Key operating mode, CAM-mode / continuous operation | <b>24</b> Display empty suction drawer |
| <b>22</b> Display continuous operation                        | <b>25</b> Enter key, save input        |
|   | <b>26</b> Display error message        |

### 3.3 Scope of Delivery

- 1 SILENT compactCAM
- 1 Quick Start Guide
- 1 Quick Reference Cardt
- 1 Power cable
- 1 Suction hose, 2 m, incl. 2 end mufflers

### 3.4 Accessories

- 2921 0003 End bushing set, 2 pieces
- 2934 0007 90° angled connector SILENT
- 90003 4240 Suction hose
- 90003 4826 Suction hose, antistatic, 3 m, incl. 2 end mufflers
- 90115 0823 Suction hose diameter 38 mm, 6 m
- 90215 0823 Suction hose diameter 38 mm, 9 m
- 90003 4305 Adapter for hose connection
- 90003 4430 Universal suction hose adapter
- 90003 4314 Y-junction
- 2925 0000 Extractor clamp
- 2925 1000 Glass screen with holder
- 2926 0000 Y-Junction for dust extractor
- 2934 0004 External air duct for SILENT compact
- 2934 0005 Interface cable Type A for vhf
- 2934 0006 Interface cable Type B for Roland DG
- 2934 0008 Interface cable Type C for imes-icore
- 2934 0009 Interface cable Type D for Amann Girrbach
- 2934 0010 Interface cable Type E for Yenadent/Origin + Nema adapter
- 2934 0011 Interface cable Type F for Zirkonzahn
- 2934 0012 Interface cable Type F for Zirkonzahn + C14 adapter
- 2934 0013 Interface cable Type G for vhf with 6 pol. connection
- 2934 0016 Interface cable Type H for Dentsply Sirona
- 2934 0017 Interface cable Type I für Up3D
- 2934 0018 Interface cable Type J für Canon
- 2934 0021 Interface cable type M for XTCERA

For further details or additional accessories, please see [www.renfert.com](http://www.renfert.com).

# 4 Setting Up

## 4.1 Unpacking

- ⇒ Remove the device and all the accessories from the delivery package.
- ⇒ Check the delivery for completeness (refer to the “Scope of Delivery” section).

## 4.2 Setup

The extraction unit is a free standing appliance which must not be operated in a lying position.

Position the extraction device so that:

- The exhaust vent (7, Fig. 1) is not blocked.
- The front of the device is easily accessible for removal of dust drawer.

If the extraction unit is kept in a closed cupboard, the warm exhaust air must be allowed to escape using one of the following methods:

- External exhaust air duct (see chapter 4.6).
- An opening in the cupboard back side min. 250 x 120 mm, directly opposite the exhaust air outlet (7, Fig. 1).
  - Distance from the cupboard back side to the wall: min 100 mm,
  - Distance from the extraction unit to the cupboard back side: max. 25 mm.
- Remove the back side of the cupboard, distance from the back of the cupboard to the wall min. 50 mm.

If the warm exhaust air is passed out of the opening vent at the back of the cupboard, ensure that the warm air can escape without hindrance.

## 4.3 Electrical Connection

**Before connecting the device, ensure that the voltage information on the identification plate corresponds with your local power supply.**

**Arrange the conducting parts (plug sockets, plugs and couplings) and install the extension cord so that the protection class is retained.**

- ⇒ Switch the device OFF at the On / Off switch (9).
- ⇒ Connect the power cable (5, Fig. 1) to the power supply (10).
- ⇒ Insert the power plug into the building installed wall socket.

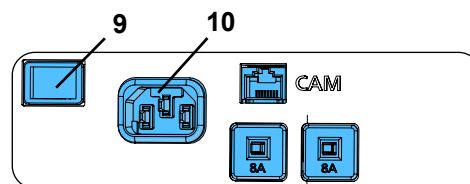
Fig. 3

## 4.4 Connection to the Extraction Point

- ⇒ Insert the suction tube (6, Fig. 1) into the suction port (8).
- ⇒ Connect the suction tube to the suction port on the CAM device.

Please observe the CAM device instructions for use.

- ⇒ If necessary, shorten the suction tube.

**Caution, risk of injury!**

**When shortening the suction tube, please ensure that the integrated wire is cut as straight as possible.**

**If the diameter size does not correspond, please use an adapter (see accessories) to prevent a loss in suction performance.**

**Long suction hoses, tight bends and kinks will considerably reduce the extraction force at the extraction point.**

**Avoid steep pitches or hanging points along the hose path.**

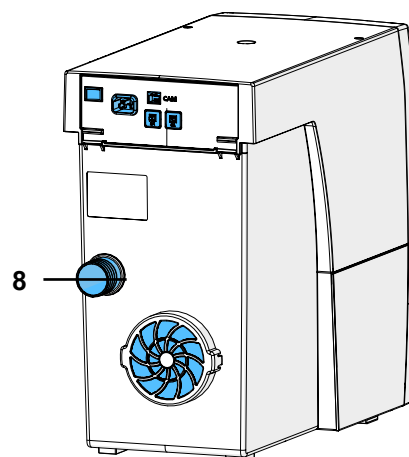
Fig. 4

## 4.5 Connection to a CAM interface

If the suction is frequently switched on or off via the CAM power supply, it may cause electronic damage to the CAM system as well as to the SILENT compactCAM. For remote control of the SILENT compactCAM it is essential to use the provided CAM interface, together with the „Interface cable type F“ (see chapter 9.3) where applicable.

The electrical connection for communication with the CAM device is provided by the interface socket (12) and an optional interface cable (see accessories).

For information on interface cables available for the CAM device or producing an individual interface cable see chapter 9.1 CAM interface.

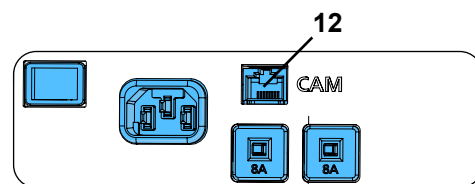
Fig. 5

Check with your CAM manufacturer, that the interface for the connection of the extraction is energy limited according to the IEC 61010-1.

Please also see the CAM device instructions for use.

## 4.6 External Exhaust Air Route

An external exhaust air route (see accessories) allows the extracted air to leave the laboratory.

The installation details are supplied with the external exhaust air route.

When the extraction unit is used in conjunction with an external ventilation system, a significant quantity of air is extracted from the room per hour.

This can create negative pressure within the room which, when using an air dependent naked flame fed by gas, liquid or solid fuel, can cause poisonous gasses (e.g. carbon monoxide) to be drawn into the working area.

It is therefore essential to ensure that the fresh air supply is sufficient and that the environmental air pressure is maintained, this should then be monitored by an authorized specialist (e.g. a certified Gas Service Engineer).

# 5 Operation

The extractor unit is operated via the buttons on the key-pad (Fig. 2).

## 5.1 Switching the Unit On

The extractor is switched ON and OFF at the On / Off switch (9).

When the unit is switched on:

- ◆ All 4 displays light up (the display carries out a functional control).
- ◆ The suction unit performs an automatic filter cleaning sequence (for approx. 8 seconds, loud vibrating noise).

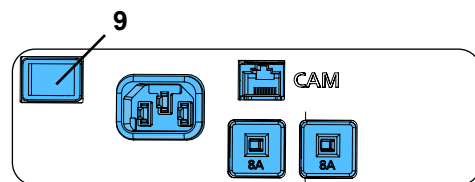
Fig 6

Then the unit returns to the last set operating mode.

## 5.2 Select Mode: CAM-mode / continuous operation

The extraction unit has two operating modes.

The set operating mode is shown in the display (20) / (22).

- CAM-mode (20):
  - ◆ The extraction unit is controlled by the signals provided by the CAM device.
- Continuous operation (22):
  - ◆ The extraction unit runs continuously.

⇒ Press the operating mode key (21).

- ◆ To change the operating mode.

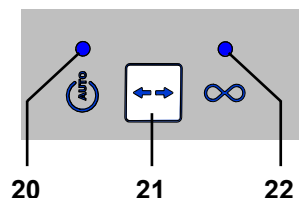
Fig. 7

The extraction unit is for use with dry dust only! When connected to a CAM device with wet and dry function, ensure that no residual moisture from the CAM device enters the extraction unit.

Check that the extraction unit is in full working order for every milling procedure:

- ▶ Display error (26, Fig. 2) is off.
- ▶ Suction turbine operates correctly without producing any peculiar noise or odor.

## 5.3 Filter Cleaning

In order to guarantee maximum suction performance, the unit is equipped with an automatic cleaning function for the filter unit.

The cleaning process is carried out:

- When the suction performance is insufficient (when the flow velocity falls below a specified level);
- When the unit is switched on;
- Before the dust drawer is removed, when the cleaning function is activated (see chapter 6.2 Empty dust drawer).

If the flow velocity falls below the specified level for more than 1 minutes, the suction is interrupted and the cleaning function is activated:

⇒ Flow velocity below specified level.

- ◆ The suction turbine is stopped.
- ◆ The cleaning function is carried out. The cleaning function takes approx. 8 seconds and is indicated by an acoustic signal.
- ◆ The suction turbine is re-started.

**After an automatic filter cleaning, in case of a renewed drop of the flow velocity, the next filter cleaning will be performed only after 2 hours of turbine running time.**

## 6 Cleaning / Maintenance

**Opening the device, other than for the processes described below, is not permissible!**

### 6.1 Cleaning

Use a damp cloth to clean the outside of the unit.

Do not use abrasive or solvent-based cleaning agents.

### 6.2 Empty Dust Drawer

After the set time period has expired (see chapter 6.2.1), a signal to empty the dust drawer will occur.

Before the dust drawer is emptied, a filter cleaning function is carried out so that loose dust particles fall into the dust drawer.

The time period has expired:

- ◆ An acoustic signal will occur 3 times.
- ◆ The display dust drawer (24) lights up.

When the suction has stopped, an acoustic signal occurs 3 times and a filter cleaning function is carried out.

When the filter cleaning function has finished:

- ⇒ Pull the dust drawer (3, Fig. 1) out to the front and empty.
- ⇒ Replace the dust drawer until it clicks into place.
- ⇒ Press the enter key (25) (the counter which determines the time period is re-set).
  - ◆ An acoustic signal confirms the input.
  - ◆ The display dust drawer (24) light disappears.

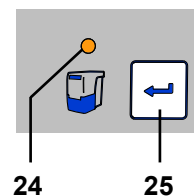
Fig. 8

If the dust drawer is not emptied, the display for the dust drawer will remain alight (24, Fig. 8). When the unit is switched off/on an acoustic signal will occur to remind that the dust drawer needs emptying.

### 6.2.1 Setting the Time Period “Empty Dust Drawer”

Five different time periods can be selected.

| Time period / h | Blink signal |
|-----------------|--------------|
| 2               | 1 x          |
| 5               | 2 x          |
| 10              | 3 x          |
| 50              | 4 x          |
| 100             | 5 x          |

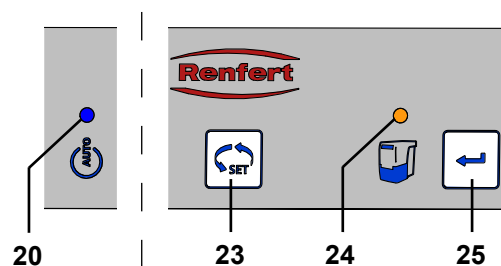
Fig. 9

To select time period:

- ⇒ Switch the unit off at the on/off switch (9, Fig. 1).
- ⇒ Switch the unit on at the on/off switch and keep the enter key (25) pressed until:
  - ◆ Display CAM-mode (20) lights up.
  - ◆ The display dust drawer (24) signal blinks in correspondence to the set time period.

When the unit is switched on all 4 displays shortly light up (Display performance check).

- ⇒ Select the required time period by pressing the selection key (23).  
By pressing this key it is possible to choose between 5 time periods. The selected time period is shown with the corresponding blink signal.
- When the required time period has been selected:
  - ⇒ Press the enter key (25).
    - ◆ The display dust drawer (24) lights up for 2 seconds.
    - ◆ An acoustic signal confirms the input.

### 6.3 Change Fine Filter

As a fine filter, it contains a two-stage filter system category M. This ensures a high level of health protection and a long service life for the extraction system.

**By monitoring the flow rate the effectiveness of the filter cleaning is determined. If a filter cleaning is repeatedly required in a 2 hour cycle, this means that the fine filter is so blocked, that the filter cleaning is inadequate.**

This is displayed as follows:

- ◆ The error message lights up (26, Fig. 2).
- ◆ 15 minutes long, acoustic signal 3 x, every 3 minutes.
- ◆ Output of the message in the CAM interface (see chapter 9).

In this case, the fine filter should be changed. In order to delete the error messages, the extraction unit must be switched off and turned on again.

- ▶ In general, the fine filter should be changed every 2 years.

**Do not clean the fine filter manually (e.g. with compressed air, brush, water, etc.), as this leads to damage of the filter material!**

When installing the fine filter please ensure that it is positioned correctly, otherwise leakages may result. See the assembly instruction at the end of the instruction manual, which is also attached to the new fine filter.

### 6.4 Safety Mechanism

The protection of the suction device is achieved by two device protection switches (11, Fig. 1).

If an overload switch is released, it can be reset by pressing the button back in.

**Repeated triggering of a device protection switch is caused by a defect in the suction device. Send the device in for repair!**

## 6.5 Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at [www.renfert.com/p918](http://www.renfert.com/p918).

Enter the following item number: 29342000

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

## 6.6 Factory Settings

⇒ Switch unit off (9, Fig.1).

⇒ Press the operating mode key (21, Fig. 2) and selection key (23, Fig. 2) at the same time and switch the unit on (9, Fig. 1).

- ◆ All 4 displays blink thrice.
- ◆ All values are now re-set to the factory settings.

Factory settings:

| Function / Feature | Setting range                   | Factory settings |
|--------------------|---------------------------------|------------------|
| Operating mode     | CAM-mode / continuous operation | CAM-mode         |
| Time period        | 2 h - 100 h                     | 50 h             |

## 7 Troubleshooting

| Trouble  | Possible Cause   | Corrective Action   |
|--|--|---|
| The error message (26, Fig. 2) blinks.   | <ul style="list-style-type: none"> <li>• The electronic device has become too hot.</li> </ul>  | <ul style="list-style-type: none"> <li>• Switch the device off and allow it to cool down.</li> <li>• Ensure sufficient cooling, e.g. with:                             <ul style="list-style-type: none"> <li>- Chap. 4.2 Setup.</li> <li>- External ventilation (see chap. 4.6).</li> <li>- Change fine filter.</li> </ul> </li> </ul>   |
| The suction performance is insufficient.   | <ul style="list-style-type: none"> <li>• There is a blockage or leak in the suction tube.</li> <li>• Dust drawer is not airtight.</li> <li>• The fine filter is full.</li> </ul> | <ul style="list-style-type: none"> <li>• Check suction tube.</li> <li>• Please observe the points in chapter 4.4.</li> <li>• Check the dust drawer is in the correct position (see chap. 6.2).</li> <li>• Clean the filter.</li> <li>• Switch the unit off and then on again, so that the filter cleaning process can be carried out. Change the fine filter (see chap. 6.3) (if the filter cleaning function does not improve the suction performance).</li> </ul> |
| Dust drawer is over full.  | <ul style="list-style-type: none"> <li>• The time period for "empty dust drawer" has been set too high.</li> </ul>   | <ul style="list-style-type: none"> <li>• Adjust to a lower time period (see chap. 6.2.1).</li> </ul>  |
| The signal to empty the dust drawer shows even though the dust drawer is not yet full. | <ul style="list-style-type: none"> <li>• The time period for "empty dust drawer" has been set too low.</li> </ul>  | <ul style="list-style-type: none"> <li>• Adjust to a higher time period (see chap. 6.2.1).</li> </ul>   |
| The display "empty dust drawer" blinks and an acoustic signal occurs 3 times.          | <ul style="list-style-type: none"> <li>• The flow velocity is insufficient; the filter cleaning function is carried out.</li> </ul>  | <ul style="list-style-type: none"> <li>• Wait until the filter cleaning has finished.</li> </ul>  |
| The error message lights up (26, Fig. 2).  | <ul style="list-style-type: none"> <li>• The fine filter is so full, that the filter cleaning function alone cannot solve the problem.</li> </ul>                                | <ul style="list-style-type: none"> <li>• Change the fine filter (see accessories and chap. 6.3).</li> </ul>   |
| The error message lights up (26, Fig. 2) and an acoustic signal occurs 3 x again.      | <ul style="list-style-type: none"> <li>• The fine filter is so full, that the filter cleaning function alone cannot solve the problem.</li> </ul>                                | <ul style="list-style-type: none"> <li>• Change the fine filter (see accessories and chap. 6.3).</li> </ul>   |

| Trouble  | Possible Cause   | Corrective Action   |
|--|--|---|
| <b>The suction process in the continuous operation or automatic operation stops abruptly and the relevant displays are still on (20 / 22, Fig. 2).</b> | <ul style="list-style-type: none"> <li>• The suction turbine is overheated.</li> <li>• The suction turbine is defect.</li> </ul> | <ul style="list-style-type: none"> <li>• Switch the unit off and allow to cool down for at least 60 min.</li> <li>• Check whether the suction tube is blocked. Eliminate blockage.</li> <li>• Switch the unit off and then on again, so that the filter cleaning process can be carried out. Change the fine filter (see chap. 6.3) (if the filter cleaning function does not improve the suction performance).</li> <li>• Change the suction turbine.</li> </ul> |
| <b>When the unit is switched on, not all of the displays light up.</b>   | <ul style="list-style-type: none"> <li>• The display which does not light up is defect.</li> </ul>                               | <ul style="list-style-type: none"> <li>• Contact Renfert / Service.</li> </ul>  |

## 8 Technical Data

|                                      | 2934 2000                                   | 2934 2500    | 2934 3000    | 2934 3500    |
|--------------------------------------|---|--------------|--------------|--------------|
| <b>Nominal Voltage:</b>              | <b>230 V</b>                                | <b>220 V</b> | <b>120 V</b> | <b>100 V</b> |
| Permissible mains voltage:           | 230 - 240 V                                 | 220 V        | 120 V        | 100 V        |
| Mains frequency:                     | 50 / 60 Hz                                  | 50 / 60 Hz   | 50 / 60 Hz   | 50 / 60 Hz   |
| Suction turbine power *):            | 490 W                                       | 460 W        | 480 W        | 480 W        |
| CAM interface:                       | RJ 45 socket                                |              |              |              |
| LpA **) (at max. flow rate):         | 55 dB(A)                                    |              |              |              |
| Flow rate, max. ***):                | 2500 l/min [1.47 ft³/s]                     |              |              |              |
| Negative pressure, max. ****):       | 219 hPa [3.2 psi]                           |              |              |              |
| Input signal:                        |   |              |              |              |
| - Low                                | 0 - 5 V                                     |              |              |              |
| - High                               | 15 - 24 V                                   |              |              |              |
| Output signal:                       |   |              |              |              |
| - max. pull-up voltage               | 24 V  |              |              |              |
| - max. switching current (I)         | 5 mA  |              |              |              |
| - internal series resistor           | 150 Ohm                                     |              |              |              |
| Filterquality:                       | Class M according to EN60335-2-69           |              |              |              |
| Weight (empty), approx.:             | 13.2 kg [29.1 lbs]                          |              |              |              |
| Dimensions (width x height x depth): | 245 x 440 x 500 mm [9.6 x 17.3 x 19.7 inch] |              |              |              |

- \*) Power consumption at nominal voltage
- \*\*) Sound pressure levels according to EN ISO 11202
- \*\*\*) Free-blowing turbine at nominal turbine voltage
- \*\*\*\*) At nominal turbine voltage

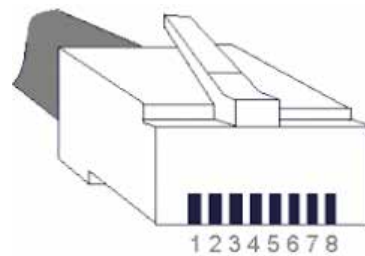
## 9 CAM interface

There is an electrically isolated, bi-directional interface available for communication with the CAM device. Via two inputs and two outputs, 4 commands can be received from the extraction unit or 4 status information can be sent to the CAM machine.

### 9.1 Pin assignment CAM interface (12, Fig. 1)

Type: RJ45

| Pin | Description |
|-----|-------------|
| 1   | Out2- GND   |
| 2   | Out2        |
| 3   | Out1- GND   |
| 4   | Out1        |
| 5   | In2 - GND   |
| 6   | In2 - 24 V  |
| 7   | In1 - GND   |
| 8   | In1 - 24 V  |


#### 9.1.1 Outputs (Out1 / Out2)

The output signals are potential-free switch outputs, in an optocoupler, with a 150 Ohm series resistor.

Please see chapter 8, Technical Data, for the maximum pull-up voltage (U) and the maximum permissible switching current (I).

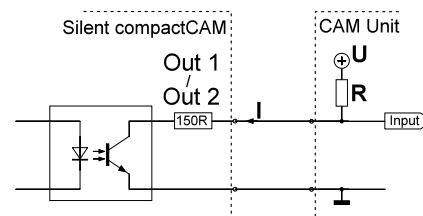The external pull-up resistor (R) must be set so that the maximum permissible switching current (I) is not exceeded.

#### 9.1.2 Inputs (In1 / In2)

The input signals go to the optocoupler LEDs. See chapter 8, Technical Data, for the specified input voltages of the two signal levels „low“ / “high“.

### 9.2 Control commands / Status information

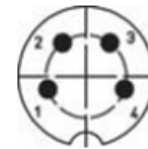
| In 1        | In 2        | Commands                  |
|-------------|-------------|---------------------------|
| 0 V (low)   | 0 V (low)   | Suction off (turbine off) |
| 24 V (high) | 0 V (low)   | Suction on (turbine on)   |
| 0 V (low)   | 24 V (high) | Carry out cleaning        |
| 24 V (high) | 24 V (high) | No allocation             |

| Out 1 | Out 2 | Signal          |
|-------|-------|-----------------|
| 24 V  | 24 V  | Error           |
| 0 V   | 24 V  | Cleaning active |
| 24 V  | 0 V   | Filter change   |
| 0 V   | 0 V   | Suction ok      |

## 9.3 Pin assignment interface cable

Interface cable Type A:


| Extractor | CAM device                           |
|-----------|--------------------------------------|
| RJ 45 pin | Lumberg SV40<br>(with screw closure) |
| Pin 8     | Pin 1 (+)                            |
| Pin 7     | Pin 4 (-)                            |

rear view


Interface cable Type B:

| Extractor | CAM device            |
|-----------|-----------------------|
| RJ 45 pin | Stereo jack<br>3,5 mm |
| Pin 8     | Pin 1 (+)             |
| Pin 7     | Pin 2 (-)             |

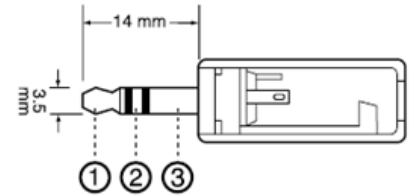
Interface cable Type C:

| Extractor | CAM device  |
|-----------|-------------|
| RJ 45 pin | 9 Pin D-Sub |
| Pin 8     | Pin 4 (+)   |
| Pin 7     | Pin 3 (-)   |

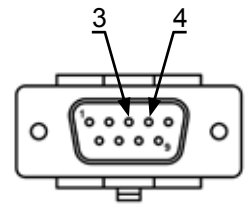
Interface cable Type D:

| Extractor | CAM device  |
|-----------|-------------|
| RJ 45 pin | 9 Pin D-Sub |
| Pin 8     | Pin 3 (+)   |
| Pin 7     | Pin 4 (-)   |

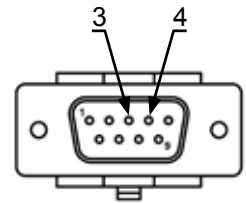Interface cable Type E:

| Extractor | CAM device  | Power supply |
|-----------|-------------|--------------|
| RJ 45 pin | 9 Pin D-Sub |              |
|           | Pin 9       | 24 V         |
| Pin 8     | Pin 8       |              |
| Pin 7     |             | GND          |

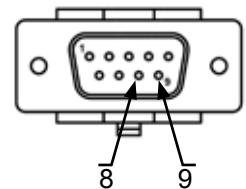For use if the CAM system only has a floating contact between pin 8 and 9.


Interface cable Type F:

| Extractor | Power supply |
|-----------|--------------|
| RJ 45 pin |              |
| Pin 8     | 24 V         |
| Pin 7     | GND          |

For use if the CAM system only has voltage for the control of the extraction unit.

Interface cable Type G:

| Extractor | CAM device                           |
|-----------|--------------------------------------|
| RJ 45 pin | Lumberg SV60<br>(with screw closure) |
| Pin 8     | Pin 1 (+)                            |
| Pin 6     | Pin 2 (+)                            |
| Pin 4     | Pin 4 (+)                            |
| Pin 2     | Pin 5 (+)                            |
| Pin 7     | Pin 6 (-)                            |
| Pin 5     |                                      |
| Pin 3     |                                      |
| Pin 1     |                                      |

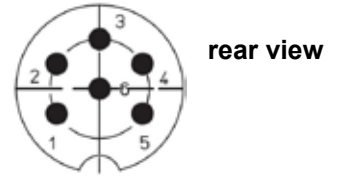
Interface cable Type H:

| Extractor | CAM device       |
|-----------|------------------|
| RJ 45 pin | 15 Pin D-Sub     |
| Pin 8     | Pin 2 (+)        |
| Pin 7     | Pin 12 (-)       |
| ---       | Pin 7 <-> Pin 10 |

Interface cable Type I:

| Extractor | CAM device         |
|-----------|--------------------|
| RJ 45 pin | 5 pol Aviation pin |
| Pin 8     | Pin 5 (+)          |
| Pin 7     | Pin 4 (-)          |

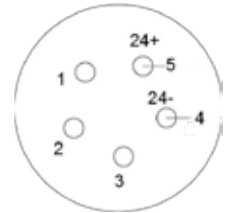
Interface cable Type J:

| Extractor | CAM device     | Power supply |
|-----------|----------------|--------------|
| RJ 45 pin | 2 pol. TMW pin |              |
| Pin 8     | ----           | 24 V         |
| Pin 7     | B              | ----         |
| ----      | A              | GND          |

Interface cable, type M:

| Extractor | CAM device             | Power supply |
|-----------|------------------------|--------------|
| RJ 45 pin | 4 pol. round connector |              |
| ---       | 1                      | 24 V         |
| Pin 7     | ---                    | GND          |
| Pin 8     | 2                      |              |
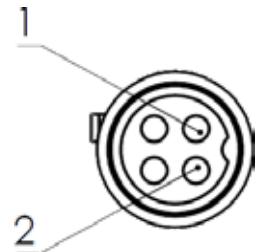
## 10 Warranty

*The suction motor has a guarantee of 3 years, with a maximum of duration of 1000 operating hours (motor running time).*

Provided the unit is properly used, Renfert warrants all components **for 3 years**.

Warranty claims may only be made upon presentation of the original sales receipt from the authorized dealer.

Parts which are subject to natural wear and tear (wear parts) and consumables are excluded from the guarantee. These parts are marked in the spare part list.

The warranty is voided in the case of improper use; failure to observe the operating, cleaning, maintenance and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed, or in case of unusual influences or influences not in compliance with the utilization instructions.

Warranty service shall not extend the original warranty.

## 11 Disposal Information

### 11.1 Disposing of Consumables

Full dust bags and filters must be disposed of under compliance with locally applicable regulations.

Depending on the material trapped by the filters, protective gear may need to be worn during disposal.

### 11.2 Disposing of the Unit

The unit must be disposed of by an authorized recycling operation. The selected firm must be informed of all possible health hazardous residues in the unit.

#### 11.2.1 Disposal Instructions for countries in the EU

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling.

**Within the European Union, units with this symbol should not therefore be disposed of in unsorted domestic waste.**

Please contact your local authorities for more information on proper disposal.

**Hochaktuell und ausführlich auf ...  
Up to date and in detail at ...  
Actualisé et détaillé sous ...  
Aggiornato e dettagliato su ...  
La máxima actualidad y detalle en ...  
Актуально и подробно на ...**

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declare under our sole responsibility, that the product(s)

| Name               | Part No.   |
|--------------------|--|
| SILENT TS          | 2921 0050  |
| SILENT TS2         | 2930 0050  |
| SILENT TC          | 2935 0000 / 2935 0000 03                             |
| SILENT TC2         | 2936 0000 / 2936 0000 03                             |
| SILENT EC2         | 2937 0000 / 2937 0000 03                             |
| SILENT compact     | 2934 0000 / 2934 0000 03                             |
| SILENT compactCAM  | 2934 2000 / 2934 2000 03<br>2934 2500 / 2934 2500 03 |
| SILENT powerCAM EC | 2939 0000  |
| SILENT powerCAM TC | 2938 0000  |
| iVAC silent        | 36 2934 2000   |

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

#### UK legislation

*Electromagnetic Compatibility Regulations 2016*

*Supply of Machinery (Safety) Regulations 2008*

*The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012*

#### Designated standards

EN 61326-1:2013

EN ISO12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,  
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

**Hochaktuell und ausführlich auf ...  
Up to date and in detail at ...  
Actualisé et détaillé sous ...  
Aggiornato e dettagliato su ...  
La máxima actualidad y detalle en ...  
Актуально и подробно на ...**

**[www.renfert.com](http://www.renfert.com)**

Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany  
Tel.: +49 7731 82 08-0 • Fax: +49 7731 82 08-70  
[www.renfert.com](http://www.renfert.com) • [info@renfert.com](mailto:info@renfert.com)

Renfert USA • 3718 Illinois Avenue • St. Charles IL 60174/USA  
Tel.: +1 6307 62 18 03 • Fax: +1 6307 62 97 87  
[www.renfert.com](http://www.renfert.com) • [info@renfertusa.com](mailto:info@renfertusa.com)  
USA: Free call 800 336 7422

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**Renfert GmbH, Industriegebiet, 78247 Hilzingen / Germany**