Picosilk

REF 1550 0000

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Descrição:

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INSTRUCTIONS FOR USE ENGLISH

Description:

Picosilk Spray is a wetting agent for wax patterns of crown and bridgework as well as chrome-cobalt denture bases. Picosilk Spray is sprayed onto the patterns before they are invested in the casting ring. Picosilk Spray considerably improves the flow properties of investment material on casting wax thus ensuring that the wax pattern is exactly reproduced in metal.

Application:

- ▶ Pull the transparent cap off the spray bottle and replace it with the spray cap supplied.
- ▶ Prepare the wax pattern for investing and spray it with Picosilk Spray from a distance of approximately 15 cm (5.91 inches).
- ▶ Ensure that the surface of the wax is wetted evenly.
- ▶ Should any excess material remain on the surface, carefully blow it off with your mouth and not with compressed air.
- ▶ Allow the Picosilk Spray to dry slightly.
- ▶ Invest the wax pattern.

Caution:

Do not shorten the spray tube on the spray cap.
An undercooling (e.g. storage in the refrigerator) may cause little precipitations which however do not influence the Picosilk's efficiency.
The precipitations dissolve within an hour with ambient temperature.

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