making work easy

de

en

fr

it

es

pt

tr

ru

zh

ko

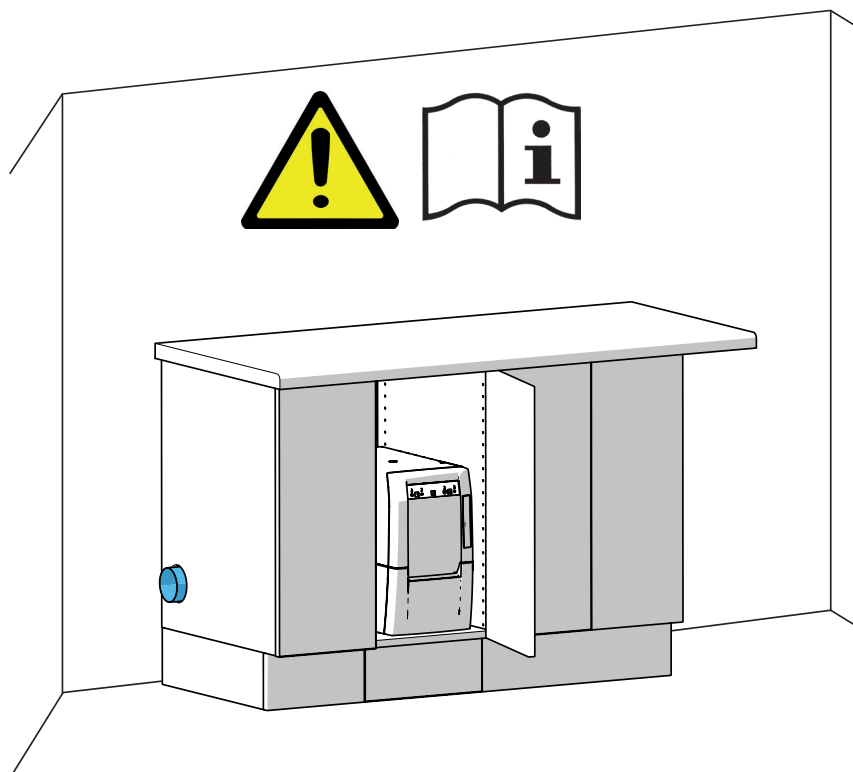
ar

# SILENT powerCAM TC

Made in Germany

21-2962 13112025

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en

# SILENT PowerCAM TC

TRANSLATION OF THE ORIGINAL INSTRUCTIONS FOR USE

Made in Germany

21-2962 13112025

# Contents

|           |                                                                    |           |
|-----------|--------------------------------------------------------------------|-----------|
| <b>1</b>  | <b>Introduction</b>                                                | <b>3</b>  |
| 1.1       | Symbols used                                                       | 3         |
| <b>2</b>  | <b>Safety</b>                                                      | <b>3</b>  |
| 2.1       | Intended use                                                       | 3         |
| 2.2       | Improper use                                                       | 3         |
| 2.3       | Ambient conditions for safe operation                              | 4         |
| 2.4       | Ambient conditions for storage and transport                       | 4         |
| 2.5       | Hazard and warning information                                     | 4         |
| 2.5.1     | General information                                                | 4         |
| 2.5.2     | Specific information                                               | 5         |
| 2.6       | Authorized persons                                                 | 5         |
| 2.7       | Disclaimer                                                         | 5         |
| <b>3</b>  | <b>Product description</b>                                         | <b>6</b>  |
| 3.1       | General description                                                | 6         |
| 3.2       | Assemblies and functional elements                                 | 6         |
| 3.3       | Scope of delivery                                                  | 7         |
| 3.4       | Accessories                                                        | 7         |
| <b>4</b>  | <b>Startup</b>                                                     | <b>8</b>  |
| 4.1       | Unpacking                                                          | 8         |
| 4.2       | Setup                                                              | 8         |
| 4.3       | Electrical connection                                              | 8         |
| 4.4       | Connection to the suction point                                    | 8         |
| 4.5       | Connection to the CAM interface                                    | 9         |
| 4.6       | External air duct                                                  | 9         |
| <b>5</b>  | <b>Operation</b>                                                   | <b>9</b>  |
| 5.1       | Switching on                                                       | 9         |
| 5.2       | Selecting the operating mode: CAM operation / continuous operation | 9         |
| 5.3       | Cleaning the filter                                                | 10        |
| <b>6</b>  | <b>Cleaning / Maintenance</b>                                      | <b>10</b> |
| 6.1       | Cleaning                                                           | 10        |
| 6.2       | Emptying the dust drawer                                           | 10        |
| 6.2.1     | Setting the time interval for emptying the dust drawer             | 11        |
| 6.3       | Replacing the fine filter                                          | 11        |
| 6.4       | Fuses                                                              | 11        |
| 6.5       | Spare parts                                                        | 11        |
| 6.6       | Factory settings                                                   | 12        |
| <b>7</b>  | <b>Troubleshooting</b>                                             | <b>12</b> |
| <b>8</b>  | <b>Technical data</b>                                              | <b>13</b> |
| <b>9</b>  | <b>CAM interface</b>                                               | <b>14</b> |
| 9.1       | Pin assignment: CAM interface (8, Fig. 1)                          | 14        |
| 9.1.1     | Outputs (Out1 / Out2)                                              | 14        |
| 9.1.2     | Inputs (In1 / In2)                                                 | 14        |
| 9.2       | Control commands / status notifications                            | 14        |
| 9.3       | Connector assignment: interface cable                              | 15        |
| <b>10</b> | <b>Warranty</b>                                                    | <b>17</b> |
| <b>11</b> | <b>Disposal information</b>                                        | <b>17</b> |
| 11.1      | Disposing of consumables                                           | 17        |
| 11.2      | Disposing of the equipment                                         | 17        |
| 11.2.1    | Disposal information for countries in the EU                       | 17        |

# 1 Introduction

## 1.1 Symbols used

Symbols with the following meanings are used in these instructions or on the equipment:

### **Danger**

**Immediate danger of injury. Consult accompanying documents!**

### **Electrical voltage**

**Danger due to electrical voltage.**

### **Caution**

**Risk of damage to the equipment if the instruction is not followed.**

### **Notice**

*Provides the operator with useful information that eases use.*

The equipment complies with applicable EU directives.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under [www.renfert.com](http://www.renfert.com).

The equipment is subject to the provisions of the EU directive 2002/96/EC (WEEE directive).

### ► **List, particular attention should be paid**

- List
- List

⇒ Instructions / appropriate action / input / operational sequence:

You are asked to carry out the specified action in the order indicated.

- ◆ Result of an action / response by the equipment / response by the program:

The equipment or program responds as a result of your actions or when a specific incident occurs.

Other symbols are explained as they occur.

# 2 Safety

## 2.1 Intended use

This extraction unit is intended for use with dental milling machines for suctioning dry dust generated when milling commercially available milling blocks and blanks.

The unit is intended for commercial use only.

The intended use also includes compliance with the conditions specified by the manufacturer concerning operation and maintenance.

## 2.2 Improper use

Flammable, easily combustible, hot, burning, or explosive materials may not be suctioned using the unit.

It is not permitted to suction liquids.

The unit is not intended for private, household use.

Any use other than specified in these instructions is deemed improper use.

The manufacturer shall not be liable for resulting damages.

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the equipment, increase the risk of serious injury and lead to damage to the environment or the product itself.

## 2.3 Ambient conditions for safe operation

The unit may only be operated:

- Indoors
- Up to an altitude of 2,000 m above sea level
- At an ambient temperature of between 5 - 40 °C [41 - 104 °F] \*)
- At a maximum relative humidity of 80 % at 31 °C [87.8 °F], decreasing linearly to 50 % relative humidity at 40 °C [104 °F] \*)
- With mains power where the voltage fluctuations do not exceed 10 % of the nominal value
- Under contamination level 2 conditions
- Under over-voltage category II conditions

\*) At between 5 - 30 °C [41 - 86 °F], the unit can be operated at a humidity of up to 80 %. At temperatures between 31 - 40 °C [87.8 - 104 °F], the humidity must decrease proportionally in order to ensure operational readiness (e.g., at 35 °C [95 °F] = 65 % humidity, at 40 °C [104 °F] = 50 % humidity). The unit may not be operated at temperatures above 40 °C [104 °F].

## 2.4 Ambient conditions for storage and transport

The following ambient conditions must be observed during storage and transport:

- Ambient temperature - 20 – + 60 °C [- 4 – + 140 °F].
- Maximum relative humidity 80 %.

## 2.5 Hazard and warning information

### 2.5.1 General information

- ▶ If the unit is not used in accordance with the supplied instructions for use, the intended safety can no longer be guaranteed.
- ▶ The unit may only be operated using a power cord equipped with a plug appropriate to the local power supply. Any necessary modifications may only be carried out by a qualified electrician.
- ▶ Only operate the unit if the information on the nameplate corresponds to the specifications of your local mains power supply.  
After removing the dust drawer, you can find the nameplate inside the unit on the lower left side.
- ▶ The unit may only be plugged into electrical sockets that are connected to the protective conductor system.
- ▶ The power plug must be easily accessible.
- ▶ Disconnect the unit from the mains power supply before carrying out work on the electrical parts.
- ▶ Check connection cables (such as power cords), tubes and housing (such as the key pad) regularly for damage (such as kinks, cracks, and porosity) or signs of aging.  
Do not operate equipment with damaged connection cables, tubes or housing parts or other defects!
- ▶ Defective equipment must be taken out of service immediately. Disconnect the power plug and secure the equipment against further use. Send the equipment in for repair!
- ▶ Please observe the national accident prevention regulations!
- ▶ It is the responsibility of the operator to ensure that national regulations are complied with during operation and regarding recurring safety inspection of electrical equipment.  
In Germany, these are DGUV Regulation 3 in conjunction with VDE 0701-0702.
- ▶ Information on REACH and SVHC is available on our website at [www.renfert.com](http://www.renfert.com), in the Support area.

### 2.5.2 Specific information

- ▶ For connection to a CAM system, observe the operating instructions for the CAM system as well as the safety instructions that they contain.
  - ▶ Observe national regulations and permitted dust exposure levels in the working environment. Refer to your national association for occupational safety and health or the responsible authority.
  - ▶ Observe the safety data sheets when suctioning hazardous materials.
  - ▶ Wear personal protective equipment when suctioning hazardous materials.
  - ▶ Depending on the type of extracted material, wear appropriate personal protective equipment when emptying the dust drawer or cleaning.
  - ▶ When disposing of the extracted material or used filters, observe local provisions and accident prevention regulations!
  - ▶ Suctioning may only be performed with the dust drawer closed.
  - ▶ Do not operate without a suction hose.
  - ▶ Do not suction flammable, easily combustible, or explosive gases or fumes.
  - ▶ The following applications are associated with increased risks and are therefore not permitted: When cleaning milling chambers and SLM printing systems by suction, a sufficiently high concentration and purity (i.e., unmixed with other dental dusts such as plaster, resin) can result in spontaneous combustion or the milling dust or printing powder exploding because of a potential exothermic reaction (e.g., due to oxidation). An alternative cleaning method must be selected, particularly for the following milling or printing media (e.g., manual cleaning):
    - Wood
    - Titanium / titanium-aluminum
    - Light metals and light metal alloys (e.g., aluminum, magnesium)
    - Cobalt-chrome powder (e.g., for use in SLM systems)
- If large quantities of light metals such as titanium alloy are processed (e.g., with sandpaper), producing very fine grinding dust, a sufficiently high concentration and purity may result in spontaneous combustion because of a potential exothermic reaction.
- ▶ Do not suction hot materials.
  - ▶ Do not suction liquids.
  - ▶ If the extraction unit is used to suction hazardous materials, appropriate personal protective equipment must be worn, and steps must be taken to ensure that the exhaust air is properly ventilated. Refer to the safety data sheets for the corresponding requirements.
  - ▶ Dispose of extracted material in accordance with statutory regulations.

## 2.6 Authorized persons

Operation and maintenance of the equipment may only be performed by trained personnel.

Minors and pregnant persons may only operate and service the extraction unit if they are wearing appropriate protective equipment, in particular when suctioning hazardous materials.

Any repairs not described in this user information may only be performed by a qualified electrician.

## 2.7 Disclaimer

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- ▶ The product is employed for any purposes other than those specified in the instructions for use.
- ▶ The product is altered in any way other than those alterations described in the instructions for use.
- ▶ The product is not repaired by an authorized dealer or if original Renfert spare parts are not used.
- ▶ The product continues to be used despite obvious safety defects or damage.
- ▶ The product is subjected to mechanical impact or is dropped.

## 3 Product description

### 3.1 General description

The equipment is an extraction unit for suctioning dust that builds up in dental CAM systems.

It can be operated manually or controlled using a CAM system.

The extraction unit is equipped with a bi-directional interface for communicating status information to the CAM system and for registering control commands from the CAM system.

### 3.2 Assemblies and functional elements

1 SILENT powerCAM TC

2 Control panel

3 On / off switch

4 Dust drawer

4A Clamp fastener

5 Fine filter

6 Suction port

7 Velcro® strip

8 CAM interface

9 Power cord

10 Protective device switch

11 Exhaust air filter / Exhaust air outlet

12 Suction hose and end bushings

13 Renfert CONNECT port

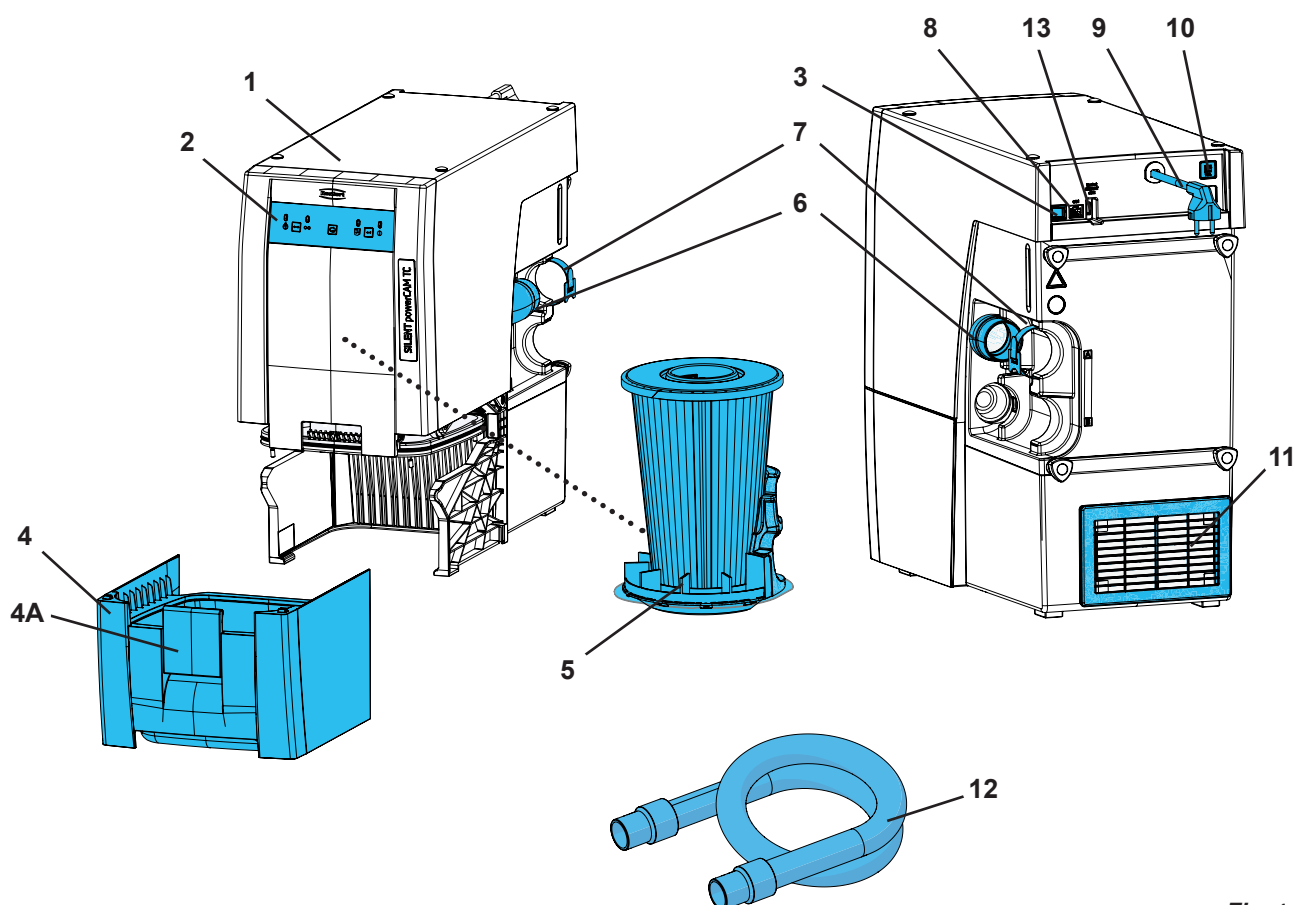
Fig. 1

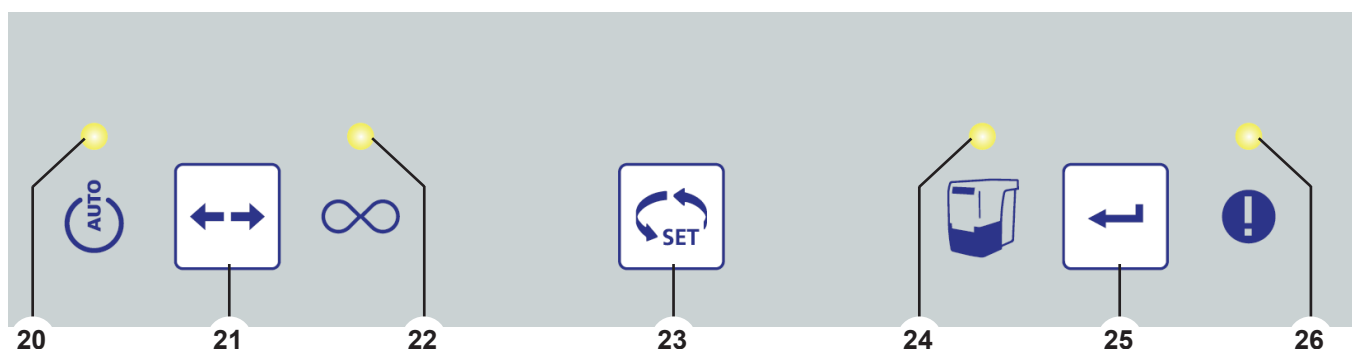
Fig. 2

- |                                                                    |                                       |
|--------------------------------------------------------------------|---------------------------------------|
| <b>20</b> Display: CAM operation                                   | <b>23</b> Selection key               |
| <b>21</b> Operating mode key: CAM operation / continuous operation | <b>24</b> Display: Empty dust drawer  |
| <b>22</b> Display: Continuous operation                            | <b>25</b> Enter key, saves input      |
|                                                                    | <b>26</b> Display: Error notification |

### 3.3 Scope of delivery

- 1 SILENT powerCAM TC
- 1 Quick Start Guide
- 1 Suction hose, 2 m, incl. 2 end bushings

### 3.4 Accessories

- 2934 0014 Waste bag for TC, TC2, EC2 & PowerCAM EC (25x)
- 2934 0015 Starter Kit waste bag for SILENT TC, TC2, EC2
- 2921 0003 End bushing set, 2x
- 90002 0097 H+Hepa filter SILENT
- 90002 1532 SILENT TC, TC2 fine filter, class M
- 90003 4240 Suction hose, 3 m, incl. 2 end bushings
- 90003 4826 Suction hose, antistatic, 3 m, incl. 2 end bushings
- 90115 0823 Suction hose, inside diameter 38 mm, 6 m
- 90215 0823 Suction hose, inside diameter 38 mm, 9 m
- 90003 4305 Adapter for hose connection
- 90003 4430 Universal suction hose adapter
- 90003 4314 Y-junction
- 2926 0000 Y-junction for dust extractor
- 2937 0002 External air duct for SILENT TC/EC/PowerCAM
- 2934 0007 90° angled connector SILENT
- 2934 0005 Interface cable, type A, for vhf
- 2934 0006 Interface cable, type B, for Roland DG
- 2934 0008 Interface cable, type C, for imes-icore
- 2934 0009 Interface cable, type D, for Amann Girrbach
- 2934 0010 Interface cable, type E, for Yenadent/Origin + Nema adapter
- 2934 0011 Interface cable, type F, for Zirkonzahn
- 2934 0012 Interface cable, type F, for Zirkonzahn + C14 adapter
- 2934 0013 Interface cable, type G, for vhf (6-pin, from K5)
- 2934 0016 Interface cable Type H for Dentsply Sirona
- 2934 0017 Interface cable Type I für Up3D
- 2934 0018 Interface cable Type J für Canon
- 2934 0021 Interface cable type M for XTCERA
- 2934 0022 Interface cable type N for Aidite

Additional details and accessories are available at [www.renfert.com](http://www.renfert.com).

## 4 Startup

### 4.1 Unpacking

- ⇒ Remove the unit and accessories from the shipping box.
- ⇒ Check the delivery for completeness (compare with the "Scope of delivery").

### 4.2 Setup

The extraction unit is a standing device. Do not operate it on its side.

Place the extraction unit so that:

- The exhaust air outlet (11, Fig. 1) is not blocked.
- The front is easily accessible for removing the dust drawer.

**If the extraction unit is placed in a closed cabinet, the warm exhaust air must be transported out of the cabinet using one of the measures shown in figures A, B, or C at the start of this document.**

- External air duct (A) (see Chap. 4.6).
  - Opening in the rear panel of the cabinet (B), min. 250 x 120 mm, directly opposite the exhaust air outlet (11, Fig. 1).
    - Distance from the rear of the cabinet to the wall: min. 100 mm,
    - Distance from the extraction unit to the rear of the cabinet: max. 25 mm.
  - Remove the rear panel of the cabinet (C), distance from the rear of the cabinet to the wall min. 50 mm.
- If the warm exhaust air is transported out of the cabinet to the rear through openings, it must be ensured that the warm exhaust air can escape from there unhindered.

### 4.3 Electrical connection

**Prior to electrical connection, check that the voltage indicated on the nameplate matches the local voltage supply.**

**Select the layout of live parts (sockets, plugs and coupling) and routing of extension cables in such a way that the protection class is maintained.**

- ⇒ Switch the unit off at the on / off switch (3, Fig. 1).
- ⇒ Unroll the power cord (9, Fig. 1) and insert the power plug into a building electrical socket.

### 4.4 Connection to the suction point

**Caution: Risk of injury!**

**When shortening the suction hose, make sure to cut the integrated wire as straight as possible.**

**Long suction hoses, tight bends and kinks greatly reduce the suction power at the suction point.**

- ⇒ Shorten the suction hose if necessary.
- ⇒ Open the Velcro® strip (7).
- ⇒ Insert the suction hose end bushing (12) into the suction port (6).
- ⇒ Secure the suction hose with the Velcro® strip (7).
- ⇒ Connect the suction hose to the suction point of the CAM system. In doing so, observe the instructions for use of the CAM system.

**If the diameter is not suitable, use an adapter (see Accessories) to avoid loss of suction.**

**Avoid steep inclines and "sagging" along the hose.**

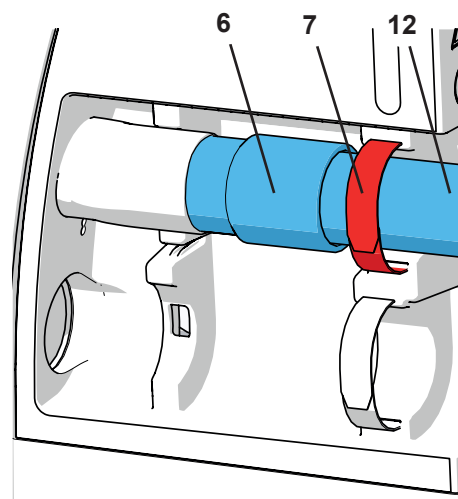
Fig. 3

## 4.5 Connection to the CAM interface

For remote control of the extraction unit, only use the CAM interface provided for this purpose.  
In the case of CAM systems that only provide a switched mains voltage, it is essential that a type F interface cable is used.

If the extraction unit is very frequently switched on and off using a power supply to the CAM system, this can damage the electronics of the CAM system and of the extraction unit.

The electrical connection for communication with the CAM system is provided via the interface connector (8) and an optional interface cable (see Accessories). For assignment of the available interface cable to the CAM system or preparation of your own interface cable, see Chap. 9.1 CAM interface.

**Confirm with your CAM manufacturer that the interface for connecting the extraction unit comprises a limited-energy circuit in accordance with IEC 61010-1.**

Also observe the instructions for use of the CAM system.

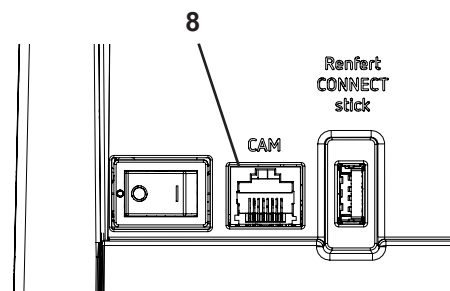
Fig. 4

## 4.6 External air duct

The exhaust air can be transported out of the laboratory via an external air duct (see Accessories).

The corresponding installation instructions are included with the external air duct.

**When using extraction units in conjunction with an external air duct, a significant quantity of air is extracted from the room per hour. This can lead to vacuum pressure, which can cause poisonous gases (e.g., carbon monoxide) to be sucked into the (working) area when using gas, liquid, or solid fuels with furnaces dependent on ambient air.**

**Depending on the structural situation, an additional air supply or vacuum pressure monitoring must therefore be ensured and checked where appropriate by the responsible institutions (e.g., chimney sweep).**

# 5 Operation

The extraction unit is controlled by the keys on the control panel (Fig. 2).

## 5.1 Switching on

The extraction unit is switched on and off via the on / off switch (3).

After switching on:

- ◆ The extraction unit performs automatic filter cleaning (for approx. 8 seconds, loud vibrating noise), see Chap. 5.3.

Afterwards, the extraction unit is in the last operating mode set.

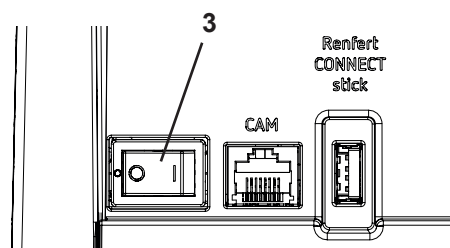
Fig. 5

## 5.2 Selecting the operating mode: CAM operation / continuous operation

The extraction unit has two operating modes.

The set operating mode is indicated by the displays (20) / (22).

- CAM operation (20):

- ◆ The extraction unit responds to the control signals of the CAM system.

- Continuous operation (22):

- ◆ The extraction unit is in continuous operation.

⇒ Press the operating mode key (21).

- ◆ Switch the operating mode.

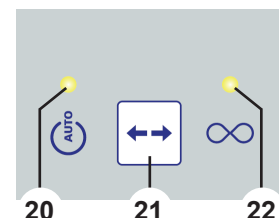
Fig. 6

**The extraction unit is suitable for dry dust only!**

**When connecting to CAM systems with a wet / dry function, it is essential to ensure that no residual moisture from the CAM system enters the extraction unit.**

**Always check that the extraction unit is in proper working order before milling:**

- ▶ Display for error notification (26, Fig. 2) is off.
- ▶ Suction turbine operates correctly without any particularly unusual noise or odor.

## 5.3 Cleaning the filter

In order to ensure maximum suction power, the extraction unit is equipped with a device for cleaning the filter of the fine filter unit. Cleaning the filter takes approx. 8 seconds.

The filter is cleaned:

- Every time the unit is switched on
- If there is insufficient suction power (flow rate falls below an internal limit value)
- Before removing the dust drawer when prompted to empty it (see Chap. 6.2)

If the flow rate falls below an internal limit value for more than 1 minute, the extraction process is interrupted, and the filter is cleaned:

⇒ Limit value has not been reached:

- ◆ Suction turbine is stopped.
- ◆ The filter is cleaned. Cleaning the filter takes approx. 8 seconds and is indicated by an acoustic signal.
- ◆ Suction turbine is started again.

**If the flow rate drops again after automatic filter cleaning, the filter is only cleaned again after the turbine has been running for two hours.**

## 6 Cleaning / Maintenance

**The unit does not contain any internal parts that require maintenance. Opening the unit is not permitted other than as described below!**

### 6.1 Cleaning

To clean the unit, only wipe the outside with a damp cloth. Do not use any solvent-based or abrasive cleaning agents.

### 6.2 Emptying the dust drawer

After a set time interval is reached (see Chap. 6.2.1), you are prompted to empty the dust drawer:

- ◆ An acoustic signal sounds three times.
- ◆ The dust drawer display (24, Fig. 2) lights up.

The next time the extraction unit comes to a stop, an acoustic signal sounds three times after which the filter is cleaned.

Once the filter has been cleaned:

- ⇒ Release the clamp fastener (4A) of the dust drawer.
- ⇒ Pull the dust drawer (4) forward and empty.

**During disposal, observe local provisions and accident prevention regulations!**

**Personal protective equipment must be worn depending on the extracted material.**

- ⇒ Insert the dust drawer again and push it all the way in.
- ⇒ Close the clamp fastener (4A).
- ⇒ Press the Enter key (25). The counter for recording the time interval is reset.
  - ◆ An acoustic signal confirms your input.
  - ◆ The dust drawer display (24) switches off.

**If the dust drawer is not emptied, the dust drawer display (24, Fig. 8) remains lit up. After switching the unit on and off, an acoustic signal sounds three times again to indicate that the dust drawer must be emptied.**

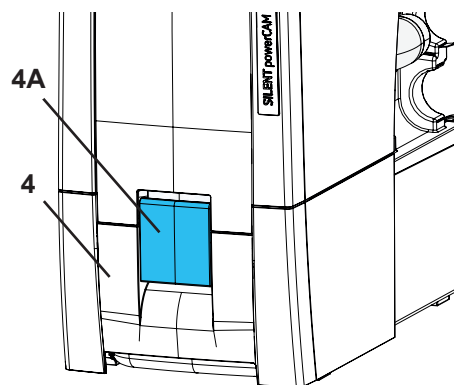
Fig. 7

## 6.2.1 Setting the time interval for emptying the dust drawer

You can choose between five different time intervals.

| Time interval / hours | Flashing signal |
|-----------------------|-----------------|
| 2                     | 1 x             |
| 5                     | 2 x             |
| 10                    | 3 x             |
| 50                    | 4 x             |
| 100                   | 5 x             |

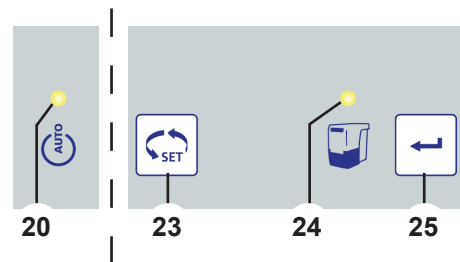
Fig. 8

Select a time interval:

⇒ Switch the unit off at the on / off switch (9, Fig. 1).

⇒ Press and hold the Enter key (25) and switch on the unit using the on / off switch.

**After switching on, all four displays light up briefly (functional check of the displays).**

⇒ Continue to hold down the Enter key (25) until:

- ◆ The CAM operation display (20) lights up.
- ◆ The dust drawer display (24) flashes in accordance with the time interval level that is set.

⇒ Choose your preferred time interval using the selection key (23).

You can toggle in sequence between the five time intervals by pressing the key.

The selected time interval in each case is indicated by the corresponding flashing signal.

Once your preferred time interval is set:

⇒ Press the Enter key (25).

- ◆ The dust drawer display (24) lights up for two seconds.
- ◆ An acoustic signal confirms your input.

## 6.3 Replacing the fine filter

How effectively the filter has been cleaned is verified by monitoring the flow rate (internal pressure sensor).

If the filter must be cleaned several times in succession after less than 2 hours, this means that the fine filter is so heavily clogged that cleaning the filter does not have a sufficient impact and the fine filter must be replaced.

This is signaled as follows:

- ◆ The error notification display (26, Fig. 2) lights up.
- ◆ For a period of 15 minutes, an acoustic signal sounds three times every three minutes.
- ◆ The notification is output to the CAM interface (see Chap. 9.2).

The error notification can be switched off by switching off the extraction unit.

**If you are prompted to replace the fine filter, or if the suction power does not significantly improve or only improves temporarily despite cleaning the filter several times, the fine filter must be replaced as soon as possible. Further operation can lead to damage to the unit.**

► **In general, the fine filter should be changed every 2 years.**

When inserting the new fine filter, pay attention to correct seating, as otherwise leaks will occur. For this purpose, see the assembly instructions at the end of the instructions for use, which are also included with the new fine filter.

## 6.4 Fuses

The extraction unit is protected by a protective device switch (10, Fig. 1).

If the protective device switch is triggered, it can be reset by pressing the button.

**If a protective device switch is triggered repeatedly, this is due to a fault in the unit. Send the equipment in for repair!**

## 6.5 Spare parts

You can find wear parts and spare parts in the spare parts list available online at

**[www.renfert.com/p918](http://www.renfert.com/p918)**.

Enter the following item number: 29380000.

The components excluded from the warranty (wear parts, consumables) are marked in the spare parts list.

## 6.6 Factory settings

⇒ Switch the unit off (3, Fig. 1).

⇒ Hold down the operating mode key (21, Fig. 2) and the selection key (23, Fig. 2) at the same time and switch on the unit (3, Fig. 1).

- ◆ All four displays flash three times.
- ◆ All values are reset to the factory settings.

**Factory settings:**

| Function / feature | Setting range                        | Factory setting |
|--------------------|--------------------------------------|-----------------|
| Operating mode     | CAM operation / continuous operation | CAM operation   |
| Time interval      | 2 - 100 hrs                          | 50 hrs          |

## 7 Troubleshooting

| Trouble                                                                                                                                                     | Cause                                                                                                                                                       | Corrective action                                                                                                                                                                                                                                                                                                                                                                                            |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Error notification display (26, Fig. 2) flashes.</b>                                                                                                     | <ul style="list-style-type: none"> <li>The electronics have overheated.</li> </ul>                                                                          | <ul style="list-style-type: none"> <li>Switch off the unit and let it cool down.</li> <li>Provide sufficient cooling, e.g., using: <ul style="list-style-type: none"> <li>Note Chap. 4.2 Setup.</li> <li>Use external air duct (see Chap. 4.6).</li> </ul> </li> </ul>                                                                                                                                       |
| <b>The error notification display (26, Fig. 2) lights up.</b>                                                                                               | <ul style="list-style-type: none"> <li>The fine filter is so heavily clogged that cleaning the filter does not have a sufficient impact.</li> </ul>         | <ul style="list-style-type: none"> <li>Replace fine filter (see Chap. 6.3).</li> </ul>                                                                                                                                                                                                                                                                                                                       |
| <b>The suction power is not sufficient.</b>                                                                                                                 | <ul style="list-style-type: none"> <li>Blockage or leakage in the suction hose.</li> <li>Dust drawer not airtight.</li> <li>Fine filter blocked.</li> </ul> | <ul style="list-style-type: none"> <li>Check suction hose.</li> <li>Please also note the information in Chap. 4.4.</li> <li>Check that the dust drawer fits correctly (see Chap. 6.2).</li> <li>Switch the unit off and then on again so that the filter is cleaned.</li> <li>Replace the fine filter (see Chap. 6.3) if cleaning the filter does not improve suction power.</li> </ul>                      |
| <b>Dust drawer is overflowing.</b>                                                                                                                          | <ul style="list-style-type: none"> <li>The selected time interval for emptying the dust drawer is too long.</li> </ul>                                      | <ul style="list-style-type: none"> <li>Set a shorter time interval (see Chap. 6.2.1).</li> </ul>                                                                                                                                                                                                                                                                                                             |
| <b>The signal to empty the dust drawer is output, even though it is not full yet.</b>                                                                       | <ul style="list-style-type: none"> <li>The time interval set for emptying the dust drawer is too short.</li> </ul>                                          | <ul style="list-style-type: none"> <li>Set a longer time interval (see Chap. 6.2.1).</li> </ul>                                                                                                                                                                                                                                                                                                              |
| <b>The display to empty the dust drawer flashes and an acoustic signal sounds three times.</b>                                                              | <ul style="list-style-type: none"> <li>The flow rate is no longer sufficient and filter cleaning is performed.</li> </ul>                                   | <ul style="list-style-type: none"> <li>Wait until the filter has been cleaned.</li> </ul>                                                                                                                                                                                                                                                                                                                    |
| <b>The extraction unit unexpectedly stops suctioning during continuous or automatic operation and the corresponding displays (20/22, Fig. 2) remain on.</b> | <ul style="list-style-type: none"> <li>Suction turbine has overheated.</li> <li>Suction turbine is faulty.</li> </ul>                                       | <ul style="list-style-type: none"> <li>Switch off the unit and leave to cool for at least 60 minutes.</li> <li>Check whether suction hose is blocked, eliminate blockage.</li> <li>Switch the unit off and then on again so that the filter is cleaned.</li> <li>Replace the fine filter (see Chap. 6.3) if cleaning the filter does not improve suction power.</li> <li>Replace suction turbine.</li> </ul> |

| Trouble                                                                       | Cause                                                                                                                                                                                                                                                                   | Corrective action                                                                                                                                                                                                                                                                                                         |
|-------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>The extraction unit does not respond to the signals of the CAM system.</b> | <ul style="list-style-type: none"> <li>• The connector to the CAM system or the extraction unit is not correctly inserted.</li> <li>• Interface cable damaged.</li> <li>• Connector assignment of the interface cable does not match that of the CAM system.</li> </ul> | <ul style="list-style-type: none"> <li>• Check the connector on the extraction unit and CAM system.</li> <li>• Check the interface cable for damage, replace if required.</li> <li>• Compare the connector assignment of the interface cable with that of the CAM. Use a suitable interface cable if required.</li> </ul> |
| <b>After switching on, not all displays light up briefly.</b>                 | <ul style="list-style-type: none"> <li>• The display that does not light up is faulty.</li> </ul>                                                                                                                                                                       | <ul style="list-style-type: none"> <li>• Contact Renfert / Service.</li> </ul>                                                                                                                                                                                                                                            |

## 8 Technical data

|                                                                                                              | <b>SILENT powerCAM TC</b>                                         |                  |
|--------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------|------------------|
|                                                                                                              | <b>2938 0000</b>                                                  | <b>2938 1000</b> |
| <b>Nominal voltage:</b>                                                                                      | <b>230 V</b>                                                      | <b>120 V</b>     |
| Permissible mains voltage:                                                                                   | 220 - 240 V                                                       | 120 V            |
| Mains frequency:                                                                                             | 50 / 60 Hz                                                        | 50 / 60 Hz       |
| Power consumption *):                                                                                        | 850 W                                                             | 800 W            |
| Mains input fuse:                                                                                            | 15 A (T)                                                          |                  |
| LpA **) (at max. volume flow):                                                                               | 54.3 dB(A)                                                        |                  |
| Ø suction port:<br>- inner                                                                                   | 42 mm [1.65 inches]                                               |                  |
| Volume flow, max. ***):                                                                                      | 3980 l/min [2.34 ft³/s]                                           |                  |
| Vacuum pressure, max. ****):                                                                                 | 281 hPa<br>[4.1 psi]                                              |                  |
| Fine filter:<br>- Filter surface, approx.<br>- Filter quality                                                | 0.9 m² [1390 sq inch]<br>Class M in accordance with EN 60335-2-69 |                  |
| Fill volume dust drawer, approx.:                                                                            | 7 l<br>[1.85 US gal]                                              |                  |
| Weight (empty), approx.:                                                                                     | 22.0 kg<br>[48.5 lbs.]                                            |                  |
| Dimensions (width x height x depth):                                                                         | 275 x 535 x 540 mm<br>[10.8 x 21.1 x 21.2 inches]                 |                  |
| CAM interface:                                                                                               | RJ45 port                                                         |                  |
| Input signal:<br>- Low<br>- High                                                                             | 0 - 5 V<br>15 - 24 V                                              |                  |
| Output signal:<br>- Max. pull-up voltage (U)<br>- Max. switching current (I)<br>- Internal series resistance | 24 V<br>5 mA<br>150 Ohm                                           |                  |

- \*) Power consumption at nominal voltage  
 \*\*) Noise level in accordance with EN ISO 11202  
 \*\*\*) Free-blowing turbine at nominal turbine voltage  
 \*\*\*\*) At nominal turbine voltage

## 9 CAM interface

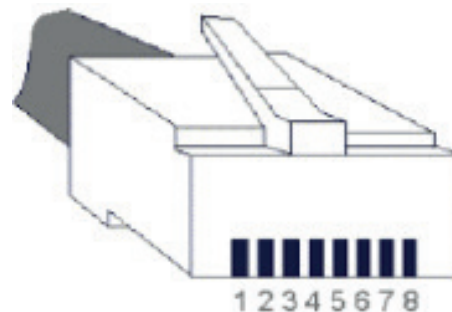
A galvanically-isolated, bi-directional interface is provided for communication with a CAM system.

Four commands each can be received from the extraction unit and four status notifications transmitted to the CAM system via two input signals and two output signals.

### 9.1 Pin assignment: CAM interface (8, Fig. 1)

Type: RJ45

| Pin | Name       |
|-----|------------|
| 1   | Out2 - GND |
| 2   | Out2       |
| 3   | Out1 - GND |
| 4   | Out1       |
| 5   | In2 - GND  |
| 6   | In2 - 24 V |
| 7   | In1 - GND  |
| 8   | In1 - 24 V |


#### 9.1.1 Outputs (Out1 / Out2)

The output signals are executed as potential-free switch outputs of an opto-coupler with a series resistance of 150 Ohm.

For maximum pull-up voltage (U) and maximum permissible switching current (I), see Chap. 8 Technical data.

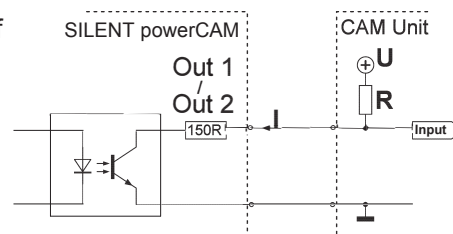The pull-up resistance (R) must be dimensioned in such a way that the maximum permissible switching current is not exceeded.

#### 9.1.2 Inputs (In1 / In2)

The input signals are transferred to the LEDs of opto-couplers. For the required input voltages of the two signal levels "low" / "high", see Chap. 8 Technical data.

### 9.2 Control commands / status notifications

| In1         | In2         | Command                           |
|-------------|-------------|-----------------------------------|
| 0 V (low)   | 0 V (low)   | Extraction unit off (turbine off) |
| 24 V (high) | 0 V (low)   | Extraction unit on (turbine on)   |
| 0 V (low)   | 24 V (high) | Clean the filter                  |
| 24 V (high) | 24 V (high) | not reserved / reserve            |

| Out1 | Out2 | Notification                           |
|------|------|----------------------------------------|
| 24 V | 24 V | Error, electronics too hot *)          |
| 0 V  | 24 V | Automatic filter cleaning is performed |
| 24 V | 0 V  | Filter must be replaced                |
| 0 V  | 0 V  | Extraction unit OK                     |

\*) An overheated or defective turbine is NOT detected and is therefore not notified as an error.

## 9.3 Connector assignment: interface cable

Interface cable, type A:

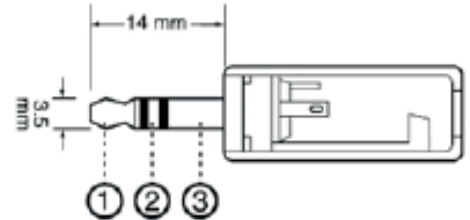
| Extraction unit | CAM system                                   |
|-----------------|----------------------------------------------|
| RJ45 connector  | Lumberg SV40<br>(with thread locking system) |
| Pin 8           | Pin 1 (+)                                    |
| Pin 7           | Pin 4 (-)                                    |

Reverse side


Interface cable, type B:

| Extraction unit | CAM system            |
|-----------------|-----------------------|
| RJ45 connector  | Stereo jack<br>3.5 mm |
| Pin 8           | Pin 1 (+)             |
| Pin 7           | Pin 2 (-)             |


Interface cable, type C:

| Extraction unit | CAM system   |
|-----------------|--------------|
| RJ45 connector  | 9-pin, D-Sub |
| Pin 8           | Pin 4 (+)    |
| Pin 7           | Pin 3 (-)    |

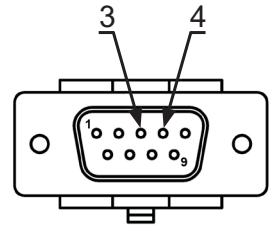Interface cable, type D:


| Extraction unit | CAM system   |
|-----------------|--------------|
| RJ45 connector  | 9-pin, D-Sub |
| Pin 8           | Pin 3 (+)    |
| Pin 7           | Pin 4 (-)    |

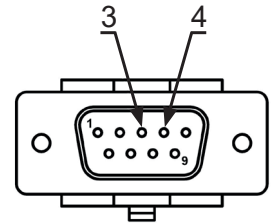Interface cable, type E:

To be used if the CAM system only provides one potential-free contact between pin 8 and 9.

| Extraction unit | CAM system   | Power supply unit |
|-----------------|--------------|-------------------|
| RJ45 connector  | 9-pin, D-Sub |                   |
| ----            | Pin 9        | 24 V              |
| Pin 8           | Pin 8        | ----              |
| Pin 7           | ----         | GND               |

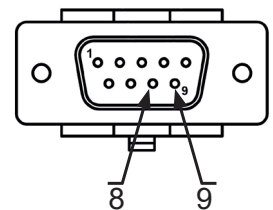
Interface cable, type F:

To be used if the CAM system only provides mains voltage for controlling the extraction unit.

| Extraction unit | Power supply unit |
|-----------------|-------------------|
| RJ45 connector  |                   |
| Pin 8           | 24 V              |
| Pin 7           | GND               |

Interface cable, type G:

| Extraction unit | CAM system                                   |
|-----------------|----------------------------------------------|
| RJ45 connector  | Lumberg SV60<br>(with thread locking system) |
| Pin 8           | Pin 1 (+)                                    |
| Pin 6           | Pin 2 (+)                                    |
| Pin 4           | Pin 4 (+)                                    |
| Pin 2           | Pin 5 (+)                                    |
| Pin 7           | Pin 6 (-)                                    |
| Pin 5           |                                              |
| Pin 3           |                                              |
| Pin 1           |                                              |

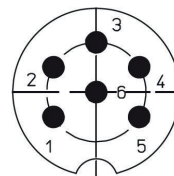
Reverse side

Interface cable Type H:

| Extractor | CAM device       |
|-----------|------------------|
| RJ 45 pin | 15 Pin D-Sub     |
| Pin 8     | Pin 2 (+)        |
| Pin 7     | Pin 12 (-)       |
| ---       | Pin 7 <-> Pin 10 |


Interface cable Type I:

| Extractor | CAM device         |
|-----------|--------------------|
| RJ 45 pin | 5 pol Aviation pin |
| Pin 8     | Pin 5 (+)          |
| Pin 7     | Pin 4 (-)          |

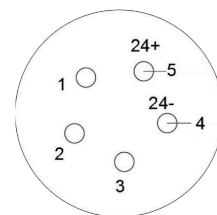
Interface cable Type J:

| Extractor | CAM device           | Power supply |
|-----------|----------------------|--------------|
| RJ 45 pin | 2 pol. TMW connector |              |
| Pin 8     | ----                 | 24 V         |
| Pin 7     | B                    | ----         |
| ----      | A                    | GND          |

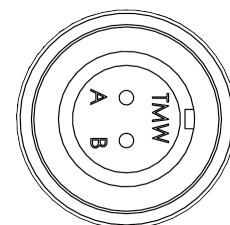
Interface cable, type M:

| Extractor | CAM device             | Power supply |
|-----------|------------------------|--------------|
| RJ 45 pin | 4 pol. round connector |              |
| ---       | 1                      | 24 V         |
| Pin 7     | ---                    | GND          |
| Pin 8     | 2                      |              |

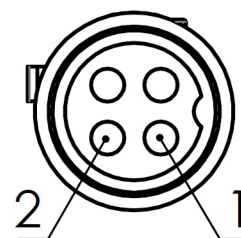Interface cable, type N:

| Extractor | CAM device                        |
|-----------|-----------------------------------|
| RJ 45 pin | 2 pol. round connector<br>(WEIPU) |
| Pin 7     | 2                                 |
| Pin 8     | 1                                 |
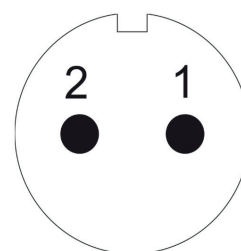
## 10 Warranty

Renfert offers a **3-year warranty** on all parts of the equipment, provided that the equipment is properly used.

**SILENT powerCAM TC:**

***A 3-year warranty is provided for the suction motor, however with a maximum of 1000 operating hours (motor running time).***

Warranty claims may only be made upon presentation of the original sales receipt from the authorized dealer.

Parts that are subject to natural wear and tear (wear parts) and consumables are excluded from the warranty. These parts are marked in the spare parts list.

The warranty is voided in the case of improper use; failure to observe the operating, cleaning, maintenance, and connection instructions; in the case of independent repairs or repairs that are not carried out by an authorized dealer; if spare parts from other manufacturers are used, or in the case of unusual influences or influences not in compliance with the utilization instructions.

Warranty claims shall not extend the warranty period.

## 11 Disposal information

### 11.1 Disposing of consumables

Full dust bags and filters must be disposed of in accordance with local regulations. When doing so, personal protective equipment must be worn, depending on the material trapped in the filter.

### 11.2 Disposing of the equipment

The equipment must be disposed of by a specialist facility. The specialist facility must be informed of any residue in the equipment that is harmful to health.

#### 11.2.1 Disposal information for countries in the EU

To conserve and protect the environment, prevent environmental pollution, and improve the recycling of raw materials, the European Commission has adopted a directive that requires the manufacturer to accept the return of electrical and electronic equipment for proper disposal or recycling.

**For this reason, equipment with this symbol may not be disposed of in unsorted domestic waste within the European Union.**

Please contact your local authorities for more information on proper disposal.

**Hochaktuell und ausführlich auf ...**  
**Up to date and in detail at ...**  
**Actualisé et détaillé sous ...**  
**Aggiornato e dettagliato su ...**  
**La máxima actualidad y detalle en ...**  
**Актуально и подробно на ...**

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declare under our sole responsibility, that the product(s)

| Name               | Part No.                                             |
|--------------------|------------------------------------------------------|
| SILENT TS          | 2921 0050                                            |
| SILENT TS2         | 2930 0050                                            |
| SILENT TC          | 2935 0000 / 2935 0000 03                             |
| SILENT TC2         | 2936 0000 / 2936 0000 03                             |
| SILENT EC2         | 2937 0000 / 2937 0000 03                             |
| SILENT compact     | 2934 0000 / 2934 0000 03                             |
| SILENT compactCAM  | 2934 2000 / 2934 2000 03<br>2934 2500 / 2934 2500 03 |
| SILENT powerCAM EC | 2939 0000                                            |
| SILENT powerCAM TC | 2938 0000                                            |
| iVAC silent        | 36 2934 2000                                         |

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

### UK legislation

*Electromagnetic Compatibility Regulations 2016*

*Supply of Machinery (Safety) Regulations 2008*

*The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012*

### Designated standards

EN 61326-1:2013

EN ISO12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,  
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

making work easy

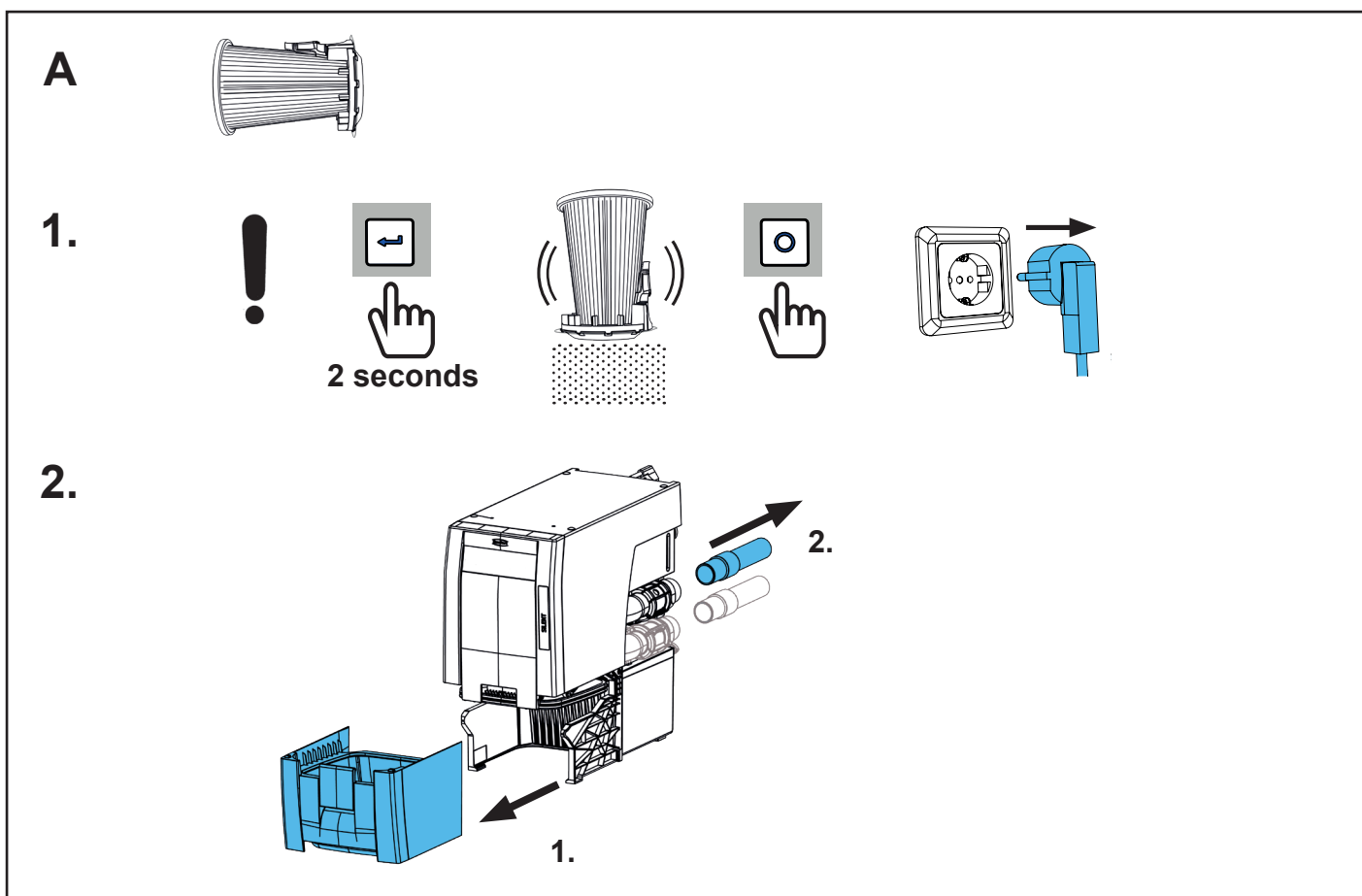
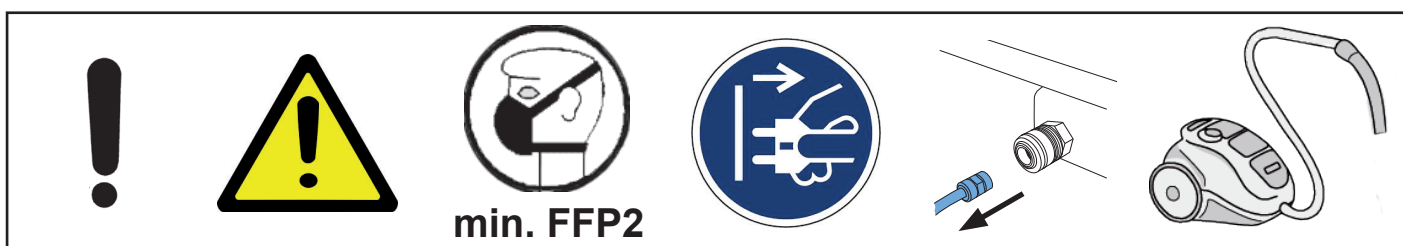
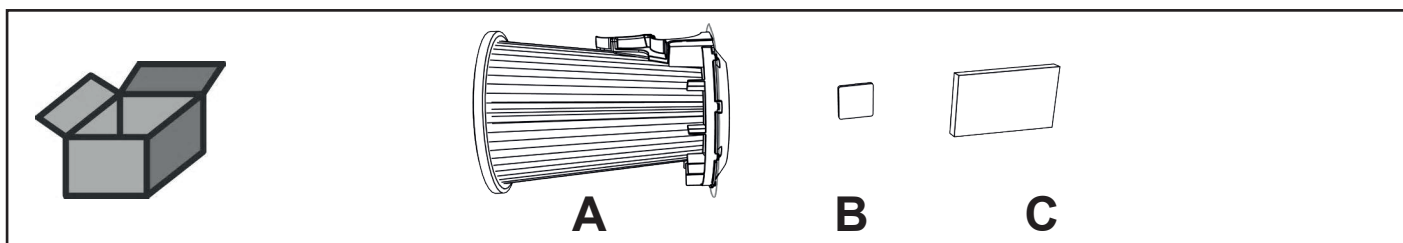
Montageanleitung • Assembly Instruction • Instruction d'installation  
Istruzioni l'installazione • Instrucciones de montaje • Руководство по  
монтажу

## SILENT TC / TC2 / powerCAM TC / EC2 / powerCAM EC


90002 1584 / 90002 0097

Feinfilter Klasse M • Fine filter class M • Filtre fin classe M • Filtro fino di classe M •  
Filtro fino clase M • Фильтр тонкой очистки, класса M



H+Hepa-Filter • H+Hepa filter • Filtre H+Hepa • Filtro H+Hepa • Filtro H+Hepa •  
Фильтр H+Hepa

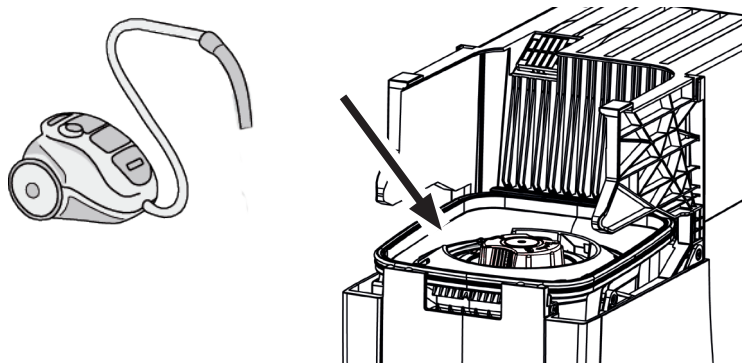
3.

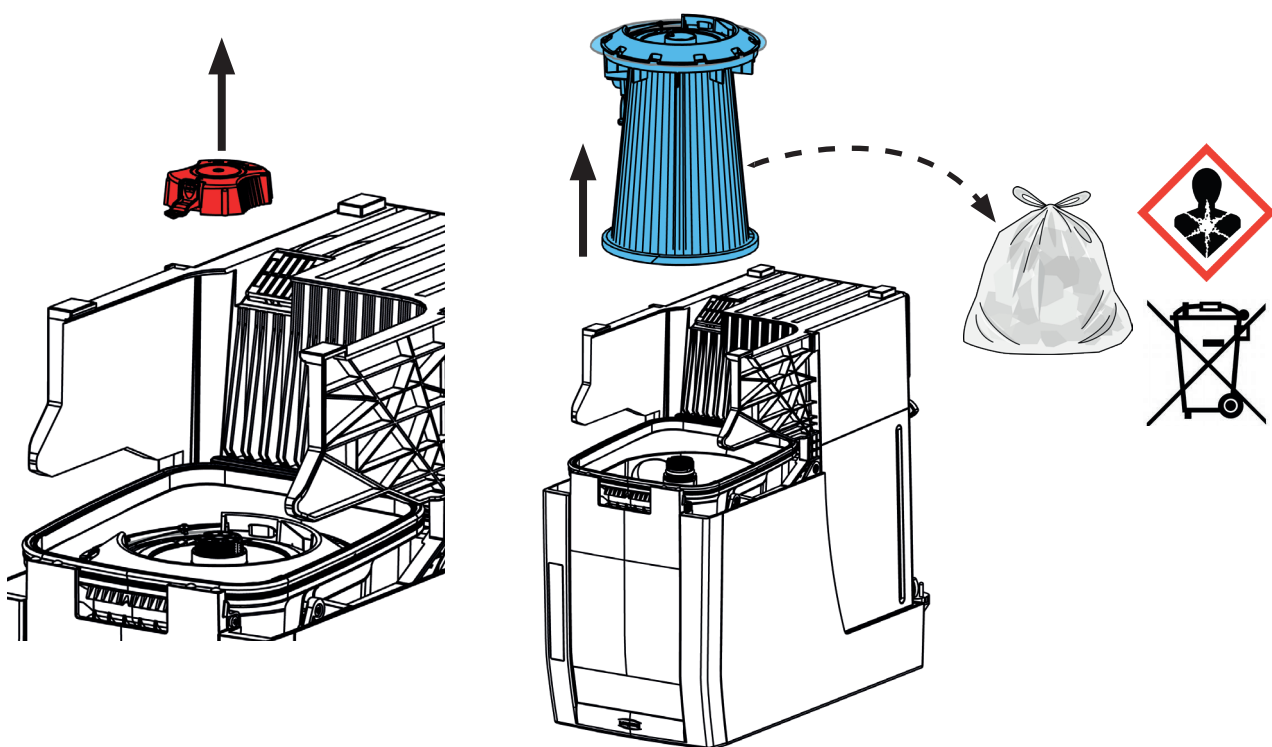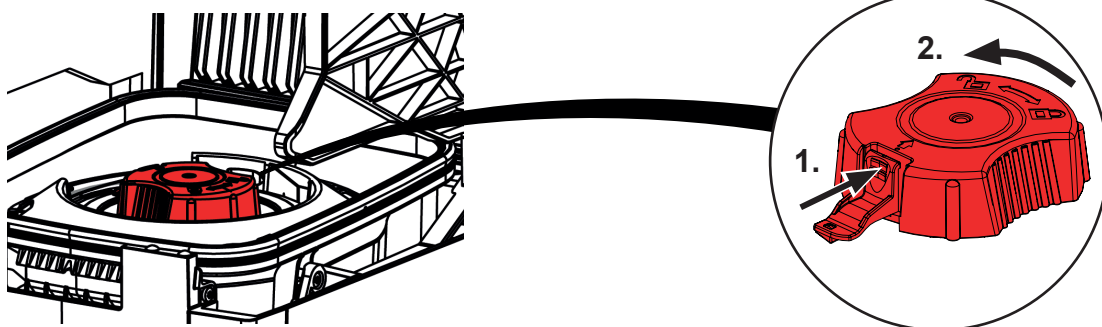
4.

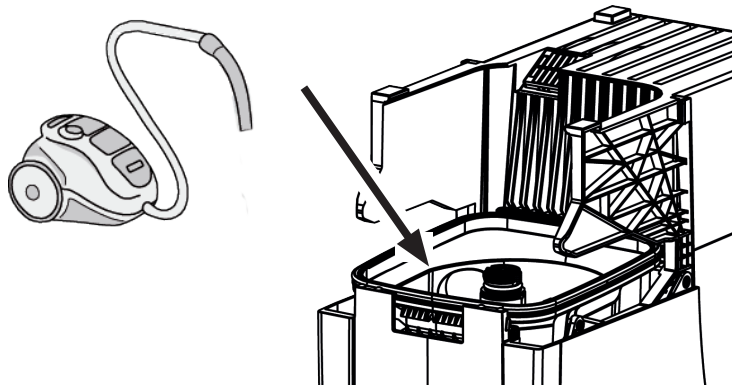
5.

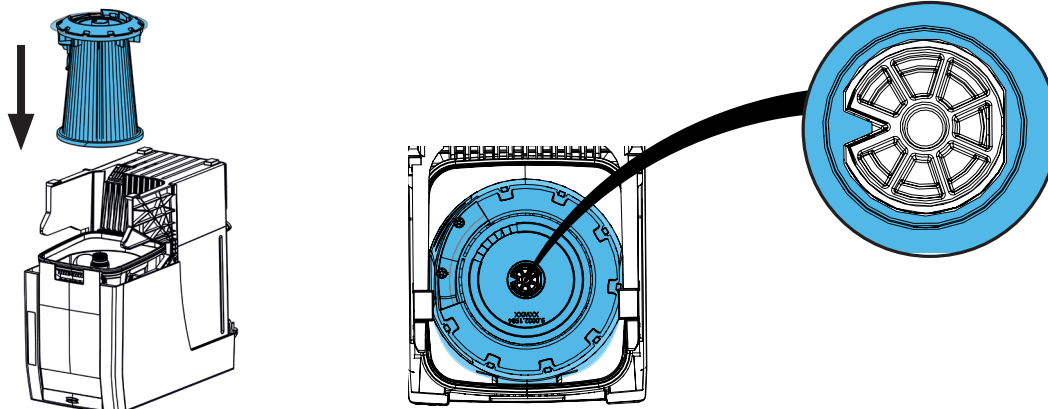6.

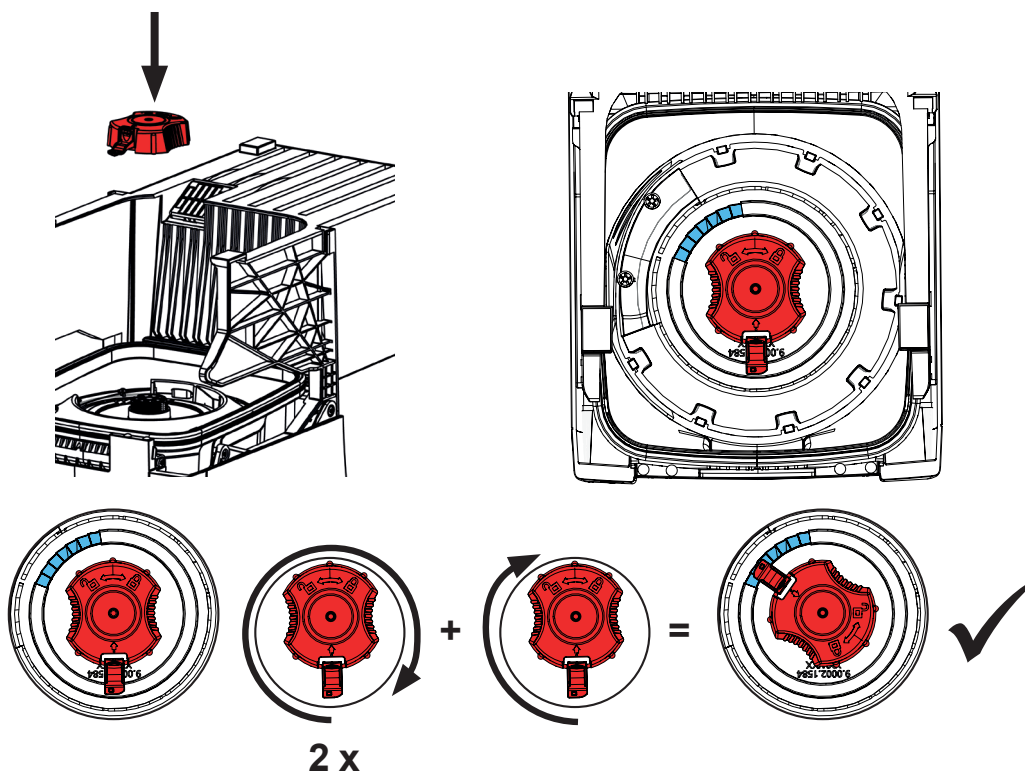
7.


B

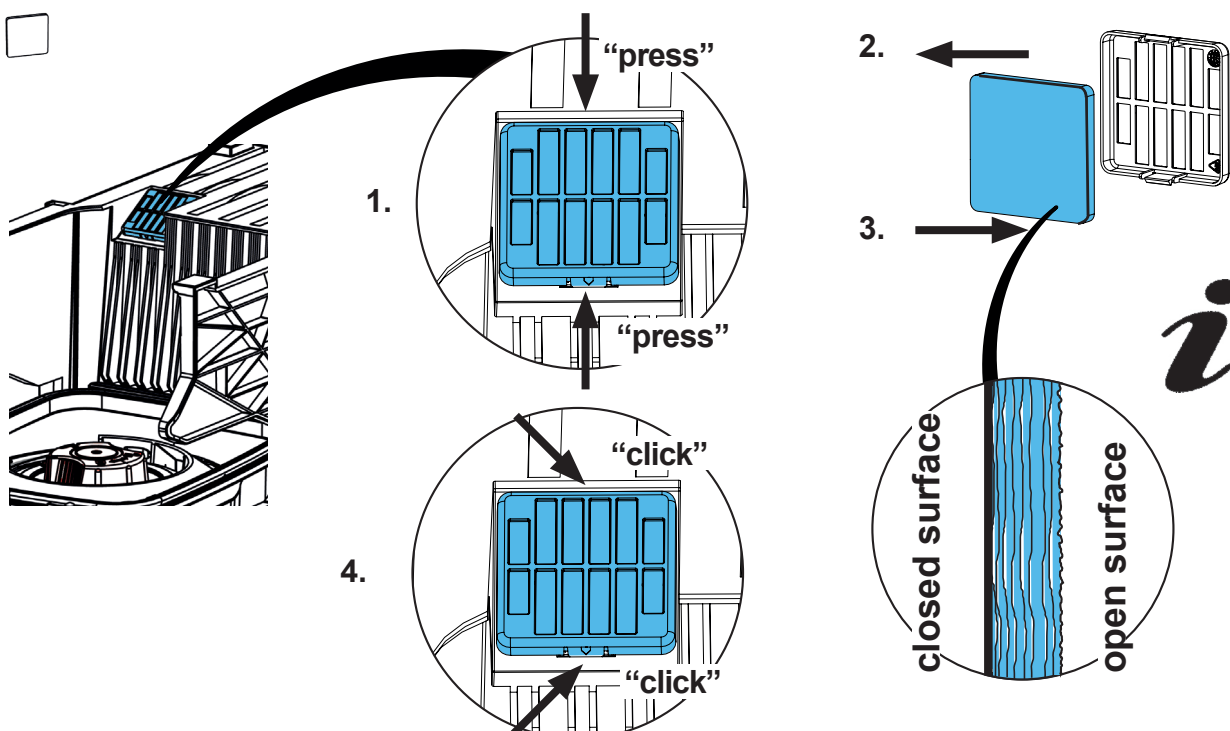
8.

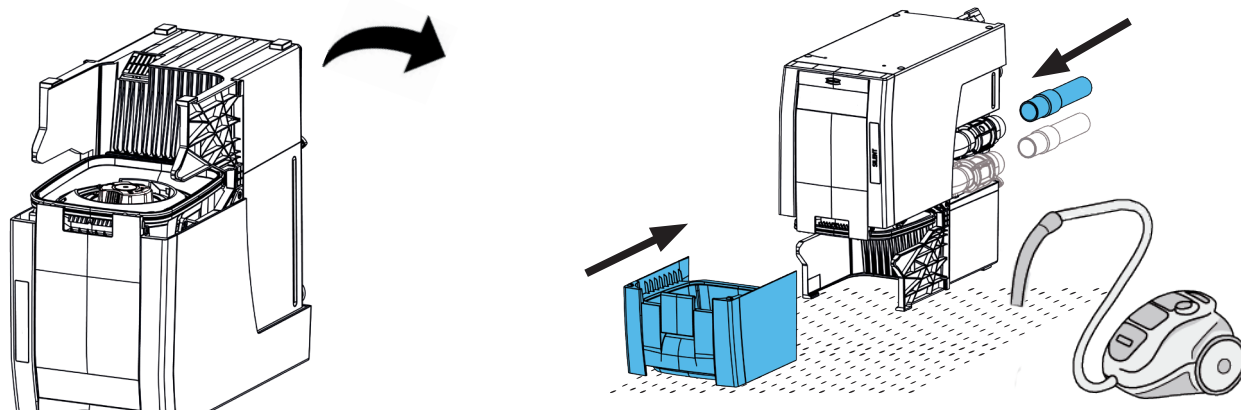9.

C



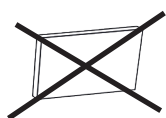
**SILENT TC / TC2 / powerCAM TC**

**SILENT EC2 / poweCAM EC**

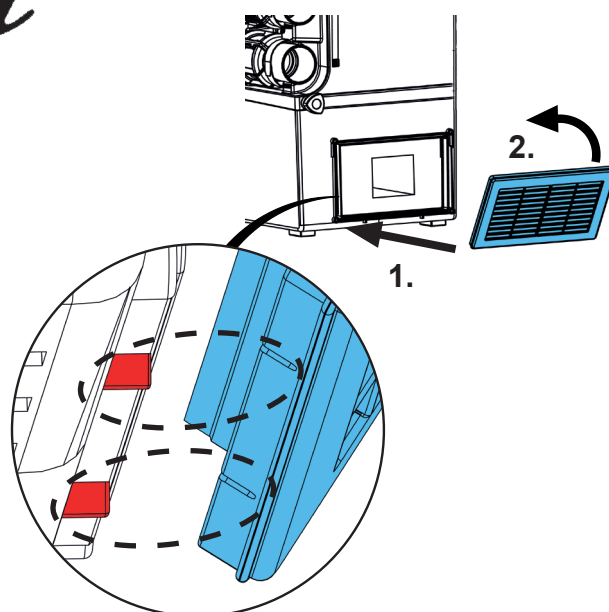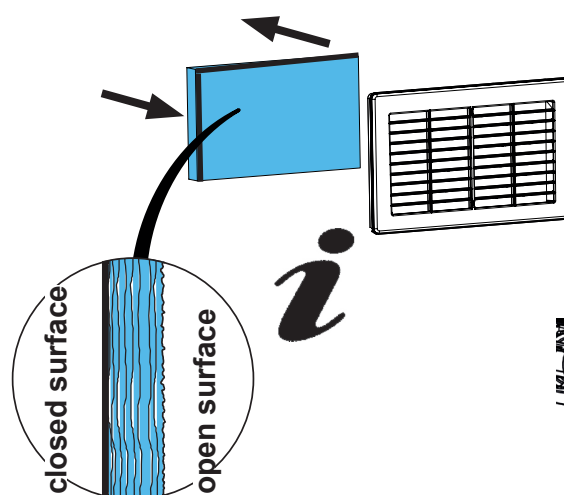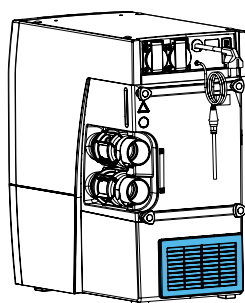10.

**Hochaktuell und ausführlich auf ...  
Up to date and in detail at ...  
Actualisé et détaillé sous ...  
Aggiornato e dettagliato su ...  
La máxima actualidad y detalle en ...  
Актуально и подробно на ...**

**[www.renfert.com](http://www.renfert.com)**

Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany  
Tel.: +49 7731 82 08-0 • Fax: +49 7731 82 08-70  
[www.renfert.com](http://www.renfert.com) • [info@renfert.com](mailto:info@renfert.com)

Renfert USA • 3718 Illinois Avenue • St. Charles IL 60174/USA  
Tel.: +1 6307 62 18 03 • Fax: +1 6307 62 97 87  
[www.renfert.com](http://www.renfert.com) • [info@renfertusa.com](mailto:info@renfertusa.com)  
USA: Free call 800 336 7422