making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Polski
- FAQ - Dansk
- FAQ - 中文
- FAQ - 日本語
- FAQ - 한국어
- FAQ - اللغة العربية
- FAQ - Українська
- FAQ - Română

SILENT powerCAM EC

FAQ - frequently asked questions

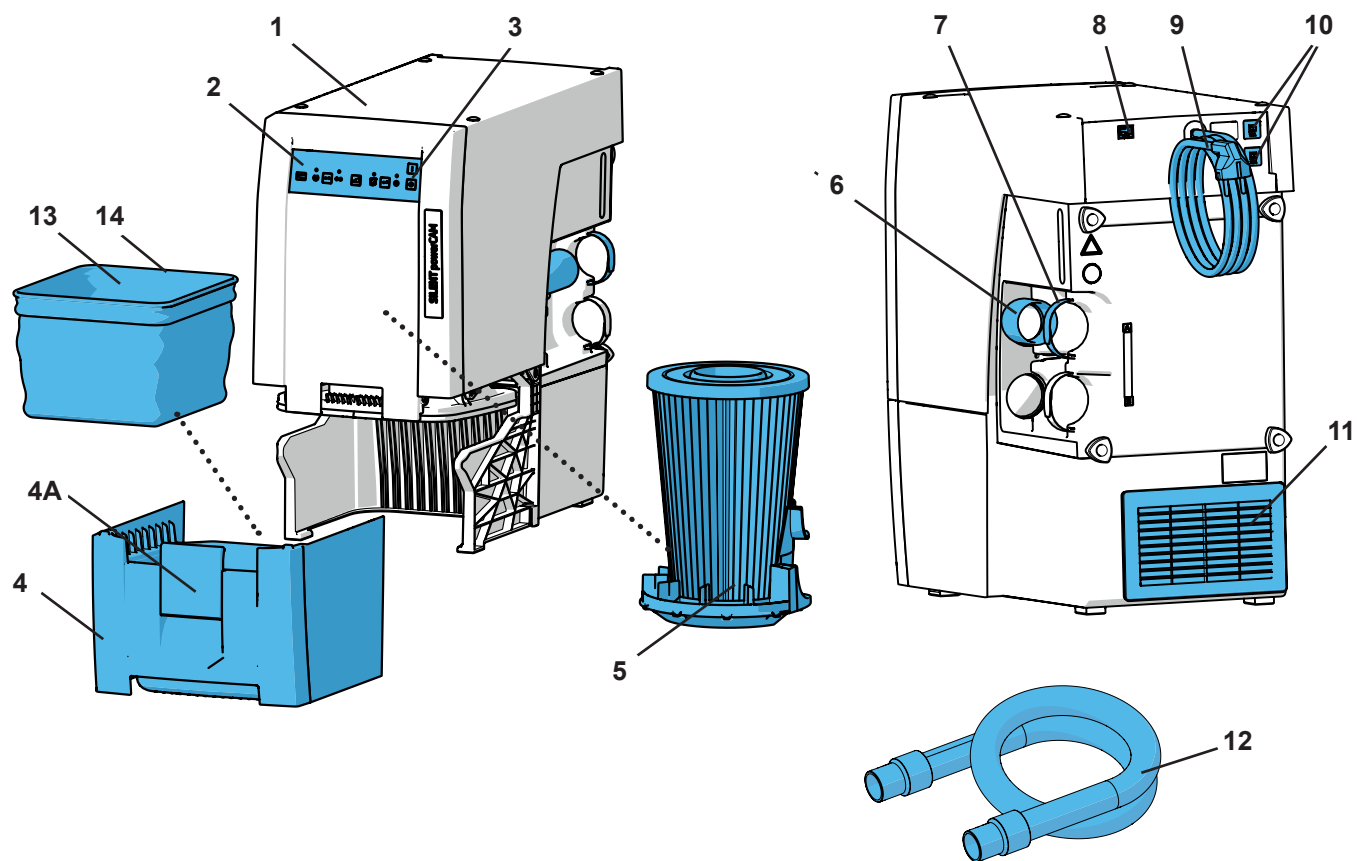
Fig. 1

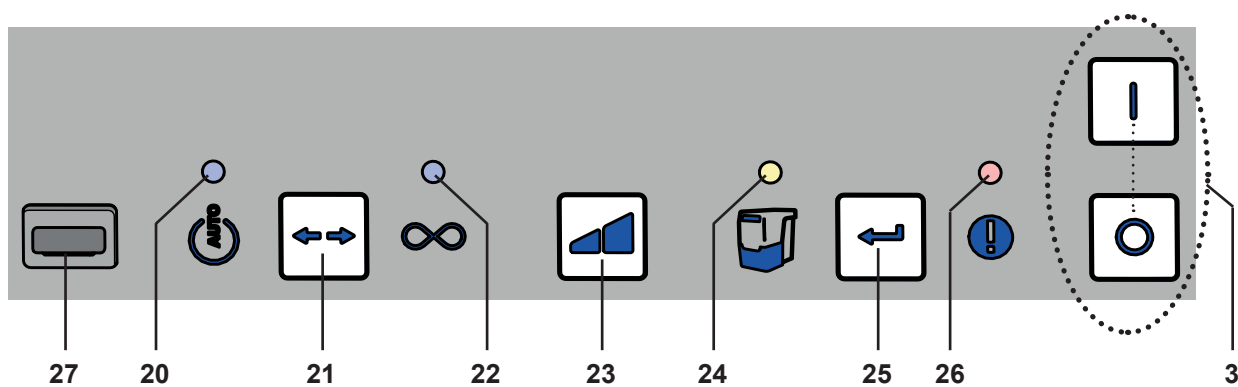
Fig. 2

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FAQ - English

For information on operating elements, see [Fig. 1](#) and [Fig. 2](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

EN

Trouble	Possible Cause	Corrective Action
An audible signal sounds, the extraction unit is switched off and filter cleaning takes place.	- The value has fallen below the internal limit for the flow velocity. - Filter cleaning takes place after 8 hours of operation (turbine running time) without being switched off in between times.	- Carry on working after the end of the cleaning process. - Switch equipment off at the end of each working day using the on / off switch (3, Fig. 1).
After it has been switched on, the dust drawer display (24, Fig. 2) lights up and an audible signal is sounded three times.	- The time period for emptying the dust drawer has elapsed and the dust drawer has not yet been emptied. - The emptying of the dust drawer has not yet been confirmed.	- Empty dust drawer and confirm this by pressing the enter key (25, Fig. 2). - Confirm that the dust drawer has been emptied by pressing the enter key (25, Fig. 2).
The error message (26, Fig. 2) blinks.	The electronic device has become too hot.	- Switch the device off and allow it to cool down. - Ensure sufficient cooling, e.g. with: - Chap. 4.2 Setup. - External ventilation (see chap. 4.6).
All 4 displays flash.	- Error in the electronics. - Suction turbine does not operate. - Leakage, because the dust drawer has not been inserted correctly.	- Contact Renfert / Service. - After changing the suction turbine, check the plug contact of the suction turbine. - Check the dust drawer is in the correct position.
The error message lights up (26, Fig. 2) and an acoustic signal occurs 2 x again.	The fine filter is so full, that the filter cleaning function alone cannot solve the problem.	Change the fine filter (see accessories and chap. 6.3).
The suction performance is insufficient.	- Selected extraction force too low. - There is a blockage or leak in the suction tube. - Dust drawer is not airtight. - The fine filter is full.	- Select higher suction level. - Check suction tube. - Please observe the points in chapter 4.4. - Check the dust drawer is in the correct position (see chap. 6.2). - Switch the unit off and back on in order to initiate the filter cleaning function. - If the filter cleaning does not improve the situation, change the fine filter (see chap. 6.3).
Dust drawer is over full.	The time period for "empty dust drawer" has been set too high.	Adjust to a lower time period (see chap. 5.5.1).
The signal to empty the dust drawer shows even though the dust drawer is not yet full.	The time period for "empty dust drawer" has been set too low.	Adjust to a higher time period (see chap. 5.5.1).
The display "empty dust drawer" blinks and an acoustic signal occurs 3 times.	The flow velocity is insufficient; the filter cleaning function is carried out.	Wait until the filter cleaning has finished.

Trouble	Possible Cause	Corrective Action
The suction process in the continuous operation or automatic operation stops abruptly and the relevant displays are still on (20 / 22, Fig. 2).	• The suction turbine is overheated. • The suction turbine is defect.	• Switch the unit off and allow to cool down for at least 60 min. • Check whether the suction tube is blocked. Eliminate blockage. • Switch the unit off and then on again, so that the filter cleaning process can be carried out. Change the fine filter (see chap. 6.3) (if the filter cleaning function does not improve the suction performance). • Change the suction turbine.
The suction unit does not respond to signals from the CAM system.	• Plug on CAM system or suction is not correctly inserted. • Interface cable damaged. • Plug assignment of the interface cable does not correspond to that of the CAM system.	• Check the plug on the suction system and the CAM system. • Check interface cable for damage, replace if necessary. • Check the connector assignment of the interface cable with that of the CAM. If necessary, use the correct interface cable.
After the suction motor was changed, the suction unit stopped functioning or stops suddenly.	• The plug from the suction motor is not sufficiently clicked in place.	• Connect the suction motor plug correctly. Ensure that the plug is fitted correctly and clicked into place by pulling it.

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