making work easy

[FAQ - Deutsch](#)

[FAQ - English](#)

[FAQ - Français](#)

[FAQ - Italiano](#)

[FAQ - Español](#)

[FAQ - Português](#)

[FAQ - Türkçe](#)

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FAQ - English

For information on operating elements, see [Fig. 10](#) and [Fig. 11](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

en

Error	Cause	Solution
Image cannot be focused.	<ul style="list-style-type: none"> Display switched to 3D and 3D glasses not used. 	<ul style="list-style-type: none"> Switch to 2D. Use 3D glasses.
Insufficient depth of field.	<ul style="list-style-type: none"> At 15x and 20x magnification, the depth of field is less than at 4x magnification. 	<ul style="list-style-type: none"> Reset to a 4x magnification.
Focus moves.	<ul style="list-style-type: none"> Head moves down on its own. 	<ul style="list-style-type: none"> Retighten the clamping screw for height adjustment (35, Fig. 11) (see Chapter 4.3.4).
Light cone moves.	<ul style="list-style-type: none"> Head folds down on its own. 	<ul style="list-style-type: none"> Retighten fastening screw (31, Fig. 10) (see Chapter 4.3.4).
No lighting.	<ul style="list-style-type: none"> The system is in stand-by mode. 	<ul style="list-style-type: none"> Move mouse or move hand under the cameras.
No network access to USB stick.	<ul style="list-style-type: none"> No network cable connected to the network extension cable. IP address has been changed. 	<ul style="list-style-type: none"> The network cable used to connect to a switch or router is not included. Check that the IP address displayed on the screen matches the one entered in the transfer program.
No 3D image can be seen.	<ul style="list-style-type: none"> 3D glasses not used. Monitor poorly aligned. 3D calibration changed. 4x magnification set. The 4x magnification does not allow for a 3D illustration. PC monitor is connected to EASY view+ 3D HDMI Splitter. 	<ul style="list-style-type: none"> Put on 3D glasses. Align the monitor so that the view+ is perpendicular to the monitor (see Chapter 4.3.3). Calibrate the display. See the description on the internet at www.renfert.com/p918. Enter the following item number: 2400 3000. Select a 15x or 20x magnification. When using EASY view+ 3D HDMI splitter, a 3D image can only be viewed on a Renfert 3D/2D monitor.
Head without function.	<ul style="list-style-type: none"> The EASY view+ 3D power supply unit was plugged into the foot switch connection. 	<ul style="list-style-type: none"> Connect the power supply unit and foot switch correctly, observing sections 4.4.1 and 4.4.2. If the head is still without function after correct connection, it was damaged by the previous incorrect connection. Contact Renfert Service.
Monitor displays image errors / stripes.	<ul style="list-style-type: none"> Cable plugs have come loose. Non-Renfert cable used. 	<ul style="list-style-type: none"> Check the cable and screw the DVI plug into the monitor. Use Renfert cable.
Head lowers by itself.	<ul style="list-style-type: none"> The fastening screw (31, Fig. 10) is not tightened tight enough. 	<ul style="list-style-type: none"> Retighten fastening screw.
Support arm can be swiveled away too easily or swivels away by itself.	<ul style="list-style-type: none"> The supporting surface of the universal clamp is uneven. 	<ul style="list-style-type: none"> Make the universal clamp even and level. Attach an adhesive tape strip to the aluminum pin of the support arm (under the black plastic sleeve).

Error	Cause	Solution
EASY view+ 3D HDMI Splitter		
No 3D image on a connected PC monitor.	<ul style="list-style-type: none"> • No error! On EASY view+ 3D HDMI Splitter, only 2D images are displayed on PC monitors. 	<ul style="list-style-type: none"> • Change EASY view+ 3D to 2D image view. • If a 3D display required, use a another Renfert 2D/3D monitor (Accessories).
No image on additional 2D device.	<ul style="list-style-type: none"> • 2D device is not fully HD capable. • Incorrect device settings. 	<ul style="list-style-type: none"> • Use full HD capable PC monitor. • Use the Renfert 2D/3D monitor (Accessories). • Check 2D device settings (e.g: 1080p60).
No image displayed on the connected TV device.	<ul style="list-style-type: none"> • No error! Only EASY view+ 3D Renfert 2D/3D monitors or PC monitors can be connected to the HDMI Splitter. 	<ul style="list-style-type: none"> • Use full HD capable PC monitor. • Use the Renfert 2D/3D monitor.
EASY view+ 3D Converter 2D/3D		
No 3D image can be viewed on the 2D/3D Renfert monitor.	<ul style="list-style-type: none"> • 2D / 3D connections are mixed up. 	<ul style="list-style-type: none"> • Check, that the Renfert 2D/3D Monitor is connected to the 3D port and the additional 2D device is connected to the 2D port.
No 3D image is displayed on the connected TV device.	<ul style="list-style-type: none"> • No error! Only 2D images can be displayed on TV devices. 	<ul style="list-style-type: none"> • If a 3D display is required on a different monitor, use the EASY view+ 3D HDMI Splitter together with a second Renfert 2D/3D Monitor (Accessories).
No image displayed on the connected 2D device.	<ul style="list-style-type: none"> • 2D device is not fully HD capable. • Incorrect device settings. • Incorrect signal input is set on 2D playback device. 	<ul style="list-style-type: none"> • Use a full HD capable 2D device. • Check 2D device settings (e.g: 1080p60). • Check settings, consult the 2D device manual if necessary.

**Note:**

For the various connection options of the EASY view+ 3D to different hardware devices, Renfert provides clear configuration operating instructions:

See description on the internet under www.renfert.com/p918.

Enter the following item number: 2400 3000.

Question	Answer
EASY view+ 3D General questions	
What is the power consumption of EASY view+ 3D?	37 watts incl. monitor (EASY view+ 3D 12 W, monitor 25 W).
Is there auto-focus on EASY view+ 3D?	<p>No.</p> <p>During development of EASY view+ 3D, various auto-focus solutions were tested and evaluated by Renfert dental technicians as well as market test customers. They found the feature to be quite problematic, so this option was not pursued further. The main reason is that in dental applications, objects are not only viewed but also worked with, putting them in constant motion. An auto-focus function means constant focusing of the object area to be viewed, which the testers found to be very stressful and annoying. EASY view+ 3D has a deep depth of field, ensuring high precision and sharpness when working with objects.</p>
Can I use any pair of 3D glasses?	<p>No.</p> <p>There are different polarization methods with different orientations on the market. The 3D glasses of EASY view+ 3D are precisely matched to the 3D monitor, ensuring flicker-free and smooth image display.</p>

Question	Answer
Can I connect my EASY view+ 3D directly to my PC?	No. In order to be able to access the images of the USB stick in the head of the EASY view+ 3D via the network, an IP address must be generated via the connection with a router or switch. To this end, a network cable (not included in the scope of delivery) must be plugged into the free network cable socket of the EASY view+ 3D and connected with a router or switch. Access is possible using free FTP transfer programs (for example, FileZilla) or directly via Windows Explorer or the Apple browser, Safari.
Can I use a WLAN stick directly?	No. Operation of a WLAN or Bluetooth stick is not supported by the program library of the EASY view+ 3D. The processor of EASY view+ 3D was designed for real-time transmission, and additional program space for corresponding drivers would have slowed down this process significantly.
Can I connect a wireless mouse?	During development of EASY view+ 3D, numerous wireless mice were tested. Unfortunately, none of them worked reliably and without errors, so Renfert does not offer this option.
When you capture a screenshot in 3D, is the photo also in 3D?	No. Photos in 3D mode are automatically converted to 2D and saved.
EASY view+ 3D lighting	
Is the lighting on EASY view+ 3D similar to daylight?	Yes. The daylight LEDs with approx. 5000 Kelvin provide you with an optimal view+ of the objects.
Can I replace the LED lighting of EASY view+ 3D on my own?	No. In the illumination of EV3D, Renfert uses the latest generation of high-quality LED lighting, featuring extremely high reliability and longevity. Should a failure nevertheless occur, our service centers will quickly help you.
Can I adjust or control the brightness of the LEDs?	No. The brightness of the LEDs is optimally matched to the two sensors, always ensuring optimal image reproduction. However, the image brightness of an object mode can be adjusted individually and according to the light and optical properties of the material using the mouse (mouse wheel press / turn).
EASY view+ 3D mounting	
What mounting options are there for EASY view+ 3D?	The scope of delivery includes a model base with universal clamp for the EASY view+ 3D support arm, and Renfert also offers a model base with screw-through attachment as an optional accessory.
How far away from the monitor can the system (support arm with head) be set up?	The support arm can be mounted at a distance of no more than 615 mm (measured from the center of the monitor).
EASY view+ 3D HDMI video recorder	
How can I record video sequences?	Using an HDMI video recorder (optional accessory), video sequences (including soundtrack) can be recorded easily and without complication in outstanding Full HD quality.
What is the max. recording time of the recorded video sequences?	Depending on the USB stick (file system format: FAT32), extended recordings are automatically split into video sequences of approximately 15 minutes (2 GB). That is, depending on the storage capacity of the USB stick used, long videos can also be recorded, with a new video sequence always being started and saved after approximately 15 minutes.
Can videos be recorded in 3D mode?	No. Videos can only be recorded in 2D mode. If you want to work in 3D mode for optimal hand / eye coordination, video recording in 2D format requires the optional EASY view+ 3D 3D/2D converter (optional accessory). In this case, the HDMI video recorder is connected to the 2D output of the converter.
Are video sequences stored directly in EASY view+ 3D?	No. The HDMI video recorder comes with a USB stick for storing the data which is plugged directly into the recorder.

Question	Answer
In what format are the videos stored?	Video format MPEG-4 --> file extension .mp4
Are the video sequences also recorded with sound?	Adding sound to video is possible using the headset included with the HDMI video recorder.
Connection to additional output devices (TV / projector / 2nd monitor / etc.)	
What type of monitor can I additionally connect to the EASY view+ 3D system? Can all Full HD / Ultra HD / 4K TVs common on the market be used as an additional output devices – for training and presentations, for example?	<p>With the EASY view+ 3D HDMI splitter (optional accessory), a 3D or 2D video signal can be sent to a second Renfert 2D/3D monitor. The signal is multiplied but remains unchanged. This can be used for training purposes, for example. For this purpose, the head is connected at the input of the HDMI splitter and one monitor is connected to each of the two outputs. When using a standard PC monitor, only a 2D display is possible.</p> <p>Important: EASY view+ Converter 3D/2D (optional accessory) is required for output to other 2D playback devices (not monitors) such as TV sets and projectors.</p> <p>Renfert tested numerous TV sets (Full-HD/UHD/4K) in conjunction with the EASY view+ Converter 3D/2D and the playback image quality of all devices was rated as good to very good.</p> <p>TV sets with Ultra HD or 4K picture quality can usually process the Full HD picture signal of the EASY view+ 3D without any problems, but Renfert recommends that you always check the operating manual of the respective TV set to be on the safe side.</p>
Accessing the USB stick via network	
Is it possible to connect to the EASY view+ 3D directly to a PC or notebook?	<p>No.</p> <p>In order to be able to access the images of the USB stick in the head of the EASY view+ 3D via the network, an IP address must be generated via the connection with a router or switch. To this end, a network cable (not included in the scope of delivery) must be plugged into the free network cable socket of the EASY view+ 3D and connected with a router or switch. Access is possible using free FTP transfer programs (for example, FileZilla), or directly via Windows Explorer or the Apple browser, Safari.</p>
Continuous operating mode for trade fairs, training courses, and presentations	
Is there a “continuous operating mode” for trade fairs or presentations so that the EASY view+ 3D does not switch to standby mode during extended breaks?	<p>Yes.</p> <p>To switch to “continuous operating mode,” proceed as follows: While holding down the left mouse button and holding down the mouse wheel, move the mouse and switch on the head, holding both down and moving until “standby disabled” is displayed on the lower right edge of the monitor (can take up to 20 sec.). If you switch off the head, this mode is automatically exited.</p>
EASY view+ 3D monitor	
What screen size does the monitor have?	21.5 inches; 460 x 270 mm
How sensitive is the monitor to metal, plastic, plaster particles and dust in the air?	The monitor is suitable for usual, typical laboratory dust levels at a dental technician workstation and was extensively tested in this environment.
How do you best clean the screen?	With a dry cloth; Renfert recommends a soft microfiber cloth.
EASY view+ 3D Software Updates	
Are there any software updates and how will I be informed?	The EASY view+ 3D comes with an information card that asks every customer to register. After registration, the customer will be automatically informed about software updates by e-mail. In addition, you can always find the latest software version for download on the Renfert homepage.

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