making work easy

FAQ - Deutsch

FAQ - English

FAQ - Français

FAQ - Italiano

FAQ - Español

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FAQ - Türkçe

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FAQ - English

For chapter indications, refer to the complete instruction manual on the internet.

Problem	Possible cause	Corrective action
No air and no abrasive.	- Blasting nozzle blocked at the hand-piece. - Foot switch connecting lines kinked.	- Unscrew the nozzle from the hand-piece and blow it out. - With the nozzle removed, blow out the blasting hose by activating the foot pedal. - Inspect/correct the connection lines.
Only air, no or very little abrasive.	- The dosage nozzle in the mixing chamber is blocked due to contaminated or moist sand. - Mixing chamber filter combination blocked. - Abrasive contains impurities or is moist.	- Cleaning the dosing nozzle, see chapter 5.4. - Change filter cartridge (Wrench SW14). - Replace the abrasive.
Unit blasts continuously.	- Foot switch connected incorrectly. - Valve in foot switch is defective.	- Inspect / correct the foot switch connection. - Replace foot switch.
Compressed air leaking from the foot switch.	Pneumatic hoses reversed.	Reverse the connections.
Foot switch fails to switch on.	- No pressure at the compressed air connection. - Pneumatic hoses are kinked.	- Inspect the compressed air lines and / or the compressor. - Reposition the hoses.

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