making work easy

de
en
fr
it
es
pt

ja

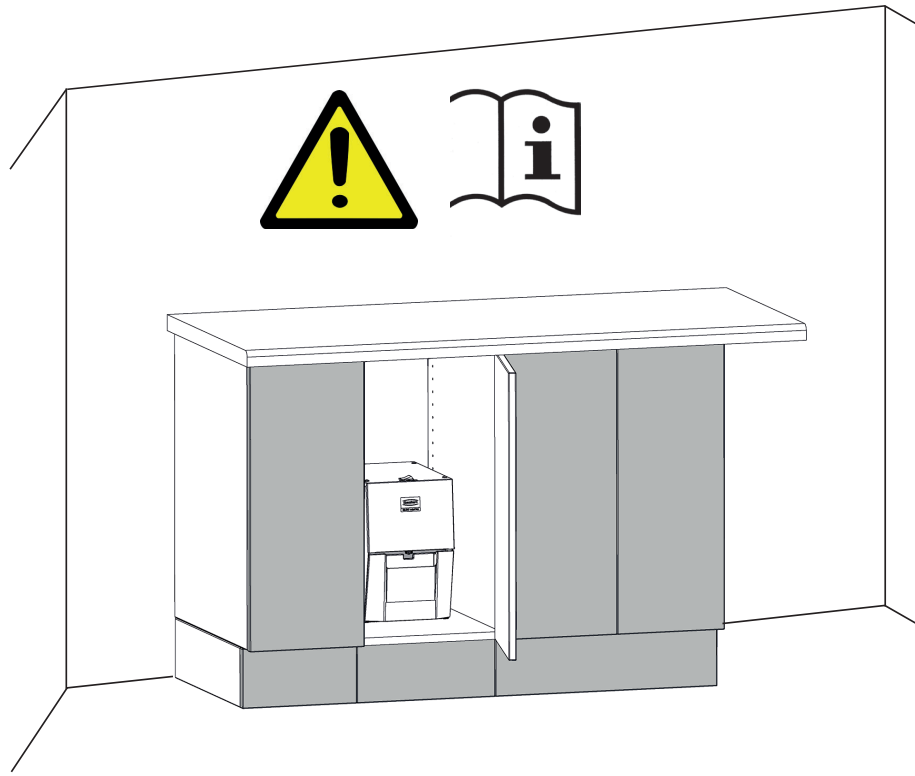
SILENT cubeCAM

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Made in Germany

20-5142 13112025

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Contents

en

1	Online help and information	3
1.1	Symbols used	3
2	Safety	3
2.1	Intended use	3
2.2	Meaning of these Instructions For Use (IFU)	3
2.3	Requirements for operators	4
2.4	Safe condition of the unit	4
2.5	Hazards as a result of modifications and repairs	4
2.6	Risk as a result of electric shock and fire	4
2.7	Risk to health as a result of dust	4
3	Product description	5
3.1	General description	5
3.2	Assemblies and functional elements	5
3.3	Scope of delivery	6
3.4	Accessories	6
4	Startup	6
4.1	Unpacking	6
4.2	Setting up the unit	6
4.3	Electrical connection	7
4.4	Connection to the suction point	7
4.5	Connection to the CAM interface	7
5	Operation	7
5.1	Mode selector switch	7
5.1.1	Off	7
5.1.2	Continuous operation	8
5.1.3	Automatic mode	8
6	Cleaning / Maintenance	8
6.1	Cleaning	8
6.2	Changing the dust bag	8
6.3	Changing the fine filter	9
6.4	Spare parts	9
7	Troubleshooting	9
8	Technical data	10
9	CAM interface	11
9.1	Pin assignment: CAM interface	11
9.2	Connector assignment: interface cable	11
10	Warranty	13
10.1	Exclusion of liability	13
11	Obligations regarding information	14
11.1	Disposal instructions	14
11.1.1	Disposal information for countries in the EU	14

1 Online help and information

www.renfert.com/silent-cubecam

en

Here you will find the following documents and information for download:

- Operating Instructions
- Quick Start Guide
- Spare parts lists and spare parts drawings
- Maintenance manual
- FAQs, cause and rectification of faults
- How-to videos
- Declarations of conformity

Please note: The documents and information listed are not available for all products.

1.1 Symbols used

In these Instructions For Use (IFU) or on the equipment you will find symbols with the following meanings:

Danger

Immediate risk of injury. Consult accompanying documents!

Electrical voltage

Danger due to electrical voltage.

Caution

Risk of damage to the equipment if the instruction is not observed.

Note

Provides useful information to make operation easier.

The equipment complies with applicable EU directives.

This product complies with the relevant UK legislation.

See the UKCA declaration of conformity on the Internet at www.renfert.com.

The equipment is subject to the provisions of the EU directive 2002/96/EC (WEEE Directive).

- List, special attention should be paid to
 - List
 - Subordinate list

⇒ Instructions / required action / input / operational sequence:

You are asked to perform the specified action in the given order.

- ◆ Result of an action / response by the equipment / response by the program:

The unit or program responds as a result of your actions or when a specific incident occurs.

Other symbols are explained as they occur in the text.

2 Safety

2.1 *Intended use*

The unit is intended for commercial use in dental labs or in-office labs in accordance with the instructions for use. The unit is intended for use in the following tasks:

- Extracting dry dust, non-explosive dusts on dental milling systems when milling commercially available milling blocks and blanks.

2.2 *Meaning of these Instructions For Use (IFU)*

These IFU will help you use the unit safely.

If these IFU are not observed, accidents may lead to life-threatening electric shock, injury, or material damage.

⇒ Only use the unit in accordance with these IFU.

⇒ Keep these IFU ready with the unit.

⇒ Provide these IFU to all subsequent users of the unit.

2.3 Requirements for operators

- ⇒ The product may only be used by persons aged 14 or over who are familiar with the procedures and rules in a dental laboratory.

2.4 Safe condition of the unit

Defective, broken or leaking components may cause injuries.

- ⇒ Check the power cord, housing, and other components such as connection cables, tubes, and the operating panel pad for damage, such as kinks, cracks, porosity, and signs of wear and tear.
- ⇒ Defective equipment must be taken out of service immediately. Disconnect the power plug. Send the unit in for repair or dispose of it safely. Until then, secure the unit to prevent it from being switched on again so that it is not used inadvertently.
- ⇒ Do not expose the unit to mechanical impact. Do not drop it.

2.5 Hazards as a result of modifications and repairs

Improper modifications and repairs to the unit may cause accidents, fire, and electric shock.

- ⇒ Only have the unit repaired by qualified specialist dealers or return it.

The use of unsuitable accessories or spare parts may result in unpredictable accidents and damage.

- ⇒ Only use accessories and spare parts supplied or approved by Renfert GmbH.

2.6 Risk as a result of electric shock and fire

Risk of electric shock and of fire as a result of excessive operating voltage or if there is no protective conductor.

- ⇒ Ensure that the power plug is easily accessible.
- ⇒ Only operate the unit using a power cord equipped with a plug appropriate to the local power supply. Only have the power cords changed by a qualified electrician.
- ⇒ Only operate the unit if the information on the type plate matches the power supply.
- ⇒ Only connect the unit to electrical sockets that are connected to the protective conductor system.
- ⇒ Do not extract flammable or explosive gasses, fumes or dust.
- ⇒ Do not extract hot materials.
- ⇒ The following applications involve increased risks and are therefore not permitted:
When cleaning milling chambers and SLM printing systems by suction, a sufficiently high concentration and purity (i.e., unmixed with other dental dusts such as plaster, resin) can result in spontaneous combustion or the milling dust or printing powder exploding because of a potential exothermic reaction (e.g., due to oxidation). An alternative cleaning method must be selected, particularly for the following milling or printing media (e.g., manual cleaning):
 - Wood
 - Titanium / titanium-aluminum
 - Light metals and light metal alloys (e.g., aluminum, magnesium)
 - Cobalt-chrome powder (e.g. for use in SLM systems)

If large quantities of light metals such as titanium alloy are processed (e.g., with sandpaper), producing very fine grinding dust, a sufficiently high concentration and purity may result in spontaneous combustion because of a potential exothermic reaction.

2.7 Risk to health as a result of dust

Dust emissions may result in damage to health.

- ⇒ If the dust extractor is employed to extract hazardous materials, appropriate personal protective gear must be worn and steps must be taken to ensure that the exhaust air is properly ventilated. Refer to the safety data sheets for the corresponding requirements.
- ⇒ When disposing of the extracted material or used filters, observe local provisions and accident prevention regulations!
- ⇒ Depending on the type of extracted material, wear appropriate personal protective equipment when emptying the dust drawer or cleaning.
- ⇒ For connection to a CAM system, observe the operating instructions for the CAM system as well as the safety instructions that they contain.
- ⇒ Please observe the Safety Data Sheets of the materials to be extracted.
- ⇒ Only extract with the dust drawer closed.

3 Product description

3.1 General description

The equipment is an extraction unit for suctioning dust that builds up in dental CAM systems.

It can be operated manually or controlled using a CAM system.

The suction power is fixed and cannot be changed.

The suction is ideal for daily use of up to 3 hours.

3.2 Assemblies and functional elements

- | | | | |
|----|--|---|---|
| 1 | Mode selector switch
Continuous / Off / Automatic | 5 | Mains connection |
| 2 | Dust drawer | 6 | Exhaust air filter / Exhaust air outlet |
| -- | Dust bag (inserted in dust drawer) | 7 | Suction port |
| 3 | Knurled nut fine filter holder | 8 | Power cord |
| 4 | CAM interface | 9 | Suction hose |

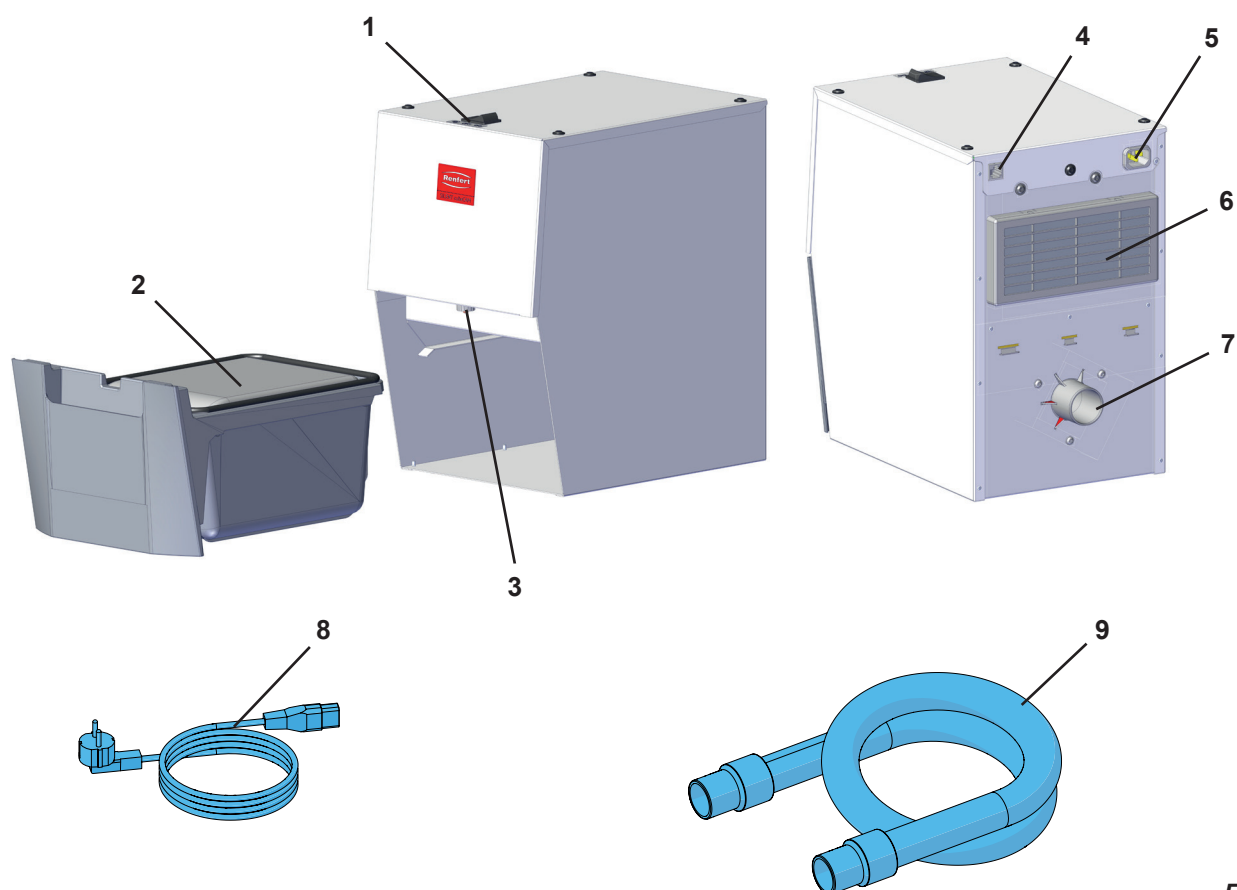
Fig. 1

3.3 Scope of delivery

- 1 SILENT cubeCAM
- 1 Dust bag (inserted ready for use in dust drawer)
- 1 Quick Start Guide
- 1 Power cord
- 1 Suction hose, 2 m, incl. 2 end bushings

3.4 Accessories

- 2921 0003 End bushing set, 2x
- 2934 0007 90° angled connector SILENT
- 90003 4240 Suction hose
- 90003 4826 Suction hose, antistatic, 3 m, incl. 2 end bushings
- 90115 0823 Suction hose, inside diameter 38 mm, 6 m
- 90215 0823 Suction hose, inside diameter 38 mm, 9 m
- 90003 4305 Adapter for hose connection
- 90003 4430 Universal suction hose adapter
- 90003 4314 Y-junction
- 2926 0000 Extractor switch
- 2934 0005 Interface cable, type A, for vhf
- 2934 0006 Interface cable, type B, for Roland DG
- 2934 0008 Interface cable, type C, for imes-icore
- 2934 0009 Interface cable, type D, for Amann Girrbach
- 2934 0010 Interface cable, type E, for Yenadent/Origin + Nema adapter
- 2934 0011 Interface cable, type F, for Zirkonzahn
- 2934 0012 Interface cable, type F, for Zirkonzahn + C14 adapter
- 2934 0013 Interface cable type G for vhf with 6-pin connection
- 2934 0016 Interface cable type H for Dentsply Sirona
- 2934 0017 Interface cable type I for Up3D
- 2934 0018 Interface cable type J for Canon
- 2934 0021 Interface cable type M for XTCERA
- 2934 0022 Interface cable type N for Aidite

4 Startup

4.1 Unpacking

- ⇒ Remove the equipment and accessories from the package.
- ⇒ Check the delivery for completeness (compare with the scope of delivery).

4.2 Setting up the unit

The extraction unit is a standing device. Do not operate it on its side.

Place the extraction unit so that:

- The exhaust air outlet (6, Fig. 1) is not blocked.
- The front is easily accessible for removing the dust drawer.

If the extraction unit is placed in a closed cabinet, the warm exhaust air must be transported out of the cabinet using one of the following measures:

- Opening in the rear panel of the cabinet, min. 250 x 120 mm, directly opposite the exhaust air outlet (6, Fig. 1).
 - Distance from the rear of the cabinet to the wall: min. 100 mm,
 - Distance from the extraction unit to the rear of the cabinet: max. 25 mm.
- Remove the rear panel of the cabinet, distance from the rear of the cabinet to the wall min. 50 mm.

If the warm exhaust air is transported out of the cabinet to the rear through openings, it must be ensured that the warm exhaust air can escape from there unhindered.

4.3 Electrical connection

Please note the information in chapter „2.6 Risk as a result of electric shock and fire“!

- ⇒ Set the mode selector switch (1) to the center position "OFF".
- ⇒ Plug the power cord (8, Fig. 1) into the power supply (5, Fig. 1).
- ⇒ Connect the power plug to an electrical socket of the building.

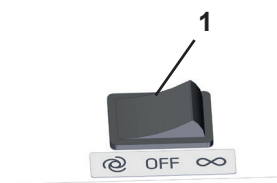
Fig. 2

4.4 Connection to the suction point

Caution: Risk of injury!

When shortening the suction hose, make sure to cut the integrated wire as straight as possible.

If the diameter is not suitable, use an adapter (see Chapter „3.4 Accessories“), to avoid loss of suction.

Long suction hoses, tight bends and kinks greatly reduce the suction power at the suction point.

Avoid steep inclines and "sagging" along the hose.

- ⇒ Plug the suction hose (9, Fig. 1) onto the suction port (7).
- ⇒ Connect the suction hose to the suction point of the CAM system. In doing so, observe the instructions for use of the CAM system.
- ⇒ Shorten the suction hose if necessary.

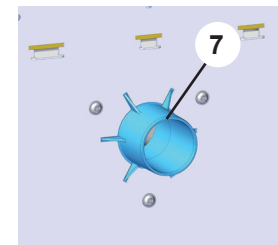
Fig. 3

4.5 Connection to the CAM interface

If the extraction unit is switched on and off too often using a power supply to the CAM system, this can damage both the electronics of the CAM system and of the extraction unit. For this reason, it is critical to use the CAM interface provided for the purpose in conjunction with a suitable interface cable to control the extraction unit (see Chapter „9 CAM interface“).

The electrical connection for communication with the CAM system is provided via the CAM interface (4) and an optional interface cable (see Chapter „3.4 Accessories“).

For assigning an available interface cable to the CAM systems, or for preparing your own interface cable, see Chapter „9.2 Connector assignment: interface cable“.

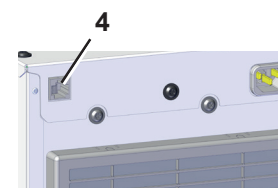
Fig. 4

Also observe the instructions for use of the CAM system.

5 Operation

Before beginning to work, check that a dust bag has been inserted. If operated without a dust bag, the intended protection against exposure to dust can no longer be guaranteed.

5.1 Mode selector switch

The mode selector switch (1) has three settings:

- OFF
Center position: OFF
- 
Pressed left: Automatic mode
- 
Pressed right: Continuous operation

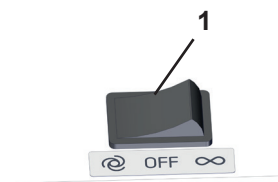
Fig. 5

5.1.1 Off

Mode selector switch in center position switches the extraction unit off. It can also not be switched on by the CAM system in this setting.

5.1.2 Continuous operation

In continuous operation, the extraction system runs regardless of whether a control signal is present at the CAM interface or not.

Continuous operation can be used, for example, to thoroughly clean and vacuum out the CAM system after a milling job.

5.1.3 Automatic mode

In automatic mode, the extraction unit runs depending on whether a control signal is active on the CAM interface or not.

6 Cleaning / Maintenance

Opening the unit is not permitted other than as described below!

Before cleaning and maintenance work:

- Switch off the unit and disconnect the power plug!

6.1 Cleaning

Do not steam clean the unit under any circumstances.

To clean the unit, only wipe the outside with a damp cloth.

Do not use any solvent-based or abrasive cleaning agents.

6.2 Changing the dust bag

NEVER operate the extraction unit without the complete filter system (dust bag + fine filter).

- ⇒ Pull the dust drawer out towards the front.
- ⇒ Leave the full dust bag in the dust drawer and take it with the dust drawer to the disposal point.
- ⇒ Remove the dust bag.
- ⇒ Remove the protective film, seal the filter opening to prevent dust escaping!
- ⇒ Dispose of the dust bag properly.

When disposing of the dust bag, observe local provisions and accident prevention regulations! Wear personal protective equipment depending on the material trapped in the filter.

- ⇒ Insert the new dust bag in the dust drawer. When doing so, make sure that the dust bag is properly inserted in the guide grooves and that the adhesive tab is facing upwards.
- ⇒ Check the seals of the dust drawer for damage and replace if necessary.
- ⇒ Insert the dust drawer straight until it is pulled into its final position by magnets. Make sure that the dust drawer is properly inserted in the guide.

6.3 Changing the fine filter

Disconnect the power plug before changing the fine filter.

NEVER operate the extraction unit without the complete filter system (dust bag + fine filter).

Check the fine filter regularly, replace it once a year.

There is a risk of increased exposure to dust when changing the fine filter.

That's why the following measures must be taken:

- ▶ Keep the plastic bag ready that contains the new filter.
 - ▶ Keep an additional vacuum cleaner or suction tube from another extraction unit ready.
 - ▶ Wear at least an FFP2 mask for respiratory protection.
- ⇒ Pull the dust drawer out straight towards the front.
- ⇒ Loosen the knurled nut (3), hold the bracket (20) and remove the knurled nut.
- ⇒ Slowly fold down the bracket (20).
- ⇒ Take out the fine filter (21), immediately put it in the plastic bag, and seal the bag tightly to prevent dust escaping.
- ⇒ Ensure proper disposal.

When disposing of the fine filter, observe local provisions and accident prevention regulations!

- ⇒ Place the new fine filter onto the bracket (20).
- ⇒ Swivel the bracket upward.
- ⇒ Position the knurled nuts and tighten hand-tight.

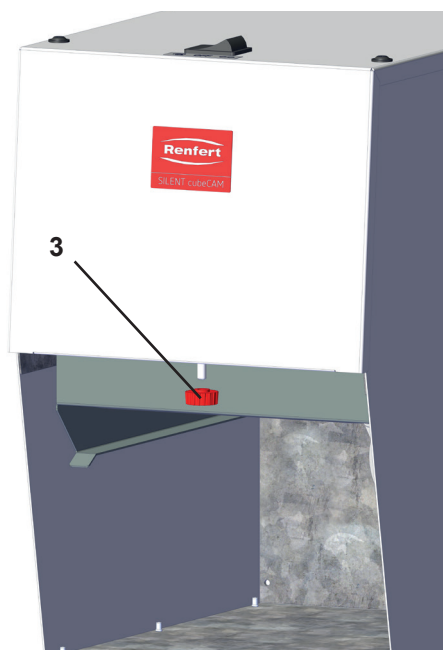
Fig. 6

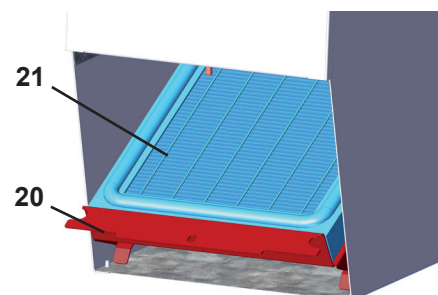
Fig. 7

6.4 Spare parts

You can find wear parts and spare parts in the spare parts list available online at, see chap. „1 Online help and information“.

The components excluded from the warranty (wear parts, consumables) are marked in the spare parts list.

The serial number, date of manufacturing and equipment version are shown on the nameplate of the equipment.

7 Troubleshooting

Trouble	Cause	Corrective action
The suction power is not sufficient.	• Blockage or leakage in the suction hose. • Dust drawer not airtight. • Dust bag is full, Filter clogged.	• Check suction hose. • Please also note the information in chapter „4.4 Connection to the suction point“ • Check that the dust drawer fits correctly (see chapter „6.2 Changing the dust bag“). • Check the dust bag and fine filter, change is necessary (see chapters „6.2 Changing the dust bag“ and „6.3 Changing the fine filter“)
Suction stops suddenly in continuous or automatic mode.	• Suction turbine has overheated. • Suction turbine is faulty.	• Switch off the unit and leave to cool for at least 60 minutes. • Check whether suction hose is blocked, eliminate blockage. • Check the dust bag and fine filter, change is necessary (see chapters „6.2 Changing the dust bag“ and „6.3 Changing the fine filter“). • When operating in a base cabinet, the ambient temperature must not exceed approx: 30 °C [86 °F] to ensure continuous operation. • Have the service team change the suction turbine.

8 Technical data

en

Item number:	2918 0000	2918 1000
Nominal voltage / V:	230	120
Permissible mains voltage / V:	220 - 240	100 - 120
Mains frequency / Hz:	50/60	
Power consumption / W [hp] **):	800 [1.07]	
Mains input fuse (internal):	10	
LpA ***) (at max. flow rate) / dB(A):	< 54	
LpA ***) (when idle) / dB(A):	< 54	
Number of suction channels:	1	
Flow rate, max. / l/min [cfm] ****):	2400 [84.8]	
Vacuum pressure, max. / hPa [psi] *****):	190 [2.76]	
Fine filter: - Filter surface, approx. / m² [sq inch]: - Filter quality:	0.43 [666] H13	
Fill volume dust drawer, approx. / l [US gal]:	4.5 [1.19]	
Ø Suction fittings / mm [inch]: - interior: - exterior:	33 - 34 [1.30 - 1.34] 38.5 - 40 [1.52 - 1.57]	
CAM interface control voltage / V:	3 - 32	
Dimensions (width x height x depth) / mm [inch]:	225 x 360 x 365 [8.6 x 14.2 x 14.4]	
Weight (empty) / kg [lbs]:	10 [22]	
Ambient conditions for safe operation		
Ambient conditions:	Indoors	
Installation height, max.:	2,000 m above sea level	
Ambient temperature:	5 - 40 °C [41 - 104 °F]	
Relative humidity:	Max. 80 % at 31 °C [87.8 °F], decreasing linearly to 50 % at 40 °C [104 °F] *)	
Voltage fluctuations in the mains power supply from the nominal value, max.:	10 %	
Contamination level:	2	
Over-voltage category:	II	
Ambient conditions for storage and transport		
Ambient temperature:	- 20 to + 60 °C [- 4 to + 140 °F]	
Maximum relative humidity:	80 %	

*) At between 5 - 30 °C [41 - 86 °F], the unit can be operated at a humidity of up to 80 %.
At temperatures between 31 - 40 °C [87.8 - 104 °F], the humidity must decrease proportionally in order to ensure operational readiness (e.g., at 35 °C [95 °F] = 65 % humidity, at 40 °C [104 °F] = 50 % humidity). The unit should not be operated at temperatures above 40 °C [104 °F].

**) Power consumption at nominal voltage

***) Noise level in accordance with EN ISO 11202

****) free blowing turbine at nominal turbine voltage

*****) at nominal turbine voltage

9 CAM interface

An electrically-isolated interface is provided for controlling with a CAM system.


The CAM system can switch the extraction unit on and off via control voltage (see chapter „8 Technical data“).

en

9.1 Pin assignment: CAM interface

Type: RJ45

Pin	Designation
1	
2	
3	
4	
5	
6	
7	In1 - GND
8	In1 - 24 V

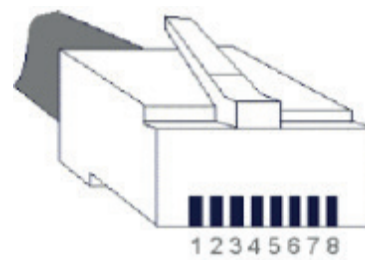9.2 Connector assignment: interface cable

Interface cable type A:


Extraction unit	CAM system
RJ45 connector	Lumberg SV40 (with thread locking system)
Pin 8	Pin 1 (+)
Pin 7	Pin 4 (-)

Rear side


Interface cable type B:

Extraction unit	CAM system
RJ45 connector	Stereo jack 3.5 mm
Pin 8	Pin 1 (+)
Pin 7	Pin 2 (-)

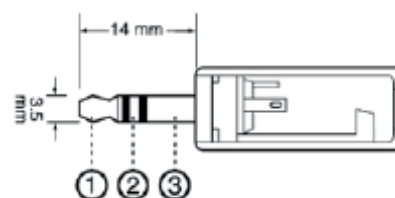
Interface cable, type C:

Extraction unit	CAM system
RJ45 connector	9-pin, D-Sub
Pin 8	Pin 4 (+)
Pin 7	Pin 3 (-)

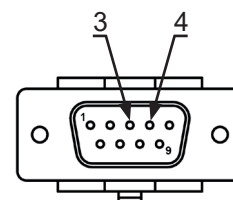
Interface cable, type D:

Extraction unit	CAM system
RJ45 connector	9-pin, D-Sub
Pin 8	Pin 3 (+)
Pin 7	Pin 4 (-)

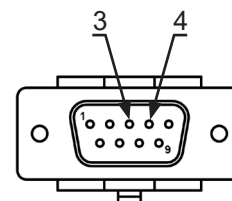Interface cable, type E:

Extraction unit	CAM system	Power supply unit
RJ45 connector	9-pin, D-Sub	
	Pin 9	24 V
Pin 8	Pin 8	
Pin 7		GND

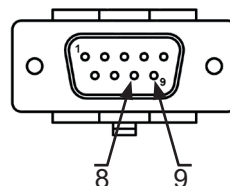To be used if the CAM system only provides one potential-free contact between pin 8 and 9.


Interface cable, type F:

Extraction unit	Power supply unit
RJ45 connector	
Pin 8	24 V
Pin 7	GND

To be used if the CAM system only provides mains voltage for controlling the extraction unit.

Interface cable, type G:

Extraction unit	CAM system
RJ45 connector	Lumberg SV60 (with thread locking system)
Pin 8	Pin 1 (+)
Pin 6	Pin 2 (+)
Pin 4	Pin 4 (+)
Pin 2	Pin 5 (+)
Pin 7	Pin 6 (-)
Pin 5	
Pin 3	
Pin 1	

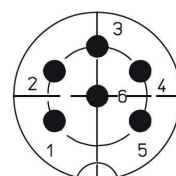
Rear side

Interface cable, type H:

Extraction unit	CAM system
RJ45 connector	15-pin, D-Sub
Pin 8	Pin 2 (+)
Pin 7	Pin 12 (-)
---	Pin 7 <-> Pin 10


Interface cable, type I:

Extraction unit	CAM system
RJ45 connector	5-pin Aviation connector
Pin 8	Pin 5 (+)
Pin 7	Pin 4 (-)

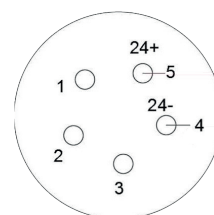
Interface cable, type J:

Extraction unit	CAM system	Power supply unit
RJ45 connector	2-pin TMW connector	
Pin 8	----	24 V
Pin 7	B	----
----	A	GND

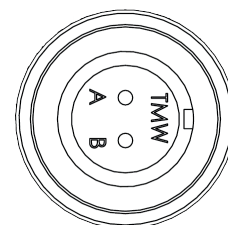
Interface cable, type M:

Extraction unit	CAM system	Power supply unit
RJ45 connector	4-pin round-plug connector	
---	1	24 V
Pin 7	---	GND
Pin 8	2	

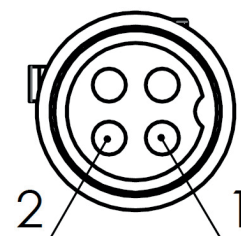Interface cable, type N:

Extraction unit	CAM system
RJ45 connector	2-pin round-plug connector (WEIPU)
Pin 7	2
Pin 8	1

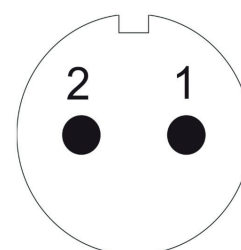10 Warranty

Renfert offers a **3-year warranty** on all parts of the equipment, provided that the equipment is properly used.

A 3-year warranty is provided for the suction motor, however with a maximum of 800 operating hours (motor running time).

Warranty claims may only be made upon presentation of the original sales receipt from the authorized dealer.

Parts that are subject to natural wear and tear (wear parts) and consumables are excluded from the warranty. These parts are marked in the spare parts list.

The warranty is voided in the case of improper use; failure to observe the operating, cleaning, maintenance, and connection instructions; in the case of independent repairs or repairs that are not carried out by an authorized dealer; if spare parts from other manufacturers are used, or in the case of unusual influences or influences not in compliance with the utilization instructions.

Warranty claims shall not extend the warranty period.

10.1 Exclusion of liability

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- The product is employed for any purposes other than those specified in the instructions for use.
- The product is altered in any way other than those alterations described in the instructions for use.
- The product is not repaired by an authorized dealer or if original Renfert spare parts are not used.
- The product continues to be used despite obvious safety defects or damage.
- The product is subjected to mechanical impact or is dropped.

11 Obligations regarding information

- Information on REACH and SVHC can be found on our website in the service area (www.renfert.com/service).

11.1 Disposal instructions

The equipment must be disposed of by a specialist facility. The specialist facility must be informed of any residue in the equipment that is harmful to health.

11.1.1 Disposal information for countries in the EU

To conserve and protect the environment, prevent environmental pollution, and improve the recycling of raw materials, the European Commission has adopted a directive that requires the manufacturer to accept the return of electrical and electronic equipment for proper disposal or recycling.

For this reason, equipment with this symbol may not be disposed of in unsorted domestic waste within the European Union.

Please contact your local authorities for more information on proper disposal.

Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...

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declare under our sole responsibility, that the product(s)

Name	Part No.
SILENT TS	2921 0050
SILENT TS2	2930 0050
SILENT TC	2935 0000 / 2935 0000 03
SILENT TC2	2936 0000 / 2936 0000 03
SILENT EC2	2937 0000 / 2937 0000 03
SILENT compact	2934 0000 / 2934 0000 03
SILENT compactCAM	2934 2000 / 2934 2000 03 2934 2500 / 2934 2500 03
SILENT powerCAM EC	2939 0000
SILENT powerCAM TC	2938 0000
iVAC silent	36 2934 2000

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Supply of Machinery (Safety) Regulations 2008

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN ISO12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...

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Renfert GmbH, Industriegebiet, 78247 Hilzingen / Germany
Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany