making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Polski
- FAQ - 中文
- FAQ - 日本語
- FAQ - 한국어
- FAQ - اللغة العربية
- FAQ - Українська
- FAQ - Română

SILENT TC

FAQ - frequently asked questions

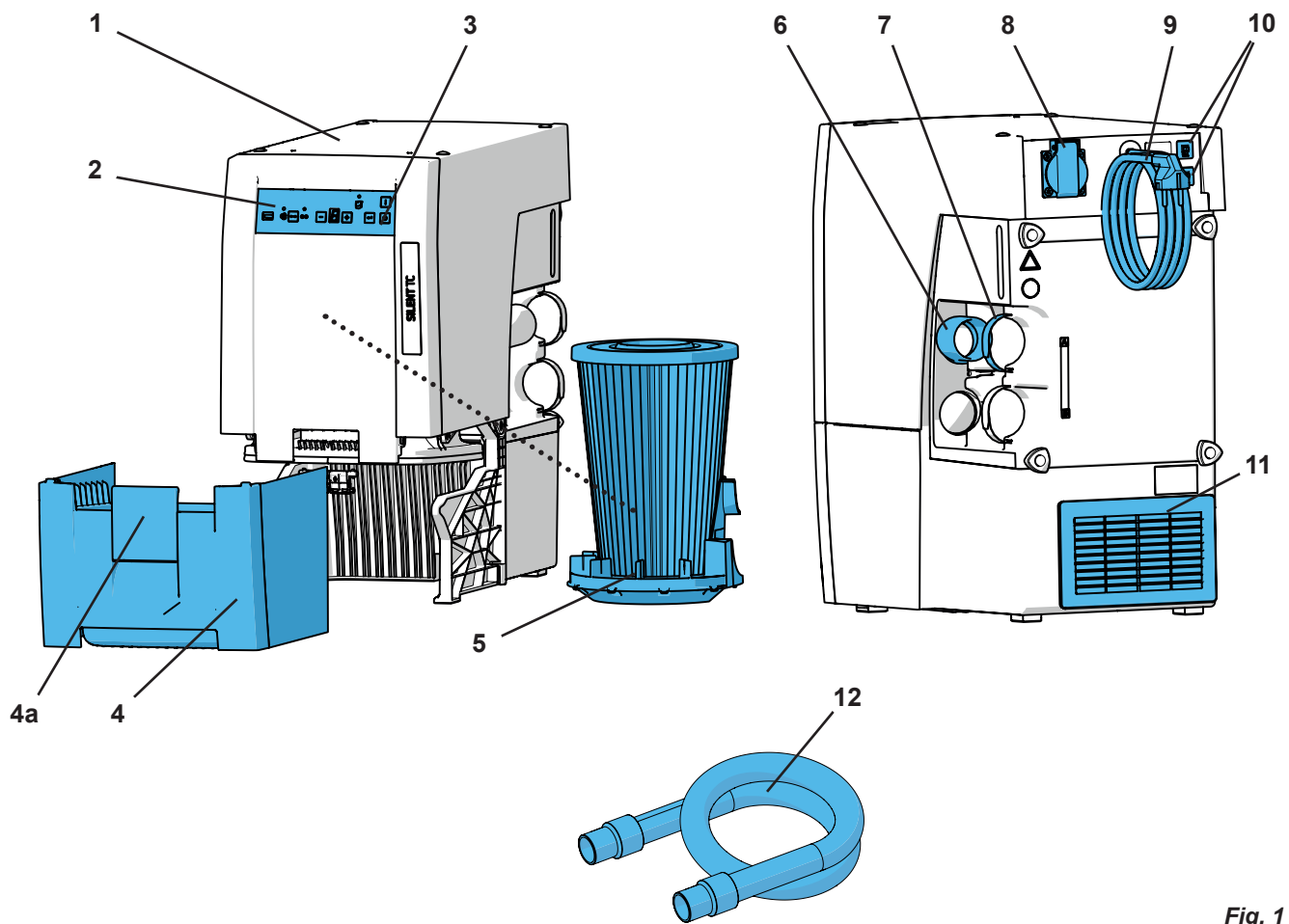
Fig. 1

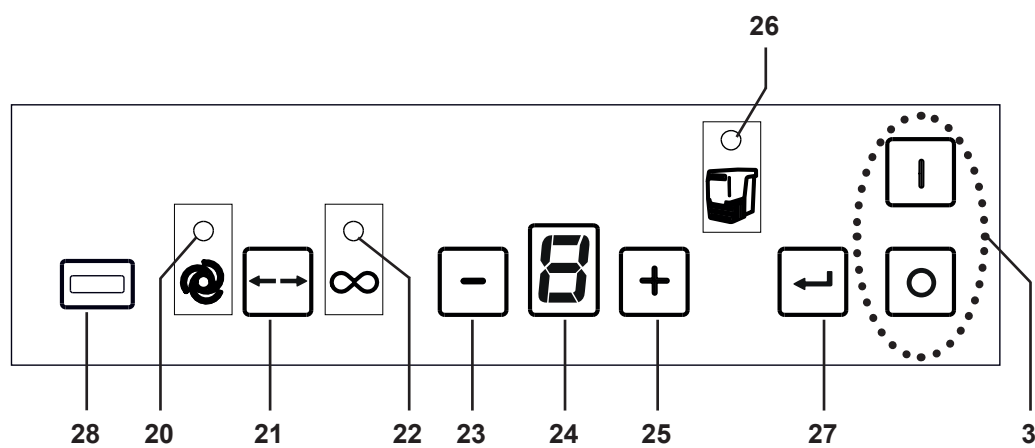
Fig. 2

FAQ - Deutsch

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FAQ - English

For information on operating elements, see [Fig. 1](#) and [Fig. 2](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

Trouble	Reason	Solution
When the device is switched on, there is a loud vibration noise for approx. 8 seconds.	The suction unit performs an automatic filter cleaning sequence.	This is functional and requires no intervention.
An acoustic signal is emitted, the suction unit is switched off and a filter cleaning is performed.	- The internal flow velocity limit was not reached. - After 8 hours of operation (turbine air time) without switching off, a filter cleaning is carried out.	- Repeat work after cleaning is finished. - Switch off devices at the end of each working day at the on / off switch (3, Fig. 1).
After switching on, the display shows "Dust drawer" (26, Fig. 2) and a 3 x acoustic signal is heard.	- The time interval for emptying the dust drawer has expired and the dust drawer has not yet been emptied. - The emptying of the dust drawer has not yet been confirmed.	- Empty Dust Drawer and confirm by pressing the Enter key (27, Fig. 2). - Confirm emptying the Dust Drawer by pressing the Enter key (27, Fig. 2).
The display shows "C".	- The automatic switch-on function cannot be set. At the selected rotational speed on the handpiece, the difference between stand-by current and operating current is too low. - The automatic switch-on function cannot be set.	- Adjust the switch-on threshold to a higher speed on the handpiece. - Device in permanent operation.
The display shows "H".	The electronics have become overheated.	- Switch off the device and let it cool down. - Provide adequate cooling, e.g. - Observe device set-up in chapter 4.2. - Use the external exhaust function (see chapter 4.6). - Change the Fine Filter (see Accessories and chapter 6.3).
The display shows "E".	- Leakage: Dust drawer not airtight. - Faults in electronics. - The suction turbine is not running.	- Check that the Dust Drawer fits correctly (see chapter 6.2). - Contact Renfert / Service. - When the suction turbine has been changed, check the plug-in contact at the suction turbine.
The display shows "F".	The fine filter is so heavily clogged that the filter cleaning does not show sufficient effect.	Change Fine Filter (see chapter 6.3).
The suction performance is insufficient.	- Suction performance is set too low. - Blockage or leakage in the Suction tube. - Dust Drawer not airtight. - Fine filter blocked.	- Select a higher suction level. - Check suction tube. - Please note the information in chapter 4.4. - Check the correct fit of the Dust Drawer (see chapter 6.2). - Clean the filter. - Switch the unit off and then on again, so that the filter cleaning process can be carried out. Change the fine filter (see chap. 6.3) (if the filter cleaning function does not improve the suction performance).
Dust Drawer over full.	Time interval to the "Empty Dust Drawer" has been set too high.	Set a shorted time interval (see chapter 5.5.2).

Trouble	Reason	Solution
The signal for emptying the dust drawer appears, even though it is not yet full.	The set time interval to the "Empty Dust Drawer" is too low.	Set a larger time interval (see chapter 5.5.2).
Suction starts even though the electrical appliance connected to the socket is not being used.	The automatic switch-on function has been set too low.	Setting the automatic switch-on (see chapter 5.5.3).
Suction does not stop when the electrical appliance is switched off.	The automatic switch-on function has been set too low.	Setting the automatic switch-on (see chapter 5.5.3).
Suction does not start even though the electrical appliance connected to the socket is in used.	The automatic switch-on function has been set too high.	Setting the automatic switch-on (see chapter 5.5.3).
Fuse triggers when a connected electrical device is switched on.	Power consumption of the connected device too high.	Observe the maximum connection value (see chapter 8).
The suction stops abruptly in automatic or continuous operation and the associated displays (20 / 22 , Fig. 2) are still on.	- Suction turbine is overheated. - Suction turbine is defect.	- Switch off the device and leave to cool for at least 60 min. - Check whether suction tube is blocked, eliminate blockage. - Switch the device off and then on again to perform a filter cleaning. Change fine filter (see chapter 6.3) if filter cleaning does not improve the performance. - Change Suction turbine.
After the suction motor was changed, the suction unit stopped functioning or stops suddenly.	The plug from the suction motor is not sufficiently clicked in place.	Connect the suction motor plug correctly. Ensure that the plug is fitted correctly and clicked into place by pulling it.

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