lay:art Anmischplatte natural

No. 1046 0000

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Instruction manual • Mode d'emploi

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ENGLISH

Instruction Manual

Congratulations, and thank you for choosing the high quality and state-of-the-art porcelain mixing palette system „*lay:art*“ by Renfert.

EN

1. Description

The system consists of a high quality plastic case with a lid and an agate mixing palette.

1.1 Components and functional elements

- | | |
|------------------|--------------------------|
| 1 Base | 6 4 x Rest plugs (tall)w |
| 2 Lid | 7 Non-permanent marker |
| 3 Mixing palette | 8 4 x Distance disc |
| 4 Labelling foil | 9 4 x Rest plugs (short) |
| 5 4 x Case feet | |

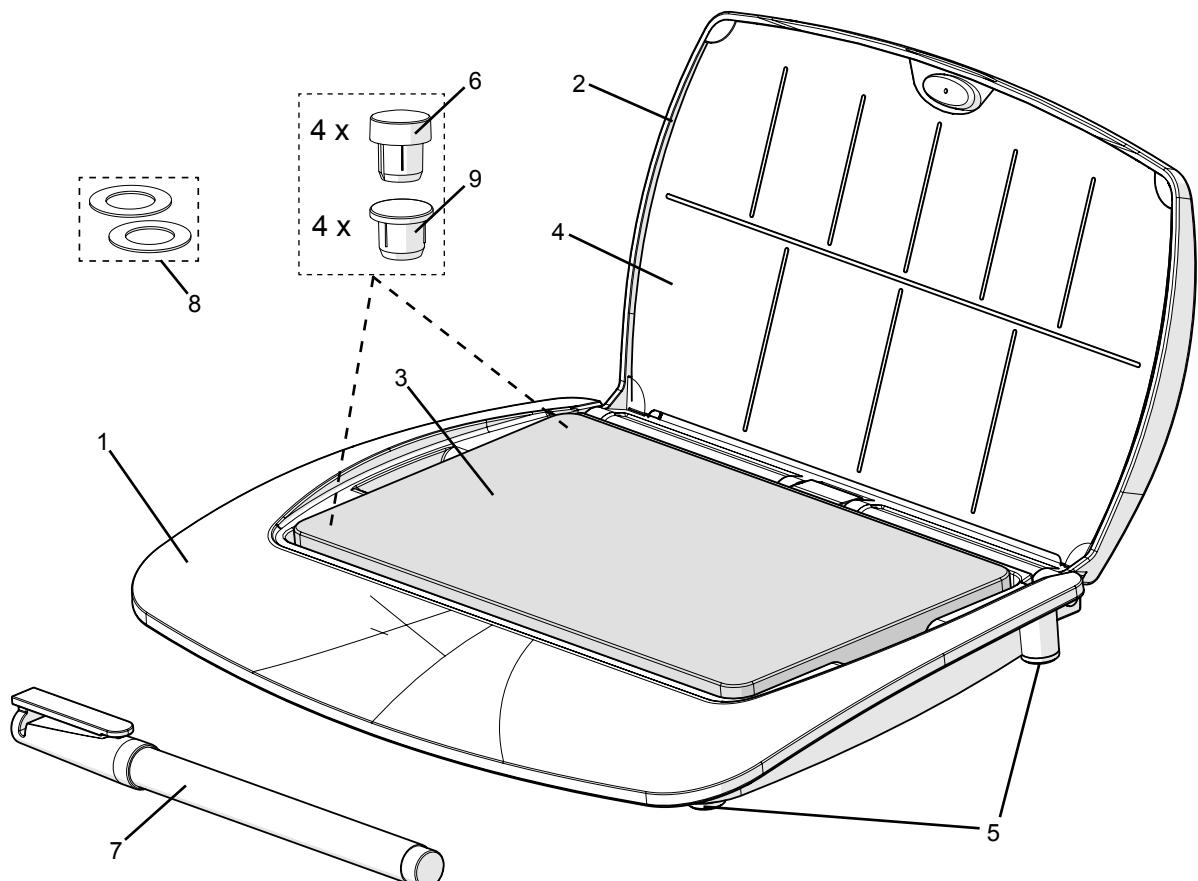
Fig. 1

2. Functional description of the case

When the lid is opened in a 90° angle it can be easily removed from the case base.

Gently press the onto the base (1) and pull the lid (2) upwards.

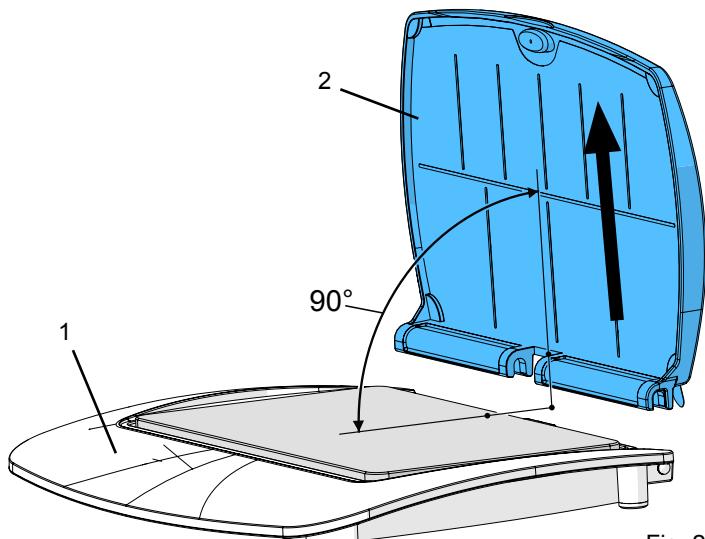
Fig. 2

At an angle of 115° there is a lid locking position, which will hold the lid open in a stable position.

Caution:
If this position is overstretched, the hinges could become damaged.

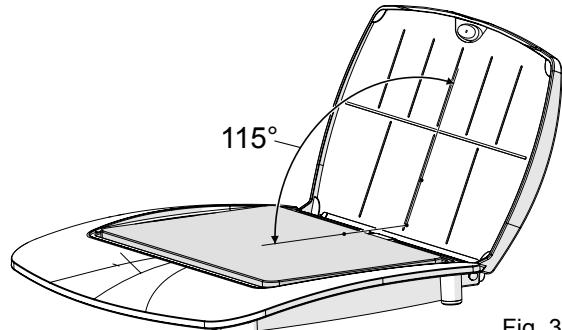
Fig. 3

3. Labelling foil

The non-permanent marker pen provided can be used to write on the labelling foil (4). The foil can be removed from the lid in order to exchange or inscribe the label.

Use the matt side of the foil to write on. When replacing the foil, clip behind the 4 mounting points which can be found on the lid. Firstly, bend the retaining flaps towards the matt surface (a).

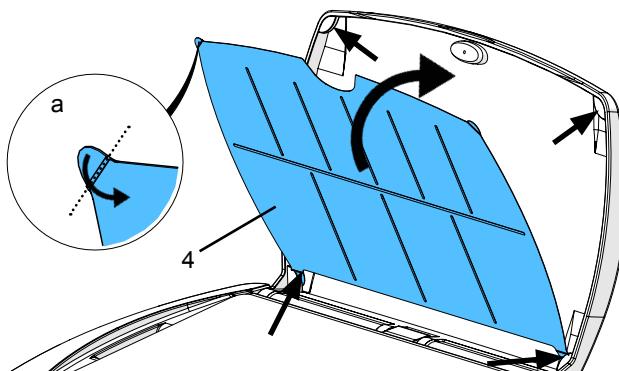
Fig. 4

4. Preparation for using the agate mixing palette

You have purchased a unique product.

Every agate mixing palette has its own individual and natural grain or colouring, which can differ slightly from the photo of the product.

Natural stone palettes are susceptible to discolouration, especially when mixing liquids with an acid content are used.

An agate palette is a hand crafted, natural product which can have divergencies in palette thickness from the production methods.

Due to this fact, we recommend using the pre-assembled shorter rest plugs (9) on thicker palettes. On thinner palettes, we recommend exchanging the rest plugs with the separately supplied taller rest plugs (6, Fig. 1).

If the palette should still wobble slightly when in position, please use the distance discs (8) to level it out.

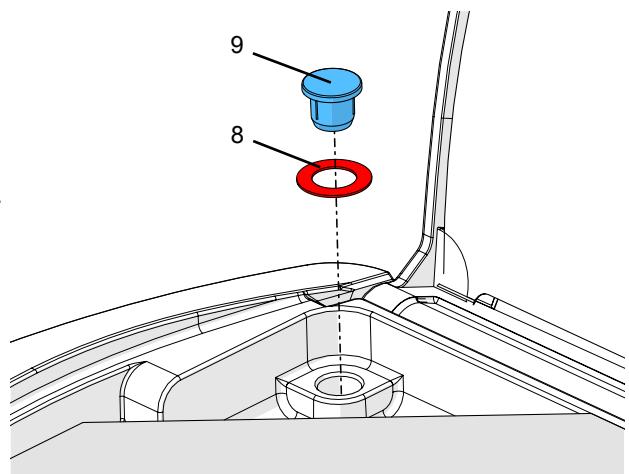
Fig. 5

EN

5. Care instructions

In order to maintain the consistent quality of the agate mixing palette, it is essential to clean on a regular basis.

To clean, take the agate mixing palette out of the case base and brush under running water, or clean carefully using a steam cleaner.

Clean the case using conventional detergent and then rinse thoroughly under running water.

Do not clean using sharp edged, abrasive or solvent based materials.

Please only use soft brushes or cleaning cloths.

When using Tricolor stains it may happen that vaporized stains fluid etches the plastic lid.

Do not clean the plastic lid with monomers or with liquids containing solvents!

6. Spare parts

1046 0100 Labelling foil S, 3 pcs.

9 1045 9991 Housing lay:art small

9 0003 0038 Case feet, 4 pcs.

9 1046 0001 Small part set lay:art natural (consisting of the Pos. 6, 8 and 9)

7. REACH

Information on REACH and SVHC is available on our website www.renfert.com, in the Support area.

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Renfert GmbH • Industriegebiet • 78247 Hilzingen/Germany