
making work easy

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- zh
- ja
- ko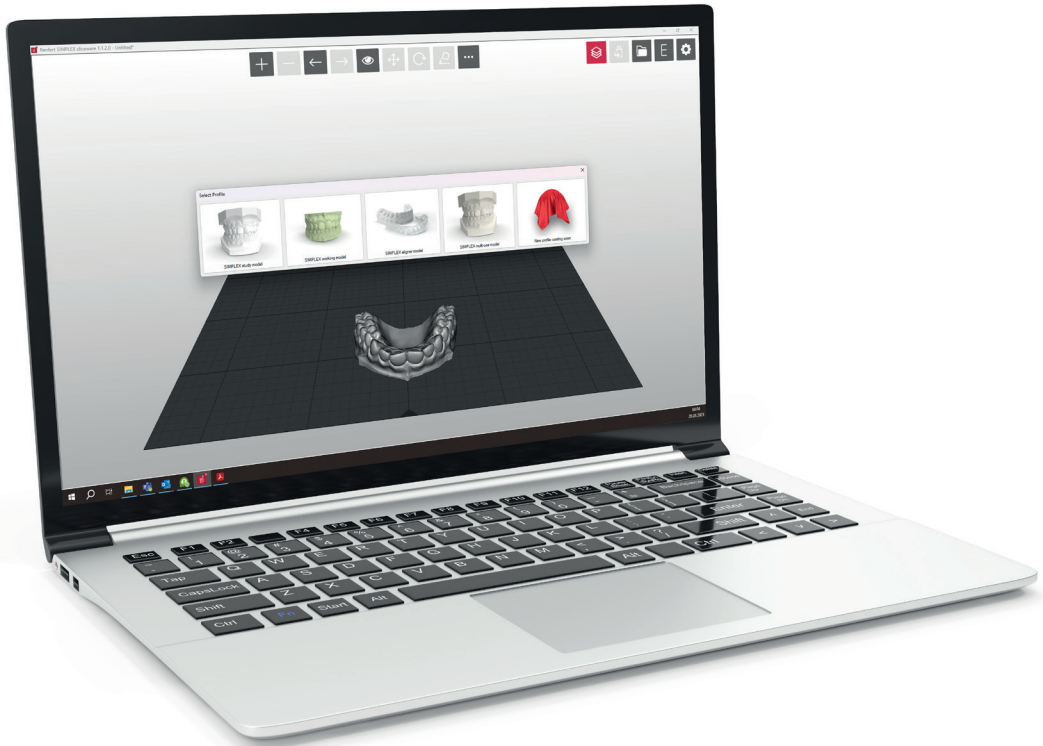

SIMPLEX sliceware

FAQ - frequently asked questions

Made in Germany

21-9052_04042025

FAQ - Deutsch

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FAQ – English

This FAQ refers to software version 1.1.3 and higher.

If you are still using an earlier version, you should update to the current version.

This can be done free of charge via myRenfert.

<https://myrenfert.renfert.com/>

The complete instructions can be found online at

<https://www.renfert.com/produkte/geraete/3d-filamentdruck/simplex#downloads>

Question / Error / Problem	Cause	Troubleshooting / Solution
What is the minimum required height (in mm) of the model base above the palate in order to facilitate an even surface during printing?		<ul style="list-style-type: none">• The model base must be approx. 3 - 4 mm above the palate.• When slicing the working model, approx. 2 mm of the SIMPLEX sliceware is removed. Remaining below this distance can result in holes.
When placing models with a thick model base, is it possible to reduce the size of the model by placing the model base slightly below the print bed?		<ul style="list-style-type: none">• Yes, this is possible. Everything that is positioned below the virtual plate is shown in orange in the SIMPLEX sliceware. These orange areas are not taken into consideration during printing.
Is the SIMPLEX sliceware only available after the customer has registered using the Renfert app?		<ul style="list-style-type: none">• Log in now on the Internet via myRenfert instead of the Renfert app.• You will receive the SIMPLEX sliceware software after logging in to myRenfert and registering your SIMPLEX 3D filament printer.
What distance is recommended between print elements? What is the recommended minimum distance?		<ul style="list-style-type: none">• The distance between the printed objects should be as small as possible in order to minimize travel times of the extruder and thus the printing time, but should be at least 5 mm.• The distance to the edge of the print bed should also be 5 mm.
Can the customer use the SIMPLEX sliceware on multiple computers, for example in a large laboratory or school?		<ul style="list-style-type: none">• Yes, the customer receives the installation program as a ZIP file. As a result, the SIMPLEX sliceware can be installed on multiple computers in a laboratory / school.
What requirements must a PC meet for downloading our printware, firmware, and sliceware?		<ul style="list-style-type: none">• Currently the sliceware is only available for Windows 10 or higher.
Registration with the Renfert CONNECT app.		<ul style="list-style-type: none">• You are now registering your SIMPLEX 3D filament printer on the Internet via myRenfert and no longer via the Renfert app.• After registration you can download and install the software package SIMPLEX print and SIMPLEX sliceware free of charge.

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