making work easy

FAQ - Deutsch

FAQ - English

FAQ - Français

FAQ - Italiano

FAQ - Español

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FAQ - Türkçe

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FAQ - English

For information on operating elements, see [Fig. 1 / 2](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

EN

Error	Possible Cause	Corrective Action
Extractor stops suddenly for no apparent reason and a error code is indicated in the display.	An error has been detected.	- See table "Error Codes" - If the instructions are of no help to you, or the error code is not listed: - Make a note of the error code. - Send the device in for repair. - Inform the repair service which error code was noted.
Dust bag over full.	Sensitivity of "dust bag full" signal is too large.	Set a smaller value (See chapter 5.3.3).
"Dust bag full" indicator goes on, even though the bag is not full.	Sensitivity of "dust bag full" signal is too low.	Set a larger value (See chapter 5.3.3).
Suction hose cannot be closed by pressing the suction hose button.	An electrical appliance which is connected to the suction hose is still in operation.	Switch the connected electrical appliance off.
"Dust bag full" indicator appears shortly after having replaced the dust bag.	- Fine particle filter is blocked. - Suction hose blocked. - Cross-section of the dust generating device's hose fitting is too small.	- Replace the fine particle filter (Chapter 6.2.1). - Remove the obstruction in the suction hose. - Reset the sensitivity of the "dust bag full" indicator (Chapter 5.3.3).
Extractor fails to start when a dust generating device is being used.	- Extractor is not switched on. - Device fuse has blown. - Dust generating device is not connected to the extractor socket. - Start-up threshold of the suction hose is too high.	- Switch the extractor on (Chapter 5.1). - Check and reinstall the fuse if necessary (Chapter 6.4). - Connect the dust generating device to the coupler socket (Chapter 4.7). - Adjust start-up threshold (Chapter 5.3.4).
Extractor starts, although a connected device is not being used.	Start-up threshold of one suction hoses is too low.	- Check, which suction hose is open, the relevant letter will be indicated in the display. - Adjust the start-up threshold of the suction hose (Chapter 5.3.4).
Extractor fails to stop when a connected device is switched off.	- The dust generating device is not plugged in to the coupler socket on the extractor and the suction hose button is switched on. - Start-up threshold is too low. - Shut-off delay is too long.	- Switch the suction hose button on the extractor off, and connect the dust generating device to the coupler socket on the extractor (Chapter 4.7). - Adjust start-up threshold (Chapter 5.3.4). - Set shorter shut-off delay (Chapter 5.3.5).
No satisfactory setting for the automatic start-up feature can be adjusted.	Some older hand pieces do not generate a sufficiently strong signal for the automatic start-up feature.	Switch to higher revs per minute (e.g. 10,000 rpm.).
The suction switches off and on in an uncontrollable manner when the hand piece is activated.	The hand piece is being used at a lower speed than the rpm to which the automatic switch-on is set.	The hand piece must at least be used at the speed at which the automatic switch-on is set.
Panels covering the fine particle filter and the dust drawer are difficult to open.	Lock bolts are dirty.	Clean lock bolts and lubricate them slightly if necessary.

Error	Possible Cause	Corrective Action
The extraction force is not readjusted when a second suction hose is switched on or off.	• The automatic extraction force re-adjustment is deactivated.	• Activate the automatic extraction force readjustment (See chapter 5.2.2).
The display indicates “OFF” above the button  (41).	• The automatic extraction force re-adjustment is deactivated.	• Activate the automatic extraction force readjustment if required (See chapter 5.2.2).
There is no audible signal.	• The audible signal is deactivated.	• Activate audible signal (Chapter 5.3.2).
Pinch valves close / open slowly.	• The filter in the compressed air connection is blocked.	• Change the filter in the compressed air connection (see chapter 6.2.4).

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