making work easy

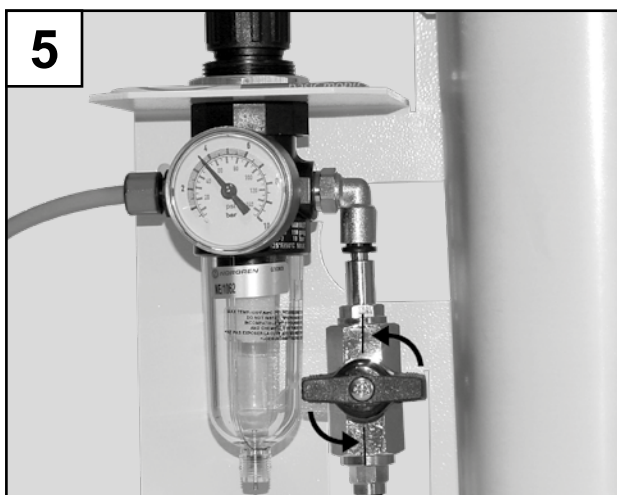
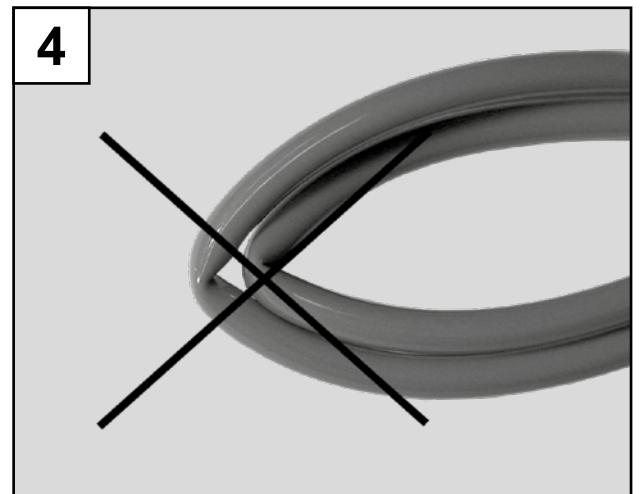
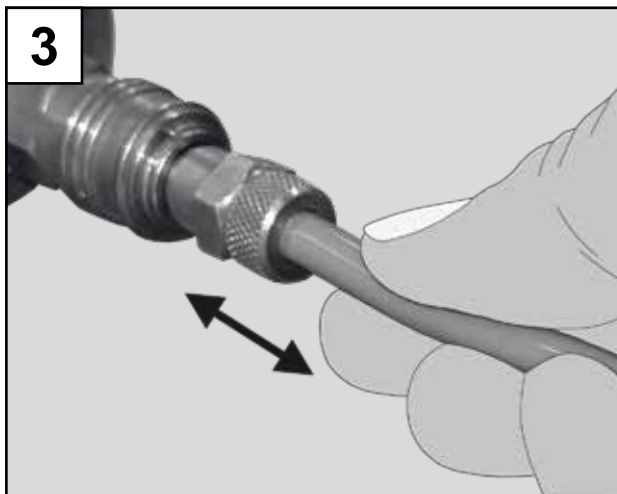
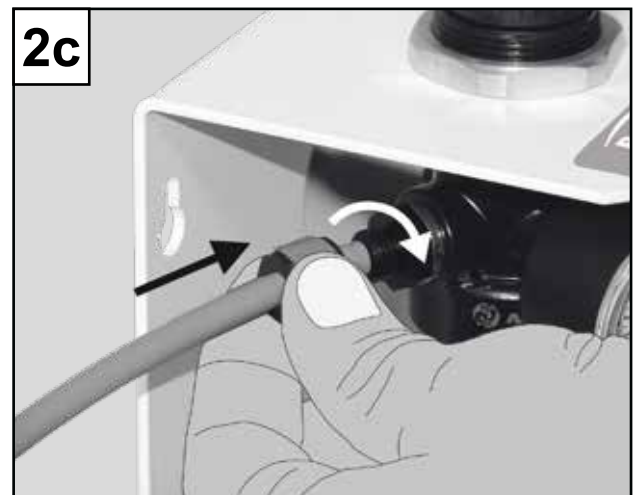
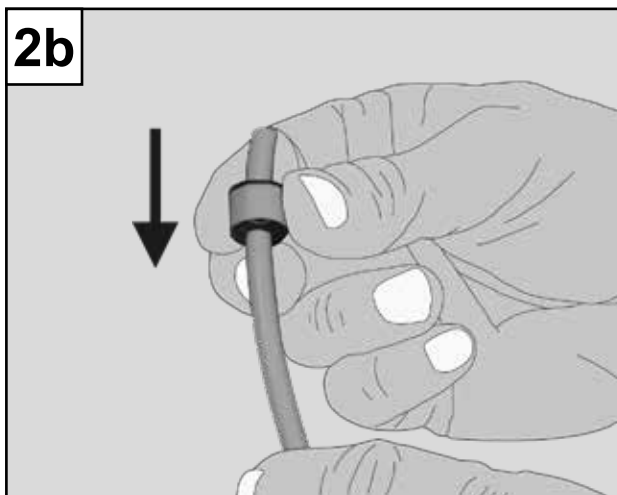
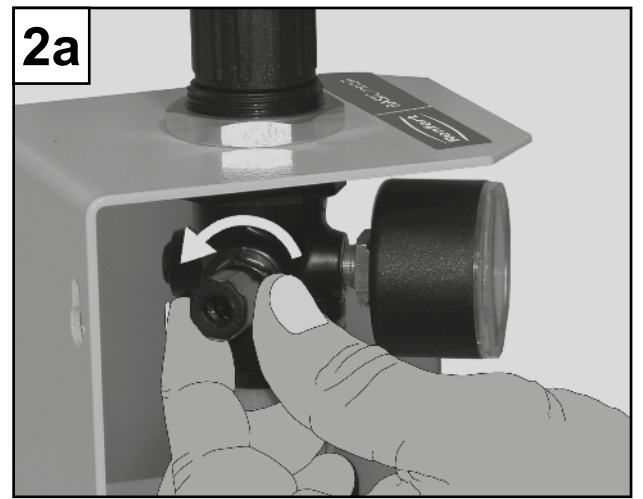
- de
- en
- fr
- it
- es
- pt
- tr
- ru
- uk
- zh
- ja
- ko
- ar

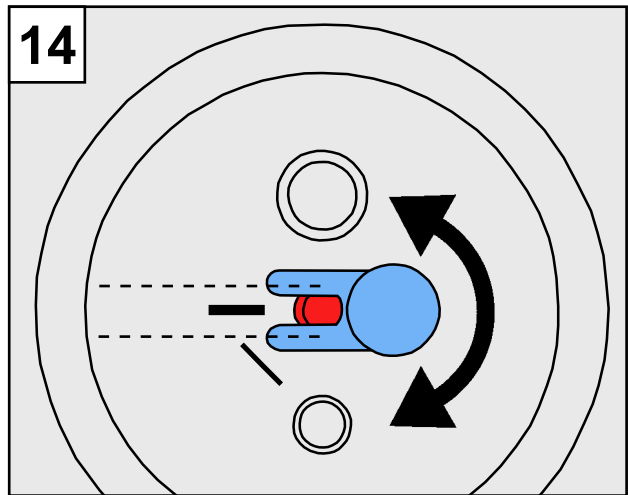
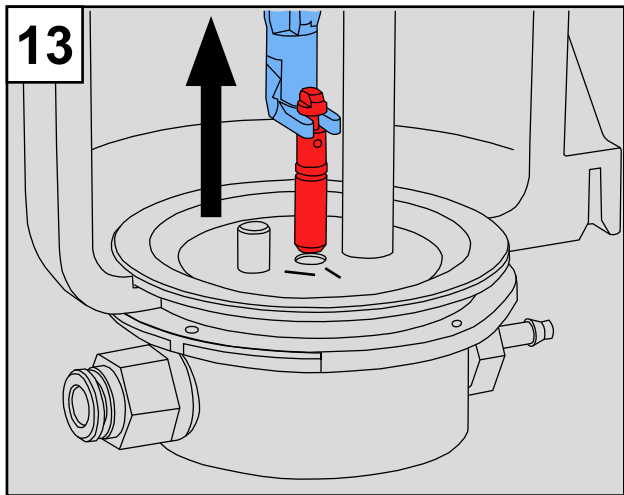
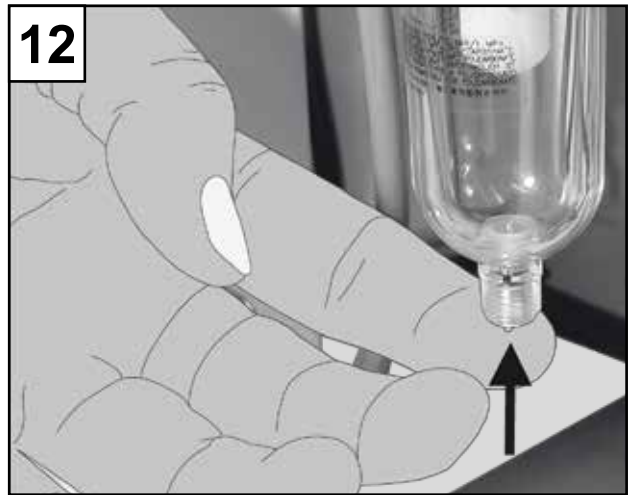
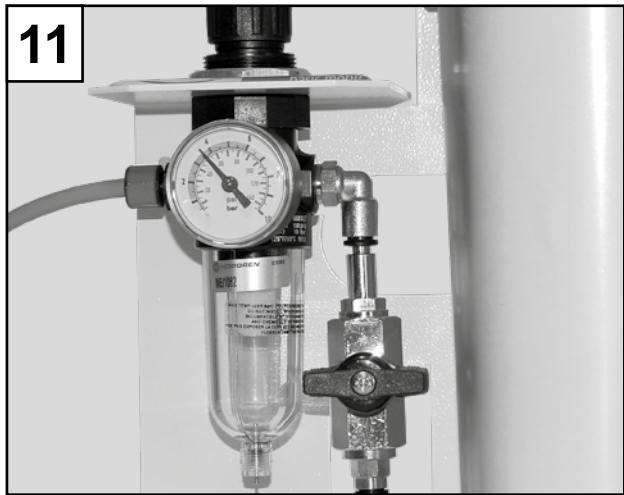
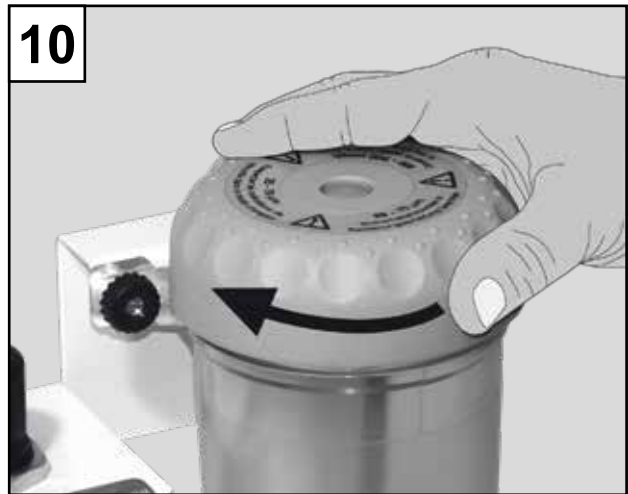
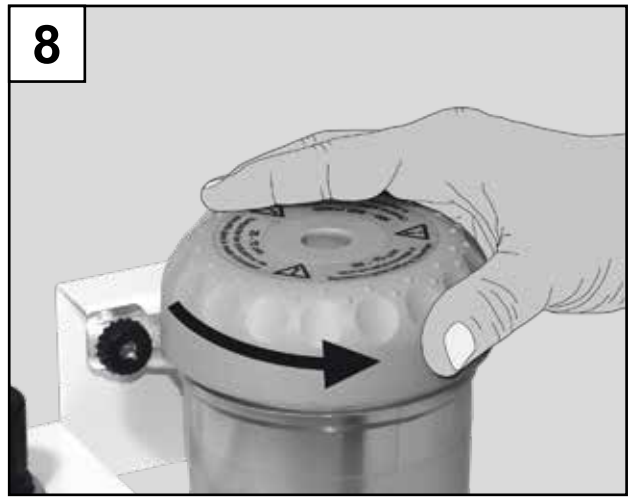
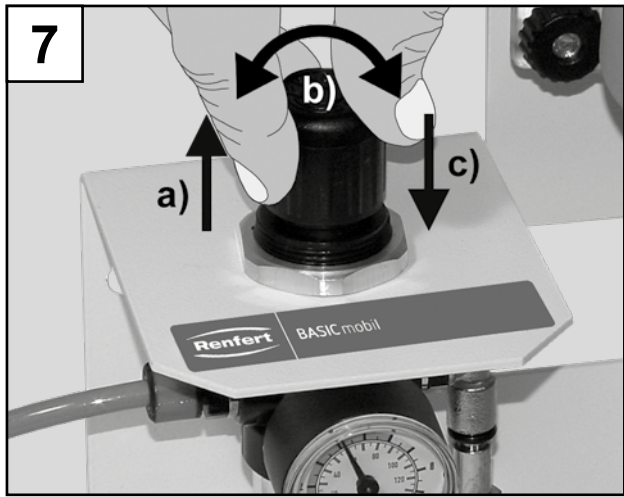
Basic mobil

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Made in Germany

21-9093 27062025

1 Einleitung

1.1 Verwendete Symbole

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1 Introduction

1.1 Symbols used

Symbols with the following meanings are used in these instructions or on the equipment:

Danger

Immediate risk of injury. Consult accompanying documents!

Caution

Risk of damage to the equipment if the instruction is not followed.

Notice

Provides the operator with useful information that eases use.

Caution!

Wear protective goggles.

- ▶ List, particular attention should be paid

- List
- Subordinate list

⇒ Instructions / required action / input / operational sequence:

You are asked to carry out the specified action in the order indicated.

- ◆ Result of an action / response by the equipment / response by the program:

The equipment or program responds as a result of your actions or when a specific incident occurs.

Other symbols are explained as they occur.

2 Safety

2.1 *Intended use*

The Basic sandblaster BASIC MOBIL is intended for commercial use in dental laboratories in accordance with the instructions for use. The unit is intended for use in the following tasks:

- Removal of residual investment material and of oxides on cast components
- Surface treatment
- Devestment of press ceramics
- Sandblasting of ceramic occlusal surfaces

2.2 *Meaning of these instructions*

These instructions for use act as a guide for safe usage of the unit.

If the operating instructions are not observed, accidents can lead to injuries and damage.

- ⇒ Only use the unit in accordance with these instructions for use.
- ⇒ Keep the instructions for use ready with the unit.
- ⇒ Provide the instructions for use to all subsequent users of the unit.

2.3 *Requirements that the operator must meet*

- ⇒ The product may only be used by persons aged 14 or over who are familiar with the procedures and rules in a dental laboratory.

2.4 *Safe condition of the unit*

Faulty or broken components, or those that are not sealed as required, may cause injuries.

- ⇒ Check the power cord, housing, and other components such as connection cables, tubes, and the key pad for damage, for example such as kinks, cracks, porosity, and signs of aging.
- ⇒ Do not expose the unit to mechanical impact. Do not drop it.

2.5 *Risk of injury to the eyes or skin*

Injury may occur to the eyes or skin as a result of high pressure or of components fracturing that are subject to high pressure.

- ⇒ Always wear appropriate protective goggles when carrying out work on the unit.
- ⇒ Never direct the blasting material towards the eyes or unprotected areas of skin.

Dust emissions may result in damage to health.

⇒ Operate the unit with a suitable extraction unit. The extraction unit must be adapted to the dust that occurs.

3 Scope of delivery

- 1 Basic mobil
- 1 Foot switch (only for no. 2914-3050 / 2914-3250)
- 1 Connection set
- 1 Operating instructions with attachments
- 1 Sample blasting abrasive

4 Accessories

Cobra aluminium oxide

1594-1105	25 µm, white 5 kg canister
1594-1205	50 µm, white 5 kg canister
1594-2220	50 µm, white 20 kg bucket
1584-1005	90 µm, white 5 kg canister
1583-1005	110 µm, white 5 kg canister
1583-1020	110 µm, white 20 kg bucket
1587-1005	125 µm, pink 5 kg canister
1587-1020	125 µm, pink 20 kg bucket
1585-1005	250 µm, white 5 kg canister
1585-1020	250 µm, white 20 kg bucket

Rolloblast glass beads

1594-1305	50 µm, 5 kg canister
1594-2312	50 µm, 12.5 kg canister
1589-1005	100 µm, 5 kg canister

5 Startup

The mobile sandblasting unit is connected to the compressed air or to the recyclable blaster Vario-Jet resp. Vario basic.

Risk of injury!

Loose tubes may hit objects around them.

Tubes must be connected first to the unit.

Only press the foot switch after all tubes have been connected.

Close the shut-off valve before start-up (Fig. 11).

5.1 Setting up and Connecting the Unit with Foot Switch

- ⇒ Inspect the tank for proper installation (Fig. 1).
- ⇒ Connect the foot switch / sandblasting unit (red) (Fig. 2a, 2b, 2c).
- ⇒ Connect the foot switch / air supply (P) (Fig. 3).
- ⇒ Do not bend or kink the hose (Fig. 4)!

5.2 Setting up and Connecting the Unit without Foot Switch

- ⇒ Inspect the tank for proper installation (Fig. 1).
- ⇒ Connect the compressed air hose / sandblasting unit (red) (Fig. 2a, 2b, 2c).
- ⇒ Connect the compressed air hose / air supply (P) (Fig. 3).
- ⇒ Do not bend or kink the hose (Fig. 4)!

5.3 Adjusting the Operating Pressure

- ⇒ Put in shut off-valve in a vertical position (Fig. 5).
- ⇒ Set the blasting pressure from 1 - 6 bar [14.5 - 87 psi] (Fig. 7) and in case you use a footswitch, it has to be pressed additionally (Fig. 6).

5.4 Filling the Sandblasting Tank

Close the shut-off valve before filling the tanks (Fig. 11).

Do not write on or stick anything to the sandblasting tank and tank lid.

Do not press the foot switch while filling is in progress!

- ⇒ Unscrew the tank cover (Fig. 8).
- ⇒ Only fill with abrasives up to the max. fill height (thickening of the tank cover underneath the thread) (Fig. 9).
- ⇒ Clean the thread on the sandblasting tank and tank lid as well as the seal.
- ⇒ Screw the tank lid closed firmly by hand (Fig. 10).

Abrasive residue on the seal causes leakage and premature wear of the seal.

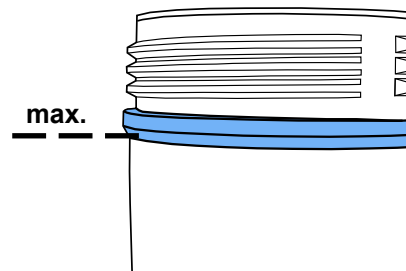
Warning: Do not tilt the tank lid when screwing it on.

Check that the tank lid is seated securely prior to use. Tank lids that are not closed securely can suddenly blow off. There is a risk of injury as a result of flying parts and sudden emission of abrasive sand.

- ⇒ Always use only clean, dry abrasive of the appropriate grain size (refer to the accessories).

The sandblasting unit is now operational.

5.4.1 Approved blasting materials

- Cobra (Renfert GmbH)
- Rolloblast (Renfert GmbH)
- ROCATEC (3M ESPE)

It is possible to process blasting materials from other manufacturers provided that the grain size, grain shape and degree of purity correspond to the above-mentioned materials.

Renfert GmbH does not assume any liability for the function and service life of the equipment if other blasting materials are used.

5.5 Information

Areal sandblasting with large-slot nozzles:

No. 90002-1207 0,3 x 3,5 mm [0.012 x 0.138 inch]

No. 90003-1739 1,5 x 3,5 mm [0.059 x 0.138 inch]

When sandblasting glass ceramic - observe manufacturer's instructions.

6 Cleaning / Maintenance

Before cleaning and maintenance work:

- ▶ Detach from compressed air connection!

6.1 Sandblasting tank

Check the sandblasting tank or tank lid regularly, at least 1x a year, by visually inspecting for damage or changes to the material (e.g., cracks ...). If in doubt, replace!

The sandblasting tank and tank lid are subject to a natural aging process depending on ambient conditions and must be replaced as standard after every 15 years of operation. Note the year of manufacture on the round stamp engraved on the sandblasting tank and in the tank lid; Our customer service team are happy to advise you.

Example of an engraved round stamp:
Date of manufacture January 2003

To clean:

DO NOT use solvent-based cleaning agents or disinfectants. Solvents and tensides can result in microcracking in the plastic (danger of explosion!).

⇒ Only clean the sandblasting tank and tank lid by rubbing with a dry cloth.

6.2 Condensation

⇒ Inspect the water separator.

⇒ Close the shut-off valve (Fig. 11).

⇒ Drain any water off through the valve (Fig. 12), employing the foot switch (Fig. 6).

6.3 Blasting Nozzle Replacement

⇒ Refer to insert no. 21-9162 for installation.

⇒ Inspect for abrasive residue.

⇒ Inspect the seal ring position.

6.4 Sandblasting Hose Replacement

⇒ Disconnect the foot switch from the air supply (Fig. 3).

⇒ Refer to insert no. 21-9162 for installation.

6.5 Cleaning the dosing nozzle

The dosing nozzle from the mixing chamber can become blocked due to contamination and moist blasting sand.

For cleaning, remove the dosing nozzle using the supplied tool and replace it after cleaning.

⇒ Vacuum clean the blasting tank completely.

⇒ Hold the dosing nozzle with the tool and pull upwards (Fig. 13).

⇒ Clean the dosing nozzle (e.g. blow with compressed air).

⇒ Replace dosing nozzle.

Before remounting, the dosing nozzle should be completely dry!

⇒ The dosing nozzle must be adjusted with blasting tanks of 25 - 70 µm (Fig. 14)!

see also supplement „Cleaning the dosing nozzle“.

7 Spare parts

Only use accessories and spare parts that have been supplied or approved by Renfert GmbH. Using other accessories or other spare parts can lead to inadvertent accidents and damage.

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

The components excluded from the warranty (wear parts, consumables) are marked in the spare parts list.

The serial number, date of manufacturing and equipment version are shown on the nameplate of the equipment.

en

8 Technical data

Item number:	2914 3050 / 2914 3250
Working pressure / kPa / bar [psi]:	100 - 600 / 1 - 6 [14.5 - 87]
External connection pressure / kPa / bar [psi]:	600 - 800 / 6 - 8 [87 - 116]
Air consumption l/min [cfm] @ kPa / bar [psi]	120 [4.24] @ 600 / 6 [87]
Ø Compressed air hose / mm [mil]:	
- inside:	4 [157]
- outside:	6 [236]
Tank capacity, each / ml [fl oz]:	1000 [34]
Dimensions (width x height x depth) / mm [inch]:	220 x 260 x 130 [8.7 x 10.2 x 5.1]
Weight (empty) / kg [lbs]:	2.5 [5.5]

9 Handling errors

Error	Cause	Corrective action
No air and no abrasive.	• Blasting nozzle blocked at the hand-piece.	• Unscrew the nozzle from the hand-piece and blow it out. • With the nozzle removed, blow out the blasting hose by activating the foot pedal.
	• Foot switch connecting lines kinked.	• Inspect / correct the connection lines.
Only air, no or very little abrasive.	• The dosage nozzle in the mixing chamber is blocked due to contaminated or moist sand.	• Cleaning the dosing nozzle, see chapter 6.5.
	• Mixing chamber filter combination blocked.	• Change filter cartridge (Wrench SW14).
	• Abrasive contains impurities or is moist.	• Replace the abrasive.
Unit blasts continuously.	• Foot switch connected incorrectly.	• Inspect/correct the foot switch connection.
	• Valve in foot switch is defective.	• Replace foot switch.
Compressed air leaking from the foot switch.	• Pneumatic hoses reversed.	• Reverse the connections.
Foot switch fails to switch on.	• No pressure at the compressed air connection.	• Inspect the compressed air lines and / or the compressor.
	• Pneumatic hoses are kinked.	• Reposition the hoses.

10 Warranty

en

Renfert gives a **3-year guarantee** on all parts of the Basic mobil provided that the product is used in accordance with the operating instructions. Warranty claims may only be made upon presentation of the original sales receipt from the authorized dealer.

Parts which are subject to natural wear and tear (wear parts) and consumables are excluded from the guarantee. These parts are marked in the spare parts list.

The guarantee is rendered void with improper use, disregard of the operating, maintenance, cleaning, and connection instructions, self-repairs or repair by unauthorized personnel, use of replacement parts from another manufacturer and with unusual influences or influences not permitted according to the instructions for use. Warranty claims shall not extend the warranty period.

10.1 Exclusion of liability

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- ▶ The product is employed for any purposes other than those specified in the instructions for use;
- ▶ the product is altered in any way other than those alterations described in the instructions for use;
- ▶ the product is repaired by other than an authorized facility or if any but Renfert OEM parts are employed;
- ▶ the product continues to be employed, despite obvious safety faults;
- ▶ The product is subjected to mechanical impact or is dropped.

11 Obligations regarding information

- ▶ Information on REACH and SVHC can be found on our website in the service area (www.renfert.com/service)

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declare under our sole responsibility, that the product(s)

Name	Part No.
Vario basic	2960 0005
Basic quattro IS	2959 0000 / 2959 0000 03
Basic quattro	2958 0000 / 2958 0000 03
Basic master	2948 2000 / 2948 2000 03 2948 2025 / 2948 2025 03 2948 2250 / 2948 2250 03
Basic classic	2947 1050 / 2947 1050 03 2947 1250 / 2947 1250 03 2947 2000 2947 2025 / 2947 2025 03 2947 2250
Basic eco	2949 1050 / 2949 1050 03 2949 1250 / 2949 1250 03 2949 2025 / 2949 2025 03
Basic mobil	2914 3050 2914 3250
Dustex master plus	2626 0105 / 2626 0105 03

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Electrical Equipment (Safety) Regulations 2016

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...

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Renfert GmbH, Industriegebiet, 78247 Hilzingen / Germany
Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany