making work easy

FAQ - Deutsch

FAQ - English

FAQ - Français

FAQ - Italiano

FAQ - Español

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FAQ - Türkçe

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FAQ - English

For information on operating elements, see [Fig. 1](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

Errors	Cause	Corrective action
No response when the ON / OFF button is pressed	The unit is not connected to the power supply.	Insert the power plug into a building electrical socket.
POWER steamer 2: No automatic filling. Red indicator flashes 2 x.	Unit not connected to the water supply, or the water supply is not on.	Connect the unit to the water supply and turn on the water supply before switching on the unit.
	Water pressure too low.	Minimum required water pressure, see tech. data.
	Water supply turned on too late after switching on.	- Turn on the water supply. - Switch the unit off then on again.
	Sieve in the water connection (16, Fig. 1) blocked.	Unscrew the water hose, remove and clean the sieve.
	The water supply provides distilled or deionized water, the conductivity of which is too low.	Add a coffee spoon of salt, approx. 10 g, to the pressure tank to increase the level of conductivity.
	Automatic filling is faulty.	Contact the repair service or send the unit in for repair.
"Tank empty" indicator (4, Fig. 1) lights up in red although water is still in the tank.	- Calcium deposits have collected in the pressure tank or too much sludge has collected. - The calcification indicator was ignored.	Rinse / de-scale the pressure tank (see Chap. 6.3 and 6.4).
Steam is discharged even after the steam button has been released.	The solenoid valve is jammed due to particles of dirt.	Remove the cover in the middle and tap the solenoid valve, for example using the handle of a screwdriver.
Tank cap cannot be opened.	The pressure tank is still pressurized.	- Do not force open the tank cap! - Release all the pressure, e.g., using the steam outlet function (see Chap. 5.4), or if the tank is empty (red indicator is on) by pressing the steam button.
	The tank cap is jammed as a result of calcium deposits and thermal stress.	- Heat the unit. - Release all of the steam. - Switch the unit off. - While cooling is in progress, check approx. every 20 minutes whether it is now possible to open the tank cap again. - If the unit is not being used for an extended period (e.g., during company vacation), leave the tank cap open.
POWER steamer 2: When a unit is cold, the manometer indicates vacuum pressure.	The vapor pressure of the water at room temperature is lower than that of air.	- No action required. - Switch on the unit and briefly press the steam button. Pressure compensation is achieved by opening the solenoid valve.
Calcium deposits in the funnel-shaped upper housing.	Excess water was not removed after filling.	- Wipe off calcium deposits. - Always remove excess water after filling.
POWER steamer 2: After switching on, the pump runs several times to fill the pressure tank.	The inlet pressure is so low that the container cannot be filled in the permitted maximum pump run time.	No action required.
Pulsating steam jet.	Simmering water is drawn into the steam outlet opening because the pressure tank has been filled to the maximum.	- This behavior resolves itself after working for a short time. - Only fill the unit with the recommended quantity of water.

Errors	Cause	Corrective action
Power steamer 2: Water is leaking from the safety valve (14, Fig. 1) during automatic filling.	• If the conductivity of the water during automatic filling is not sufficient (e.g., untreated, distilled / deionized water), it is not possible to correctly detect the fill level in the boiler. The result is that automatic filling by the pump does not stop. The excess pressure is released through the safety valve.	• Establish conductivity by adding a teaspoon (10 g) of salt to the boiler.

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