making work easy

FAQ - Deutsch

FAQ - English

FAQ - Français

FAQ - Italiano

FAQ - Español

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FAQ - Türkçe

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FAQ - English

For chapter indications, refer to the complete instruction manual on the internet.

Fault	Cause	Solution
It is not possible to adjust the drill length, as the Allen key cannot be used.	• The hexagon socket is full of drilling dust.	• Remove drill jig and chuck and use a pointed instrument to remove the drilling dust from the hexagon socket.
The unit does not switch on.	• The sensor surface has been touched too briefly or too lightly.	• Place the palm of your hand onto the marked surface for approximately 1 second. It is not necessary to use any pressure.
The drilling depth suddenly changes.	• The drill bit is not sufficiently fixed in place; it stops during drilling and therefore alters the drill stop depth. • The coating on the drill stop (threaded pin) is worn out..	• Re-set the drill stop (see section 5.6); tighten the drill jig by hand (without tools). • Insert replacement threaded pin (see last page of the instructions for use).
The drill bit does not turn.	• The shaft diameter of the drill bit is too small.	• Use a drill bit with the correct shaft diameter size (see section 8 Technical Data). Do not turn the chuck further with tools!
The drilling is not effective.	• The drill bit is worn.	• Replace the drill bit.
The drill holes are too big or not round.	• The drill jig is not tight enough.	• Tighten the jig by hand.
The unit switches off abruptly and the operation display blinks briefly.	• The model was pushed too hard into the drill bit.	• Depress the drill table quickly but not suddenly.

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