Renfert

making work easy

de

en

fr

it

es

pt

tr

ru



pl

zh

ja

ko

ar

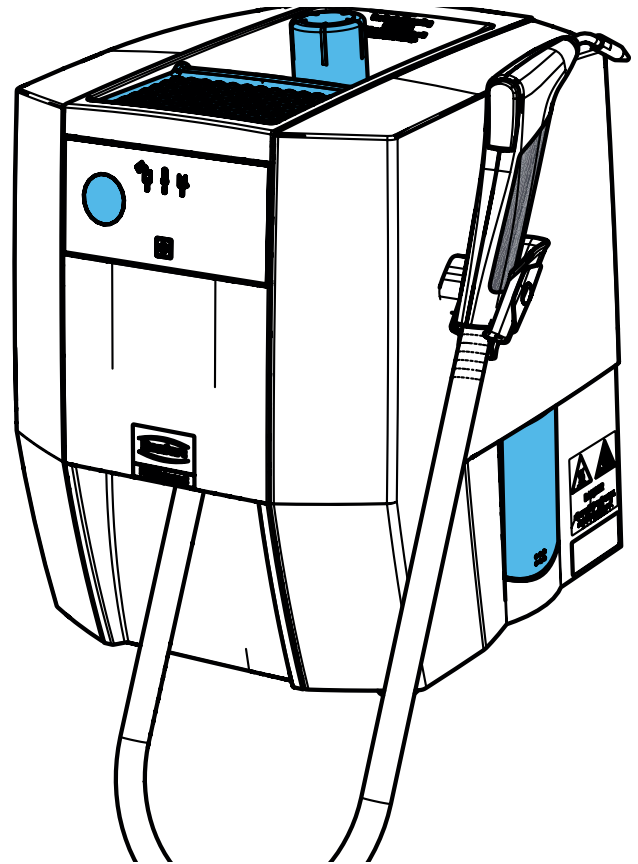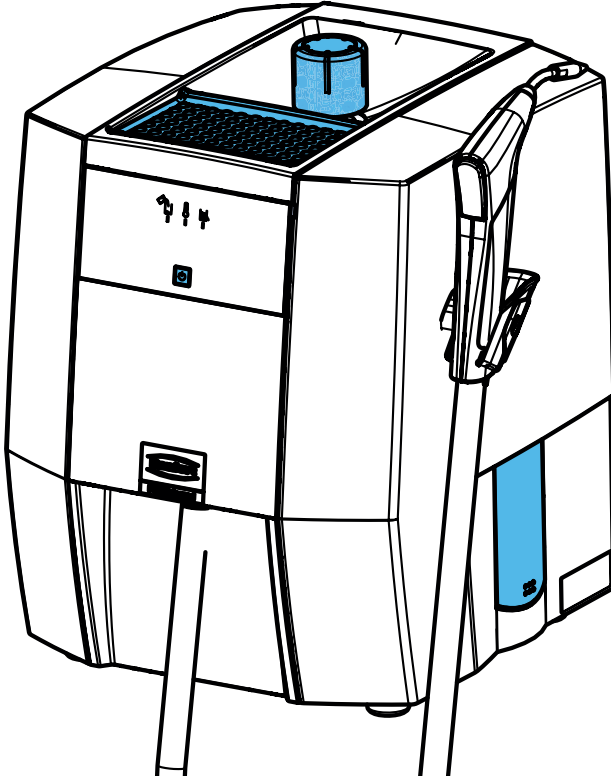POWER steamer 1 & 2

Wartungsplan • Maintenance schedule • Calendrier de maintenance

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Maintenance instructions

making work easy

- ▶ Renfert GmbH shall be absolved from all claims for damages or warranty if: the prescribed cleaning and maintenance intervals are not observed, or a descaling solution is used that is not approved by Renfert.
- ▶ The warranty is voided if the cleaning and maintenance intervals are not observed and if descaling solutions are used that are not approved.

i Document the maintenance work carried out in the maintenance plan provided separately. This will be requested in the case of a warranty claim and is part of the evaluation.

! Before startup, determine the water hardness and calculate the resulting rinsing intervals. Test strips for determining water hardness are part of the scope of delivery.

Chap. 4.5

Rinsing

! Weekly rinsing is mandatory if a salt-based softening system is used, regardless of the water hardness measured.

Water from a softening system results in water with a high pH value in the tank. This can corrode the tank and may result in total failure of the unit.

i How quickly a pressure tank becomes calcified and how this affects the energy efficiency of the heating system depends on a variety of parameters, not all of which can be taken into consideration by the calcification indicator.

For this reason, descaling must be carried out every three months at the latest regardless of the calcification indicator.

Descaling

! Do not use citric acid for descaling. If filling while warm (residual heat or residue when heating), citrate powder is formed that clogs the pipes and equipment, causing failure of the unit.

! Descaling solutions that are not authorized could damage the pressure tank and mechanical components, quickly causing complete failure.

The warranty is voided if a descaling solution is used that is not approved.

- ▶ When descaling, observe the instructions for use of the descaling solution.

Sealing

- ▶ Check the sealing of the tank cap and the service cap regularly for damage. Replace damaged sealing.
- ▶ Replace seals after 3 months at the latest.
- ▶ After rinsing or descaling, clean the sealing surface on the inspection closure.

Checking the overpressure valve

The correct function of the overpressure valve must be checked once a year by an approved specialist company.

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany