making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - 中文
- FAQ - 한국어
- FAQ - اللغة العربية

SILENT powerCAM TC

FAQ - frequently asked questions

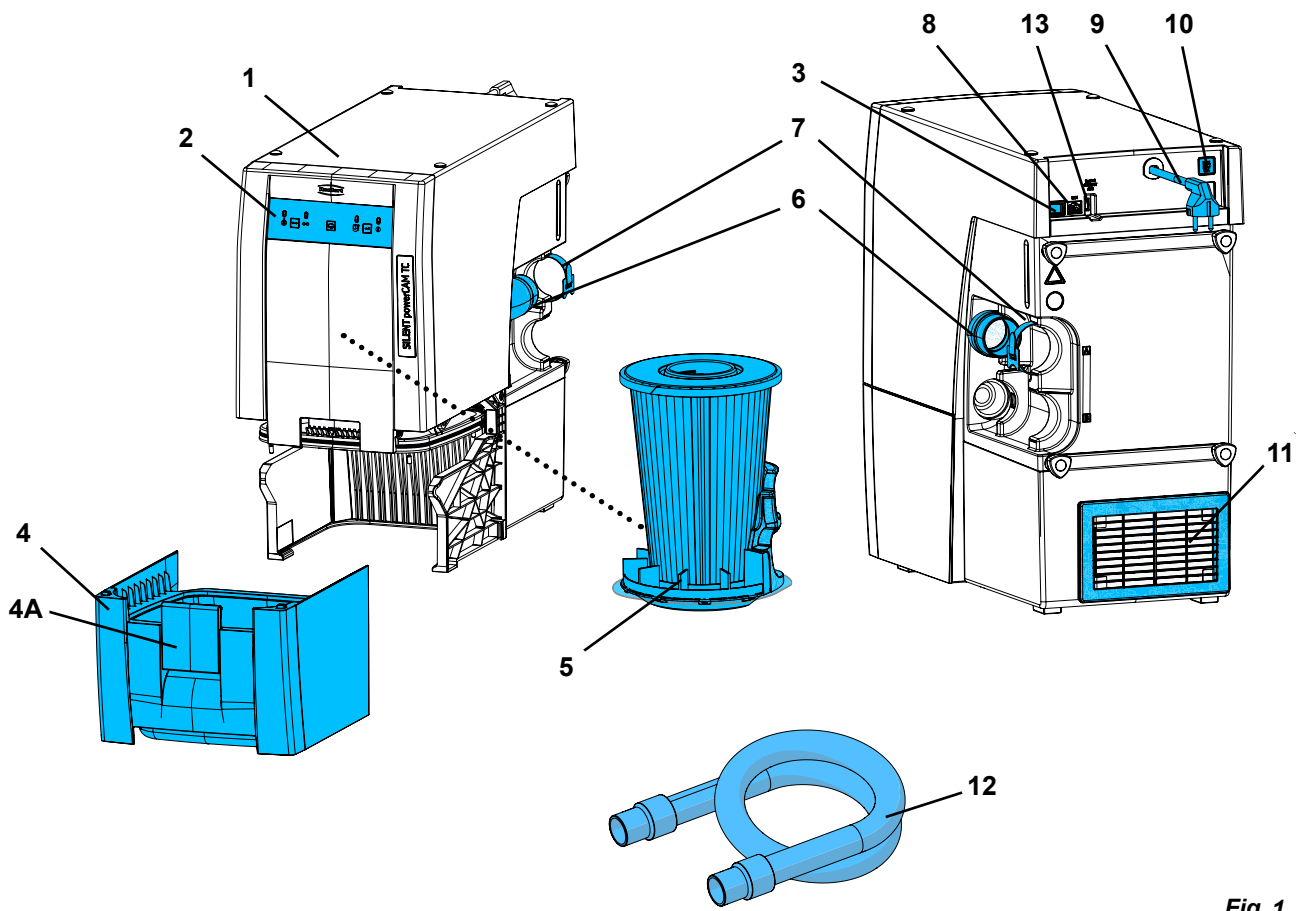
Fig. 1

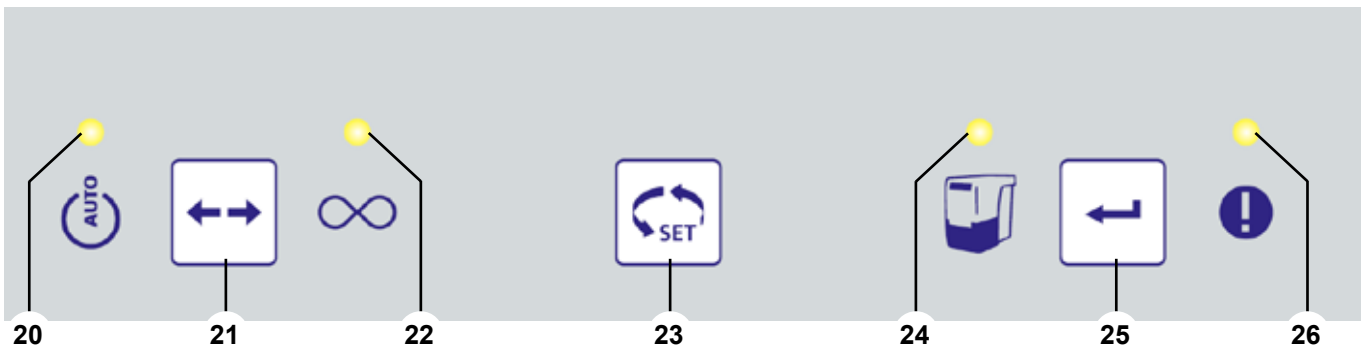
Fig. 2

FAQ - Deutsch

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FAQ - English

For information on operating elements, see [Fig. 1](#) and [Fig. 2](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

en

Trouble	Cause	Corrective action
Error notification display (26, Fig. 2) flashes.	<ul style="list-style-type: none"> The electronics have overheated. 	<ul style="list-style-type: none"> Switch off the unit and let it cool down. Provide sufficient cooling, e.g., using: <ul style="list-style-type: none"> Note Chap. 4.2 Setup. Use external air duct (see Chap. 4.6).
The error notification display (26, Fig. 2) lights up.	<ul style="list-style-type: none"> The fine filter is so heavily clogged that cleaning the filter does not have a sufficient impact. 	<ul style="list-style-type: none"> Replace fine filter (see Chap. 6.3).
The suction power is not sufficient.	<ul style="list-style-type: none"> Blockage or leakage in the suction hose. Dust drawer not airtight. Fine filter blocked. 	<ul style="list-style-type: none"> Check suction hose. Please also note the information in Chap. 4.4. Check that the dust drawer fits correctly (see Chap. 6.2). Switch the unit off and then on again so that the filter is cleaned. Replace the fine filter (see Chap. 6.3) if cleaning the filter does not improve suction power.
Dust drawer is overflowing.	<ul style="list-style-type: none"> The selected time interval for emptying the dust drawer is too long. 	<ul style="list-style-type: none"> Set a shorter time interval (see Chap. 6.2.1).
The signal to empty the dust drawer is output, even though it is not full yet.	<ul style="list-style-type: none"> The time interval set for emptying the dust drawer is too short. 	<ul style="list-style-type: none"> Set a longer time interval (see Chap. 6.2.1).
The display to empty the dust drawer flashes and an acoustic signal sounds three times.	<ul style="list-style-type: none"> The flow rate is no longer sufficient and filter cleaning is performed. 	<ul style="list-style-type: none"> Wait until the filter has been cleaned.
The extraction unit unexpectedly stops suctioning during continuous or automatic operation and the corresponding displays (20/22, Fig. 2) remain on.	<ul style="list-style-type: none"> Suction turbine has overheated. Suction turbine is faulty. 	<ul style="list-style-type: none"> Switch off the unit and leave to cool for at least 60 minutes. Check whether suction hose is blocked, eliminate blockage. Switch the unit off and then on again so that the filter is cleaned. Replace the fine filter (see Chap. 6.3) if cleaning the filter does not improve suction power. Replace suction turbine.
The extraction unit does not respond to the signals of the CAM system.	<ul style="list-style-type: none"> The connector to the CAM system or the extraction unit is not correctly inserted. Interface cable damaged. Connector assignment of the interface cable does not match that of the CAM system. 	<ul style="list-style-type: none"> Check the connector on the extraction unit and CAM system. Check the interface cable for damage, replace if required. Compare the connector assignment of the interface cable with that of the CAM. Use a suitable interface cable if required.
After switching on, not all displays light up briefly.	<ul style="list-style-type: none"> The display that does not light up is faulty. 	<ul style="list-style-type: none"> Contact Renfert / Service.

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