making work easy

de
en
fr
it
es
pt
tr
ru

zh
ja
ko
ar

MT premium

Quick Start Guide

Made in Germany

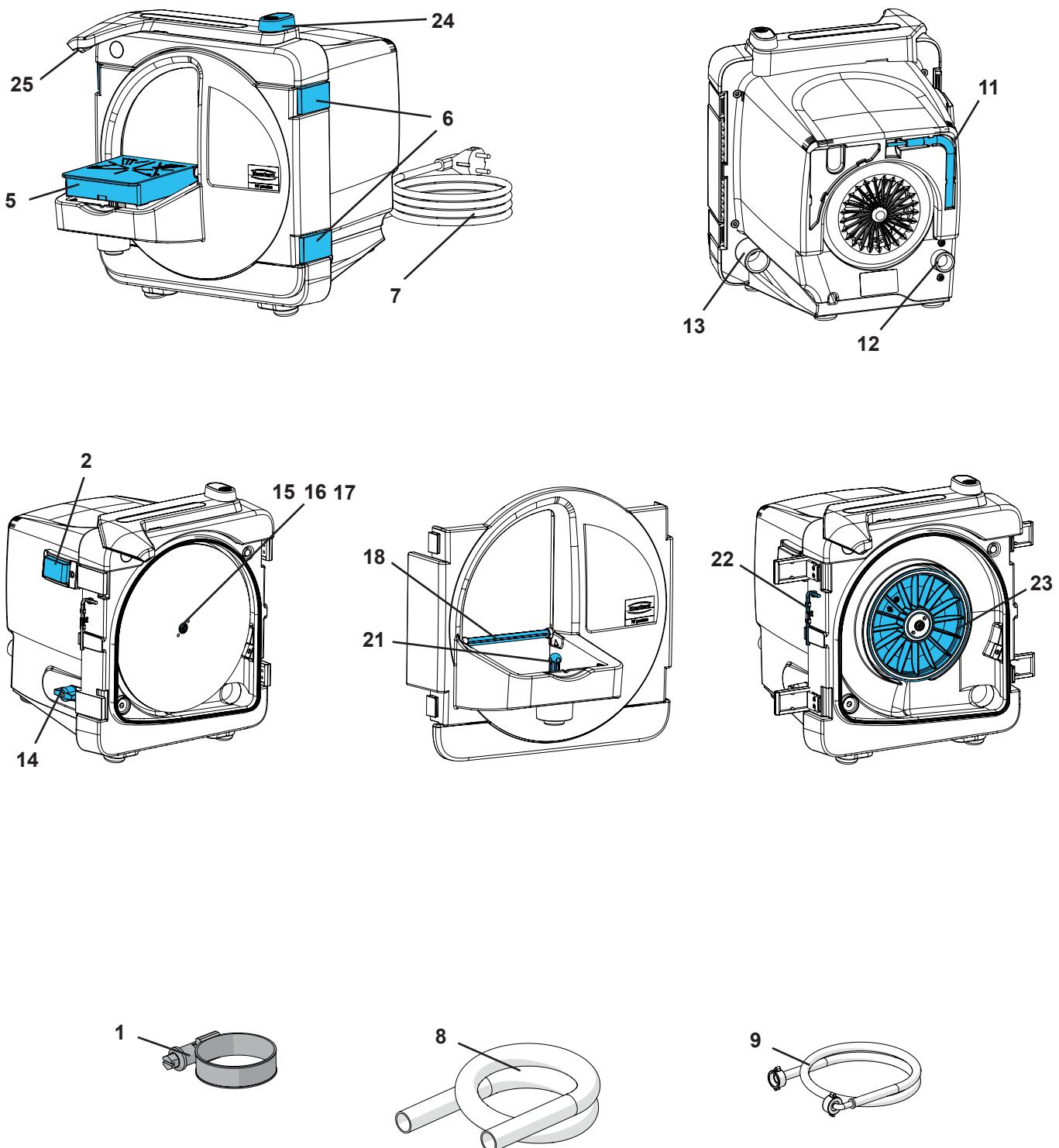
Fig. 1

Einleitung

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Introduction

This Quick Start Guide contains instructions on the operation of the unit and relevant safety information.

Detailed information about the unit is provided in the instructions for use available online at www.renfert.com/p918.

Enter the following item number: 18070000.

en

Symbols used

Symbols with the following meanings are used in these instructions or on the equipment:

Danger
Immediate danger of injury. Consult accompanying documents!

Electrical current
Danger due to electrical current.

Caution
Risk of damage to the equipment if the instruction is not followed.

Notice
Provides the operator with useful information that eases use.

Caution!
Wear protective goggles.

IPX2 An inclination of 15° prevents dripping water entering the unit.

CE The equipment complies with applicable EU directives.

UK This product complies with the relevant UK legislation.
CA See UKCA Declaration of Conformity in the Internet under www.renfert.com.

The equipment is subject to the provisions of the EU directive 2002/96/EC (WEEE directive).

Other symbols are explained as they occur.

Safety

Intended use

The *MT premium* trimmer is only used for wet trimming of plaster models.

Improper use

Use of the unit for dry trimming is improper use and not permitted.

The unit is not suitable for grinding investment materials. The resulting dust is carcinogenic.

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the equipment, increase the risk of serious injury and lead to damage to the environment or the product itself.

Hazard and warning information

General information

- If the unit is not used in accordance with the supplied operating instructions, the intended safety can no longer be guaranteed.
- The unit may only be operated using a power cord equipped with a plug appropriate to the local power supply. Any necessary modifications must be carried out by a qualified electrician.
- Only operate the unit if the information on the nameplate corresponds to the specifications of your local mains power supply.
- The unit may only be plugged into electrical sockets that are connected to the protective conductor system.
- The power plug must be easily accessible.
- Disconnect the unit from the mains power supply before carrying out work on the electrical parts.
- It is the responsibility of the operator to ensure that national regulations are complied with during operation and regarding recurring safety inspection of electrical equipment. In Germany, these are DGUV Regulation 3 in conjunction with VDE 0701-0702.
- Check connection cables (such as power cords), tubes and housing (such as the key pad) regularly for damage (such as kinks, cracks and porosity) or signs of ageing.
Do not operate equipment with damaged connection cables, tubes or housing parts or other defects!
- Defective equipment must be taken out of service immediately. Disconnect the power plug and secure the equipment against further use. Send the equipment in for repair!
- Only operate the equipment under supervision.
- Please observe the national accident prevention regulations!
- Information on REACH and SVHC is available on our website at www.renfert.com, in the Support area.

Specific information

- When working with the trimmer, do not wear loose or wide clothing and keep long hair tied back. There is a risk of injury if these are pulled into or become entangled in the equipment!
- Never open the front cover while the motor is running. The startup protection does not cause the grinding disc to stop.
- Only open the front cover after the grinding disc has come to a complete stop.
- Always wear protective goggles when grinding.
- Do not touch the trimmer disc while it is rotating.
- Do not work without the model table installed and locked into position ("click"). There is a significant risk of injury from small parts or fingers becoming caught in the unit.
- Only lift the trimmer table when the disc is stationary.

Authorized persons

Operation and maintenance of the equipment may only be performed by trained personnel.

Product description

Assemblies and functional elements

See Fig. 1 (on inside cover)

- | | | | |
|-----------|------------------|-----------|-----------------|
| 1 | Clamp | 14 | Water valve |
| 2 | On/off switch | 15 | Protective cap |
| 5 | Model table | 16 | Fastening screw |
| 6 | Door locks (4 x) | 17 | Lock pins |
| 7 | Power plug | 18 | Spray tube |
| 8 | Drain hose | 21 | Adjustment fork |
| 9 | Water hose | 22 | Allen key, SW4 |
| 11 | Puller | 23 | Flange |
| 12 | Water connection | 24 | Aqua stop |
| 13 | Water outlet | 25 | Work light |

Scope of delivery

- 1 Trimmer (*MT premium*)
- 1 Water hose (not with 1807 2x00)
- 1 Drain hose
- 1 Allen key
- 1 Puller
- 1 *Marathon* trimmer disc, only with no. 1807x500
- 1 Operating instructions

Startup

Unpacking

⇒ Check the delivery for completeness (compare with the "Scope of delivery").

Setup

- ⇒ Place the unit on a stable, level work table.
- ⇒ Place the unit near to a water connection / drain.
- ⇒ Place the unit so that the power plug can be accessed easily.

Connect

- ⇒ Slide the drain hose (8) onto the connection piece (13) with a twisting movement.
- ⇒ Secure with a clamp (1).
- ⇒ Lay the drain hose at an incline.
- ⇒ Connect the water hose (9) to the water connection (12) of the unit.
- ⇒ Connect the water hose to a water supply with a water shut-off valve.
- ⇒ Carefully open the water shut-off valve and check the leak-tightness of the connections.
- ⇒ Insert the power plug into a building electrical socket.

Switching on / off

The unit is switched on and off via the on / off switch (2).

**After switching on, wait until the trimmer disc has reached its nominal speed (approx. 2 - 3 seconds).
Do not begin trimming beforehand.**

Operation

Trimming

⇒ Set or check the table inclination as required.

⇒ Switch on the unit.

After switching on, wait until the trimmer disc has reached its nominal speed (approx. 2 - 3 seconds).

Do not begin trimming beforehand.

⇒ Set the amount of water at the water valve (14).

⇒ Using both hands, secure the model on the trimmer table and carefully press it against the disc.

During short breaks from work, e.g., to examine the grinding result, the unit can remain switched on and the water supply can be interrupted using aqua stop (24).

⇒ Aqua stop must be opened again when resuming grinding work.

After work has been completed:

⇒ Switch the unit off.

Close the water shut-off valve of the water supply in the case of longer interruptions in work and if the unit is left unattended (e.g., overnight).

Cleaning / Maintenance

The unit does not contain any internal parts that require maintenance.

Opening the unit is not permitted other than as described below!

⇒ Switch off the unit and disconnect the power plug before cleaning or servicing.

⇒ Close the water shut-off valve of the water supply.

Never clean the unit under running water or immerse in water.

Do not steam clean the housing!

⇒ Only wipe the housing with a damp cloth.

⇒ Open the door locks and remove the trimmer door.

⇒ Clean the trimmer door under running water with a brush or cloth.

⇒ Wipe out the inside of the trimmer with a brush and/or cloth.

⇒ Pull the model table upwards from the ball head at the front edge.

⇒ Clean the model table using a brush as appropriate under running water.

⇒ Unscrew the spray tube (18) backwards on the curved side.

⇒ Place the spray tube in descaling solution overnight.

Spare parts

You can find wear parts and spare parts in the spare parts list available online at www.renfert.com/p918.

Enter the following item number: 18070000.

The components excluded from the warranty (wear parts, consumables) are marked in the spare parts list.

The serial number, date of manufacturing and equipment version are shown on the nameplate of the equipment.

For this reason, equipment with this symbol may not be disposed of in unsorted domestic waste within the European Union.

Please contact your local authorities for more information on proper disposal.

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declare under our sole responsibility, that the product(s)

Name	Part No.
MT plus	1803 0000 1803 0500
MT3	1808 0000 / 1808 0000 03 1808 0500 / 1808 0500 03
MT premium	1807 0000 1807 0500

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Supply of Machinery (Safety) Regulations 2008

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN 61326-1:2013

EN ISO12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN 60825-1: 2014

EN 61000-3-2: 2014

EN 61000-3-3: 2013

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

Technical Data

Item no.:	1807 0000 1807 0500	1807 1000 1807 1500	1807 2000 1807 2500
Nominal voltage / V:	230	120	100
Permissible mains voltage / V:	220 - 240	100 - 120	100 - 120
Mains frequency / Hz:	50/60	50/60	50/60
Power consumption / W [hp]:	1300 [1.74]	1325 [1.78]	
Speed / 1/min bei 50 Hz: / 1/min bei 60 Hz:		2880 3450	
Water consumption, max. l/min [cfm]:		7 / [0.25]	
Necessary water pressure min. / kPa / bar [psi]: max. / kPa / bar [psi]:		100 / 1 [14.5] 500 / 5 [72.5]	
Connection piece diameters for external extraction / mm [inch]:		36 [1.42]	
Dimensions (W x H x D) / mm [inch]:		305 x 330 x 410 [12.0 x 13.0 x 16.1]	
Ø Trimmer disc dimensions / mm [inch]:		234 [9.2]	
Weight (without trimmer disc) / kg [lbs]:		13,2 [29]	
LpA *) (im Leerlauf) /dB(A):		74	
Energy efficiency class of the light source provided:		G	

*) Sound pressure level in accordance with EN ISO 11202

Measurement of motor noise. Depending on the workpiece and working methods, the grinding noise can be louder.

Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany