making work easy

FAQ - Deutsch

FAQ - English

FAQ - Français

FAQ - Italiano

FAQ - Español

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FAQ - Türkçe

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FAQ - English

For information on operating elements, see [Fig. 1](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

Error	Possible cause	Corrective action
Dust extractor stops suddenly, for no apparent reason.	- Dust extractor continued to be used, despite "Change Filter" indicator. - Unit fuse blown. - Motor protection switch (M, Fig. 4) has been activated due to overload (temperature, electricity). - Motor over-temperature protector switch activated. - Plug on grey motor wire out of its socket or loose. - Suction turbine fault. - In-house fuse blown.	- Check and replace the dust bag and fine particle filter (Chapter 3.2 and 3.3). - Check and reinstall the fuse. - Check the motor protection switch, if necessary reset. - Change the motor if repeatedly activated. - Check dust bag and filter, replace as required. (Sec. 3.2 and 3.3). - Check exhaust air filter, replace as required. (Sec. 3.3.2). - Check whether suction hose is blocked. Remove obstruction as required. - Dust extractor can be restarted after being allowed to cool off for approx. 1 hour. - Check plug (Fig. 26). - Carry out self-diagnosis (chapt. 3.5), at fault „E2“ exchange suction turbine (see spare parts list). - Replace the in-house fuse. - Check the total power consumption of the dust extractor plus dust-generating devices.
Dust extractor stops suddenly, for no apparent reason. E4 is indicated on the display.	The electronics may be overheated due to excessively high ambient temperature, e.g., if the unit is installed in a cabinet.	- Allow the extractor to cool down. - Attempt to improve the ventilation conditions. Refer also to Sec. 1.1.
After switching on the unit, the display immediately indicates E4.	- Temperature sensor fault. - After overheating, the device has not yet cooled down sufficiently.	- Have the device repaired. - Disconnect the device and let it continue to cool down.
The self-test returns a suction turbine error, even though the turbine is running.	- Filter full. - Suction hose blocked. - Suction turbine fault.	- Check dust bag and fine particle filter, replace as required. (Sec. 3.2 and 3.3.1). - Remove obstruction from the suction hose. - Replace the suction turbine (Sec.3.6).
Insufficient extraction force.	- Extraction force level too low. - Filter blocked. - Suction hose blocked. - Dust drawer seal not tight. - Upper front panel fails to close properly. - Suction hose damaged.	- Increase the extraction level (Sec. 2.3). - Check dust bag and fine particle filter, replace as required. (Sec. 3.2 and 3.3.1). - Remove obstruction from the suction hose. - Check the proper seat of the dust drawer (Sec. 3.2). - Check the dust drawer seal for damage and replace as required (Sec. 3.1). - Check the proper seat of the upper front panel (Sec. 3.3.1). - Check the upper front panel seal for damage and replace as required (Sec. 3.1). - Inspect the suction hose and replace as required.

Error	Possible cause	Corrective action
“Change Filter” indicator goes on shortly after filters were re-placed.	• Fine particle filter blocked. • Suction hose blocked. • Cross-section of the dust-generating device's hose fitting is too small. • Dust is too fine. • Flow sensor defect.	• Replace the fine particle filter (Sec. 3.3.1). • Remove obstruction from the suction hose. • Adjust the sensitivity of the „Replace filter“ indicator(Chapter 2.7.1.). • Adjust the sensitivity of the „Replace filter“ indicator (Chapter 2.7.1.) • Carry out self-diagnosis (chapt. 3.5), at fault „E1“ equipment requires professional repair.
Dust bag ruptures.	• The “Change Filter” indicator's sensitivity has not been adjusted for the material being extracted.	• Reduce the sensitivity of the „Replace filter“ indicator (Chapter 2.7.1.).
Dust extractor fails to start when a dustgenerating device is being used.	• Dust extractor not switched on. • Dust-generating device not plugged into the unit power outlet on the SILENT TS. • Wrong operating mode selected. • Automatic start feature's threshold value too high. • Device fuse has blown. • Automatic start feature fails to detect the attached device's operation.	• Switch the dust extractor on (Sec. 2.2). • Connect the dust-generating device to the SILENT TS (Sec. 1.4). • Select the automatic mode (Sec. 2.5). • Adjust the automatic start feature (Sec. 2.6). • Check and reinstall the fuse.(Sec. 3.4). • Some older handheld instruments do not generate a sufficiently strong signal for the automatic start feature. Switch the dust extractor to the continuous mode.
Dust extractor starts, although a connected device is not being used.	• Automatic start feature's threshold value too low.	• Adjust the automatic start feature (Sec. 2.6).
Dust extractor fails to stop when a connected device is switched off.	• Dust-generating device not plugged into the unit power outlet on the SILENT TS. • Wrong operating mode selected. • Activation threshold too low.	• Connect the dust-generating device to the SILENT TS (Sec. 1.4). • Select the automatic mode (Sec. 2.5). • Adjust the automatic start feature (Sec. 2.6).
No satisfactory setting of the automatic start feature can be adjusted.	• Some older handheld instruments do not generate a sufficiently strong signal for the automatic start feature.	• Switch the dust extractor to the continuous mode.
Extraction briefly starts up when the unit is switched on.	• Activation sensitivity too low.	• Adjust the automatic start feature (Sec. 2.6).
Panels covering the fine particle filter and the dust drawer are difficult to open.	• Lock bolts dirty.	• Clean the lock bolts and lubricate them slightly, if required.

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