making work easy

FAQ - Deutsch

FAQ - English

FAQ - Français

FAQ - Italiano

FAQ - Español

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FAQ - Türkçe

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FAQ - English

For chapter indications, refer to the complete instruction manual on the internet.

Error	Possible cause	Corrective action
Display does not go on, unit does not operate.	<ul style="list-style-type: none">Control unit not switched on.Power supply interrupted.The plug is not fully inserted into the control unit. .Faulty control unit.Faulty transformer.Short circuit in the tip.	<ul style="list-style-type: none">Switch the control unit on at key A.Check the power supply and reconnect, as required.Insert the plug until it clearly snaps into position.Have unit replaced by Service Department.Replace with spare (refer to the spare parts list).Clean the tip and handpiece (use compressed air).Replace the tip.Faulty heater handle --> Replace with spare (refer to the spare parts list).
Unit operating, but no temperature at the tip.	<ul style="list-style-type: none">Tip not properly screwed in.Tip / handpiece contact contaminated.Faulty tip.Handpiece cable not properly plugged into the control unit.Plug on handpiece cable shows signs of corrosion.	<ul style="list-style-type: none">Hand tighten tip.Clean the contact (use compressed air).Replace the O-ring (refer to the spare parts list).Replace with a new tip (refer to Sec. 13 / "Accessories").Fully insert the handpiece cable into the socket until the cable locks in place.Reinsert the plug, clean the contacts, if required.
The tip fails to reach the desired temperature.	<ul style="list-style-type: none">Incorrect power supply to the tip because the OEM mains power supply unit is not being used.	<ul style="list-style-type: none">Use the OEM mains power supply unit.
Values are not saved.	<ul style="list-style-type: none">Memory chip faulty.	<ul style="list-style-type: none">Have the unit repaired.
Pressing the P key always displays the same temperature.	<ul style="list-style-type: none">The same temperature has been saved to each memory location.	<ul style="list-style-type: none">Assign different temperatures to each memory location (refer to Sec. 5.6).

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