making work easy

FAQ - Deutsch

FAQ - English

FAQ - Français

FAQ - Italiano

FAQ - Español

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FAQ - Türkçe

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FAQ - English

For information on operating elements, see [Fig. 1](#) and [Fig. 2](#) on page 2 of this document.

For chapter indications, refer to the complete instruction manual on the internet.

EN

Problem	Possible cause	Corrective action
No air and no abrasive.	- Blasting nozzle blocked at the hand-piece. - Foot switch connecting lines kinked. - Tank selection switch in an intermediate position.	- Unscrew the nozzle from the hand-piece and blow it out. - With the nozzle removed, blow out the blasting hose by activating the foot pedal. - Inspect / correct the connection lines. - Turn the switch until you feel it click into place.
Only air, no or very little abrasive.	- The dosage nozzle in the mixing chamber is blocked due to contaminated or moist sand. - Mixing chamber filter combination blocked. - Abrasive contains impurities or is moist.	- Cleaning the dosing nozzle, see chapter 6.5. - Change filter cartridge (17. Fig. 1) (wrench SW14). - Replace the abrasive.
Pressure control cannot be adjusted.	The pressure control button is locked.	Pull the pressure control button upwards to unlock.
Unit blasts continuously.	- Foot switch connected incorrectly. - Valve in foot switch is defective.	- Inspect / correct the foot switch connection. - Replace foot switch.
Lamp is not alight.	- Power cord not plugged in. - Illumination defect.	- Plug the power cord into the wall outlet. - Change light using the service or see spare part list.
Compressed air leaking from the foot switch.	Pneumatic hoses reversed.	Reverse the connections.
Foot switch fails to switch on.	- No pressure at the compressed air connection. - Pneumatic hoses are kinked.	- Inspect the compressed air lines and/or the compressor. - Reposition the hoses.
Foot switch hoses cannot be disconnected from the unit.	Push-in connection is not released (Fig. 2 / 3).	Press the push-in connection ring on the hose side firmly while pulling on the hoses.

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