making work easy

DE
EN
FR
IT
ES
PT
TR
RU
PL
ZH
JA
KO
AR

EASY view 3D - Kalibrierung

DE

making work easy

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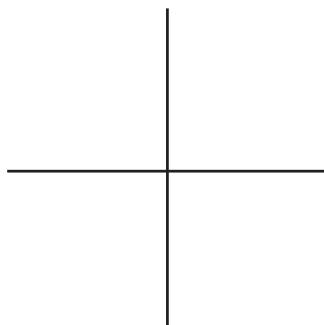
EN

making work easy

EASY view 3D - Calibration Checking and adjusting the 3D calibration

EN

- i** If no clear 3D image is apparent even when wearing 3D glasses, this may be due to a misaligned 3D calibration.

Check the 3D calibration:

Carry out the following work steps **without** using 3D glasses:

- ⇒ Print out page 2.
- ⇒ Place page 2 on the working surface so that the cross depicted is located in the center of the EASY view 3D light beam and is therefore visible on the monitor.
- ⇒ Switch to 2D display.
- ⇒ Adjust the magnification to 15x.
- ⇒ Align the head parallel to the table surface.
- ⇒ Rough adjustment of the image sharpness is achieved by lifting and lowering the support arm.
- ⇒ Fine adjustment of the image sharpness is achieved by adjusting the height (6) of the head until optimum focusing of the cross is detected
- ⇒ Switch to 3D display.
 - ◆ If only one sharp cross (a) can be detected then the 3D calibration is correct.
 - ◆ If a second cross (b, c) is detected, then the 3D calibration is misaligned.

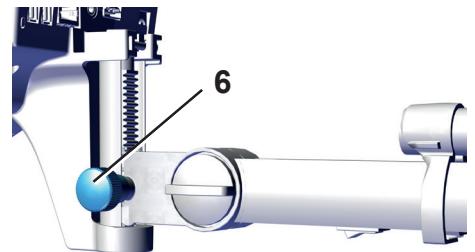
Fig. 1

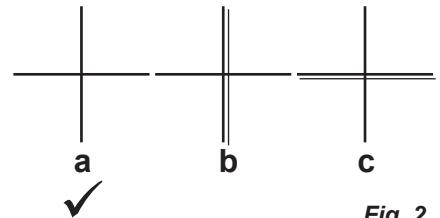
Fig. 2

- i** The height setting may have to be readjusted for the main axes (x axis / y axis) to be clearly in focus.

If the 3D calibration is misaligned, it must be reset:

Setting 3D calibration without using 3D glasses:

- ⇒ Leave the EASY view 3D in the previously described position.
- ⇒ Switch the head off at the switch (7).
- The following procedure can take up to 20 seconds:
- ⇒ While holding down the right mouse button and the mouse wheel, switch on the head again, keep holding both the right mouse button and the mouse wheel down and simultaneously move the mouse until the service menu appears in the top right edge of the monitor.

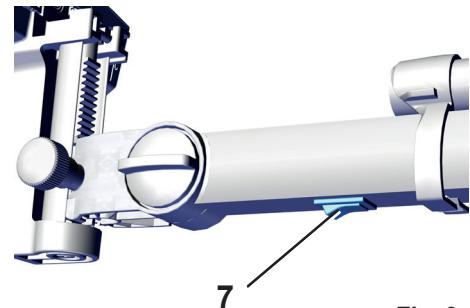
Fig. 3

Service menu:

- Firmware version
- Offset calibration

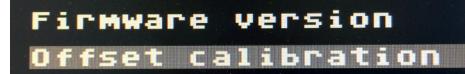
Fig. 4

- ⇒ Using the mouse wheel, select "Offset calibration" and open with the left mouse button.

Setting x axis:

- ⇒ Using the mouse wheel, select the x axis and open with the **left** mouse button.
 - ◆ The offset value of the x axis will be displayed in a monitor window in the top right-hand corner.
- ⇒ Use the mouse wheel to change the value.
 - ◆ The vertical bar of the cross is visible in duplicate.
- ⇒ Use the mouse wheel to set the value so that the two bars are placed precisely over one another.
- ⇒ Confirm the setting using the **left** mouse button.

Fig. 5

Setting the y axis:

- ⇒ Using the mouse wheel, select the y axis and open with the **left** mouse button.
 - ◆ The offset value of the y axis is displayed in a monitor window in the top right-hand corner.
- ⇒ Use the mouse wheel to change the value.
 - ◆ The horizontal bar of the cross is visible in duplicate.
- ⇒ Use the mouse wheel to set the value so that the two bars are placed precisely over one another.
- ⇒ Confirm the setting using the **left** mouse button.

Fig. 6

Leaving the 3D calibration:

- ⇒ Click the **right** mouse button.
 - ◆ Jump back to the service menu.
- ⇒ Click the **right** mouse button.
 - ◆ Leaving the service menu.
- ⇒ Switch the head off and then back on again (7, Fig.).

Check the 3D calibration using 3D glasses:

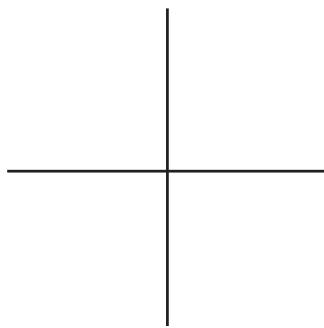
- ⇒ Switch to 3D display.
 - ◆ If a clear 3D image is detected, then the 3D calibration is correct.
 - ◆ If no clear 3D image is detected, repeat the 3D calibration.

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Renfert GmbH • Industriegebiet • 78247 Hilzingen/Germany