making work easy

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- zh
- ja
- ko

SIMPLEX 3D-Filament-Drucker

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Made in Germany

21-7139 28022024

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Contents

en

1	Introduction	4
1.1	General description	4
1.2	Symbols used	4
2	Safety	5
2.1	Intended use	5
2.2	Improper use	5
2.3	Ambient conditions for safe operation	5
2.4	Ambient conditions for storage and transport	5
2.5	Hazard and warning information	5
2.5.1	General information	5
2.5.2	Specific information	6
2.6	Authorized persons	6
2.7	Disclaimer	6
3	Product description	7
3.1	Assemblies and functional elements	7
3.2	Scope of delivery	8
3.3	Accessories	8
4	Startup	8
4.1	Unpacking and setup	8
4.2	Printer assembly	9
4.2.1	Installing the filament spool holder	9
4.2.2	Installing the filament sensor (FMS)	9
4.2.3	Installing the filament guiding tube	9
4.2.4	Attaching the door	9
4.2.5	Attaching the cover	10
4.2.6	Connecting the power cord	10
4.3	Switching on	10
5	Getting started	11
5.1	Setting the language	11
5.2	Setting the temperature unit	12
5.3	Wi-Fi connection	12
5.4	Leveling the print bed	13
5.5	Moving the print head manually	14
5.5.1	Attaching the filament spool	15
5.6	Loading the filament	15
5.6.1	Selecting the filament / starting heating	15
5.6.2	Drawing in / extruding the filament	15
5.7	FMS system	17
5.7.1	Calibrating the FMS system	17
5.8	Printing an object	18
5.8.1	Loading an object from a USB flash drive	18
6	Other settings	19
6.1	Setting the date/time / resetting the operating hours meter	19
6.2	Setting the display backlight	20
6.3	Activating the fan	20
6.4	Audio settings	20
6.5	The cover menu	20
6.6	Using the SIMPLEX Wi-Fi	21
6.7	The FMS system	21
6.8	Resetting settings	21
7	Operation	22
7.1	Saved filaments	22
7.2	Setting a custom print and bed temperature	22
7.3	Heating the printer	22
7.4	Loading the filament	23
7.5	Extrusion	23
7.6	Removing the filament from the extruder	23

- 8 Moving the print head and the print bed 24
- 9 SIMPLEX sliceware / SIMPLEX print 24
- 10 Printing objects 25
 - 10.1 Cleaning the Kapton film 25
 - 10.2 Printing using a USB flash drive 25
 - 10.3 Actions during USB printing 26
 - 10.3.1 Stop 26
 - 10.3.2 Settings / optimization 27
 - 10.3.3 Pause next layer 27
 - 10.3.4 Pause 28
 - 10.4 Printing with SIMPLEX print 28
 - 10.4.1 Connecting the printer via USB cable 28
 - 10.4.2 Automatically starting SIMPLEX print 28
 - 10.4.3 Print process using SIMPLEX print 29
 - 10.4.4 Starting SIMPLEX print manually 29
- 11 Cleaning / Maintenance 30
 - 11.1 Cleaning 30
 - 11.2 Lubrication 30
 - 11.2.1 Lubrication of the z-axis 30
 - 11.2.2 Lubrication of the x/y-axis 31
 - 11.3 Unblocking the extruder 31
 - 11.4 Replacing the nozzle 32
 - 11.4.1 Attaching the nozzle 33
 - 11.5 Applying a new Kapton film 33
 - 11.6 Firmware – update 34
 - 11.7 Replacing the backup battery 34
 - 11.8 Tighten X- and Y-axis traction belt 35
 - 11.9 Spare parts 35
- 12 Troubleshooting 35
- 13 Technical data 38
- 14 Warranty 39
- 15 Disposal information 39
 - 15.1 Disposing of the equipment 39
 - 15.1.1 Disposal information for countries in the EU 39

1 Introduction

1.1 General description

Congratulations on the purchase of your SIMPLEX 3D filament printer and welcome to the world of 3D printing!

The SIMPLEX 3D filament printing system was developed by Renfert with the goal of simplifying and perfecting the 3D printing process with filaments. It offers an excellent manufacturing experience without the need for prior knowledge of programming or 3D printing. To print a 3D object, you need the SIMPLEX 3D filament printer, SIMPLEX sliceware, and a SIMPLEX filament of your choice.

"App Store", "Apple", "Google", "Play Store", and "Android" are registered trademarks of Apple or Google.

1.2 Symbols used

Symbols with the following meanings are used in these instructions or on the equipment:

Danger

Immediate danger of injury. Consult accompanying documents!

Electrical current

Danger due to electrical current.

Hot surface

Danger of burns due to hot surfaces and hot components.

Danger of hand injury

Danger of the user's hands getting crushed by moving parts.

Caution

Risk of damage to the equipment if the instruction is not followed.

Wear protective gloves!

Notice

Provides the operator with useful information that eases use.

The equipment complies with applicable EU directives.

This product complies with the relevant UK legislation.

See UKCA Declaration of Conformity in the Internet under www.renfert.com.

The equipment is subject to the provisions of the EU directive 2002/96/EC (WEEE directive).

► **List, particular attention should be paid**

- List
- Subordinate list

⇒ Instructions / appropriate action / input / operational sequence:

You are asked to carry out the specified action in the order indicated.

- ◆ Result of an action / response by the equipment / response by the program:

The equipment or program responds as a result of your actions or when a specific incident occurs.

Other symbols are explained as they occur.

2 Safety

Before using the equipment, read this user manual carefully and keep it for future reference. Renfert GmbH reserves the right to modify or revise this user manual at its own discretion at any time.

2.1 Intended use

The SIMPLEX 3D filament printer produces three-dimensional objects from melted filament in just one printing step. First you must create the objects using a CAD program and save them as *.stl, *.obj or *.3mf files. You must then use SIMPLEX sliceware to convert your 3D CAD files into print objects for the SIMPLEX 3D filament printer. Transfer your files to your SIMPLEX 3D filament printer using a USB flash drive or a USB cable.

2.2 Improper use

Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the equipment, increase the risk of serious injury and lead to damage to the environment or the product itself.

2.3 Ambient conditions for safe operation

The unit may only be operated:

- Indoors
- Up to an altitude of 2,000 m above sea level
- At an ambient temperature of between 15 - 32 °C [59 - 90 °F]
- At a maximum relative humidity of 80 % at 32 °C [90 °F]
- With mains power where the voltage fluctuations do not exceed 10 % of the nominal value
- Under contamination level 2 conditions
- Under over-voltage category II conditions

2.4 Ambient conditions for storage and transport

The following ambient conditions must be observed during storage and transport:

- Ambient temperature of -20 - +60 °C [-4 - +140 °F]
- Maximum relative humidity 80 %

2.5 Hazard and warning information

2.5.1 General information

- ▶ If the unit is not used in accordance with the supplied operating instructions, the intended safety can no longer be guaranteed.
- ▶ The unit may only be operated using a power cord equipped with a plug appropriate to the local power supply. Any necessary modifications must be carried out by a qualified electrician.
- ▶ Detachable power cords must not be replaced with inadequately rated power cords. Only use original Renfert spare parts.
- ▶ Only operate the unit if the information on the nameplate corresponds to the specifications of your local mains power supply.
- ▶ The unit may only be plugged into electrical sockets that are connected to the protective conductor system.
- ▶ The power plug must be easily accessible.
- ▶ Disconnect the unit from the mains power supply before carrying out work on the electrical parts.
- ▶ It is the responsibility of the operator to ensure that national regulations are complied with during operation and regarding recurring safety inspection of electrical equipment. In Germany, these are DGUV Regulation 3 in conjunction with VDE 0701-0702.
- ▶ Check connection cables (such as power cords), tubes and housing (such as the key pad) regularly for damage (such as kinks, cracks and porosity) or signs of ageing. Do not operate equipment with damaged connection cables, tubes or housing parts or other defects!
- ▶ Defective equipment must be taken out of service immediately. Disconnect the power plug and secure the equipment against further use. Send the equipment in for repair!
- ▶ Please observe the national accident prevention regulations!
- ▶ Information on REACH and SVHC is available on our website at www.renfert.com, in the Support area.

2.5.2 Specific information

- ▶ The cover and the door are equipped with a safety function. Operation without a cover and/or door is prohibited.
- ▶ Keep the power cord away from hot surfaces.
- ▶ Do not touch moving parts or heated elements. This can cause injury.
- ▶ Never reach into the unit while the unit is switched on during printing. Various parts of the machine (primarily the extruder and heated print bed) operate at very high temperatures and can cause severe burns.
- ▶ Before maintenance work, ensure that the unit is switched off and the power cord is disconnected from the mains power supply. After disconnecting the power plug, wait at least 5 minutes for the unit to cool before reaching into the printer for maintenance purposes.
- ▶ Always turn off the printer and disconnect it from the computer when not in use.
- ▶ The SIMPLEX 3D filament printer melts plastic filament during printing. Plastic odors/gases are released during printing. Ensure that the SIMPLEX 3D filament printer is placed in a well-ventilated location.
- ▶ Use SIMPLEX filaments, as they do not result in any gases during printing that are harmful to health.
- ▶ SIMPLEX-filaments are not suitable for the application in the patients mouth
- ▶ If using alternative filaments, ensure that no gases are released during printing that are harmful to health. Observe the Safety Data Sheets.
- ▶ When using filaments which are medical products, the used parameters must be coordinated with the manufacturer.
- ▶ This equipment is not suitable for use in locations where children may be present.

2.6 Authorized persons

The unit may

- only be used, cleaned and maintained by persons who have been instructed in its use and who have been expressly assigned to operate it,
- only be operated under supervision,
- not be used by children,
- not be used by persons who are unable to operate the unit safely due to limited physical, sensory, or mental capabilities.

Any repairs not described in these operating instructions may only be performed by a qualified electrician.

2.7 Disclaimer

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- ▶ The product is employed for any purposes other than those specified in the operating instructions.
- ▶ The product is altered in any way other than those alterations described in the operating instructions.
- ▶ The product is not repaired by an authorized dealer or if original Renfert spare parts are not used.
- ▶ The product continues to be used despite obvious safety defects or damage.
- ▶ The product is subjected to mechanical impact or is dropped.

3 Product description

3.1 Assemblies and functional elements

- 1 Touchscreen LCD display / main menu
- 2 Filament guiding tube
- 3 Nozzle
- 4 Print bed (removable glass plate with Kapton film applied)
- 5 Leveling knobs
- 6 Z-Axis rod
- 7 Filament spool holder
- 8 Filament spool
- 9 Filament sensor
- 10 Extruder
- 11 USB A port for USB flash drive
- 12 USB B port for connection to a PC
- 13 ON/OFF switch / connection for mains power supply
- 14 Port for filament sensor
- 15 Port for cover fan
- 16 Level gauge
- 17 Menu - TEMPERATURE 
- 18 Menu - EXTRUDER 
- 19 Menu - USB PRINT 
- 20 Menu - AXES 
- 21 Menu - SETTINGS 

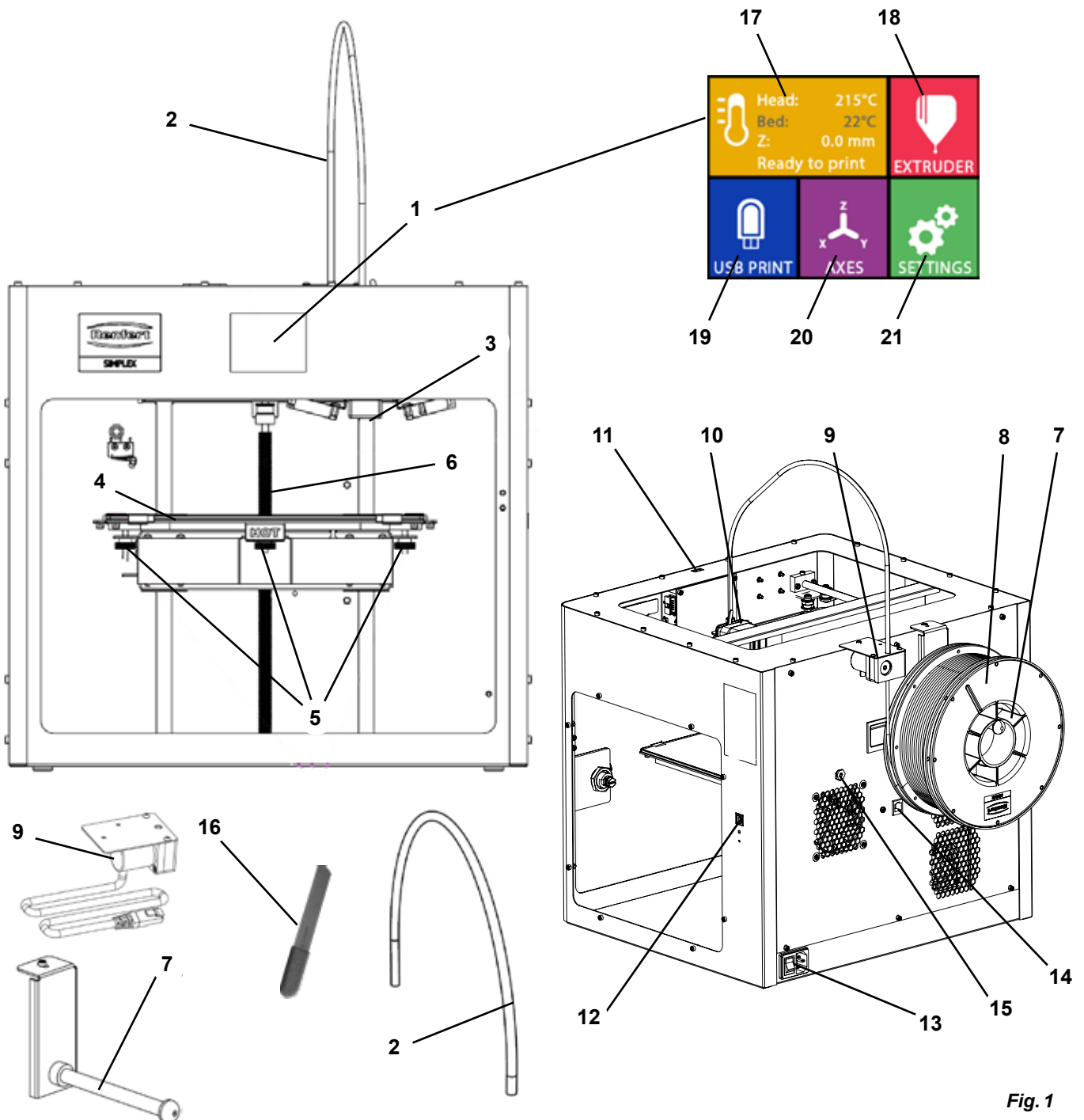
Fig. 1

3.2 Scope of delivery

1 SIMPLEX 3D filament printer	1 USB A-B cable
1 Filament guiding tube	1 Allen keys (set of 5)
1 Filament spool holder	1 Level gauge
1 Filament spool: SIMPLEX study model	1 Replacement nozzle 0.4 mm
1 Filament sensor (FMS)	2 Nozzle keys
1 Cover	1 Spatula
1 Door including lock and key	1 Wire cutters
1 USB flash drive	1 Conductive paste (in dosing syringe)
1 Power cord	1 Operating instructions SIMPLEX 3D filament printer
1 Power adapter set	1 Operating instructions SIMPLEX sliceware

3.3 Accessories

1735 0002	SIMPLEX nozzle 0,4 mm / steel	1735 0300	SIMPLEX aligner model 800 g
1735 0100	SIMPLEX study model 800 g	1735 0600	SIMPLEX multi-use model 800 g
1735 0200	SIMPLEX working model 800 g	900038974	SIMPLEX nozzle 0,25 mm

4 Startup

The unit must be stored at room temperature for at least 3 hours before startup.

Do not apply force to the unit during unpacking, setup, operation, or maintenance.

4.1 Unpacking and setup

A video on this topic is provided under the following link www.renfert.com/simplex-1-en and via the QR code shown.

Select a setup location where the printer is not exposed to direct sunlight or drafts.

- ⇒ Place the shipping box on a flat surface on the floor. Ensure that you have enough space available.
- ⇒ Open the shipping box. It contains three boxes:
 - Unit
 - Cover
 - Accessories box
- ⇒ Remove the three boxes.
- ⇒ Carefully open the box with the unit.
- ⇒ Remove the foam insert.
- ⇒ Carefully remove the accessories box provided. It protects the extruder during transport. This box contains the accessories for the SIMPLEX 3D filament printer.
- ⇒ Lift the SIMPLEX 3D filament printer out of the shipping box by gripping the insides of the frame.

Please note the weight of the printer and ask a second person to assist you if necessary.

When removing, be careful not to touch the extruder and the electronics.

The SIMPLEX 3D filament printer is not suitable for setup in the treatment room.

- ⇒ Place the SIMPLEX 3D filament printer on a stable and level surface with sufficient space around the unit.

Do not cover the ventilation openings on the rear of the unit

- ⇒ Ensure that the on/off switch and power cord are easily accessible.
- ⇒ Using the wire cutters, remove the two cable ties that secure the print bed.

Caution: Do not damage the Kapton film!

- ⇒ Remove the door, which is located under the print bed.
- ⇒ Carefully open the box with the cover so as not to damage the contents.
- ⇒ Remove the cover and set it aside carefully for subsequent assembly.

4.2 Printer assembly

4.2.1 Installing the filament spool holder

- ⇒ Remove the screw (31) on the top rear using the Allen key provided.
- ⇒ Secure the filament spool holder (7) with the screw in this position.

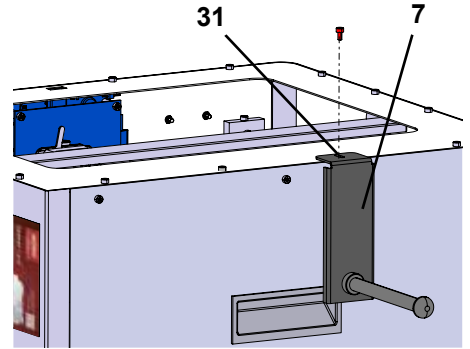
Fig. 2

4.2.2 Installing the filament sensor (FMS)

- ⇒ Remove the screw (30) on the top rear using the Allen key provided.
- ⇒ Secure the FMS (9) with the screw in this position.
- ⇒ Plug the FMS connection cable into the rear of the unit (14).

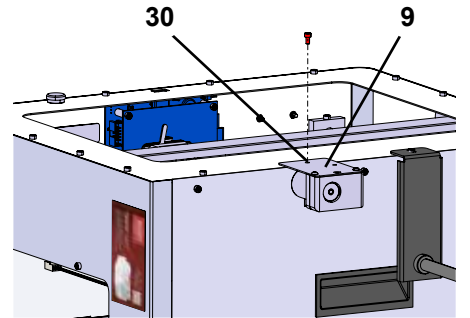
Fig. 3

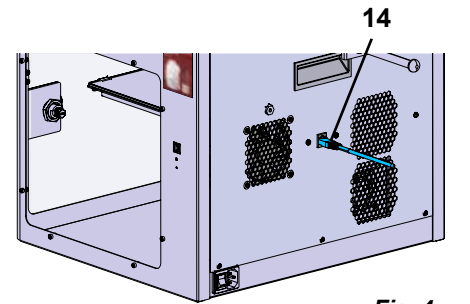
Fig. 4

4.2.3 Installing the filament guiding tube

- ⇒ Insert one end of the filament guiding tube (2) into the FMS (9) from above.
- ⇒ From above, insert the other end into the opening on the front of the extruder (10).

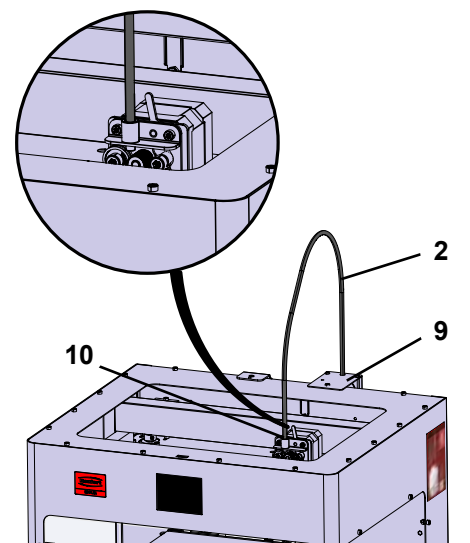
Fig. 5

4.2.4 Attaching the door

- ⇒ Hold the door at right angles to the unit on the hinge bolts.
- ⇒ Lower the door until the hinges thread into both hinge bolts.
- ⇒ Close the door.

4.2.5 Attaching the cover

- ⇒ Remove the four screws (32) on the top of the unit using the Allen key provided.
- ⇒ Place the cover into position.
- ⇒ Connect the fan to the socket (15) on the rear of the unit.

Also ensure subsequently that the SIMPLEX unit is switched off when you connect the fan as otherwise a short circuit could occur.

The cover does not necessarily have to be secured again with the screws.

A safety system detects when the cover is removed and then prevents any of the axes from moving.

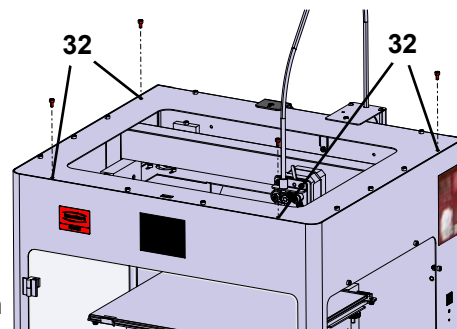
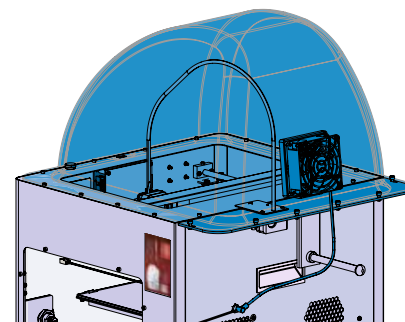
Fig. 6

15

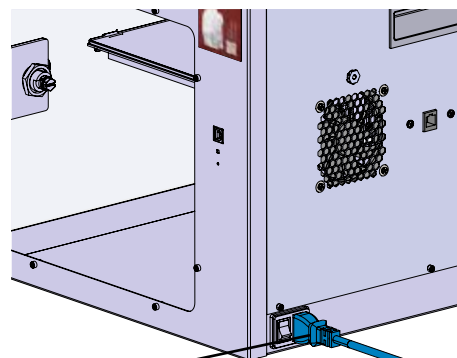
Fig. 7

4.2.6 Connecting the power cord

Ensure that the on/off switch (13) is in the OFF position.

- ⇒ Plug the power cord into the connection for the mains power supply (13) on the rear of the unit.
- ⇒ Insert the power plug into a building electrical socket.

13

Fig. 8

4.3 Switching on

- ⇒ Switch the unit on using the on/off switch (13, Fig. 1) on the rear of the unit.
 - ◆ The LCD display on the front lights up.
 - ◆ The main menu is shown:
 - Menu - TEMPERATURE 
 - Menu - EXTRUDER 
 - Menu - USB PRINT 
 - Menu - AXES 
 - Menu - SETTINGS. 

Fig. 9

5 Getting started

The following chapters describe the workflow that is required for initial startup in order to print an object that is provided on the supplied USB flash drive.

A prerequisite is the assembly of the printer as described above.

In addition, all of the following steps must be carried out in the order described.

The first steps include:

- Basic settings in the SETTINGS menu
 - Setting the language
 - Setting the temperature unit
 - Connecting the printer with a Wi-Fi network
- Leveling the print bed
- Moving the print head manually
- Loading the filament
 - Selecting a filament
 - Heating the print head
 - Drawing in the filament
 - Extruding the filament manually
- Calibrating the filament sensor (FMS)
- Loading the model from a USB flash drive
- 3D printing

Additional setting and control options offered by the printer are described in subsequent chapters.

Note on using the menu:

Use the lower left arrow ← to go back one page, exit a setting, cancel an operation, or return to the main menu.

5.1 Setting the language

A video on this topic is provided under the following link www.renfert.com/simplex-2-en and via the QR code shown.

- ⇒ Select the SETTINGS menu. .
- ⇒ Use the right arrow → to go to page 2 (as shown here in the image).
- ⇒ Press the button with the language icon .
- ⇒ Use the arrows ↑ / ↓ to select the required language.
- ⇒ Confirm your selection with OK.
- ⇒ Press the lower left arrow ← until you are back in the main menu.

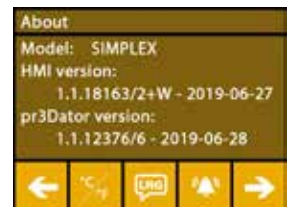
Fig. 10

Fig. 11

5.2 Setting the temperature unit

A video on this topic is provided under the following link www.renfert.com/simplex-3-en and via the QR code shown.

- ⇒ Select the SETTINGS menu.
- ⇒ Use the right arrow to go to page 2 (as shown here in the image).
- ⇒ Press the button with the temperature icon
- ⇒ The unit can be changed by pressing the arrow buttons / at the top.

The factory setting is °Celsius.

- ⇒ Press the lower left arrow until you are back in the main menu.

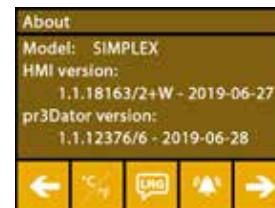
Fig. 12

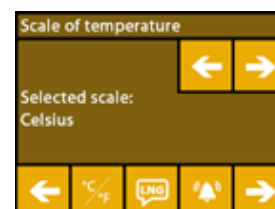
Fig. 13

5.3 Wi-Fi connection

A video on this topic is provided under the following link www.renfert.com/simplex-8-en and via the QR code shown.

The Wi-Fi module is only available in EU member states and in the USA. The button with the Wi-Fi icon can only be operated if the Wi-Fi module is available.

The Wi-Fi module may only be used as an information tool for monitoring printing. Print jobs cannot be started, paused, or stopped using the Wi-Fi module.

- ⇒ Select the SETTINGS menu.
- ⇒ Use the right arrow to go to page 4 (as shown here in the image).
- ⇒ Press the button with the Wi-Fi icon .
- ⇒ Press the button with the search icon to search for a Wi-Fi network.
 - ◆ A list of Wi-Fi networks is shown
- ⇒ Use the up/down arrows / and select the required Wi-Fi network using the checkmark .
- ⇒ Enter the password of the selected Wi-Fi network.
- ⇒ Confirm the password using the checkmark .

A connection was successfully established to the selected Wi-Fi network:

- ◆ The name of the connected Wi-Fi network is shown
- ◆ The IP address (LAN IP) allocated for the SIMPLEX 3D filament printer is shown.

A connection could not be established to the selected Wi-Fi network:

- ◆ An error message is shown:
"No connection to host <Name of the WiFi network>"

- ⇒ Press the lower left arrow until you are back in the main menu.

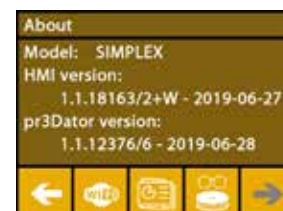
Fig. 14

Fig. 15

5.4 Leveling the print bed

A video on this topic is provided under the following link www.renfert.com/simplex-4-en and via the QR code shown.

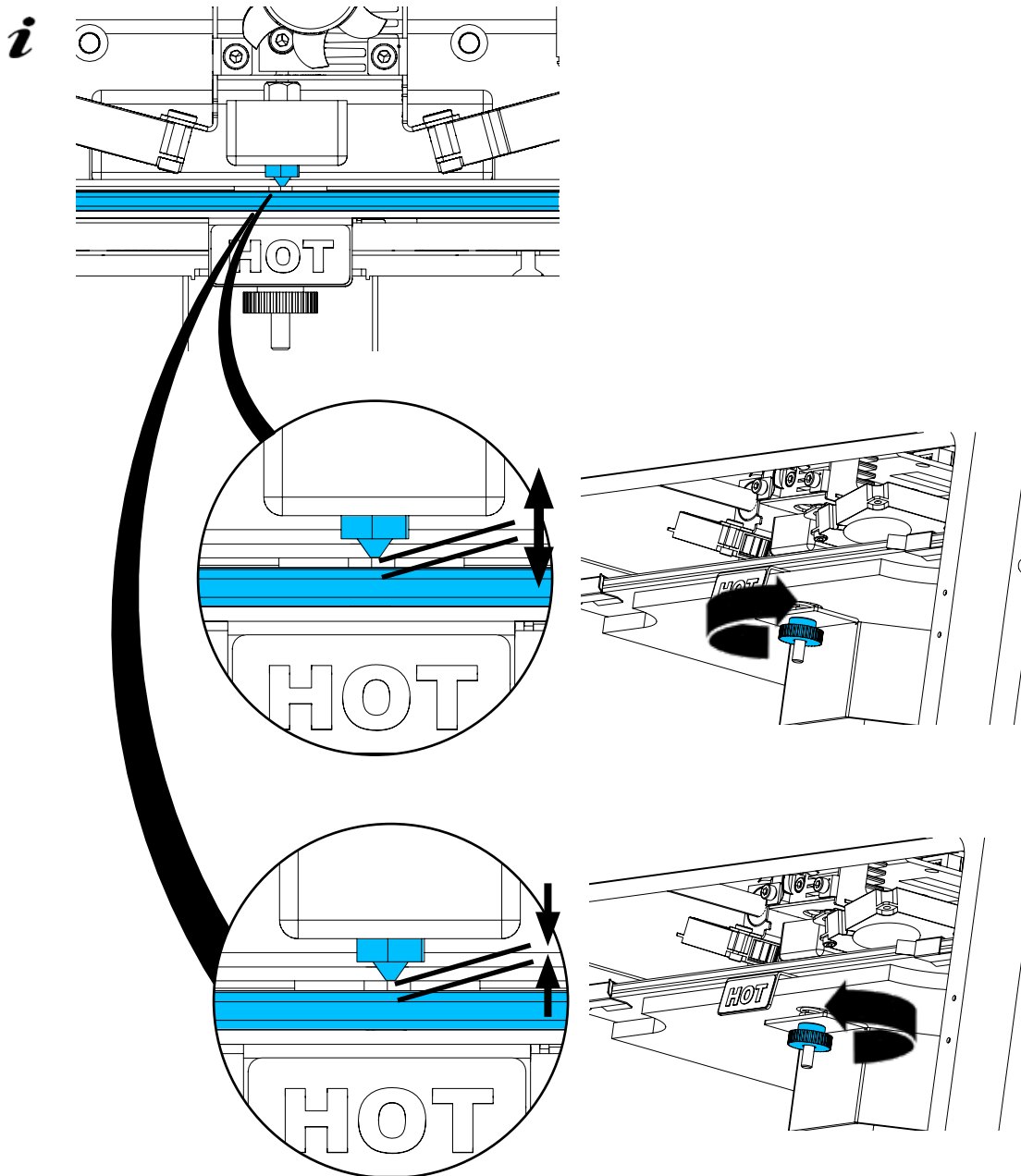
en

The SIMPLEX unit features a leveling process that ensures that the extruder and heated print bed are optimally spaced and leveled uniformly.

A smaller gap or uneven leveling can result in damage to the Kapton film (orange colored tape on the print bed), the print bed, the extruder, or in printing issues.

The gap between the extruder and the print bed is set using three adjustment screws under the print bed:

The print bed should be leveled after initial installation and approx. 1x every month.

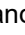
i You can ignore the "Please clean the nozzle!" message the first time the print bed is leveled.

! When you level the print bed again subsequently, ensure beforehand that a filament is not loaded, and that the nozzle is clean. Clean the nozzle carefully using a soft cloth. Wear gloves as the nozzle is at operating temperature at this point.

i You can cancel leveling at any time using the left arrow ← and return to the menu above.

i When prompted to do so, tighten the three leveling knobs (knurled screws) under the print bed. Do not overtighten them. You need to be able to adjust them for leveling.

i Alignment to three reference points is performed (one point after the other). Alignment to the three reference points is performed 2 x. You can correct leveling if required during the second run.

- ⇒ Ensure that there is no filament in the extruder!
- ⇒ Keep the level gauge (16, Fig. 1) ready at hand.
- ⇒ Select the SETTINGS menu. 
- ⇒ Press the button for the calibration menu .
- ⇒ Using the right arrow , scroll through the messages on the display and follow the instructions.
- ⇒ If necessary, wait until the extruder has reached the set temperature.
- ⇒ Once the temperature has been reached, use the right arrow  to perform leveling step by step.
- ⇒ Follow the instructions on the LCD display.
 - ◆ Alignment to three reference points is performed (one point after the other).
- ⇒ Set the corresponding leveling knob in each case so that a level gauge can slide between the nozzle and the print bed with minimum resistance.
- ⇒ Close the door after each step in order to proceed using the right arrow.
 - ◆ Alignment to the three reference points is performed 2 x.
- ⇒ At the second run, check the setting of the leveling knobs and correct it if required.
 - ◆ Once this operation has been completed, the print bed should be evenly leveled.
- ⇒ Using the right arrow , exit leveling and return to the menu above.
- ⇒ Press the lower left arrow  until you are back in the main menu.

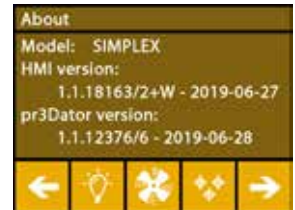
Fig. 16

Fig. 17

5.5 Moving the print head manually

A video on this topic is provided under the following link www.renfert.com/simplex-5-en and via the QR code shown.

The print head must be moved to an appropriate position for the following steps.

- ⇒ Select the AXES  menu.

i If the house icon is shown in black instead of white, first press the XYZ house icon  to move the print head to a defined home position.

i Ensure that there are no objects on the print bed when moving to the home position.

- ⇒ Press the button with the axis icon .
- ⇒ Press the Y+ and X+ arrow buttons to position the print head to the side outside of the print bed.
- ⇒ Press the lower left arrow  until you are back in the main menu.

Fig. 18

Fig. 19

5.5.1 Attaching the filament spool

⇒ Place the filament spool (8) on the filament spool holder (7) in such a way that the filament can be inserted into the filament sensor from underneath.

Ensure that the filament spool is placed on the filament spool holder in such a way that the filament can be inserted clockwise into the FMS from underneath (Fig. 20).

Only use Renfert filaments, as they do not result in any gases during printing that are harmful to health.

If using alternative filaments, ensure that no gases are released during printing that are harmful to health.

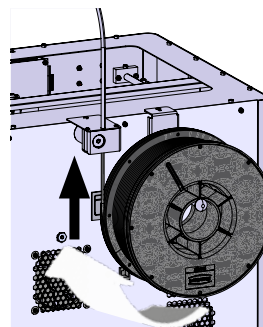
Fig. 20

5.6 Loading the filament

A video on this topic is provided under the following link www.renfert.com/simplex-6-en and via the QR code shown.

5.6.1 Selecting the filament / starting heating

For loading, the extruder must be pre-heated to operating temperature (= printing temperature).

The temperature values for the extruder (= head temp:) and the print bed (= bed temp:) of four different filaments are stored in the SIMPLEX 3D printer.

- TEMP1: SIMPLEX study model / working model / multi-use
- TEMP2: SIMPLEX aligner model
- USER1: Freely configurable
- USER2: Freely configurable

Chap. 7.2. explains how you can store customer temperature values for the USER1 and USER2 filaments.

⇒ Select the TEMPERATURE menu.

⇒ Using the filament spool icon , select the required filament.

Select the TEMP1 filament for the study model filament that is included with the delivery.

⇒ Press the extruder icon to start heating.

- ◆ On the display, the current temperature and the target temperature change from gray to white.

⇒ Press the lower left arrow until you are back in the main menu.

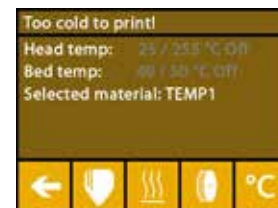
Fig. 21

5.6.2 Drawing in / extruding the filament

⇒ Remove the printer cover. When doing so, pay attention to the cable of the fan.

⇒ Pull the end of the filament through the safety hole of the filament spool.

⇒ Cut off the end of the filament in such a way that the filament is straight without any damage, bends, kinks, or melted parts (Fig. 22).

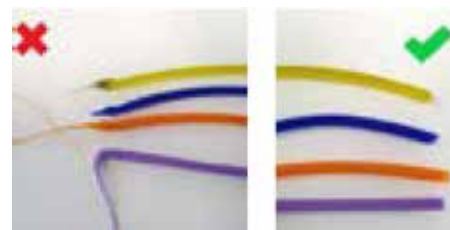
Fig. 22

- ⇒ Remove the filament guiding tube (2) through the opening on top of the extruder (10).
- ⇒ Insert the end of the filament from underneath through the FMS into the filament guiding tube until it exits on the other side.

Ensure that the filament does not spring off the spool or become tangled. The filament must be neatly wound on the spool. Improperly wound or knotted filaments can lead to printing errors or even misprints.

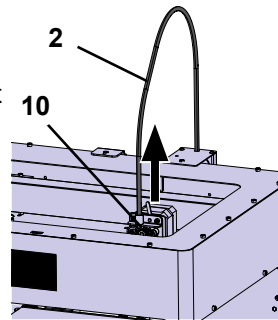
Fig. 23

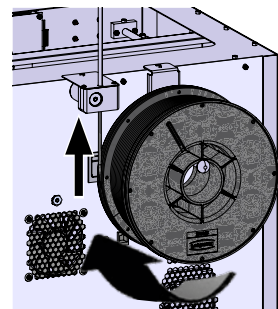
Fig. 24

- ⇒ Move the lever at the top of the extruder to the left (Fig. 25).

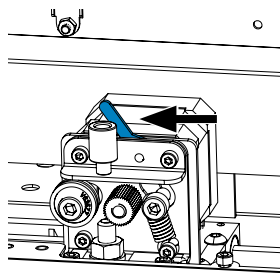
Fig. 25

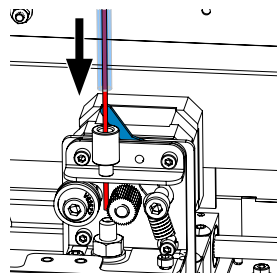
Fig. 26

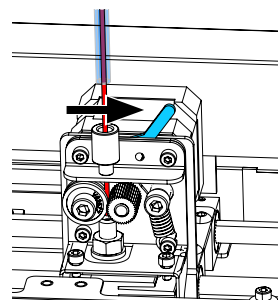
Fig. 27

- ⇒ Push the filament through the top opening of the extruder until the filament is between the two guide wheels (Fig. 26).
- ⇒ Now move the lever of the extruder to the right (Fig. 27)
- ⇒ Ensure that the filament is held by the two guide wheels by gently pulling on the filament and feeling resistance.

- ⇒ Heating of the extruder was already started in the previous step.
 - ◆ The head temperature is shown in white in the TEMPERATURE menu.
- ⇒ Wait until "Ready to print" is shown in the TEMPERATURE menu.
- ⇒ Select the EXTRUDER menu.

Fig. 28

If you select the EXTRUDER menu too early and the extruder has not yet reached the required temperature, a warning message is shown.

- ⇒ Skip the warning message with the right arrow →.

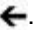
Fig. 29

⇒ Wait until the extruder has reached the required temperature. The current temperature and the required temperature are shown at the top of the display.

⇒ The filament can be extruded using the "EXTRUDE"  button. The cogwheel will only turn for as long as you press the button.

Alternatively:

⇒ Press the load filament  button. This automatically extrudes 15 cm of filament. This can be canceled by pressing the lower left arrow .

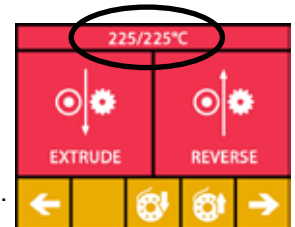
Fig. 30

⇒ Now insert the end of the filament guiding tube into the upper opening of the extruder again.

⇒ Place the cover on the printer. Pay attention to the cable of the fan.

If the fan cable has come loose, first switch off the printer before you insert the connector into the unit again.

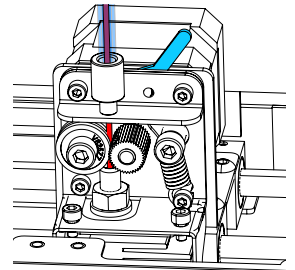
Fig. 31

5.7 FMS system

The filament sensor (FMS) controls the filament flow during printing.

If a jam is detected during printing, the printer attempts to fix this itself as part of an automatic fault correction process.

You must ensure that the FMS system is always calibrated with the currently loaded filament and the corresponding temperature. If this is not the case, the filament flow cannot be properly monitored during printing.

The FMS system is negatively impacted, and printing is not correctly monitored.

5.7.1 Calibrating the FMS system

A video on this topic is provided under the following link www.renfert.com/simplex-7-en and via the QR code shown.

The FMS system must be calibrated in order for it to operate correctly.

As a result of the previous steps, the extruder is already heated up and the filament is loaded.

⇒ Select calibration with the  icon.

The FMS system must be calibrated with the filament used correctly selected.

If you replace the filament, i.e., use a different type, the FMS system must be recalibrated.

⇒ If necessary, change the selected filament with the  icon.

⇒ Start the calibration using the start icon .

- ◆ The print head and print bed are moved to the home position.
- ◆ The print head is moved to a position beside the print bed.
- ◆ A preset quantity of material is extruded.

After successful calibration, this is shown on the display.

If calibration is not successful, an error message is shown.

⇒ In this case, repeat calibration.

After calibration has been completed, the FMS system is adapted to the filament used and can detect and display errors in the filament flow.

⇒ Press the lower left arrow  until you are back in the main menu.

Fig. 32

Fig. 33

5.8 Printing an object

A video on this topic is provided under the following link www.renfert.com/simplex-10-en and via the QR code shown.

To print a 3D object, SIMPLEX requires toolpath information that is generated from 3D object CAD files such as .obj or .stl. Toolpath information is known as a "G-code".

Sample G-codes are stored on the USB flash drive provided. You can use these to immediately carry out test prints.

Make sure to use the G-code for the currently loaded filament. Using an incorrect G-code can cause misprints or damage to the printer.

The following G-codes are available:

G-Code SIMPLEX study model, working model, multi-use model ==> Filament TEMP1

G-Code SIMPLEX aligner model ==> Filament TEMP2

5.8.1 Loading an object from a USB flash drive

- ⇒ Switch off the SIMPLEX unit (13).
- ⇒ Insert the USB flash drive into the USB port (11) on the SIMPLEX unit.
- ⇒ Switch on the SIMPLEX unit (13).

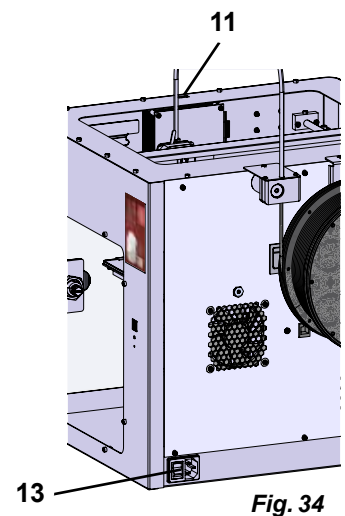
Fig. 34

- ⇒ Select the USB  menu.
- ⇒ Use the arrows  /  to select the file name of the object.

Using the right arrow  and the "up directory"  icon, you can navigate through the directory structure of the USB flash drive.

- ⇒ Press the printer icon  to start printing.

Fig. 35

- ◆ The unit warms the extruder and the print bed to the temperatures stored in the G-code.
- ◆ As long as the temperatures have not yet been reached, the two letters "H" and "B" are shown in red on the display.
 - H: Head = Print head
 - B: Bed = Print bed
- ◆ As soon as the stored temperatures are reached with sufficient accuracy, the color of the letters changes to white, and printing begins.

Fig. 36

During printing, the following information is shown on the display:

- Name of the file to be printed
- Head (H) and bed (B) temperature
- Z-position information
- Printing speed
- Filament: remaining / total
- Progress bar with information in % as well as elapsed time / total time required.

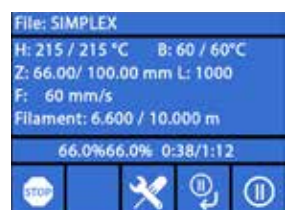
Fig. 37

Printing is complete:

- ◆ For easier removal of the object, the print bed is automatically lowered a little and the print head is moved to the home position.
- ◆ A corresponding message is shown on the display. This can be confirmed with "OK".

Fig. 38

Caution! Danger of burns!

The print head and print bed could still be hot.

Wear protective gloves!

- ⇒ If necessary, you can use the "AXES" menu (see Chap. 5.5) to lower the print bed further.
- ⇒ Remove the object from the print bed. Carefully remove the object with one hand using a slight twisting motion while holding the print bed with the other hand. If the object is difficult to remove, use the spatula provided.

Take care not to scratch the Kapton film.

So that removal of the object is as easy as possible, wait until the print bed has cooled.

If necessary, you can also remove the build plate from the printer to remove the object.

To do this, loosen the fixing clasps on the front right and left of the print bed and remove the build plate. When reinserting, ensure that the build plate has been inserted in the correct end position and that the fixing clasps are closed.

An incorrectly seated build plate can cause misprints or damage to the printer.

6 Other settings

In the "Getting started" section, the following items from the SETTINGS menu are already explained:

- Setting the language.
- Setting the temperature unit.
- Connecting to a Wi-Fi network.
- Leveling the print bed.
- Calibrating an FMS system.

Additional options in the SETTINGS menu are presented below.

6.1 Setting the date/time / resetting the operating hours meter

A video on this topic is provided under the following link www.renfert.com/simpl-11-en and via the QR code shown.

You can set the date/time and reset the operating hours meter. This function helps you to track operating hours between maintenance work.

- ⇒ Select the SETTINGS menu. .
- ⇒ Use the right arrow → to go to page 3 (as shown here in the image).
- ⇒ Press the button with the clock icon .
- ⇒ Use the arrows < / > to toggle between hour / minute / year / month / day.
- ⇒ Use the arrows ↑ / ↓ to set the date and time.
- ⇒ Press the lower left arrow ← until you are back in the main menu.

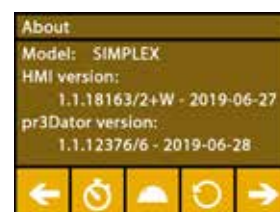
Fig. 39

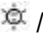
The operating hours meter, which can be used to track the time elapsed between two maintenance sessions, can be reset with the reset button . The operating hours of the last reset are saved under "Last reset".

Fig. 40

6.2 Setting the display backlight

- ⇒ Select the SETTINGS menu. .
- ⇒ Press the button with the light bulb icon .
- ⇒ Adjust the brightness using the slide control or switch it on or off using the  /  buttons.
- ⇒ Press the lower left arrow  until you are back in the main menu.

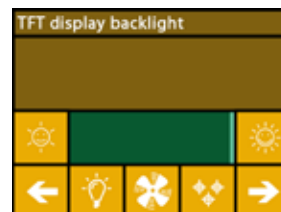
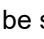
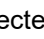
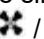
Fig. 41

6.3 Activating the fan

- ⇒ Select the SETTINGS menu. .
- ⇒ Press the button with the fan icon .
- ⇒ The fan can be selected by pressing the arrow buttons  /  at the top:
 - ◆ The name of the fan to be controlled is shown on the display:
 - Object fan
 - Extruder fan
 - Case fan
- ⇒ Set the rotational speed using the slide control or switch it off / set the maximum rotational speed using the  /  buttons.

i *The rotational speed of the extruder fan can only be reduced when the extruder temperature is below 60 °C [140 °F].*

i *The extruder and case fans are automatically switched off again after 60 seconds.*

i *The setting of the object fan can also be modified during printing.*

i *In Renfert mode, the printing parameters are stored for a perfect printing result. All of the necessary settings for orthodontic applications are optimally pre-configured and tailored to the filament. This ensures a high level of process reliability and simplicity. If the fan settings are changed, the pre-installed parameters are modified during printing. This can impair print quality, cause misprints, or cause damage to the printer.*

- ⇒ Press the lower left arrow  until you are back in the main menu.

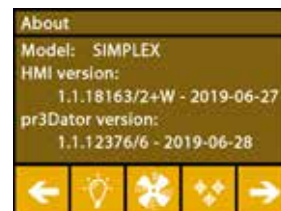
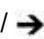
Fig. 42

Fig. 43

6.4 Audio settings

- ⇒ Select the SETTINGS menu. .
- ⇒ Use the right arrow  to go to page 2 (as shown here in the image).
- ⇒ Press the button with the bell icon .
- ⇒ Use the arrows  /  to choose between three options:
 - Sound: On
 - Sound: Off
 - Warnings only
- ⇒ Press the lower left arrow  until you are back in the main menu.

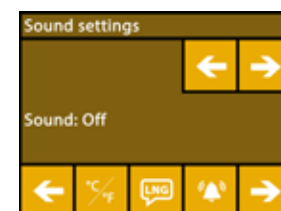
Fig. 44

6.5 The cover menu

- ⇒ Select the SETTINGS menu. .
- ⇒ Use the right arrow  to go to page 3 (as shown here in the image).
- ⇒ Press the button with the cover icon .
- ◆ When the cover is mounted, you can activate the cover fan here by pressing the ON/OFF button.
- ◆ If the cover fan has been enabled, it is switched on during a print job.
- ⇒ Press the lower left arrow  until you are back in the main menu.

i *The cover fan is enabled by default on delivery. If it is deactivated, heat can build up during the printing process causing misprints.*

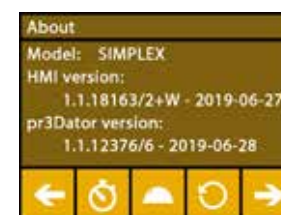
Fig. 45

6.6 Using the SIMPLEX Wi-Fi

i *The Wi-Fi module is only available in EU member states and in the USA. The button with the Wi-Fi icon  can only be operated if the Wi-Fi module is available.*

i *The Wi-Fi module may only be used as an information tool for monitoring printing. Print jobs cannot be started, paused, or stopped using the Wi-Fi module.*

SIMPLEX also provides Wi-Fi as a host.

In order to use it, make the following settings:

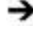
- ⇒ Select the SETTINGS menu. .
- ⇒ Use the right arrow  to go to page 4 (as shown here in the image).
- ⇒ Use the button with the SSID icon  to enter the name of the SIMPLEX Wi-Fi. The entry is completed with a checkmark .
- ⇒ Set a password for this network using the button with the key icon . The password must be at least 8 characters long. The entry is completed with a checkmark .
- ⇒ Use the button with the eye icon  /  to switch visibility of the network on/off:
 - ◆  : The Wi-Fi is visible.
 - ◆  : The Wi-Fi is not visible.

Fig. 46

6.7 The FMS system

The filament sensor (FMS) controls the filament flow during printing.

i *You must ensure that the FMS system is always calibrated with the currently loaded filament and the corresponding temperature. If this is not the case, the filament flow cannot be properly monitored during printing.*

The FMS system is negatively impacted, and printing is not correctly monitored.

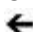
If a jam is detected during printing, the printer attempts to fix this itself as part of an automatic fault correction process.

Automatic fault correction process:

- The printer switches to pause status and the print head moves to the pause position.
- The filament is retracted and extruded.
- This operation is repeated three times.
- The printer attempts to extrude 6 cm of filament to see if this causes blockages again.
- If not, printing is automatically resumed.
- If the blockage still remains, an error message is shown on the LCD display if the sensor is activated.
- If the error has been corrected by the user, they can press resume printing to continue printing immediately.

6.8 Resetting settings

You can reset all settings.

- ⇒ Select the SETTINGS menu. .
- ⇒ Use the right arrow  to go to page 3 (as shown here in the image).
- ⇒ Press the button with the reset icon .
- ⇒ Press the  button again.
- ⇒ Confirm that you want to reset the settings with  or select the lower left arrow  to cancel.

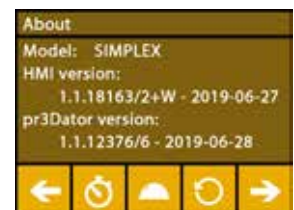
Fig. 47

Fig. 48

7 Operation

7.1 Saved filaments

The temperature values for the extruder (= head temp:) and the print bed (= bed temp:) of four different filaments are stored in the SIMPLEX 3D printer.

- TEMP1: SIMPLEX study model / working model / multi-use
- TEMP2: SIMPLEX aligner model
- USER1: Freely configurable
- USER2: Freely configurable

7.2 Setting a custom print and bed temperature

The values for the TEMP1 and TEMP2 filaments are optimized and cannot be adjusted! Only the values for the USER1 and USER2 filaments can be changed.

The temperatures set here only apply for loading/unloading the filament and for manual extrusion. When printing a model, the temperatures stored in the G-code of the model are used.

Changing the values for the USER1 and USER2 filaments:

- ⇒ Select the TEMPERATURE menu or the temperature icon in the EXTRUDER menu.
- ⇒ Press the filament spool button until the required filament (USER1 / USER2) is shown.
- ⇒ Press the °C button
- ◆ The settings window for the extruder and print bed temperature is opened.
- ⇒ Use the arrow keys / to set the required temperature for the extruder and the print bed .
- ⇒ Exit the settings window with the lower left arrow .

Fig. 49

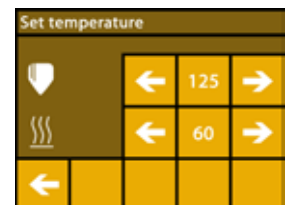
Fig. 50

7.3 Heating the printer

A video on this topic is provided under the following link www.renfert.com/simplex-6-en and via the QR code shown.

To load/unload the filament and for manual extrusion, the printer must be heated in accordance with the type of filament used.

If heating is switched on (extruder or print bed), this is signaled by showing the corresponding temperature in white on the displays that show temperature.

If the temperature is shown in gray, the corresponding heating is switched off.

To load/unload the filament, only the extruder should be pre-heated.

At room temperatures < 18 °C [64,4 °F] heat the extruder and print bed for 10 minutes before you begin printing. This ensures that the temperature in the printing chamber is optimal for printing.

When using TEMP1 and TEMP2 group filaments, always load/unload the filament with the TEMP2 setting.

- ⇒ Select the TEMPERATURE menu or the temperature icon in the EXTRUDER menu.
- ⇒ Using the filament spool icon , select the required filament.
- ⇒ Press the extruder icon to start heating.
 - ◆ On the display, the current temperature and the target temperature change from gray to white.
 - ◆ As soon as the extruder has reached the target temperature of the filament, you can load/unload the filament or start printing.
- ⇒ You can return to the main menu using the lower left arrow .

Fig. 51

7.4 Loading the filament

A video on this topic is provided under the following link www.renfert.com/simplex-6-en and via the QR code shown.

en

The procedure for loading a filament can be found in the "Getting Started" section.

7.5 Extrusion

A video on this topic is provided under the following link www.renfert.com/simplex-6-en and via the QR code shown.

The procedure for manual extrusion can be found in the "Getting Started" section.

7.6 Removing the filament from the extruder

A video on this topic is provided under the following link www.renfert.com/simplex-6-en and via the QR code shown.

Never remove the filament from the extruder as long as the extruder is cold. You must always pre-heat the extruder before removing or replacing filaments. Removing the filament from a cold extruder can damage the unit.

You do not need to remove the filament between the print runs.

When using TEMP1 and TEMP2 group filaments, always load/unload the filament with the TEMP2 setting.

⇒ Select the TEMPERATURE  menu.

⇒ Using the filament spool icon , select the filament that is currently loaded in the printer.

⇒ Press the extruder icon  to start heating.

◆ On the display, the current temperature and the target temperature change from gray to white.

⇒ Press the lower left arrow  until you are back in the main menu.

⇒ Select the EXTRUDER  menu.

If you select the EXTRUDER menu too early and the extruder has not yet reached the required temperature, a warning message is shown.

⇒ Skip the warning message with the right arrow .

Fig. 52

⇒ Remove the printer cover. Pay attention to the cables of the fan.

⇒ Wait until the extruder has reached the required temperature. The current temperature and the required temperature are shown at the top of the display.

⇒ Press the backwards button . The cogwheel will only turn each time you press the button.

Alternatively:

⇒ Press the button for unloading the filament  to automatically perform the operation.

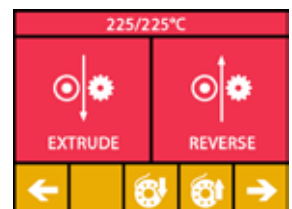
Fig. 53

With the function for unloading the filament, the filament is briefly drawn into the extruder before it is automatically unloaded. This heats up the filament in the front area slightly so that it is easier to unload and pull out of the guiding tube.

When the cogwheels start to pull back the filament, you can safely remove the rest of the filament from the guiding tube and connect the end of the filament to the spool for safe storage.

8 Moving the print head and the print bed

In the AXES menu, you can manually move the print head and the print bed as required or automatically move them to their home position.

Ensure that there are no objects on the print bed when moving to the home position.

The home position of the print head is in the front left corner of the build plate.

The home position of the print bed is the uppermost position.

⇒ Select the AXES  menu.

Moving to the home positions

⇒ Press the button with the house icon :

⇒ The axes can be activated individually using the button with the corresponding house icon  /  /  and moved to the home position.

- ◆ Black house icons indicate motors that have been switched off.
- ◆ By pressing the button, the corresponding motors are switched on and alignment to the home position is initiated.

Automatic alignment to the home position can be canceled using the button with the motor icon . Motors that are currently in motion are stopped and switched off.

Move the axes as required

⇒ Press the button with the axis icon .

⇒ The axes can be moved individually as required by pressing the arrow keys.

Here, the designation X / Y / Z means:

X: The print head moves to the right (+) / to the left (-)

Y: The print head moves forwards (-) / backwards (+)

Z: The print bed moves up (-) / down (+)

Motors that have been switched off cannot be moved. The corresponding arrow keys are gray.

In order to be able to move an axis as required, it must first be moved to its home position.

If you want to move the print head by hand, you must switch off the motors of the X and Y axes beforehand using the button with the motor icon .

⇒ Press the lower left arrow  until you are back in the main menu.

Fig. 54

Fig. 55

9 SIMPLEX sliceware / SIMPLEX print

The SIMPLEX print software is required to print 3D objects directly from a PC.

To print a 3D object, the SIMPLEX 3D filament printer requires tool path information (G-codes).

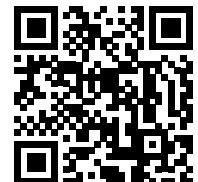
Use SIMPLEX sliceware to generate this G-code from 3D object files (e. g., *.stl or *.obj).

You can download both softwares via “myRenfert”.

To do this, you must log in to “myRenfert” and register your SIMPLEX 3D filament printer

You can find a description of the procedure under:

<https://qrco.de/QSGSIMPLEXSliceware>

10 Printing objects

To print a 3D object, SIMPLEX requires toolpath information that is generated from 3D object CAD files such as .obj or .stl. Toolpath information is known as a "G-code".

The SIMPLEX sliceware program converts 3D design files into a G-code.

You can obtain the SIMPLEX sliceware program via the Renfert CONNECT app (see Chap. 9)

You can use the SIMPLEX print program to send the generated G-code to the printer via a USB cable and print it out, or copy it to a USB flash drive and use it to create a print object.

Ensure that the print bed has been correctly leveled before printing (see Chap. 5.4).

If you use a different filament for printing than for the previous print object, you must recalibrate the FMS system (see Chap. 5.7.2).

10.1 Cleaning the Kapton film

The Kapton film applied to the glass plate improves the adhesion of the model to the print bed and prevents partial or complete detachment of the model.

Make sure the Kapton film is clean, undamaged, and facing up when you insert the glass plate into the printer.

⇒ Clean the Kapton film before each print job.

⇒ Wear gloves when cleaning to avoid grease residue.

⇒ Move the print bed to a convenient position for cleaning ("AXES" menu, Chapter 8).

⇒ Clean the Kapton film only with spirit and/or denatured alcohol and wipe it with a dry cloth or paper towel.

10.2 Printing using a USB flash drive

⇒ Connect the USB flash drive to your PC.

⇒ Save the G-code generated with SIMPLEX sliceware on the USB flash drive provided.

The file name must not contain any special characters or umlauts. These cannot be read by the printer. Printing cannot be started, and an error message is shown.

⇒ Switch off the SIMPLEX unit (ON/OFF switch 13, Fig. 1).

⇒ Insert the USB flash drive into the USB port (11, Fig. 1) on the SIMPLEX unit.

⇒ Switch on the SIMPLEX unit (ON/OFF switch, 13, Fig. 1).

⇒ Select the USB  menu (Fig. 9).

⇒ Use the arrows  /  to select the file name of the object.

Using the right arrow  and the "up directory"  icon, you can navigate through the directory structure of the USB flash drive.

Ensure that you have the right filament and enough of it for the selected project. The quantity was calculated by SIMPLEX sliceware.

⇒ Press the printer icon  to start printing.

◆ The unit warms the extruder and the print bed to the temperatures stored in the G-code.

◆ As long as the temperatures have not yet been reached, the two letters "H" and "B" are shown in red on the display.

H: Head = Print head

B: Bed = Print bed

◆ As soon as the stored temperatures are reached with sufficient accuracy, the color of the letters changes to white, and printing begins.

During printing, the following information is shown on the display:

- Name of the file to be printed
- Head (H) and bed (B) temperature
- Z-position information
- Printing speed
- Filament: remaining / total
- Progress bar with information in % as well as elapsed time / total time required.

Fig. 56

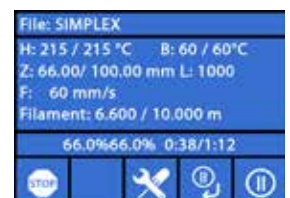
Fig. 57

Printing is complete:

- ◆ For easier removal of the object, the print bed is automatically lowered a little and the print head is moved to the home position.
- ◆ A corresponding message is shown on the display. This can be confirmed with "OK".

Fig. 58

Caution! Danger of burns!

The print head and print bed could still be hot.
Wear protective gloves!

- ⇒ If necessary, you can use the "AXES" menu to lower the print bed further.
- ⇒ Remove the object from the print bed. Carefully remove the object with one hand using a slight twisting motion while holding the print bed with the other hand. If the object is difficult to remove, use the spatula provided.

Take care not to scratch the Kapton film.

So that removal of the object is as easy as possible, wait until the print bed has cooled.

If necessary, you can also remove the build plate from the printer to remove the object.

To do this, loosen the fixing clasps on the front right and left of the print bed and remove the build plate. When reinserting, ensure that the build plate has been inserted in the correct end position and that the fixing clasps are closed.

An incorrectly seated build plate can cause misprints or damage to the printer.

10.3 Actions during USB printing

When printing using a USB flash drive, you can intervene in the following ways:

- : Stop
- : Settings / optimization
- : Pause next layer
- : Pause

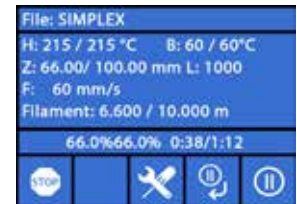
Fig. 59

10.3.1 Stop

- ⇒ To cancel printing, press the stop button.
 - ◆ A confirmation prompt asks you whether you really want to cancel the printing process.
- ⇒ To resume printing, press the left arrow .
- ◆ Printing continues without further interruption.
- ⇒ To cancel printing, press the stop button again.
 - ◆ Printing is stopped immediately.
 - ◆ The print head heating and print bed heating are switched off.
 - ◆ You are in the main menu (Fig. 9).
- ⇒ Select the AXES menu (Fig. 9) and the axes icon in that menu:
- ⇒ Move the print bed down a little (Z+) in order to be able to remove the object that has been printed up to that point.

Fig. 60

Caution! Danger of burns!

The print head and print bed could still be hot. Wear protective gloves!

- ⇒ Remove the object.

10.3.2 Settings / optimization

By repeatedly pressing the settings/optimization button , you can make successive changes to the following settings:

- Temperature
- Printing speed / flow rate
- Backlight / brightness of the display
- Object fan control
- FMS settings

Changes you make here overwrite the settings in the G-code.

Caution!

In Renfert mode, the printing parameters are stored for a perfect printing result. All of the necessary settings for orthodontic applications are optimally pre-configured and tailored to the filament. This ensures a high level of process reliability and simplicity. Changing these settings modifies the pre-installed parameters. In this case, Renfert does not guarantee printing quality.

Temperature:

The nozzle and print bed temperature can be changed via the object fan of the head. Set the object fan speed. This is unlocked by default. When locked, the G-code settings cannot be overwritten.

Fig. 61

Printing speed / flow rate:

The printing speed and extrusion volume can be changed as a percentage of the original. Use the reset button to return to the G-code values.

Caution!

In Renfert mode, the printing parameters are stored for a perfect printing result. All of the necessary settings for orthodontic applications are optimally pre-configured and tailored to the filament. This ensures a high level of process reliability and simplicity. Changing these settings modifies the pre-installed parameters. In this case, Renfert does not guarantee printing quality.

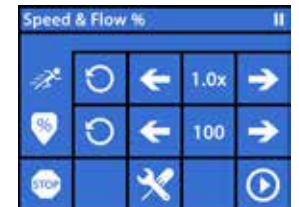
Fig. 62

Backlight / brightness of the display:

Changing the brightness of the display backlight.

Object fan control

Set the rotational speed using the slide control or set a minimum/maximum rotational speed using the  /  buttons.

Fig. 63

10.3.3 Pause next layer

The "pause next layer" function stops printing when printing of the current layer has finished.

Activating the "pause next layer" function:

- ⇒ Press the  button (see Fig. 59).
 - ◆ The "pause next layer" function is activated.
 - ◆ The icon shown changes to .
 - ◆ Once the layer is complete, the printer switches to the pause menu.

Deactivating the "pause next layer" function:

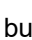
- ⇒ Press the  button (see Fig. 59).
 - ◆ The "pause next layer" function is deactivated again.
 - ◆ The icon shown changes to .
 - ◆ Printing is resumed without interruption.

10.3.4 Pause

The pause function can be used, for example, to replace the filament.

⇒ Press the  button (see Fig. 59).

- ◆ The icon changes to .
- ◆ Printing is interrupted
- ◆ The print head is moved to the left edge so that the object is not damaged when a new filament is inserted.

If you would like to replace the filament, use the  /  or  /  buttons to unload / load the filament and proceed as described in Chap. 7.6 / 7.4.

Resuming printing:

⇒ Press the  icon.

- ◆ The print head moves back to the object
- ◆ Printing is resumed.

Fig. 64

10.4 Printing with SIMPLEX print

If the SIMPLEX 3D-filament-printer is connected directly to a computer using the USB cable, objects can be printed with SIMPLEX sliceware using SIMPLEX print.

i *The print process using SIMPLEX print cannot be followed on the display of the SIMPLEX 3D-filament-printer.*

For this reason, the alternative is available of printing using a USB flash drive. (Chap 10.2 / 10.3).

If printing is carried out using the SIMPLEX print program, printing is controlled exclusively via SIMPLEX print.

Only the main menu is shown on the SIMPLEX 3D filament printer.

Actions should not be performed via the display.

The options described in the previous chapter (10.3) are only available with SIMPLEX print.

Fig. 65

10.4.1 Connecting the printer via USB cable

- ⇒ Connect the USB cable to the USB outlet of the SIMPLEX.
- ⇒ Connect the cable to a USB port on your computer.
- ⇒ Switch on the printer.

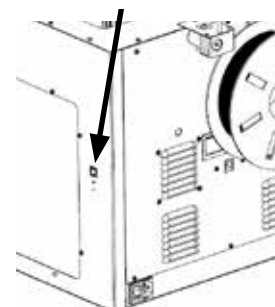
Fig. 66

10.4.2 Automatically starting SIMPLEX print

When you click the "Print" command in SIMPLEX sliceware, SIMPLEX print is automatically started.

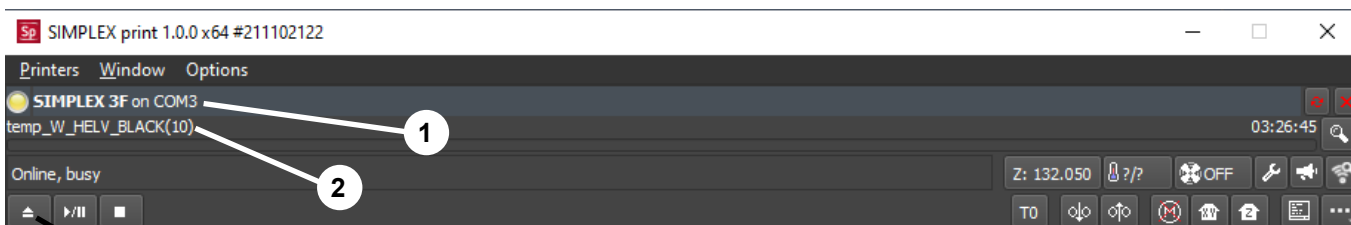
Fig. 67

- ⇒ Close the error message with "OK".
- ⇒ Click "Printers" and "Detect SIMPLEX 3D printers" to activate the connected SIMPLEX 3D-filament-printer (alternatively, press CTRL+R).
 - ◆ The SIMPLEX is shown in the status bar (1, Fig. 67)
- ⇒ Select and load the G-Code by clicking the load button (3, Fig. 67).
 - ◆ The corresponding file name is shown in the status bar (2, Fig. 67)

You can use the control keys at the bottom right of the program window to remotely control the SIMPLEX 3D printer.

The following actions are possible:

- Adjust the print bed to a specified height.
- Adjust the print bed to the home position.
- Adjust the tool head to the home position.
- Switch off the motors.
- Manual extrusion

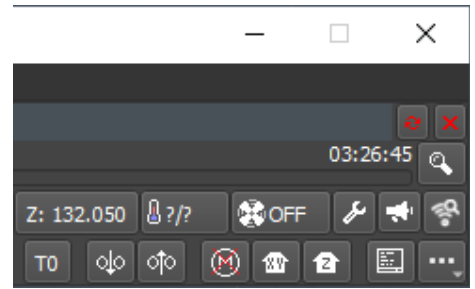
Fig. 68

10.4.3 Print process using SIMPLEX print

Before starting the print process, check that the correct filament has been placed in the SIMPLEX!

The print process is started in SIMPLEX print by clicking the start / pause button.

The extruder and print bed are heated to the temperatures specified in the G-Code.

Printing begins as soon as the required temperatures have been reached.

You can pause printing using the pause button and cancel it using the stop button.

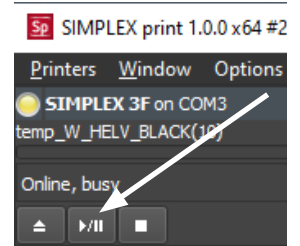
Fig. 69

You can use the control keys at the bottom right of the program window to control the print process.

The following actions are possible:

- Modify the printing speed
- Adjust the quantity to be extruded
- Switch the cover fan on / off
- Modify the extruder temperature
- Modify the temperature of the print bed

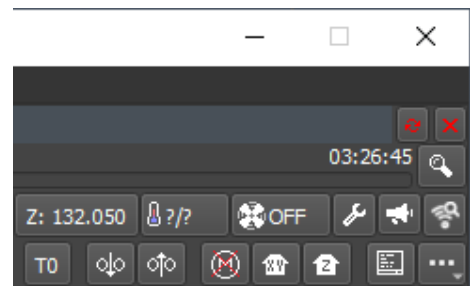
Fig. 70

For printing, a larger amount of G-Code data is always sent to the SIMPLEX.

There is no response to the pause / stop button or control key commands until the G-Code that has already been sent has been fully processed. In some cases, this can take several minutes.

10.4.4 Starting SIMPLEX print manually

Alternatively, you can also start and use SIMPLEX print manually using the start menu of your computer.

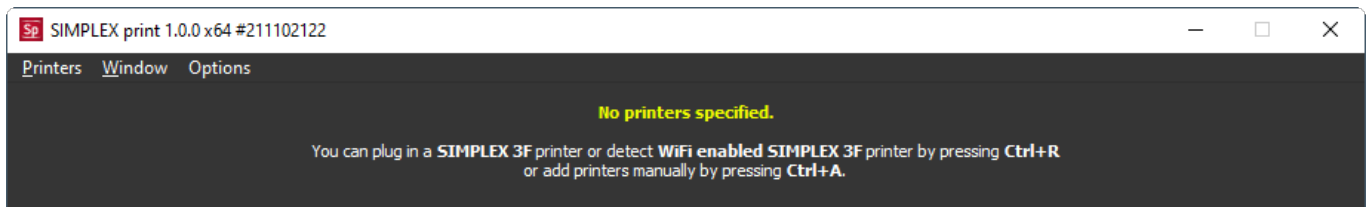
Fig. 71

It is also the case here that you must activate the connected SIMPLEX 3D-filament-printer and load the G-Code (see description in Chapter 10.4.2).

11 Cleaning / Maintenance

Opening the unit is not permitted other than as described below!

Caution! Danger of burns!

The print head / extruder and print bed could still be hot.

Allow the SIMPLEX unit to cool down for at least 30 minutes before cleaning and maintenance work.

Only carry out cleaning and maintenance work on the SIMPLEX unit if it has been switched off.

After 1,000 hours of operation, the printer must be cleaned, and used and worn parts in the printer replaced.

The printer shows a corresponding warning message.

If this display is shown, at least the following components must be replaced:

- Nozzle
- Kapton film

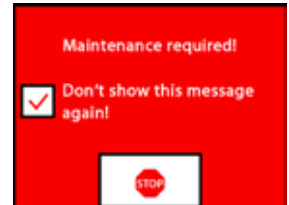
Fig. 72

Service and oil the recommended parts as often as recommended in the Cleaning section. Only use the maintenance materials recommended by Renfert.

11.1 Cleaning

⇒ Clean the housing and the cover of the SIMPLEX 3D filament printer with a damp cloth.

Cleaning the print bed:

⇒ Move the print bed into a position that facilitates cleaning (AXES , menu), Chap. 7).

⇒ Only clean the print bed with spirit-based glass cleaner or pure alcohol and wipe with a dry cloth or paper towel.

11.2 Lubrication

The SIMPLEX 3D filament printer should be lubricated at least once after every 50 hours of operation or every 6 months.

Only lubricate the following parts:

For the print bed:

- Guide rods of the z-axis
- Threaded rod of the z-axis

For the print head:

- Guide rods of the x and y-axis

The following tools are required for lubrication:

- PTFE-based spray lubricant or grease
- 2 lint-free cloths or thicker paper towels
- Gloves and safety glasses
- Light to adequately illuminate the interior of the SIMPLEX unit.

11.2.1 Lubrication of the z-axis

The guide rod and threaded rod of the z-axis are located inside the printer at the rear.

⇒ Select the AXES , menu (Fig. 9).

⇒ Adjust the print head and the print bed to the home position .

⇒ Switch off the printer and disconnect the power plug.

⇒ Slide a folded paper towel or lint-free cloth between the threaded rod and the back to protect other areas of the printer from any overspray that may occur.

⇒ Spray the threaded rod at a relatively short distance to avoid overspraying. Ensure that the lubricant reaches the inside of each thread. If you use grease, use gloves to spread it.

⇒ Be careful not to over-lubricate, only spray the minimum amount required.

⇒ Do not spray the two guide rods directly! Spray the lubricant onto a cloth and use it to lubricate the guide rods.

⇒ Remove the cloth.

- ⇒ Connect the power plug and switch on the printer.
- ⇒ Select the AXES  menu (Fig. 9).
- ⇒ Move the print bed to the lowest position .
- ⇒ Switch off the printer and disconnect the power plug.
- ⇒ Slide a folded paper towel or lint-free cloth between the threaded rod and the back to protect other areas of the printer from any overspray that may occur.
- ⇒ Place a second clean cloth on the print bed to protect the components and the Kapton film from spatter.
- ⇒ Lubricate the threaded rods and guide rods again as previously described.
- ⇒ Connect the power plug and switch on the printer.
- ⇒ Select the AXES  menu (Fig. 9).
- ⇒ Adjust the print head and the print bed to the home position .
- ⇒ Switch off the printer and disconnect the power plug.
- ⇒ Remove excess lubricant or grease underneath the print bed.
- ⇒ Connect the power plug and switch on the printer.
- ⇒ Select the AXES  menu (Fig. 9).
- ⇒ Move the print bed to the lowest position .
- ⇒ Switch off the printer and disconnect the power plug.
- ⇒ Remove excess lubricant or grease above the print bed.

11.2.2 Lubrication of the x/y-axis

- ⇒ Connect the power plug and switch on the printer.
 - ⇒ Select the AXES  menu (Fig. 9).
 - ⇒ Adjust the print head and the print bed to the home position .
 - ⇒ Switch off the printer and disconnect the power plug.
- With the motors switched off, the print head can be moved forwards/backwards and left/right by hand.
- ⇒ Lubricate the x and y-axis guide rods with an oiled cloth.

Do not spray lubricant or oil into the unit from above!

- ⇒ If the print head moves easily in all directions, lubrication is sufficient.

11.3 Unblocking the extruder

A blocked extruder is the most common problem with any 3D filament printer and cannot be prevented entirely.

When loading/unloading the filament, a blockage can often be avoided by waiting until the extruder has reached the temperature corresponding to the filament.

It is especially important to remove old filament residue between using different filaments.

Different filament properties (even printing the same material in a different color) can cause blockages due to inconsistencies.

Do not switch off the SIMPLEX 3D filament printer until the nozzles have cooled down (< 60°C). (Note: Extruder fan shuts down at < 60°C). If you switch off the SIMPLEX 3D filament printer when the extruder is hot, the temperature can spread from the extruder into the heatbreak, melt the filament and lead to clogging after the extruder has cooled..

You can clear a blockage in the extruder with the smallest of the Allen keys provided.

**Caution! Danger of burns!
The blockage must be cleared while the print head / extruder is hot.
Wear protective gloves!**

- ⇒ If you have not already done so, remove the filament (see Chap. 7.6).
- ⇒ Move the lever at the top of the extruder to the left.
- ⇒ Heat the extruder to 250 °C.
- ⇒ Press out excess filament by inserting the Allen key into the opening. Ensure that you do this as necessary but not forcefully as this would damage the machine.

If the blockage cannot be cleared, you must replace the nozzle (see the following chapter).

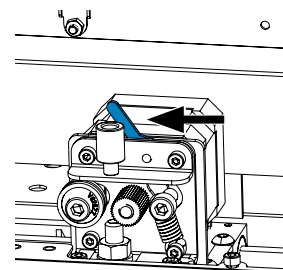
Fig. 73

11.4 Replacing the nozzle

To replace the nozzle, proceed as follows:

- ⇒ If you have not already done so, remove the filament (see Chap. 7.6).
- ⇒ Turn off the heating of the extruder and print bed and allow both to cool down.
- ⇒ Move the print bed into the lowest position (AXES  menu), Chap. 8).
- ⇒ Place a cloth or a cardboard box on the print bed to protect the Kapton film.
- ⇒ Switch off the printer.
- ⇒ Manually move the print head to a working position that is convenient for you.
- ⇒ Loosen the two screws holding the fan assembly approx. 1 turn (Fig. 74).
- ⇒ Pull the fan assembly forwards to lift it off.
- ⇒ Disconnect the connector and set the fan assembly aside.

Fig. 74

- ⇒ Loosen the left screw of the clamping plate approx. 1 turn and remove the right screw (Fig. 75).
- ⇒ Rotate the clamping plate towards the back so that there is space to move the heater cartridge cable.

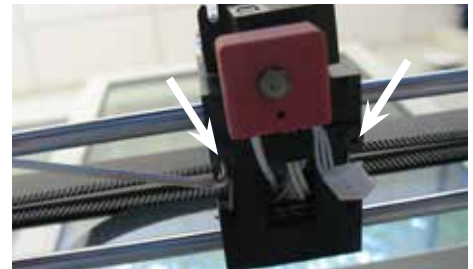
Fig. 75

- ⇒ Loosen the countersunk screw behind the nozzle by approx. 2 turns.

Fig. 76

- ⇒ Pull the temperature sensor and heater cartridge out towards the back in a straight line.

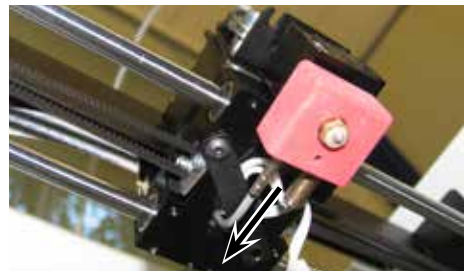
Fig. 77

- ⇒ Loosen the two screws on the heat sink and pull the nozzle unit out downwards.
Only loosen the screws, to not unscrew them!

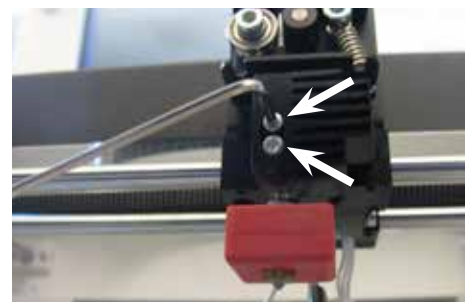
Fig. 78

- ⇒ Insert the largest Allen key into the bore hole in the heater cartridge and unscrew the heatbreak.
- ⇒ Then unscrew the nozzle, countering it again with the Allen key.

Fig. 79

11.4.1 Attaching the nozzle

- ⇒ Screw in the new nozzle and heatbreak, countering it again with the Allen key.
- ⇒ Clean the heatbreak and lubricate it with a conductive paste (see scope of delivery).

Recommended:

- Halnziye HY810 paste
 - Thermal conductivity: > 4.63 W/m-K
 - Thermal resistance: < 0.0078 °C/W
 - Operating temperature: -30 - 280

- ⇒ Slide the nozzle unit into the heatsink from underneath and tighten the two screws again.
- ⇒ For the rest of the assembly, proceed in reverse order as previously described.

Fig. 80

11.5 Applying a new Kapton film

The Kapton film of the print bed wears out over time or can be damaged by the nozzle due to incorrect calibration or removal of the printed models.

Replacing the Kapton film is important to maximize print quality when the old Kapton film has problems or is damaged.

- ⇒ Select the AXES  menu (Fig. 9).
 - ⇒ Move the print bed into a position that facilitates removal of the build plate.
 - ⇒ Fold the two metal clasps on the front of the print bed to the side.
 - ⇒ Lift the build plate slightly at the front and pull it forwards out of the rear metal clasps.
 - ⇒ Remove the old Kapton film by peeling it off the glass plate.
 - ⇒ Clean the surface of the build plate with acetone.
 - ⇒ Remove the plastic protective film (save for future use) from the sticky side of the new Kapton film.
 - ⇒ Apply a little window cleaning fluid to the build plate and to the sticky side of the Kapton film so that you can position the Kapton film properly on the build plate.
 - ⇒ Place the Kapton film on the build plate so that the distance to the opposite sides is approximately the same.
 - ⇒ Once the positioning is correct, place the protective film (removed at the beginning) on the new Kapton film to avoid scratches and damage during the following operation.
 - ⇒ Squeeze out the excess liquid between the build plate and the Kapton film with a thin but firm object (e.g., with a used credit card).
 - ⇒ Start in the center of the build plate and move horizontally and vertically. Once all the liquid has been squeezed out and the Kapton film has been properly applied, set the build plate aside to dry.
 - ⇒ Leave the build plate with the new Kapton film for a day before use.
 - ⇒ Cut off any overhanging film flush using a knife.
- To insert the build plate into the print bed:
- ⇒ Place the build plate on the print bed.
 - ⇒ Slide the build plate towards the back into the metal clasps.
 - ⇒ Close the two metal clasps on the front of the print bed.

11.6 Firmware – update

You can only obtain a firmware update via the Renfert CONNECT app (see Chap. 9).

Using the Renfert CONNECT App it is ensured, that you get informed about the latest firmware version.

A new firmware will improve the efficiency of the SIMPLEX 3D-filament-printer and should always be installed using the update.

To execute a firmware update, you need to install the SIMPLEX print software on your computer first (see Chap. 9).

You will receive a push notification via your Renfert CONNECT app as soon as new firmware is available for your printer.

If you want to load the firmware, press the load button,

You will receive a link to the mail address, stored during your registration

⇒ Open the mail with a mail client on your computer.

⇒ Click on the link in the mail.

⇒ Download the firmware ZIP file to your computer and extract it.

⇒ Read the release notes provided.

⇒ Connect the SIMPLEX 3D filament printer with the USB A-B cable (see Chap. 3.2 Scope of delivery) to your computer.

⇒ Open the SIMPLEX print program.

⇒ In the "... " menu, select the option "Load custom firmware from file".

⇒ Select the *.firmware file and open it.

⇒ Click OK in the message to confirm that the firmware was loaded successfully.

⇒ Open the "... " menu again and select "Update SIMPLEX firmware from downloaded file".

⇒ When prompted to do so, turn off the printer for at least 5 seconds.

⇒ Turn the printer back on and wait until the update has been completed.

⇒ When the update has been completed, turn the printer off and on again.

⇒ Check the SIMPLEX firmware version on the printer.

11.7 Replacing the backup battery

The backup battery may only be replaced by an electrical specialist or an equivalently qualified person.

⇒ Remove filament from the extruder, roll it up on the filament spool and put it aside.

**Switch off the unit!
Disconnect the power plug!**

⇒ Disconnect the connection cable from the FMS system.

⇒ Disconnect the connection cable from the fan.

⇒ Unscrew the filament spool holder and FMS system.

**Pay attention to the protective conductor cable and the cable to the fan and FMS system connector when removing the rear panel!
Do NOT disconnect the cable.**

⇒ Loosen the screws on the rear panel and remove it.

⇒ Remove the backup battery from the holder and insert a new backup battery.

⇒ Pay attention to correct polarity.

⇒ Screw the rear panel on again.

⇒ Assemble the filament spool holder and the FMS system.

⇒ Connect the connection cable of the FMS system.

⇒ Connect the connection cable of the fan.

⇒ Connect the power plug.

⇒ Switch on the unit

⇒ Set the date/time (see Chap. 6.1).

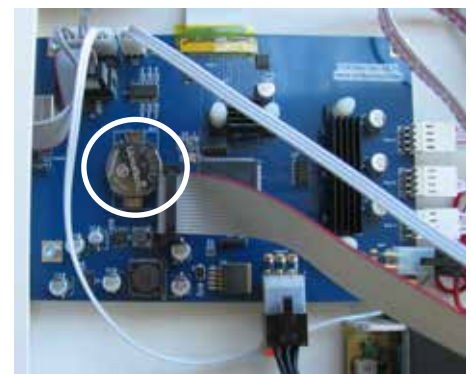
Fig. 81

11.8 Tighten X- and Y-axis traction belt

⇒ Remove filament from the extruder, roll it up on the filament spool and put it aside.

**Switch off the unit!
Disconnect the power plug!**

- ⇒ Remove the cover.
- ⇒ Unscrew the filament spool holder and FMS system.
- ⇒ Remove all remaining screws of the upper panel.
- ⇒ Carefully lift the upper panel at the front and disconnect the interior lighting.
- ⇒ Put the upper panel aside.
- ⇒ Loosen the screws of the right or left belt drive slightly and move drive to the back, until the belt is tense.
- ⇒ Tighten the screws again.
- ⇒ Put upper panel back in place and connect the interior lighting
- ⇒ screw on the upper panel.

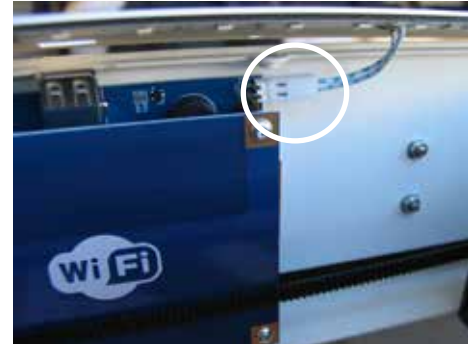
Fig. 82

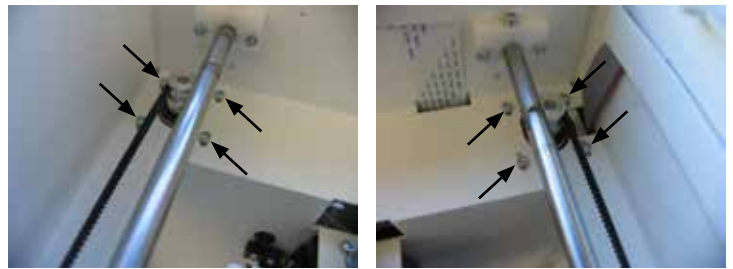
Fig. 83

11.9 Spare parts

You can find wear parts and spare parts in the spare parts list available online at www.renfert.com/p918.

Enter the following item number: 17350000.

The components excluded from the warranty (wear parts, consumables) are marked in the spare parts list.

The serial number, date of manufacturing and equipment version are shown on the nameplate of the equipment.

12 Troubleshooting

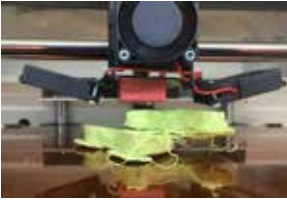
The FAQ's correspond to the status at printing.

An up-to-date overview of the FAQ's can be found on our website at www.renfert.com/p918

Enter the following item number: 1735 0000.

Errors	Cause	Corrective action
No material flow.	• Extruder blocked.	• Unblock the extruder, see Chap. 10.3.
Extruder blocked.	• Temperature too low when loading or unloading the filament.	• Choose the right temperature for the filament. • When loading or unloading the filament, wait for the extruder to fully reach the temperature.
	• Not all the filament was removed from the hotend when replacing/ removing.	• When removing the filament, proceed as follows: - To facilitate filament extraction, press the extrude function for 5 seconds followed immediately by the reverse function. - Use your fingers to slightly withdraw the end of the filament that is sticking out of the extruder. Extrusion is necessary, all the previously melted filament must be melted in the hotend.
Printer "crashes" when plugging in the jack plug (fan) of the cover.	• Glitch interrupts software.	• Only insert when the unit is switched off.
Error message: Fatal error! Short circuit on the 12 V line!	• The cover fan connector is not properly inserted.	• Insert the connector completely. • Only insert when the unit is switched off.

Errors	Cause	Corrective action
Spool covers the connection for the filament sensor (RJ45 socket).	<ul style="list-style-type: none"> • Spool too large. 	<ul style="list-style-type: none"> • Use a smaller spool. Use Renfert filament.
Date / time is incorrectly set.	<ul style="list-style-type: none"> • Backup battery of the clock is flat. 	<ul style="list-style-type: none"> • Replace the backup battery (see Chap. 11.7).
The FMS system reports a filament jam wrongly.	<ul style="list-style-type: none"> • The filament guiding tube (2, Fig. 1) is rough on the inside and hinders the material transport. 	<ul style="list-style-type: none"> • The filament guiding tube is a wearing part. Replace the filament guiding tube.
	<ul style="list-style-type: none"> • The filament guiding tube has not been completely pushed into the aperture at the FMS or the extruder. 	<ul style="list-style-type: none"> • Check the correct assembly of the filament guiding tube at the FMS and the extruder, see Chap. 4.2.3.
Object is printed crooked or incomplete.	<ul style="list-style-type: none"> • The filament guiding tube (2, Fig. 1) is rough on the inside and hinders the material transport. 	<ul style="list-style-type: none"> • The filament guiding tube is a wearing part. Replace the filament guiding tube.
File cannot be loaded from the USB flash drive.	<ul style="list-style-type: none"> • Umlauts / special characters in the file name. 	<ul style="list-style-type: none"> • Change the file name (no umlauts / special characters).
Poor adhesion between the individual layers, gaps in the layers.	<ul style="list-style-type: none"> • Not enough filament is fed. 	<ul style="list-style-type: none"> • Clean the cogwheel of the extruder. • Unblock the extruder (see Chap. 11.3).
Print object does not adhere to the print bed.	<ul style="list-style-type: none"> • The gap between the nozzle and the print bed is too large. • Kapton film of the print bed is dirty. 	<ul style="list-style-type: none"> • Calibrate the print bed (bed leveling). • Thoroughly clean the Kapton film with isopropyl alcohol.
Model printed at an incorrect angle. 	<ul style="list-style-type: none"> • Model misaligned in CAD software. The SIMPLEX sliceware is not the cause. SIMPLEX sliceware cannot influence the orientation of the model. 	<ul style="list-style-type: none"> • Review the situation in the CAD software. If necessary, the orientation must be corrected in the CAD software before the STL file is sliced again with the SIMPLEX sliceware.
Misprints / side shift 	<ul style="list-style-type: none"> • Print bed was not repositioned correctly in the printer after removal and repositioning. The latches on the build plate were not correctly closed. The print bed moved during printing, thus impairing printing. 	<ul style="list-style-type: none"> • Before printing, ensure that the print bed is installed correctly. The latch clasps must be closed to secure the print bed.

Errors	Cause	Corrective action
<p>Model loses adhesion to the print bed. The print job is interrupted.</p> 	<ul style="list-style-type: none"> • If the model detaches from the print bed, this is usually not due to the filament, but to the print bed, the printing environment, the settings, the STL file used or how the model was placed on the virtual platform. • Print bed not aligned / leveled. The gap between the nozzle and the print bed has changed. Distance too large. The nozzle is too far away from the print bed. Filament not in contact with the print bed during extrusion. • Print bed not clean. For example, due to fingerprints. Grease residue prevents the filament from adhering to the print bed. • Model not flatly aligned in the CAD software. • Contact surface to the print bed is too small for optimal adhesion. 	<ul style="list-style-type: none"> • Print bed leveling. • Clean the print bed with pure alcohol / spirits. • Model must be aligned again to the zero position in the CAD software.
<p>The first layers do not stick properly to the print bed and loosen. Unwanted lines are shown on the underside.</p> 	<ul style="list-style-type: none"> • The model was not placed flat on the virtual print bed in the SIMPLEX sliceware. Model has little contact with the print bed and therefore insufficient adhesion. 	<ul style="list-style-type: none"> • Make sure the models are dropped correctly. • When enlarging, a gap should not be visible between the model and the print bed. The model must be placed flat and fitted on the virtual print bed. • Tip: Use "Drop" in Renfert mode.

13 Technical data

en

Item no.	1735 0000 / 1735 1000
Nominal voltage	100 - 240 V
Permissible mains voltage:	90 - 264 V
Mains frequency:	50/60 Hz
Power consumption:	3 A
Dimensions (width x height x depth), including all attachments:	415 x 635 x 500 mm [16.3 x 25.0 x 19.7 inches]
Weight:	16.3 kg / [36 lbs.]
Light source:	This product contains a light source of energy efficiency class G.

Printing	
Printing technology	Fused Filament Fabrication (FFF)
Size	250 x 200 x 200 mm / [10 x 8 x 8"]
Layer resolution	≥ 50 microns
Positioning precision X, Y, Z:	4 x 4 x 2 microns
Filament diameter	1.75 mm
Nozzle diameter	0.4 mm
Printing speed	50 - 200 mm/s
Number of extruders	Single
Type of extruder	All-metal hotend
Nozzle operating temperature	180 - 260 °C / [356 - 500 °F]
Operating temperature of the build plate	50 - 110 °C / [122 - 230 °F]
Ambient temperature	15 - 32 °C / [59 - 90 °F]
Storage temperature	0 - 32 °C / [32 - 90 °F]
Software	
Software package	SIMPLEX sliceware / SIMPLEX print
File types	OBJ / STL / CWPRJ
Supported systems	Windows 7 and higher

14 Warranty

Renfert offers a 3-year warranty on all parts of the equipment, provided that the equipment is properly used.

Warranty claims may only be made upon presentation of the original sales receipt from the authorized dealer.

Parts which are subject to natural wear and tear (wear parts) and consumables are excluded from the warranty. These parts are marked in the spare parts list.

The warranty is voided in the case of improper use; failure to observe the operating, cleaning, maintenance, and connection instructions; in the case of independent repairs or repairs that are not carried out by an authorized dealer; if spare parts from other manufacturers are used, or in the case of unusual influences or influences not in compliance with the utilization instructions.

Warranty claims shall not extend the warranty period.

15 Disposal information

15.1 Disposing of the equipment

The equipment must be disposed of by a specialist facility. The specialist facility must be informed of any residue in the equipment that is harmful to health.

15.1.1 Disposal information for countries in the EU

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic equipment for proper disposal or recycling.

For this reason, equipment with this symbol may not be disposed of in unsorted domestic waste within the European Union.

Please contact your local authorities for more information on proper disposal.

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declare under our sole responsibility, that the product(s)

Name	Part No.
SIMPLEX 3D filament printer	1735 0000

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Supply of Machinery (Safety) Regulations 2008

Radio Equipment Regulations 2017

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Designated standards

EN ISO12100: 2010

EN 300 328 V2.2.2

EN 62368-1:2014/AC:2015

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 02.11.2022

The Technical Documentation has been retained by Renfert GmbH

**Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...**

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Renfert GmbH, Industriegebiet, 78247 Hilzingen / Germany
Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany