making work easy

de

en

fr

it

es

pt

ru

ja

ko

SILENT XS

Quick Start Guide

Made in Germany

205139 18122025

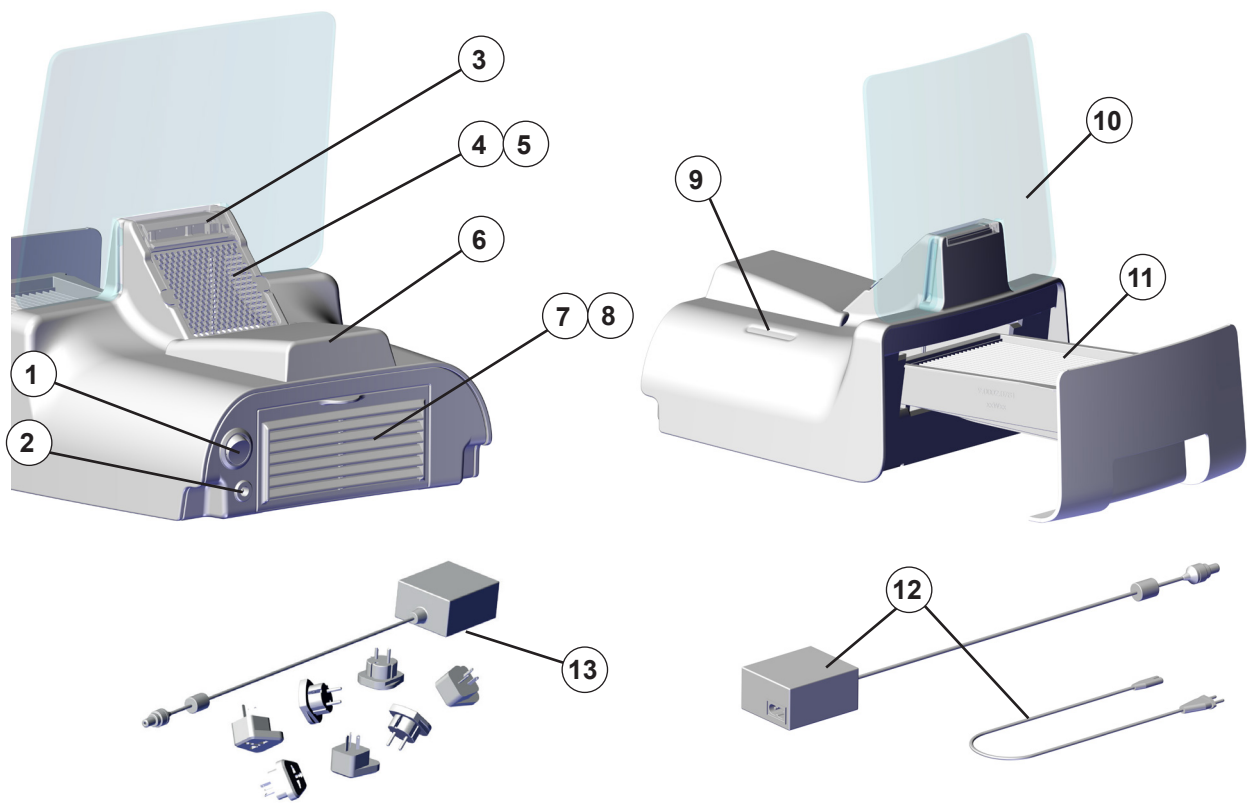
Fig. 1

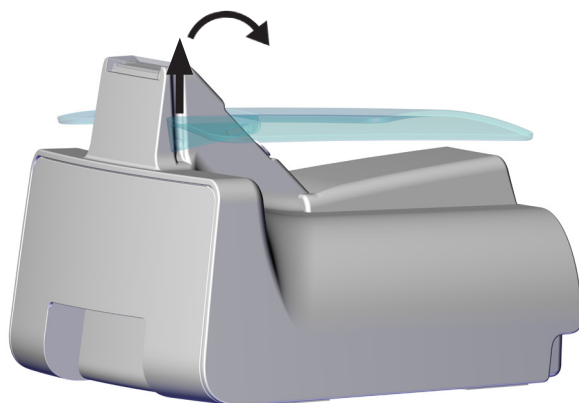
Fig. 2

<https://www.renfert.com/silent-xs>

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<https://www.renfert.com/silent-xs>

en

Here you will find the following documents and information for download:

- Instructions For Use
- Quick Start Guide
- Spare parts lists and spare parts drawings
- Maintenance manual
- FAQs, cause and rectification of faults
- How-to videos
- Declarations of conformity

Please note: The documents and information listed are not available for all products.

Symbols used

In these Instructions For Use (IFU) or on the equipment you will find symbols with the following meanings:

Danger

Immediate risk of injury. Consult accompanying documents!

Electrical voltage

Danger due to electrical voltage.

Caution

Risk of damage to the equipment if the instruction is not observed.

Note

Provides useful information to make operation easier.

The equipment complies with applicable EU directives.

This product complies with the relevant UK legislation.

See the UKCA declaration of conformity on the Internet at www.renfert.com.

The equipment is subject to the provisions of the EU directive 2002/96/EC (WEEE Directive).

Other symbols are explained as they occur in the text.

Safety

Intended use

The unit is intended for commercial use in dental labs or in the environment of dental offices in accordance with the IFU (Instructions For Use). The unit is intended for use in the following tasks:

- Suction of dry, non-explosive dust during minor fitting work on disinfected objects.

Meaning of these Instructions For Use (IFU)

These IFU will help you use the unit safely.

If these IFU are not observed, accidents may lead to life-threatening electric shock, injury, or material damage.

⇒ Only use the unit in accordance with these IFU.

⇒ Keep these IFU ready with the unit.

⇒ Provide these IFU to all subsequent users of the unit.

Requirements for operators

⇒ The product may only be used by persons aged 14 or over who are familiar with the procedures and rules in a dental laboratory.

Safe condition of the unit

Defective, broken or leaking components may cause injuries.

- ⇒ Check the power cord, housing, and other components such as connection cables, tubes, and the operating panel pad for damage, such as kinks, cracks, porosity, and signs of wear and tear.
- ⇒ Defective equipment must be taken out of service immediately. Disconnect the power plug. Send the unit in for repair or dispose of it safely. Until then, secure the unit to prevent it from being switched on again so that it is not used inadvertently.
- ⇒ Do not expose the unit to mechanical impact. Do not drop it.

Hazards as a result of modifications and repairs

Improper modifications and repairs to the unit may cause accidents, fire, and electric shock.

- ⇒ Only have the unit repaired by qualified specialist dealers or return it.

The use of unsuitable accessories or spare parts may lead to unpredictable accidents and damage.

- ⇒ Only use accessories and spare parts supplied or approved by Renfert GmbH.

Risk as a result of electric shock and fire

Risk of electric shock and of fire as a result of excessive operating voltage or if there is no protective conductor.

- ⇒ Ensure that the power plug is easily accessible.
- ⇒ Only operate the unit using a power cord with a plug that matches the local power supply. Only have the power cords replaced by a qualified electrician.
- ⇒ Only operate the unit if the information on the type plate matches the power supply.

Risk of fire if the wrong charger or batteries are used.

- ⇒ Use only the charger supplied to charge the batteries.
- ⇒ Do not use rechargeable lithium-ion batteries.
- ⇒ Do not use normal batteries (non-rechargeable).

Risk of injury to the eyes or skin

Risk of glare!

- ⇒ Do not look directly into the object illumination!

Risk to health and fire hazard due to dust

Dust emissions may result in damage to health.

- ⇒ The unit is not suitable for sucking in dust classified as hazardous, such as cristobalite and quartz (investment material), or CrCo.
- ⇒ When disposing of the sucked in material or used filters, observe local provisions and accident prevention regulations!
- ⇒ Depending on the type of sucked in material, wear appropriate personal protective equipment when emptying the dust drawer or during cleaning.

Sucking in non-approved types of dust can cause a fire.

- ⇒ Do not suck in flammable or explosive gasses, fumes or dust.
- ⇒ Do not suck in hot materials.

Risk to health due to infection

Infection due to cross-contamination may result in damage to health.

- ⇒ Disinfect the dental restoration with a suitable product listed by the VAH (Association for Applied Hygiene) before using the suction unit (see "Hygiene guidelines").
- ⇒ Repeat disinfection before and after each patient contact (see "Hygiene guidelines").
- ⇒ Disinfect the unit before use and after each treatment (see "Hygiene guidelines").
- ⇒ It is the user's responsibility to separately comply with country-specific regulations regarding compliance with reprocessing procedures.

Product description

General description

SILENT XS is a mobile suction unit. Thanks to battery operation, it is not tied to a single workstation and can easily be used in different locations.

Hygiene guidelines

The hygiene guidelines must be observed during startup and cleaning/maintenance.

The hygiene guidelines can be found online.

Scan this QR code.

Assemblies and functional elements

See Fig. 1 (on inside cover)

- | | |
|----------------------------------|--|
| 1 On/off switch | 8 Exhaust air filter (behind exhaust air grille) |
| 2 Charging socket | 9 Operating and charging status indication |
| 3 Object illumination | 10 Inspection window |
| 4 Suction grille | 11 Dust drawer with fine filter |
| 5 Suction opening | 12 Charger, desktop power supply with power cord |
| 6 Clamp (available as accessory) | alternatively |
| 7 Exhaust air grille | 13 Charger, plug-in power supply with plug set |

Scope of delivery

- 1 Suction unit SILENT XS
- 1 Inspection window
- 1 Charger (desktop power supply or plug-in power supply)
- 1 Power cord (only with desktop power supply)
- 1 L-key T20
- 1 Quick Start Guide

Accessories

Information on accessories and how to use them is provided in the instructions for use available online.

Startup

Insert batteries

Do not use rechargeable lithium-ion batteries!

8 rechargeable batteries are required.

Rechargeable batteries are not included in the scope of delivery and must be obtained locally prior to startup.

Recommended batteries:

Type:	Varta Endless
	Duracell Recharge
Capacity:	2500 mAh


Battery life is significantly affected by charging status, use and charging cycles, age, manufacturer and production batches.

Use of non-recommended batteries may result in significant variations in run time.

Please refer to the technical data for the minimum battery requirements.

- ⇒ Switch off the unit at the on/off switch (1).
- ⇒ Turn the unit over.
- ⇒ Remove the screws from the battery compartment (Torx 20).
- ⇒ Slide the battery compartment cover in the direction of the printed arrow and remove it.
- ⇒ Slide the holding clip to the side and remove the battery holder.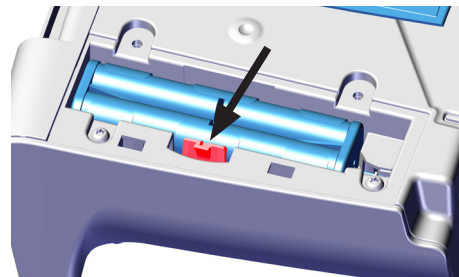

Caution
Do not pull on the battery holder cable!

- ⇒ Insert the batteries.

Pay attention to correct polarity!
The negative pole of the batteries always points towards the contact spring in the battery holder.

When inserting the batteries, first insert the batteries with the positive pole next to the contact into the battery holder and push them up to the contact. Then insert the batteries whose negative pole is pointing towards the contact spring.

- ⇒ Insert the battery holder until it rests under the retaining clip.
- ⇒ Replace the battery cover and slide it against the direction of the arrow until the screw holes are accessible.
- ⇒ Screw the screws back in.

Do not overtighten the screws!
Tighten the screws finger-tight.

Connecting the charger / charging the batteries

Note the instructions in Chapter Risk as a result of electric shock and fire!

Desktop power supply (12):

- ⇒ Plug the power cord into the desktop power supply.
- ⇒ Connect the power plug.

Plug-in power supply (13):

- ⇒ Select the appropriate country-specific plug adapter from the plug set.
- ⇒ Insert the plug adapter into the power supply until it clicks into place.

- ⇒ Connect the charging plug to the charging socket (2).
 - ♦ The batteries are charged.
 - ♦ It takes approx. 5 hours to recharge empty batteries.

Installing the inspection window

- ⇒ Slide the inspection window (10) straight from above into the grooves on the side of the suction opening.

The inspection window can be folded down to store the unit in a drawer, for example. To do this, pull the inspection window up a bit and fold it back, see Fig. 2 (on inside cover).

Operation

The unit must be disinfected before first use (see chapter „Disinfection“).
The procedure is described in the hygiene guidelines.

Switching On / Off

Use the On/Off switch (1) to switch the unit on and off.

- ⇒ Switching on
 - ♦ The suction fan starts to run.
 - ♦ The object illumination is switched on.

Charging the unit

When not in use, the unit should be connected to the charger to keep it fully charged.

Charging status

The operating and charging status is indicated by 4 LEDs (9).

LED 1 (green):


Is lit when the charger is connected.

The inserted batteries are charging.

LED 2-4 (white):

Signal the charging status during the charging process (flashing) and during operation (illuminated).

All LEDs will light continuously when all batteries are fully charged.

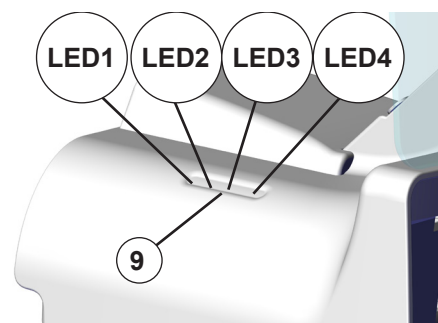LED 4 indicates a remaining run time of approx. 5 – 10 minutes.

Emptying the dust drawer

Empty the dust drawer after every grinding operation.

Caution


Do not beat out the dust, as this will only increase the dust accumulation!

⇒ Switch the unit off.

⇒ Pull the dust drawer (11) straight out.

⇒ Take the dust drawer to the disposal site and empty it carefully so that no dust is stirred up.

⇒ Insert the dust drawer again and push it all the way in.

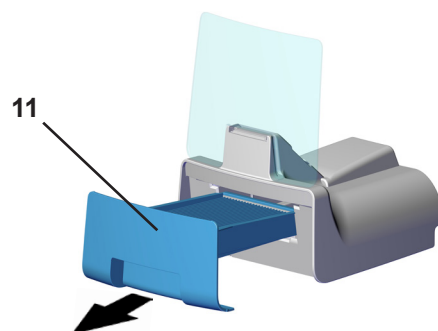Cleaning / Maintenance

The unit does not contain any internal parts that require maintenance. Do not open the unit except in the situations described below!

Before cleaning and maintenance work:

- Switch off the unit and disconnect the charging plug!

Cleaning

Do not steam clean the unit under any circumstances.

Do not use compressed air.

To clean the unit, wipe the entire surface (e. g. including the window) with a dry cloth.

Do not use any solvent-based, alkaline or abrasive cleaning agents.

Do not use solvent-based disinfectants (e. g. BIB forte/ Alpro Medical).

Disinfection

Disinfect the unit in accordance with hygiene guidelines.

The following steps are only an excerpt from the hygiene guidelines.

Caution

Do not spray disinfectant directly onto or into the unit.

Use disinfectant wipes.

For disinfection, use a disinfectant wipe listed by the VAH (Association for Applied Hygiene), e. g. minutenwipes / Alpro Medical or Caviwipes / Metrex.

It is recommended to disinfect the unit before initial use and after each use.

To do this, carefully wipe the entire surface of the housing (including the grilles and window) with the disinfectant wipe.

Follow the manufacturer specifications for handling and exposure time.

Changing the fine filter

If the suction power decreases significantly despite emptying, the fine filter in the dust drawer (11) must be replaced.

See the complete operating instructions on the Internet, see the “Online help and information” section.

Replacing the exhaust air filter

The exhaust air filter (8) must be replaced every six months.

See the complete operating instructions on the Internet, see the “Online help and information” section.


Removing/inserting the suction grille

Removal

- ⇒ Slide the suction grille (4) upwards (a).
- ⇒ Swing the suction grille out at the bottom (b) and remove it.
- ⇒ Remove any stuck particles.

Insertion:

- ⇒ Slide the suction grille up under the object illumination cover.

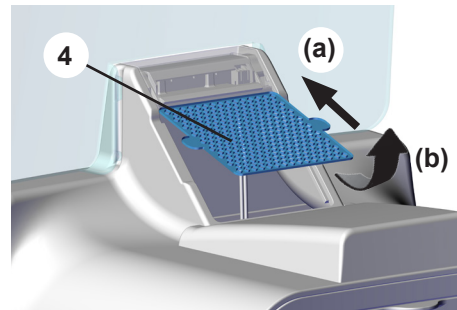Spare parts

You can find information on wear parts and spare parts in the spare parts list available online, see chapter “Online help and information”.

The components excluded from the warranty (wear parts, consumables) are marked in the spare parts list.

The serial number, date of manufacturing and equipment version are shown on the nameplate of the equipment.

Obligations regarding information

- Information on REACH and SVHC can be found on our website in the service area (www.renfert.com/service)

Disposal instructions

The equipment must be disposed of by a specialist company. If there are any residues in the equipment that are hazardous to health, the specialist company must be informed.

Disposal information for countries in the EU

Disposal of rechargeable batteries / batteries

The product contains rechargeable batteries or batteries. They must be removed before disposal and then disposed of properly. These may not be disposed of with household waste.

Rechargeable batteries / batteries are not supplied with the product and must be disposed of by the operator on their own responsibility.

Dispose of rechargeable batteries / batteries according to local regulations!

To conserve and protect the environment, prevent environmental pollution, and improve the recycling of raw materials, the European Commission has adopted a directive that requires the manufacturer to accept the return of electrical and electronic equipment for proper disposal or recycling.

For this reason, equipment with this symbol may not be disposed of in unsorted domestic waste within the European Union.

Please contact your local dealer for more information on proper disposal.

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declare under our sole responsibility that the product(s)

Name	Part No.
SILENT TS	2921 0050
SILENT TS2	2930 0050
SILENT TC	2935 0000 / 2935 0000 03
SILENT TC2	2936 0000 / 2936 0000 03
SILENT EC2	2937 0000 / 2937 0000 03
SILENT compact	2934 0000 / 2934 0000 03
SILENT compactCAM	2934 2000 / 2934 2000 03 2934 2500 / 2934 2500 03
SILENT powerCAM EC	2939 0000
SILENT powerCAM TC	2938 0000
iVAC silent	36 2934 2000
SILENT cubeCAM	2918 0000
SILENT XS	2922 0000 / 2922 3000

is (are) in conformity with the relevant regulatory requirements by compliance with the UK designated standards.

UK legislation

Electromagnetic Compatibility Regulations 2016

Supply of Machinery (Safety) Regulations 2008

The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012

Ecodesign for Energy-Related Products Regulations 2010

Designated standards

EN 61326-1:2013

EN ISO12100:2010

EN 61010-1:2010/A1:2019/AC:2019-04

EN IEC 63000:2018

Tilo Burgbacher,
Engineering Director

Hilzingen, 10.10.2025

The Technical Documentation has been retained by Renfert GmbH

Technical data

Item number:	2922 0000 / 2922 1000 / 2922 3000
Nominal voltage / VDC:	12
Power consumption *1) / A:	1.4
LpA *2) (at max. flow rate) / dB(A):	52.7
Flow rate, max. / l/min [cfm]:	510 [18]
Fine filter: – Filter surface, approx. / m² [sq inch]: – Filter quality:	0.11 [170.5] HEPA H13
Fill volume dust drawer, approx. / l [US fl oz]:	0.22 [7.44]
Rechargeable battery: Quantity Type Size Voltage / V Capacity, minimum / mAh	8 NiMH (rechargeable) AA 1.2 1200
Dimensions (width x height x depth) / mm [inch] (incl. Inspection window):	223 x 248 x 193 [8.8 x 9.8 x 7.6]
Weight (empty) / kg [lbs]:	1.12 [2.47]
This product contains a light source of energy efficiency class:	E
Charger	
Permissible mains voltage / VAC:	100 - 240
Mains frequency / Hz:	50 / 60
Output voltage / VDC:	12
Output power / W:	25
Ambient conditions for operation *3)	
Ambient temperature / °C [°F]:	5 - 30 [41 - 86]
Ambient conditions for storage and transport	
Ambient temperature / °C [°F]:	- 20 to + 60 [- 4 to + 140 °F]
Maximum relative humidity:	80 %
Ambient conditions according to EN 61010-1 *3)	
Ambient conditions:	Indoors
Installation height, max.:	2,000 m above sea level
Ambient temperature:	5 - 40 °C [41 - 104 °F]
Relative humidity:	Max. 80 % at 31 °C [87.8 °F], decreasing linearly to 50 % at 40 °C [104 °F] **)
Voltage fluctuations in the mains power supply from the nominal value, max.:	10 %
Contamination level:	2
Over-voltage category:	II

*1) Max. 1.8 A in suction mode and when charging empty batteries.

*2) Noise level in accordance with EN ISO 11202

*3) The product performs as specified in the technical data when operated within ambient conditions. If the unit is operated outside these ambient conditions, safety devices contained in the product may stop operation, the optimum performance specified in the technical data may not be achieved, and the service life of components may be reduced.

The product fulfills the safety requirements of standard EN 61010-1 when operated within ambient conditions. Operation outside of these ambient conditions is not permitted, as electrical safety may be compromised.

Hochaktuell und ausführlich auf ...
Up to date and in detail at ...
Actualisé et détaillé sous ...
Aggiornato e dettagliato su ...
La máxima actualidad y detalle en ...
Актуально и подробно на ...

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany