making work easy

- [FAQ - English](#)
- [FAQ - Français](#)
- [FAQ - Italiano](#)
- [FAQ - Español](#)
- [FAQ - Português](#)
- [FAQ - Türkçe](#)
- [FAQ - Русский](#)
- [FAQ - Polski](#)
- [FAQ - Dansk](#)
- [FAQ - 中文](#)
- [FAQ - 日本語](#)
- [FAQ - 한국어](#)
- [FAQ - اللغة العربية](#)
- [FAQ - Українська](#)

# Pillo / Power pillo

[FAQ - frequently asked questions](#)

# FAQ - Deutsch

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# FAQ - English

*For chapter indications, refer to the complete instruction manual on the internet.*

Error	Possible cause	Corrective action
<b>Renfert deflasking chisel fails to start; no sound of compressed air.</b>	<ul style="list-style-type: none"><li>Compressed air supply not established.</li><li>Ball valve on the Renfert deflasking chisel not properly / fully open.</li><li>Kink in connecting hose.</li><li>No compressed air.</li></ul>	<ul style="list-style-type: none"><li>Check and correct the compressed air connection.</li><li>Fully open the ball valve.</li><li>Straighten the connecting hose.</li><li>Check the compressed air system and in case switch the compressor on.</li></ul>
<b>Renfert deflasking chisel fails to start but compressed air can be heard.</b>	<ul style="list-style-type: none"><li>Insufficient connecting pressure.</li><li>Motor gummed up or blocked.</li><li>Damaged piston.</li></ul>	<ul style="list-style-type: none"><li>Increase the connecting pressure to 4 - 6 bar.</li><li>The unit will need to be repaired by our Service Department.</li><li>The unit will need to be repaired by our Service Department.</li></ul>
<b>Renfert deflasking chisel produces insufficient power.</b>	<ul style="list-style-type: none"><li>Insufficient connecting pressure.</li><li>Ball valve not fully open.</li><li>Filter in the connector fitting blocked.</li></ul>	<ul style="list-style-type: none"><li>Increase the connecting pressure to 4 - 6 bar.</li><li>Fully open the ball valve.</li><li>Clean the filter.</li></ul>

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