Power-Pillo

Nr. 5022-3000

DEUTSCH

1. Einleitung

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ENGLISH

1. Introduction

We are pleased with your decision to purchase a *Power-Pillo*.

The *Power-Pillo* is a robust device operating at a high impact rate. Power regulation permits the unit to be precisely adjusted to meet your specific requirements.

Please observe the following operating instructions carefully and observe all the information they contain in order to ensure long and trouble-free service. These operating instructions must always be available at the work area.

2. Application Area

The *Power-Pillo* is a pneumatically powered chisel for deflasking acrylic dentures or cast objects in dental technology operations.

The plasma coating on the pistons eliminates the need for an oil atomizer.

We recommend the use of a filter pressure regulator (Art. No. 2929-0000).

2.1 Ambient Conditions

(in accordance with IEC 60664-1)

The *Power-Pillo* may only be operated:

- Indoors;
- Up to an altitude of 2,000 m [6,500 ft] above sea level;
- At an ambient temperature range of 5 - 40°C [41 - 104°F];
- At a maximum relative humidity of 80% at 31°C [87.8°F], dropping linearly to 50% relative humidity at 40°C [104°F];
- At contamination level 2.

3. Hazard Information

3.1 Employed Symbols

The following symbols are used in these instructions and on the unit:

Danger

This indicates an imminent risk of injury.

Attention

Wear eye protection.

Attention

Wear hearing protection.

Attention

Failure to observe the associated information can result in damage to the unit.

Note

The operating instructions contain useful information to make handling easier.

Note the operating instructions.

3.2 Hazard Information

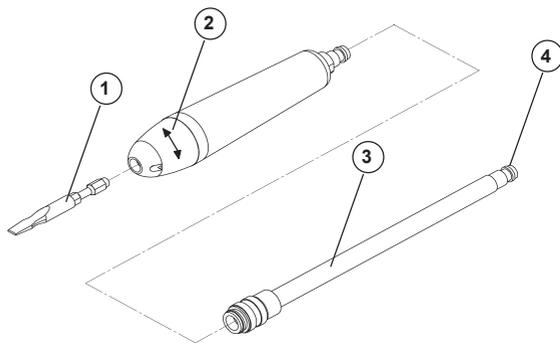
- **Working with brittle materials can result in the generation of health-hazardous dusts. Always wear appropriate personal protection and ensure the work area is properly ventilated.**
- **Always wear eye protection. Working with brittle materials can generate splinters.**

- Always wear hearing protection. The unit's sound level typically lies on the range of 78 dB(A). During some tasks, the sound level can exceed 85 dB(A).
- Always work away from yourself. Ground chisels are very sharp. There is a high risk of injury should the chisel slip off the piece being worked.
- Always disconnect the unit from the compressed air supply before exchanging or replacing chisel inserts. There is a high risk of injury should the unit start unexpectedly.
- Before starting the unit, compare the information on the nameplate with the compressed air connection values.
- Always disconnect the unit from the compressed air supply before beginning any maintenance tasks.
- Regularly examine the connecting hoses for damage (e.g., kinks, cracks, porosity).
- Never operate a unit with damaged connecting hoses or other defects.
- The risk of mechanical vibrations is dependent on the duration and the intensity of the vibrations. The typical maximum vibration is 1.93 m/s².

3.3 Liability Exclusion

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- The product is employed for any purposes other than those cited in the operating instructions;
- The product is altered in any way other than those alterations described in the operating instructions;
- The product is repaired by other than an authorized facility or if any but Renfert OEM parts are employed;
- The product continues to be employed, despite obvious safety faults.

4. Commissioning / Operation

! Do not exceed the maximum connection pressure of 6 bar [87 psi]!

4.1 Connecting the Unit

- Insert the desired chisel (1) into the tool retainer. The chisel must palpably lock into place in the retainer.
- Turn the adjusting collar (2) completely counter clockwise.
- Reduce the compressed air supply connection pressure at the pressure regulator to 4 to 6 bar [58 – 87 psi].
- Establish the connection to the compressed air supply with the connecting hose (3). The connecting kit contains an assortment of the most commonly used connecting nipples (4).

4.2 Switching the Unit On and Adjusting the Chisel Force

Working with brittle materials can generate splinters.

The unit's sound level typically lies on the range of 78 dB(A). During some tasks, the sound level can exceed 85 dB(A).

Always work away from yourself.

Turn the adjusting collar (2) to switch the chisel on and off and to adjust the chisel force:

- Clockwise: Switch the chisel on and increase its force.
- Counter clockwise: Reduce the chisel force and switch it off.
- During short breaks: Turn the adjusting collar (2) completely counter clockwise.
- During longer interruptions or once you are finished with the unit: Disconnect the compressed air hose from the compressed air supply.

4.3 Chisel Replacement

Always disconnect the *Power-Pillo* from the compressed air supply before exchanging or replacing chisel inserts.

- To replace an insert, pull the chisel (1) forwards and out of the tool retainer.
- Insert the new chisel into the tool retainer. The chisel must clearly lock into place in the retainer.

5. Cleaning / Maintenance

Always disconnect the unit from the compressed air supply before beginning any maintenance tasks.

The *Power-Pillo* does not require the use of an oil atomizer. However, it may not be operated at a pressure in excess of 6 bar [87 psi] otherwise the unit will be damaged.

Occasionally wipe down the *Power-Pillo* surfaces with a soft cloth.

6. Spare Parts

Please refer to the enclosed spare parts list for all consumables and spare / replacement parts.

7. Warranty

Provided the unit is properly used, Renfert warrants the *Power-Pillo* for a **period of 3 years**. The original sales receipt from the

dealer must be provided when making warranty claims. Components subject to natural wear are excluded from this warranty.

The warranty is voided in case of improper use; failure to observe the operating, cleaning, maintenance, and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed, or; in case of unusual influences or influences not in compliance with the utilization instructions. Warranty service shall not extend the original warranty.

8. Technical Specifications

Operating pressure: 4 – 6 bar
[58 – 87 psi]

Compressed air consumption, maximum: 40-50 l/min.,
Hand / arm vibration in accordance with
DIN EN 28662, Part 1/2: 1.93 m/s²

Sound pressure level LpAep,
in accordance with DIN EN 45635,
Page1/Part 20: 78 dB(A)

Weight: 300 g

Dimensions: Ø 30 x 155 mm

9. Standard Delivery

- 1 *Power-Pillo*
- 1 Wall unit
- 1 Chisel insert, no. 5022-0200
- 1 Connecting hose
- 1 Connecting kit
- 1 Spare parts list
- 1 Operating instructions

10. Accessories

Nr. 5022-0100 1 flat chisel, narrow
Nr. 5022-0200 1 flat chisel, wide
Nr. 5022-0300 1 hollow-ground chisel

Nr. 5022-0400 1 pointed chisel
Nr. 5022-0500 1 rivet chisel
Nr. 5022-0600 1 chisel blank for grinding
Nr. 2929-0000 1 filter pressure regulator

11. Error List

Error	Possible cause	Corrective action
Power-Pillo fails to start; no sound of compressed air.	<ul style="list-style-type: none">• Compressed air supply not established.• Adjusting collar not properly open.• Kink in connecting hose.• Air compressor not switched on.	<ul style="list-style-type: none">• Check and correct the compressed air connection.• Completely open the adjusting collar.• Straighten the connecting hose.• Switch the air compressor on.
Power-Pillo fails to start but compressed air can be heard.	<ul style="list-style-type: none">• Insufficient connecting pressure.• Damaged piston.	<ul style="list-style-type: none">• Increase the connecting pressure to 4-6 bar [58 – 87 psi].• The unit will need to be repaired by our Service Department.
Power-Pillo producing insufficient power.	<ul style="list-style-type: none">• Insufficient connecting pressure.• Adjusting collar not fully open.	<ul style="list-style-type: none">• Adjust the connecting pressure to 4-6 bar [58 – 87 psi].• Open the adjusting collar further.

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Renfert GmbH • Industriegebiet • 78247 Hilzingen