de|en|fr|it|es|pt|ru|zh(s)|ko|ja

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* elFU: www.renfert.com/downloads

INSTRUCTIONS FOR USE ENGLISH

Intended use:
SIMPLEX model isolation is an alcohol-based isolating gel for model isolation. The model isolating agent is especially adapted for separating resin models (printed using FFF/FDM 3D printing) from dental resins (auto-curing polymer resins and cold-curing polymer resins) such as PMMA. A perfect, ultra-fine film of isolating agent is applied on the surface of the model using a brush. After application of the ready-to-use model isolating agent, removable orthodontic appliances can be easily fabricated using orthodontic dental resin.

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Note: Only suitable for extraoral use.**Risk information and safety advice:**

Please read the Safety Data Sheet before using the product.

Download at www.renfert.com.

Highly flammable liquid and vapor. Causes severe eye irritation.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. Do not smoke or drink.

Wear protective gloves and eye/face protection.
In case of persistent eye irritation: seek medical advice/seek medical assistance.
Store in a well-ventilated place. Keep container tightly closed.**Application area:**

Especially adapted model isolating agent for models printed using FFF/FDM printing. Also suitable as a model isolating agent for resin models printed using SLA/DLP 3D printing.

**Processing:**

Optimum isolation is already achieved with one or two coats of the model isolating agent. For this purpose, squeeze two pea-sized drops of isolating agent (the quantity of isolating agent varies depending on the size and shape of the model) out of the pump bottle onto the model and spread generously over the model using any brush (e.g., Kolinsky size 08, item no. 17131008 or Takanishi size 08, item no. 17140008). Leave to dry thoroughly. Each layer requires approx. 30 minutes to dry.

Note:

As soon as the isolating agent that you have applied changes from shiny to matt and no longer feels sticky when gently touched, it is sufficiently dry to allow further processing of the model.

TIPP: By using a light-curing unit or the circulating air at the suction mouth, you can significantly reduce the drying time, by up to 90%.**Note:**

The max. exposure time of 180 seconds should not be exceeded. Too much heat can result in warping and damage.

Before isolation, block undercuts as usual using wax. Process resin in accordance with the instructions for use for orthodontic technology. Once polymerization has been completed, carefully lift the orthodontic appliance from the 3D model using an instrument (e.g., ERGO Wax Instrument No. 2, Art. Nr. 10342002). After polymerization, residue from the isolating agent on the 3D model and orthodontic appliance can be effortlessly removed under lukewarm running water. Slight wax residue on the 3D model and orthodontic appliance can be carefully removed using warm water or a steam cleaning unit at a minimum distance of 5 cm while moving continuously from side to side.

Note: Reduce the impact of heat during cleaning as far as possible.**Do not expose SIMPLEX filament models to the steam jet for long periods, do not scald them, and do not pour hot water over them. Risk of warping and damage.****TIPP:** Remove large pieces of wax residue manually in order to prevent damage and warping of the 3D model through overheating as a result of extended cleaning using a steam jet or warm water.

Finish and polish the orthodontic appliance in the usual way.

Notes on processing and storage:

Always work with a clean brush so that the isolating agent is not impacted negatively by impurities, thus reducing the isolation effect.

Store SIMPLEX model isolation out of direct sunlight at temperatures between 5°C and 40°C.

All information on processing SIMPLEX model isolation in any form (spoken, written, or demonstrated) are provided to the best of our knowledge and are for information purposes only. Usage and processing of SIMPLEX model isolation are beyond the scope of our control and are the responsibility of the user. We are liable only for the quality of the product.

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