making work easy

FAQ - Deutsch

FAQ - English

FAQ - Français

FAQ - Italiano

FAQ - Español

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FAQ - Türkçe

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FAQ - English

For information on operating elements, see [Fig. 1](#) on page 2 of this document.

Fault	Cause	Remedy
Unit doesn't blast, warning lamp flashing.	- Glass pane is not closed. - Magnet (Fig. 42-A) is missing. - Protective switch is defect.	- Close the pane. - Replace the magnet (see spare parts list). - Exchange during service.
Air flow present but no sand coming through.	- Amount of sand too low. - Filter on suction point of suction pipe is blocked. - Suction pipe defect.	- Fill with more sand. - Clean the filter. - Exchange the suction pipe.
No air, no sand.	Nozzle is blocked.	Open the blasting nozzle unit (Fig. 15-A) and clean the nozzle.
On / Off switch doesn't work.	- No power connection. - Unit fuse (Fig. 13-B) is defect.	- Connect to the power supply. - Change fuse (Fig. 13-B) - replacement fuse in set. If it happens again >>> repair.
Basket shakes, doesn't move.	Basket is stuck mechanically.	Release by removing and replacing the basket.
Blasting chamber gets too dusty.	- Too little suction power. - Extraction unit blocked.	- Raise suction power on extraction unit. - Increase suction performance by closing the air inlet boreholes under the glass holder (Fig. 42-B) (with the supplied magnetic strip). - Clean extraction unit.
Consumption of blasting material too high.	Suction power is too high.	Reduce suction power. Warning: too little suction power can lead to dirty blasting material.
Objects fall out of the basket.	Too many objects.	Reduce to a maximum of 6 objects.
Blasting time too long / reduced blasting effect.	- The blasting cones are sheltering the objects. - Too little working pressure. - Too great distance from nozzle. - Too little blasting material in unit. - Blasting material lead pipe is blocked / defect. - Blasting nozzle is split. - Air power of the compressor is too low.	- Remove blasting cones before starting. - Increase working pressure >>> recommended working pressure: 4 - 5 bar [58 - 72.5 psi]. - Move nozzle further forward. - Unit should be filled with min. 4 - 5 kg blasting material. - Remove sand and check / replace lead pipe. - Replace nozzles. - Dehydrate the compressor. Use a more powerful unit.

Fault	Cause	Remedy
Extraction unit still runs even when blasting has been interrupted.	• A certain run-on time is normal in order to remove any dust still present in the chamber. • The switch for the external extraction unit (Fig. 18-F) was reactivated.	• Regulate the run-on time between 5 and 30 seconds via the control on the back of the unit (Fig. 21). • Switch off the external extraction unit.
Heavy dust build-up during blasting.	• No extraction unit connected. • Extraction power too weak.	• Connect an extraction unit. • Close the air holes (Fig. 42-B). • Increase extraction power.
Switch for external extraction does not work.	• Extraction unit is not controlled by blasting unit.	• Connect the extraction unit power cable to the blasting unit.
Foot switch doesn't work.	• Unit is in automatic mode.	• Interrupt automatic mode using HOLD button (Fig. 18-I). • Switch to manual (Fig. 18-E).
Extraction unit doesn't work.	• Extraction unit is not controlled by the sandblasting unit. • Extraction unit is not switched to continuous mode. • Fuse for extraction unit (Fig. 13-C) is defect.	• Connect the extraction unit power cable to the blasting unit. • Switch the extraction unit to continuous mode. • Change the fuse (Fig. 13-C) - replacement fuse found in set. If it happens again >>> repair.
Pulsating sand flow.	• Blasting material is used or too impure.	• Renew blasting material.
Unusually large amount of blasting material in blasting sand.	• Sieve is in an incorrect position.	• Replace the sieve correctly, observe the insertion sequence: 1. Insert right hand sieve. 2. Insert left hand sieve.
Independent switching off while working.	• Too low supply voltage. • Inadequate grounding because the device is located too far from the nearest ground connection.	• Checking of the power capacity of the mains supply. • Temporary switching off of other more powerful units. • Decrease the distance to the nearest ground connection.

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