making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Polski
- FAQ - Čeština
- FAQ - 中文
- FAQ - 日本語
- FAQ - 한국어
- FAQ - اللغة العربية
- FAQ - Українська

VORTEX compact 3L

FAQ - frequently asked questions

FAQ - Deutsch

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FAQ - English

For chapter indications, refer to the complete instruction manual on the internet.

Problem	Cause	Remedy
Motor won't run.	<ul style="list-style-type: none"> • Fuse blown in mains socket. • Overload protection triggered. • Motor defective. 	<ul style="list-style-type: none"> • Reactivate mains fuse. • Turn off dust extractor and allow to cool down for 5 minutes. Contact the Renfert Service Team if the extractor still fails to restart. • If the extractor is to be situated in a base cabinet, ensure there is sufficient air cooling. • Have Renfert Service Team replace the motor.
Motor won't run in Auto mode.	<ul style="list-style-type: none"> • Electrical appliance defective or not connected properly. • Low power consumption of electrical appliance. 	<ul style="list-style-type: none"> • Check operability of electrical appliance and check if connector is tight. • Note that the min. rated power consumption is $P > 20 \text{ W}$.
Reduced suction power.	<ul style="list-style-type: none"> • Suction control turned down too low. • Suction hose / nozzle clogged. • Disposal bag or dust bag full. • Fine particle filter dirty. • Seal between container and top of extractor housing defective or not there. • «Free-Filter-System» mechanism defective. • Activation button for the filter cleaning function is pressed due to coiled cable. 	<ul style="list-style-type: none"> • Set suction control as described in "Suction control adjustment". • Clean suction hose / nozzle. • See section "Replacing the disposal bag, dust bag." • See section "Cleaning the fine particle filter", "Replacing the fine particle filter". • Replace seal. • Contact Renfert Service Team. • Wind the cable up so that the activation button is not actuated.
Extractor won't pick up wet substance.	<ul style="list-style-type: none"> • Dust retainer full. 	<ul style="list-style-type: none"> • Turn off unit. Empty dust retainer.

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