making work easy

- FAQ - Deutsch
- FAQ - English
- FAQ - Français
- FAQ - Italiano
- FAQ - Español
- FAQ - Português
- FAQ - Türkçe
- FAQ - Русский
- FAQ - Polski
- FAQ - Dansk
- FAQ - 中文
- FAQ - 日本語
- FAQ - 한국어
- FAQ - اللغة العربية
- FAQ - Українська

Pillo / Power pillo

FAQ - frequently asked questions

FAQ - Deutsch

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FAQ - English

For chapter indications, refer to the complete instruction manual on the internet.

Error	Possible cause	Corrective action
Renfert deflasking chisel fails to start; no sound of compressed air.	<ul style="list-style-type: none"> • Compressed air supply not established. • Ball valve on the Renfert deflasking chisel not properly / fully open. • Kink in connecting hose. • No compressed air. 	<ul style="list-style-type: none"> • Check and correct the compressed air connection. • Fully open the ball valve. • Straighten the connecting hose. • Check the compressed air system and in case switch the compressor on.
Renfert deflasking chisel fails to start but compressed air can be heard.	<ul style="list-style-type: none"> • Insufficient connecting pressure. • Motor gummed up or blocked. • Damaged piston. 	<ul style="list-style-type: none"> • Increase the connecting pressure to 4 - 6 bar. • The unit will need to be repaired by our Service Department. • The unit will need to be repaired by our Service Department.
Renfert deflasking chisel produces insufficient power.	<ul style="list-style-type: none"> • Insufficient connecting pressure. • Ball valve not fully open. • Filter in the connector fitting blocked. 	<ul style="list-style-type: none"> • Increase the connecting pressure to 4 - 6 bar. • Fully open the ball valve. • Clean the filter.

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